EN
 Insert each leg into frame tubing, making sure to align the holes. Secure the assembly by ensuring that the push-buttons have fully protruded through the holes.

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EN

This AquaSense® product is guaranteed to be free from defects in materials and workmanship for the life of the original user. This warranty is valid for the original user only and does not extend to non-durable components which are subject to normal wear and replacement. Any alterations, abuse, misuse, or accidental damage voids this warranty. For replacement or repair under warranty, please call:

CANADA & INTL: 1-800-363-2381 (Monday to Friday, 8 am to 5 pm EST)

USA: 1-866-820-2086 (Monday to Friday, 8 am to 5 pm EST)

For more information on this, or any other AquaSense® product, visit: www.myAquaSense.com

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Read carefully before use
Lire attentivement avant usage

1

Tool-free assembly (continued) Assemblage sans outils (suite)

EN

- Determine on which side of the unit you want to install the backrest. Make sure the unit will be facing the faucets with the armrest on the inside of the bathtub (fig. 1).

NOTE: To reverse the seat on model 770-434, grab the edge of the seat opposite of the commode opening and pull upward until the seat brackets snap out. Slide out and remove the seat, rotate the seat position and snap back into place (fig. 2).

- Install the 2 backrest extension pieces into the main backrest part, ensuring that the push-buttons deploy fully through the corresponding holes (backrest assembly will differ slightly depending on model, as illustrated) (fig. 3).

- Slide the backrest into the seat structure and ensure that the push-buttons have fully protruded through the holes (fig. 4).

NOTE: To reverse the backrest position, press on the push-buttons to disengage, slide out the backrest extension tubes and repeat the above step from the other side of the transfer bench.

- Snap the armrest into position, making sure the push-buttons protrude fully through the corresponding holes.

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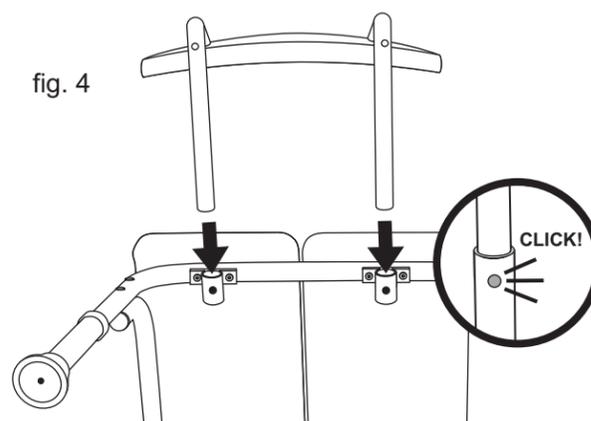
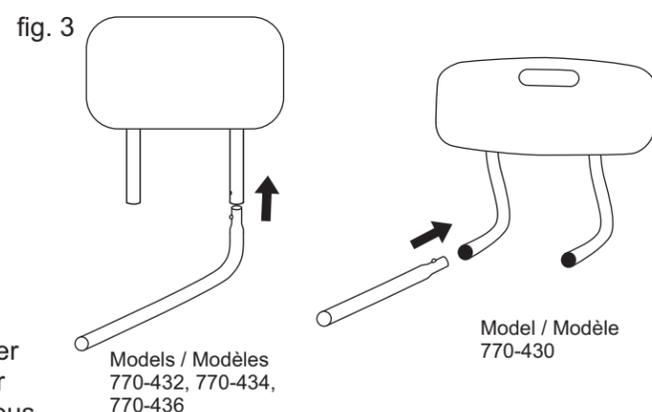
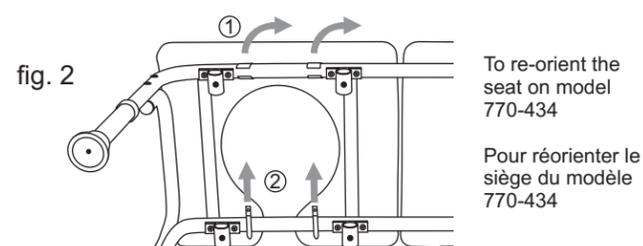
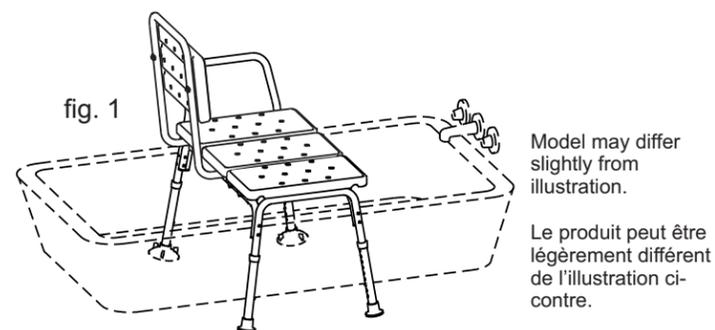
EN

To adjust height

- Adjust the unit to the desired height by pressing in the leg push-button and sliding the leg upward or downward until the button snaps into desired hole.

NOTE: The transfer bench should be level. It is therefore possible that the legs inside the bathtub may be adjusted to a different height from the legs outside the bathtub.

To use

- The inner legs should be as close to the far wall of the tub as possible.

To use (models 770-430, 770-432 and 770-434)

- Place the transfer bench with the suction tips legs inside the bathtub.
- Push down gently on the transfer bench to securely affix the suction tip feet to the tub floor.
- To remove the unit from the bathtub, always release the suction tips by first pinching the tabs to break the suction. Failure to do so will make it difficult to lift the unit.

User safety instructions

- Make sure all the push-buttons are popped out and fully engaged in selected holes.
- Before using the transfer bench, always test for stability by vigorously attempting to move the unit.
- Weight limit is 350 lbs (158 kg).

Maintenance

- We recommend that you regularly verify that all bolts and screws are properly tightened.
- Clean the pads with soap and water or mild liquid household cleaners. Avoid the use of harsh cleaning solvents, such as acetone, which may attack the plastic parts.
- If a mechanical defect appears, do not attempt to use the unit.

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