User Guide

XL40D Amplified Telephone with ClarityLogic™ and DCP™

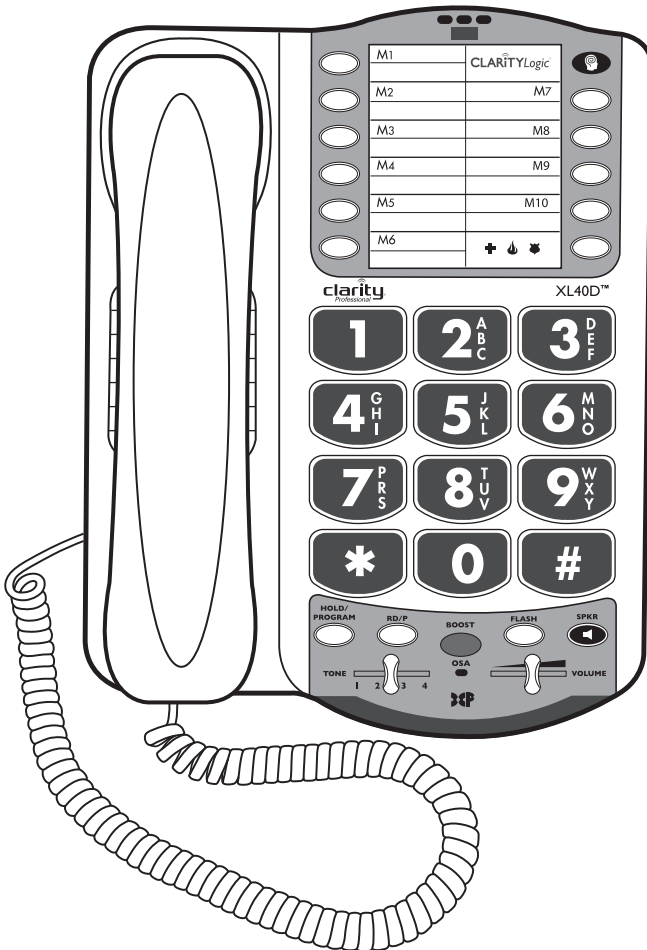
TABLE OF CONTENTS

Important Safety Instructions	4
Packaging Contents	7
Basic Installation	8
Wallmount Installation	9
XL40D Features List	10
Operations	14
Boost Function/Amplifying Incoming Sounds	14
Adjust Volume	14
Adjust Tone	14
Setting Sound Levels	15
Boost Override	15
Memory Storage	16
Outgoing Speech Amplification (OSA)	16
Technical Specifications	18
Clarity Logic™	19
ClarityLogic Memory Dial®	19
Privacy Policy	20
Troubleshooting	21

IMPORTANT SAFETY INSTRUCTIONS

When using your telephone equipment, basic safety precautions should always be followed to reduce the risk of fire, electric shock and injury to persons, including the following:

1. Read and understand all instructions.
2. Follow all warnings and instructions marked on the telephone.
3. Do not use this telephone near a bathtub, wash basin, kitchen sink or laundry tub, in a wet basement, near a swimming pool or anywhere else there is water.
4. Avoid using a telephone (other than a cordless type) during a storm. There may be a remote risk of electrical shock from lightning.
5. Do not use the telephone to report a gas leak in the vicinity of the leak.
6. Unplug this telephone from the wall outlets before cleaning. Do not use liquid cleaners or aerosol cleaners on the telephone. Use a damp cloth for cleaning.
7. Place this telephone on a stable surface. Serious damage and/or injury may result if the telephone falls.
8. Do not cover the slots and openings on this telephone. This telephone should never be placed near or over a radiator or heat register. This telephone should not be placed in a built-in installation unless proper ventilation is provided.
9. Operate this telephone using the electrical voltage as stated on the base unit or the owner's manual. If you are not sure of the voltage in your home, consult your dealer or local power company.
10. Do not place anything on the power cord. Install the telephone where no one will step or trip on the cord.
11. Do not overload wall outlets or extension cords as this can increase

SAVE THESE INSTRUCTIONS

IMPORTANT SAFETY INSTRUCTIONS

the risk of fire or electrical shock.

12. Never push any objects through the slots in the telephone. They can touch dangerous voltage points or short out parts that could result in a risk of fire or electrical shock. Never spill liquid of any kind on the telephone.
13. To reduce the risk of electrical shock, do not take this phone apart. Opening or removing covers may expose you to dangerous voltages or other risks. Incorrect reassembly can cause electric shock when the appliance is subsequently used.
14. Unplug this product from the wall outlet and refer servicing to the manufacturer under the following conditions: when the power supply cord or plug is frayed or damaged; if liquid has been spilled into the product; if the telephone has been exposed to rain or water; if the telephone has been dropped or the case has been damaged; if the telephone exhibits a distinct change in performance; if the telephone does not operate normally by following the operating instructions. Adjust only those controls that are covered by the operating instructions. Improper adjustment may require extensive work by a qualified technician to restore the telephone to normal operation.
15. Never install telephone wiring during a lightning storm.
16. Never install telephone jacks in wet locations unless the jack is specifically designed for wet locations.
17. Never touch uninsulated telephone wires or terminals unless the telephone line has been disconnected at the network interface.
18. Use caution when installing or modifying telephone lines.

SAVE THESE INSTRUCTIONS

IMPORTANT SAFETY INSTRUCTIONS

BATTERY SAFETY

IMPORTANT: Dispose of used batteries according to the instructions.

1. Do not dispose of the battery in a fire as it may explode. Check with local codes for possible special disposal instructions.
2. Do not open or mutilate the battery. Released electrolyte is corrosive and may cause damage to the eyes and skin. Additionally, it may be toxic if swallowed.
3. Exercise care in handling batteries in order not to short the battery with conducting materials such as rings, bracelets and keys. The battery or conduction material may overheat and cause burns.

SAVE THESE INSTRUCTIONS

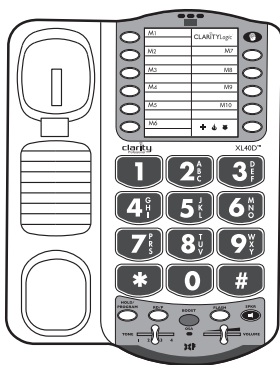
PACKAGING CONTENTS

Your package includes all the items as shown below. If there is anything missing, please contact our Customer Care Department immediately.

NOTE: Please retain a copy of proof of purchase for your records.


Handset

Base unit

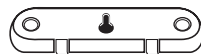
Telephone line cords
(one long cord and
one short cord)


Mounting bracket

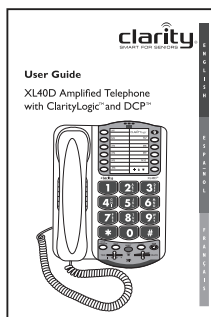DC adapter

User Guide and
Warranty/FCC Regulations

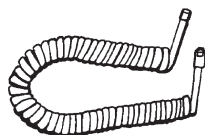Handset coil cord

XL40D INSTALLATION

Basic Installation

- Step 1:** Install the backup batteries (optional) by removing the battery cover as shown in **Figure 1**. Connect your telephone as shown in **Figure 2**.
- Step 2:** Connect one end of the phone cord into the (square) **LINE** port on the back of the phone; connect the other end of the phone cord into your wall telephone outlet.
- Step 3:** Connect one end of the (curly) handset cord into the handset jack on the left side of your base; connect the other end of the handset cord into the bottom of the handset. Place handset in cradle.
- Step 4:** Connect the DC adapter cord into the (round) **DC 9V 500MA** port on the back of your phone; plug the adapter into your wall's electrical outlet.

IMPORTANT: Use only with Class (Classe) 2 power source.

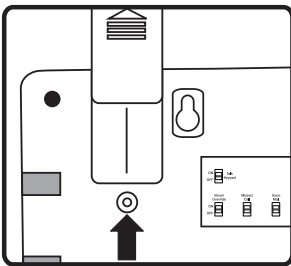
Figure 1

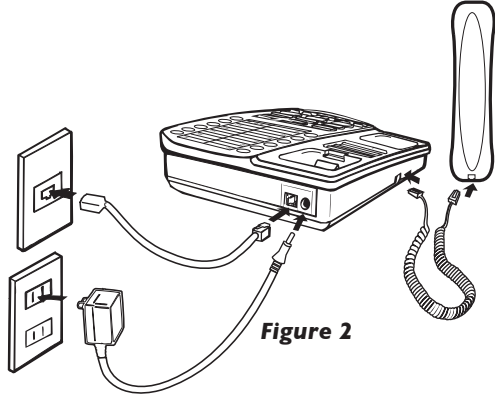
Figure 2

Note: Remove battery cover using a thin object (like the tip of a pen) to push down the tab shown by the arrow above.

When finished, lift handset to test. If you hear a dial tone, your telephone is ready for use. If not, check all connections again.

If there is a power outage, the XL40D will maintain its full amplification for up to 48 hours with four AA alkaline backup batteries (not included).

XL40D INSTALLATION

The phone operates as a regular phone if there is no power or backup batteries. In this case, the indicator lights, the loud ringer, the lighted keypad and the amplification will be disabled; you will still be able to make and receive phone calls, but the extra amplification will not be engaged. We recommend installing batteries if you live in an area with frequent power outages.

Wall Mount Installation

- Step 1:** Pull the handset hook out of the base and reverse its position so that the hook points up and will hold the handset when you hang up. (See *Figure 3*)
- Step 2:** Plug one end of the short flat telephone line cord into the jack marked **LINE** located on the back of the telephone. Then run the cord through the groove on the back of the phone. Insert the tabs of the mounting bracket into the slots in the back of the phone. Push the mounting bracket in and down firmly until it snaps into place. (See *Figure 4*)
- Step 3:** Plug the flat telephone line cord into the wall plate jack, then align the mounting bracket's keyhole slots with the wall plate studs and slide the base of the phone downward to secure it on the wall. (See *Figure 5*)

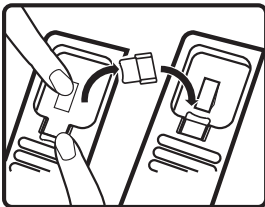
Figure 3

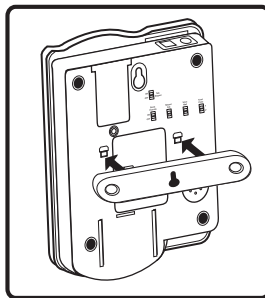
Figure 4


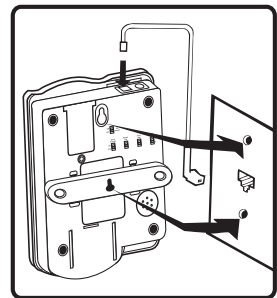
Figure 5

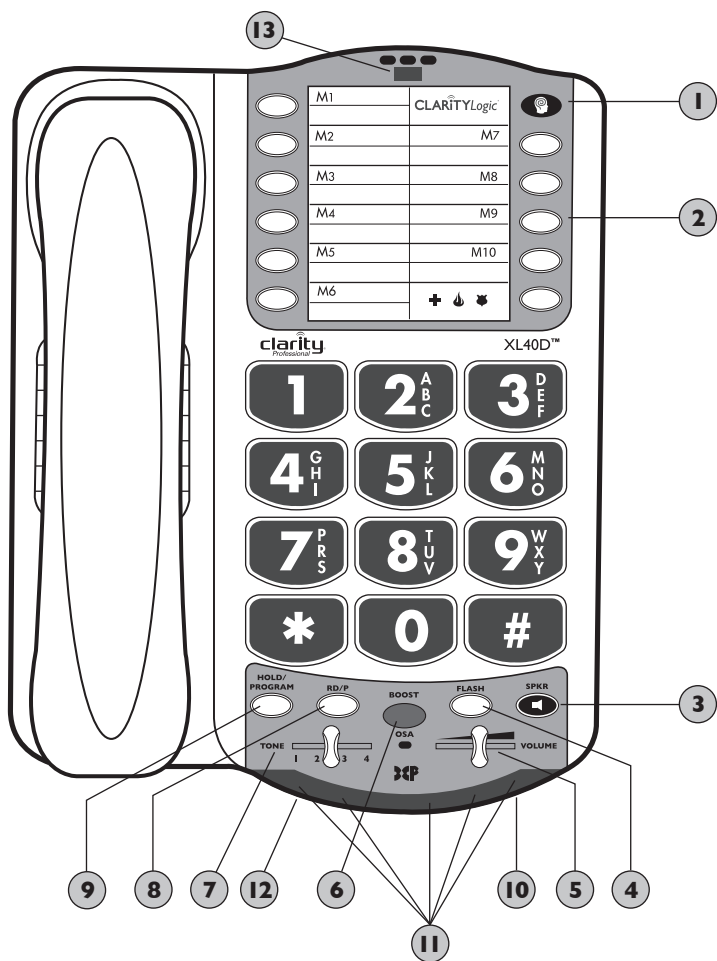XL40D FEATURE LIST


- 1 Blue ClarityLogic Button:** press the blue button for direct access to a Customer Care Representative.
- 2 Memory Buttons (M1-M10 and Emergency):** for saved numbers (see MEMORY STORAGE section for details).
- 3 SPKR (Speaker) Button:** press SPKR to engage the speakerphone function; press again to turn the feature off.
- 4 FLASH Button:** use this button to engage the Call Waiting Caller ID functions (feature provided by local phone company).

XL40D FEATURE LIST

- 5 VOLUME Slider:** move to right or left in order to increase or decrease the volume of the incoming voice.
- 6 BOOST/OSA Button:** press BOOST to engage the extra amplification. In BOOST OFF mode, the phone has a volume range of up to 16dB of amplification. In BOOST ON mode, the phone has a volume range of up to 50dB of amplification. When active, the BOOST button is backlit. Press BOOST again to turn off the extra amplification. Press and hold the button for three seconds to engage the OSA function (to enhance the level of your voice). See OSA section for details.
NOTE: When OSA is enabled, the light at the top of the phone turns orange when the phone is engaged in a call.
- 7 TONE Switch:** use this switch to pick one of the four tone settings available.
- 8 RD/P (Redial/Pause) Button:** use this button to redial the last number dialed (up to 32 digits). When in programming mode, press this button to insert a two second pause.
- 9 HOLD/PROGRAM Button:** during a call, use this button to put the call on hold; press the button again to release the call from hold. Use this button to program the memory keys during idle state or during a call. (See MEMORY STORAGE section for details.)
- 10 Low Battery Indicator:** light turns on when the backup batteries are low.
- 11 Visual Ringer Lights:** lights turn on and off when the phone rings.
- 12 Missed Call Light:** light turns on when there was an unanswered call. (Active when Missed Call switch is ON – located on bottom of phone.)
- 13 Voicemail/Ringer/OSA Light:** **a.** Light turns on GREEN when there is a voice mail left with your voicemail feature (from local phone company). **b.** Light turns RED when the ringer switch is in the OFF position. **c.** Light turns ORANGE when the OSA feature is active.


XL40D FEATURE LIST

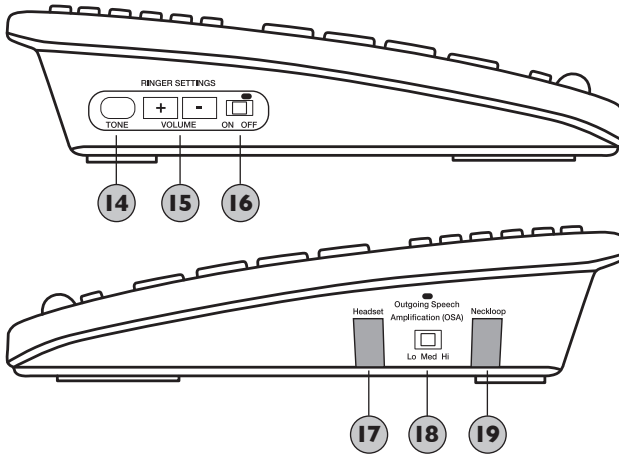SIDE VIEWS


24

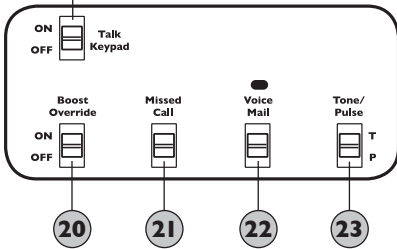BOTTOM VIEW

BACK VIEW

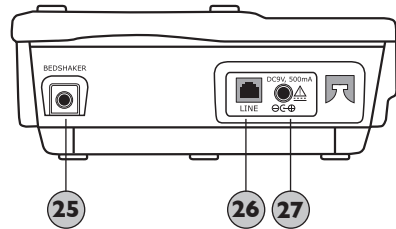- 14 (Ringer) TONE:** press this button to choose the desired incoming ring tonality. Choose one of the six available ring tones to select the most favorable ringer.
- 15 (Ringer) Volume +/-:** press the keys to increase/decrease the volume of the incoming ring.
- 16 (Ringer) ON/OFF switch:** use this switch to turn the ringer on or off. When the switch is set to OFF, the light at the top of the phone lights-up RED.
- 17 Headset Port:** insert a standard headset (2.5mm plug) to use instead of handset. For more information on purchasing please contact Clarity at 1-800-426-3738

XL40D FEATURE LIST

- 18 Outgoing Speech Amplification (OSA) Switch:** use the switch to set the level of the outgoing speech amplification level. To enable/disable the OSA switch, press and hold the BOOST/OSA button for three seconds (see OSA section).
- 19 Neckloop Port:** insert a standard neckloop (mono, 3.5mm plug) to use with your T-coil equipped hearing aid. We recommend the Clarity CE30 neckloop. For more information on purchasing please contact Clarity at 1-800-426-3738.
- 20 Boost Override Switch:** set to ON if you wish for the BOOST function to be enabled at the beginning of all calls.
- 21 Missed Call Switch:** set to ON if you wish to have the Missed Call light enabled.
- 22 Voicemail Switch:** set to ON if you wish for the light at the top of the phone to turn GREEN when you have voicemail (service from your phone service provider).
- 23 Tone/Pulse Switch:** set this switch to the desired dialing method.
- 24 Talk Keypad Switch:** set to ON if you wish to hear the numeric keys pressed during dialing - both in handset and speakerphone modes.
NOTE: “star” and “pound” keys are not part of this feature.
- 25 BEDSHAKER Port:** insert the optional bedshaker (sold separately, model number C2210). It will vibrate to indicate incoming rings. For more information on purchasing please contact Clarity at 1-800-426-3738.
- 26 LINE Port:** see INSTALLING YOUR XL40D section.
- 27 POWER Port (9V,500mA):** see XL40D INSTALLATION section.

OPERATIONS

Boost function – Amplifying incoming sounds

1. While on a call, **press the BOOST button** to engage the amplifier. The XL40D has a BOOST button that controls the loudness of the receiver. (See Figure 6) Once the BOOST button is pressed, an extra level of amplification is added over the entire range of the Volume Control. During a call, the BOOST button can toggle the Clarity Power function on or off according to the user's needs. The BOOST button will reset automatically every time the phone is hung up for at least five seconds. The BOOST light will illuminate (blue) while the BOOST function is activated.

WARNING: Volume may be at a high level. To protect hearing, lower the Clarity Power Volume Control before using the phone.

2. Adjust the **VOLUME slider** to achieve the desired sound level (up to 50dB of amplification). The Clarity Power Volume Control adjusts the level of volume in the handset both in Boost or non-Boost modes. The volume dial provides up to 16dB of volume before the BOOST button is activated. Once the BOOST button is pressed, the XL40D will provide up to 50dB of gain. (See Figure 7)

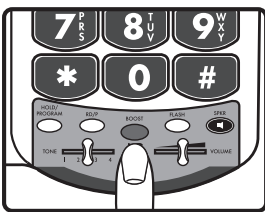
Figure 6

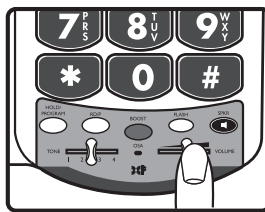
Figure 7

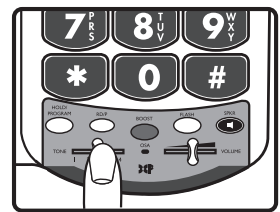
Figure 8

3. Adjusting the **TONE** switch allows you to customize your hearing experience. Since we all have different hearing preferences, we suggest exploring each setting to find the one most comfortable for you. These four settings, along with our volume amplification, are part of the Digital Clarity Power (DCP™) technology. (See Figure 8)

OPERATIONS

The available settings are:

- 1 – Low frequency emphasis: best used to hear low pitched sounds;
- 2 – Flat frequency response: best for hearing aid users;
- 3 – Clarity Power/High frequency emphasis response: best used to hear high frequency sounds;
- 4 – Multi Band Compression with high frequency emphasis response: best setting for speech intelligibility.

NOTE: Boost button is not active in speakerphone mode.

Boost Override

On the bottom of the XL40D, there is a **Boost Override** switch. When this is set to **ON** the Boost function is automatically enabled every time the phone is picked up. In standard use, when the **Boost Override** switch is set to **OFF**, the Boost function will turn off after the phone is hung up. Setting this switch to ON is suggested if all the users of the phone need the amplification. Setting this switch to OFF is useful when people with varying hearing ability use the phone. (See Figure 9)

WARNING: Volume may be loud. Proceed with care.

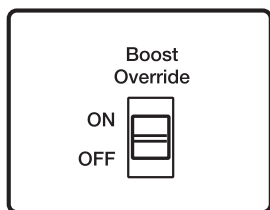
Figure 9

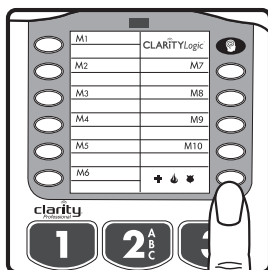
Figure 10

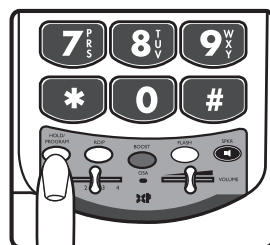
Figure 11

OPERATIONS

Memory Storage

1. Press and hold the desired memory location (**MI-M10 and Emergency**) to store the number. The phone will beep to confirm you are in memory programming mode. (See *Figure 10*)
2. Enter the telephone number you wish to store (up to 31 digits). If no button is pressed within 30 seconds, the phone will time out and go back to idle or talk mode.
3. Press the **HOLD/PROGRAM** button to save the desired number. The phone will beep to confirm the number has been saved. (See *Figure 11*)
4. To change a Memory location, reprogram using the same procedure with the button to be changed.

To dial the saved numbers:

1. In handset mode, lift the handset and press the appropriate memory button.
2. In speakerphone mode, press the appropriate memory button with the handset in the cradle.

Outgoing Speech Amplification (OSA)

This function amplifies your speech so it becomes louder to the person on the other end of the call. To enable this function, **press and hold** the **BOOST/OSA** button for three seconds. (See *Figure 12*) The phone beeps; the function can be enabled/disabled while on a phone call or in standby mode.

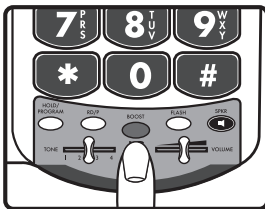
Figure 12

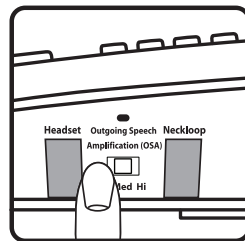
Figure 13

OPERATIONS

To verify the function is active, lift the handset and check the light at the top of the phone. If the light is **ORANGE**, the function is active. To adjust the level of amplification for your voice, use the switch on the side of the phone labeled “**Outgoing Speech Amplification**” (**OSA**). You can choose between three levels: Low, Med and High. (See *Figure 13*)

To return to the default setting for your voice volume level, **press and hold** the **BOOST/OSA** button again for three seconds. You will hear a beep confirming that the function is disabled. To verify, the light at the top of the phone should not be orange when you are on a call.

NOTE: In order to experience the best audio quality for both parties during a call, we recommend you turn on this function only if your voice is too quiet. OSA is not active in speakerphone mode.

TECHNICAL SPECIFICATIONS

Amplified dB Level: 50dB/124 dBSPL

Tone control range: From 300 to 3000Hz using one slide control

Dimensions:

Size: 9 1/2" x 7" x 3 1/4" (24.13cm x 17.78cm x 8.25cm)

Weight: 2.52 lbs. (1.14 kg)

Power Requirements:

DC Adapter: 9VDC, 500 mA

Batteries: 4 AA alkaline batteries (not included)

CLARITY LOGIC™

Introduction to Clarity Logic

ClarityLogic is the name of our simplified and easy-to-use customer support service. While talking to a Clarity representative, the telephone sends information such as its model name and settings to Clarity. The representative is immediately aware of the type and current settings of the phone. This will allow any troubleshooting call to be quick and simple, helping you find exactly what you are looking for. Just call 1-877-LOGIC24 (1-877-564-4224) to find out how Clarity is there to make life easier.

ClarityLogic® Memory Dial

For your convenience, Clarity has pre-programmed the customer support number into your phone's blue ClarityLogic button.

NOTE: ClarityLogic has been designed to work with the Plain Old Telephone Service (POTS). If your phone service is provided by a broadband service provider, ClarityLogic may not work. ONLY CLARITYLOGIC MAY BE DISABLED. ALL OTHER FEATURES WORK AS DESCRIBED IN THIS USER GUIDE.

PRIVACY POLICY

Subscription Data and Privacy Policy

Your device will automatically send Clarity various types of information as part of the Clarity services, which may include the phone number associated with your device. This information may be combined with account and other information we have on record regarding you or your device or may be transmitted to us by your device. To assist with remote diagnostics and other customer services functions, your device may also regularly send us certain information associated with your device, including software and hardware settings, the phonebook, caller ID history and other information you have stored on the device. All of such information, together with other information that we have on record regarding you and your device, is collectively referred to as the “Subscription Data.” We use this information to diagnose and troubleshoot any device issues as well as to provide you with the services.

You agree that Clarity and its subsidiaries, affiliates, partners, suppliers and agents (collectively, “Affiliates”) may collect, store, access, disclose, transmit, process and otherwise use your subscription data to provide you with the Clarity services, address your requests, provide technical support, process transactions for your account and otherwise use such subscription data in accordance with Clarity’s privacy policy (<http://clarityproducts.com/about-clarity/privacy-policy/>), which is hereby incorporated by reference. Clarity may also provide or enable certain services through your device that rely upon location information. In order to provide such services, Clarity and its Affiliates may collect, store, access, disclose, transmit, process and otherwise use your location data (including real time geographic information) in accordance with Clarity’s privacy policy.

TROUBLESHOOTING

No Dial Tone/Phone Will Not Operate

1. Check all phone cords and connections. Make sure they are plugged-in securely.
2. Plug unit into a different phone jack to help determine if the difficulty is with the phone or the phone jack.
3. Switch out handset or telephone line cords.
4. Disconnect any other equipment that may be attached to the phone.

Unable To Dial Out

1. Make sure the TONE/PULSE switch is set correctly.
2. Phone may be at the end of a long line of phones (loop). Phone may not be getting enough power from the phone line. It may be necessary to wait for a few seconds after lifting the handset to begin dialing.
3. The XL40D is not compatible with Digital PBX Systems. If a standard, single line phone works on the system, your XL40D should as well.
4. There is an advanced computer running inside the XL40D. Sometimes after the handset is picked up, it will take a little less than a second to engage the phone. Since it is not immediate, if you dial a number before the computer engages, it will not receive the entire phone number that is dialed.

The Person On The Other End Cannot Hear You

1. Too many phones or phone devices on the line may affect your phone's transmission. As a guideline, more than five phones or phone devices may over load the phone line. Disconnect one or two devices to see if that eliminates the problem. If not, contact your local phone company for load guidelines.
2. You may not be speaking directly into the transmitter. Always speak directly into the mouthpiece in your normal tone of voice.

TROUBLESHOOTING

Unable To Hear Phone Ringer

1. Too many phone or phone devices on a line may cause your phone to not ring. Disconnect one or two of the devices to see if the problem is eliminated. If not, contact your local phone company for load guidelines.
2. If power is not being supplied to the phone, the extra loud ringer will not work.
3. Check the Ringer ON/OFF switch to see if the ringer is off.

Volume Control Does Not Work

1. BOOST must be activated to turn on the additional amplification.
2. Check power connections. BOOST will not work without power or backup batteries.

Unable To Access Automated Systems

Automated systems used by banks, long distance voicemail and other applications require that a phone be set to TONE dialing. Check the switch on the bottom of the phone labeled TONE/PULSE and set to TONE. This will enable the phone to be compatible with these automated systems.

If the power goes out, or the AC Adapter is not properly connected to the phone, the amplification, extra loud ringer and the lighted keypad will not work. If you have backup batteries installed, amplification will still work, but the phone will drain the backup batteries very quickly. This will cause the need for the batteries to be changed frequently and the Low Battery indicator light to constantly illuminate.

TROUBLESHOOTING

Do I Need to Install Batteries?

Batteries are optional. They provide backup in case of a power outage. They are recommended in areas of frequent power outages. This phone uses four alkaline batteries. When installed and fully charged, the batteries maintain the high amplification for your telephone.

Still Not Working?

If, after going through this checklist, the XL40D is still not functioning properly, call ClarityLogic at 1-877-LOGIC24 (1-877-564-4224).

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Clarity, a Division of Plantronics, Inc.
6131 Preservation Drive
Chattanooga, TN 37416

Tel: 800-426-3738 Fax: 800-325-8871
E-mail: claritycs@plantronics.com
Website: www.clarityproducts.com

©2012 Clarity, a Division of Plantronics, Inc. All rights reserved. Clarity, Clarity Professional, Clarity, the amplification device, Clarity and the amplification device combined, ClarityLogic, Digital Clarity Power, DCP, XL40D, and Smart for Seniors are trademarks or registered trademarks of Plantronics, Inc.

Rev. N (9-12)
Printed in China