ALCOFIND

Breath Alcohol Tester

Semiconductor sensor
Personal use alcohol tester

DA-3000

Instruction Manual

- ENGLISH • FRENCH
- DUTCH • GERMAN
- POLISH

Digital Breathalyzer ALCOFIND DA-3000


Precise breath test using semiconductor sensor
Thank you for your purchasing our product. ALCOFIND DA-3000 is a digital breathalyzer with an advanced semiconductor sensor to analyze a breath sample to detect alcohol in the lungs and convert the measured value into %BAC, g/l(%) or mg/l.
We proudly present ALCOFIND DA-3000 which will give you high satisfaction thanks to its simple and neat design as well as its excellent quality.

Products and Models

Contents

- Digital Breathalyzer ALCOFIND DA-3000
- Two AAA size Batteries
- Mouthpiece Holder Cap (Mouthpiece adaptor)
- Two Mouthpieces
- User's Manual

Name of Parts

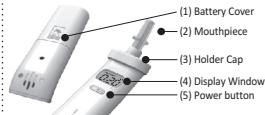Product Specifications

Indication of B.A.C.	0.00 ~ 0.40%BAC
	0.00 ~ 4.00‰
	0.00 ~ 2.00mg/l
Accuracy	+/- 0.1% at 0.5‰ at 25 °C
Warm up time	Within 20 seconds at 0.50 g/l(‰)
Sensor	Semiconductor sensor
Power supply	Two AAA size alkaline batteries
Battery Life	Approx. 300 tests
Dimensions (W x H x D)	32mm x 105mm x 23mm
Weight	52g including batteries
Calibration	every 6 months or after 200 tests
Operating temperature	5 ~ 40 °C

The specifications are subject to change without prior notice for functional improvements.

ENGLISH

How to Use

1. Before Use:

- Press down the battery cover to open it and install two AAA size 1.5V alkaline batteries paying attention to the battery polarity. Close the battery cover.
- Cover the holder cap on top of the unit and insert the mouthpiece.

2. Breath Alcohol Test

Display	Description	Display
Start	• Press the power button, and the unit is turned on with a beep and displays the accumulated test count. [00~99. In case of Re-test, test count will not be displayed.]	
Start-up	• The unit will commence a start-up process to restabilize the alcohol sensor automatically. The display window will undergo a countdown sequence from [19] to [00] for about 19 seconds. [From [08] to [00] for about 10 seconds in case of Re-test]	
Breath Sampling	• Blow steadily to the mouthpiece with one deep breath for at least 3 seconds, until you hear two short beeps after a long beep. Please be careful not to let the saliva enter into the unit, as it may damage the sensor.	
Analysis and Display g/l(%)	• When sampling is over, the unit displays [] during its analysis • After completion of analysis, the unit will display measured Blood Alcohol Concentration in three digits [0.xx] for 15 seconds and shut off the power.	
Re-Start	• If the power button is pressed within 2 minutes, the unit will start up in Re-test mode.	

Alerts

Display	Meaning	Description
	Battery Low	The battery power is low. Please replace the batteries
	Flow Error	The breath sample was not steadily enough. Please try sampling again with Soft and steady breath
	Time Out	If sampling is not performed in 20 seconds at Ready-to-Test mode, the unit displays [Er] and shuts down.
	Calibration	The unit requires recalibration or maintenance. Please get in touch with the nearest service center.

ENGLISH

Important safety information

- Allow at least 20 minutes between the last consumption of alcohol and testing.
- For more accurate testing, it is recommended to recalibrate the sensor every 6 months or after 200 tests.
- For protection of the sensor, avoid direct ray of light, extreme temperature, humid place, keep the device in a well ventilated place and keep out of reach of children.
- When not used for a long time, keep the device after taking out the batteries. In case of reuse after long-term storage, there might be high tolerance of test result for about first 3 tests.
- Do NOT leave the device together with newspaper or alcohol containing materials.
- Do NOT blow smoke, saliva or other contaminants into the mouthpiece, as the sensor may be damaged.
- Do NOT block the air hole opposite to the mouthpiece during the test, as it may damage the sensor.
- We recommend you to replace the disposable mouthpiece for sanitary purpose.
- Do NOT use the device as a tool to drink and drive. The manufacturer and seller will not take any legal responsibility for the measured test value.

Warranty Card

Product			
Digital Breathalyzer ALCOFIND DA-3000			
Purchasing Date	(DD-MM-YY)	Seller	Shop name Owner name
Warranty Period	<ul style="list-style-type: none"> ■ The manufacturer warrants the product to be free from defects in workmanship or material (excluding calibration) under normal use either for (1)one year from date of purchase or 200 breath tests whichever comes earlier. The manufacturer's obligation under this warranty is limited to replacing, adjusting or repairing the unit at manufacturer's option, if returned along with proof of purchase. ■ For any defects due to natural disaster of customer's negligence, misuse, reconstruction and repair by unauthorized person, reasonable cost shall be charged for repair or part replacement. 		

Customer Service

For any question, after sales service, product calibration or purchasing mouthpiece, please get in contact with the nearest distributor in your country.

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