## Völker Diagnostic System

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First steps

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**Deutsch**

ab Seite  3

**English**

from page  11

**Français**

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**Nederlands**

vanaf pagina  27

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The screenshot shows the Völker Diagnostic System website with a registration form. At the top right is the Völker logo: "VÖLKER A H-Bus Company". The page has a search bar and language selection. The main content area has a title "Völker Diagnostic System optimiert den technischen Service." followed by descriptive text about the system's features. Below this is a sidebar with links: "Service", "Serviceverträge", "Völker Diagnostic System", and "Anweisungen". There are also three checked checkboxes: "Völker Webserver", "Anleitung zur Registrierung am Völker Webserver", and "Technik-Seminare". At the bottom of the sidebar are links to "Impressum", "Sitemap", "Hilfe", and "Kontakt". On the right side of the page, there is a photograph of a laptop displaying the Völker logo, sitting on a desk next to a car key fob and a small electronic device. At the bottom right of the page are zoom controls ("A+", "A-", "A"). The registration form itself has fields for Vorname, Nachname, Straße, PLZ, Land, Firma, Telefon, Nummer, Ort, eMail, Hobby, and a CAPTCHA field containing "fr wW 3 w".

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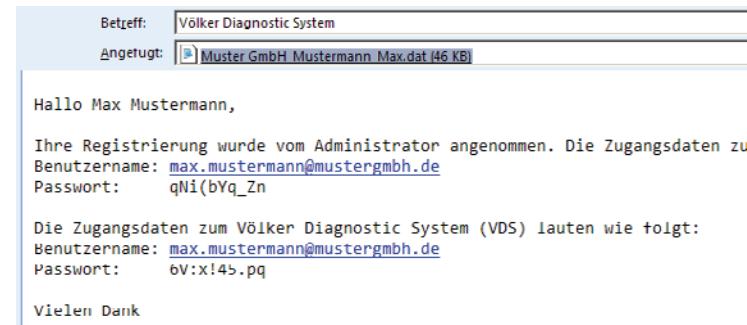
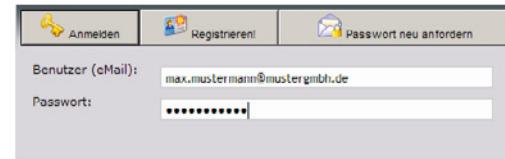
### Völker Diagnostic System

#### Anmeldung

The screenshot shows the login interface of the Völker Diagnostic System. At the top, there are three buttons: "Anmelden" (Login) with a key icon, "Registrieren!" (Register!) with a person icon, and "Passwort neu anfordern" (Request new password) with an envelope icon. Below these buttons is a horizontal line. The main area contains two input fields: "Benutzer (eMail):" with the value "max.mustermann@mustergrb.de" and "Passwort:" with the value "\*\*\*\*\*".

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# Table of content

- Version and imprint  11
- Forward and contents of delivery  12
- Registration on the Völker web server  
 13
- Registration confirmation with login data and licence file  14
- Login at Völker web server  15
- Download the VDS-software  16
- Installation  17
  - Driver  17
  - Runtime Components  17
  - VDS software  17
- Start VDS software/import licence file  
 18

# Version and imprint

**Brief instructions for use KA192**  
**Version 2.0 (2013-10-15)**  
**Völker Diagnostic System**

© by Völker GmbH 2011-2013

Völker GmbH  
Wullenner Feld 79  
58454 Witten

**GERMANY**

Phone: +49 2302 96096-0  
Fax: +49 2302 96096-16  
E-Mail: vds.help@voelker.de  
Internet: www.voelker.de

All rights reserved. Reproduction - even in part - is not permitted.

We reserve the right to make changes to reflect technical advances. All specifications are non binding.

The contents of this document are subject to change without prior notification.  
Printing errors excepted

We accept no liability for damage and operational faults caused as a result of misuse and/or non-observance of these instructions for use.

# Foreword and contents of delivery

## Foreword

Thank you very much, that you have decided for the Völker Diagnostic System (VDS). The VDS simplifies the diagnostic of Völker beds with Völker own drives and eases the repair process based upon guided repair instructions. This plus the included spare part ordering system optimises the availability of the Völker beds. Additionally the standardized documents for the technical inspection and the possibility to view all bed data via the Völker web server lead to a better documentation of the beds.

## Contents of delivery

The transport case contains the gateway, two cables to establish a link between bed and PC as well as the necessary mounting tools.

## Software

The necessary VDS software can be downloaded from the Völker web server after the registration was confirmed. The operating manual as well as Frequently Asked Questions (FAQ) can also be found there.

# 1. Registration on the Völker web server

Open your internet browser and go to site [www.volker.com](http://www.volker.com). Click in the upper menu on the topic “Service” and then on the left side on „Völker Diagnostic System“. After clicking on „Völker Webserver“ the login window will open. Select the item “Register” in this window.

Please fill in all necessary fields (identified with \*) as precisely as possible. Use the address of your institution for the address fields. This optimises the assignment of your data and the order process. After filling in all fields and the security code click on „Send data“.

If the security code should not map your entry a message will be shown. The process can be repeated as often as needed. The inserted data will remain.

After successfully entering and data transmission a message will be shown and all fields will be cleared.

The screenshot shows the Völker Diagnostic System website. At the top, there is a search bar and a language selection dropdown set to English. The header includes links for Start, About Volker, News, Products, Media, and Service. On the left, a sidebar lists Service, Service contracts, Völker Diagnostic System, and Manuals. The main content area features a heading about the Völker Diagnostic System optimizing technical service, mentioning the VDS (Völker Diagnostic System) which reads out and checks information from Völker beds via a LIN-Bus system for faster diagnosis and error correction. It also mentions guided repairs and technical inspections, software updates, and spare part orders. Below this, there are two links: "Volker Webservice" (with a gear icon) and "Instructions for the registration at the Volker Webservice" (with a document icon). At the bottom of the page, there are links for Legal notice, Sitemap, Help, and Contact. On the right, there is a photograph of a Völker bed with a laptop and a handheld device connected to it. The footer contains icons for A+, A-, and a print symbol.

## Völker Diagnostic System

### Registration

Send data   Back to Login

First Name*: John	Last name*: Doe
Street*: Streetname	Number*: 123
ZIP*: 45678	Place/City*: Cityname
Country*: United Kingdom	
Company*: Yourcompany	eMail*: john.doe@yourcompai
Telefon*: 0123456789	Mobil:

Please enter the character pattern:  
Yoss a6  
yossae

## 2. Registration confirmation with login data and license file

After the transmission of the data it will be verified by a Völker staff member. This process may take up to 2 working days.

After verification of the data you will receive a confirmation e-mail. It contains two sets of login data for the Völker web server and for the VDS software. Additionally it holds a license file in the attachment. The included passwords are generated by random and may be modified in both systems after login.

Store the license file on your disk in a place where it does not get lost – e.g. “*My documents*”. Right click on the file and select “Save as”. The license file is needed during first start of the VDS software.

Meanwhile please ensure, that the technical prerequisites for using the VDS software are given:

### Hardware:

- Processor with min. 1,4 GHz
- 1 GB RAM
- 100 MB free disk storage

### Software:

- Windows XP, Service Pack 3, .NET Framework 3.5 SP1
- Windows Vista, Service Pack 2, .NET Framework 3.5 SP1
- Windows 7, Service Pack 1, .NET Framework 4.0

For additional questions ask your IT responsible.

**NOTE** Please install the recommended Microsoft updates regularly to keep your system up to date.

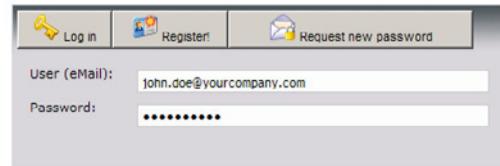
### 3. Login at Völker web server

Open your browser and go again to site „<https://diagnose.voelker.de/vbb/>“. Enter your user (e-mail address) and password you received with the confirmation e-mail and click on „Login“.

To change the random generated password to a personal password click on the right top side on “User / Change password”.

#### Völker Diagnostic System

### Login

The screenshot shows the login interface of the Völker Diagnostic System. At the top, there are three buttons: "Log in" (with a key icon), "Register" (with a person icon), and "Request new password" (with an envelope icon). Below these buttons is a form with two fields. The first field is labeled "User (eMail):" and contains the value "john.doe@yourcompany.com". The second field is labeled "Password:" and contains a series of asterisks "\*\*\*\*\*".

## 4. Download the VDS-Software

You can find the installation file for the VDS software, the driver for the gateway, as well as other downloads by selecting "Downloads / Show downloads".

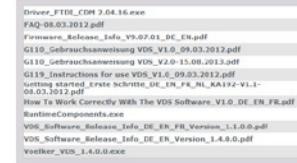
Click on the file VDS\_Setup\_2.x.xx.exe (x.xx = current version no.) to download the software. Then select "Save" to indicate a path on the hard disk.

**IMPORTANT** For the usage of the gateway an additional driver may be needed. Connect the gateway to any of the USB ports of your PC. If some LEDs are lit the gateway can be used without further action. If no LEDs are lit a driver has to be downloaded and installed. Click on the file „CDM 2.04.16.exe“ and store it on your hard disk as described before.

### Völker Diagnostic System

Diagnosis data	Firmware	Client software	Spare parts	Downloads
Show downloads				
Bed list				

### Downloads

## 5. Installation

### 5.1. Driver

When it is clear that the driver for the gateway is necessary, this one should be installed first. The installation starts with a double click on the file. Acknowledge possible security message with “Run”. The installation is performed automatically and results positive, when some LEDs are lit.

CDM 2.04.16

Runtime Components

VDS\_Setup\_2.x.xx

### 5.2. Runtime Components

After installing the driver, please install the „Runtime Components“.

### 5.3. VDS software

Start the installation of the VDS-Software and acknowledge possible security message with “Run”. Follow the instructions shown on the screen.

## 6. Start VDS-Software/Import licence file

Click on the VDS icon which is placed on the desktop after the installation.

In the following login screen the license file has to be imported. Click on “*Import license file*” and select the path under which the file has been stored before. There will be a message shown, when the import was successful.

After the import enter the login data out of the confirmation mail received before and confirm with “*Done*” to start the VDS software.

**NOTE** For further information in using the Völker web server and the VDS software please read the detailed instructions for use and the FAQ. These could be downloaded from the Völker web server under „*Downloads*“.

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The screenshot shows a registration form on a website. At the top right is the Völker logo with the text "A H-Baum Company". The page has a header with links: Départ | A propos de Völker | Actualités | Produits | Supports visuels | Maintenance. The main content area has two columns. The left column contains links: Maintenance, Contrat de maintenance, Völker système de diagnostic, and Manuels. The right column is titled "Le système de diagnostic Völker permet d'optimiser le service technique." It describes how the system allows for remote diagnostics and maintenance via a bus LIN. Below this is a section for "Völker Webserver" and "Instructions pour l'enregistrement sur le Völker Webserver". At the bottom are links for Mentions légales, Plan du site, Aide, and Contact. There are also zoom controls (A+, A-, A) and a print icon.

## Völker Diagnostic System

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The registration form has two buttons at the top: "Envoyer les données" and "Retourner au Login". The form fields are as follows:

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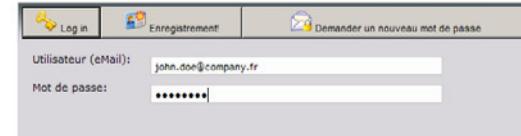
Völker Diagnostic System

## Login

The screenshot shows the login interface of the Völker Diagnostic System. At the top, there is a horizontal menu bar with three items: "Log in" (with a key icon), "Enregistrement!" (with a person icon), and "Demander un nouveau mot de passe" (with an envelope icon). Below the menu, there are two input fields. The first field is labeled "Utilisateur (eMail):" and contains the value "john.doe@company.fr". The second field is labeled "Mot de passe:" and contains a series of six asterisks ("\*\*\*\*\*").

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The screenshot shows the Völker Diagnostic System website with the following details:

- Header:** A search bar labeled "zoeken" and a language dropdown set to "Nederlands". The logo "VÖLKER A Hilti Company" is in the top right corner.
- Breadcrumbs:** Start | Over Völker | Actueel | Producten | Media | Service
- Left sidebar:** Service, Servicecontracten, Völker Diagnostic System, Aanwijzingen.
- Main Content:** A section titled "Völker Diagnostic System optimaliseert de technische service." It describes how the system integrates with Völker's own diagnostic systems to provide faster service. Below this are links for "Volker Webservices" and "Instructies voor het registreren op de Volker Webservices".
- Footer:** Colofon | Sitemap | Help | Contact. On the far right are zoom controls (A+, A-, A).
- Registration Form:** A modal window titled "Registration". It contains fields for First Name, Last Name, Street, Number, Place/City, ZIP, Country (dropdown), Company, eMail, Telefon, Mobil, and a CAPTCHA field containing "yoss a6". There are "Send data" and "Back to Login" buttons.

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Dear John Doe,

Your registration has been accepted by an administrator. You login data  
Username: [john.doe@yourcompany.com](mailto:john.doe@yourcompany.com)  
Password: qNi(bYq\_Zn

Your login data to Völker Diagnostic System (VDS) are:  
Username: [john.doe@yourcompany.com](mailto:john.doe@yourcompany.com)  
Password: 6V:x!45.pq

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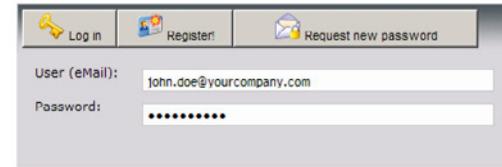
#### Völker Diagnostic System

#### Login

The screenshot shows a login interface for the Völker Diagnostic System. At the top, there are three buttons: "Log in" (with a key icon), "Register" (with a person icon), and "Request new password" (with an envelope icon). Below these buttons, there are two input fields. The first input field is labeled "User (eMail):" and contains the value "john.doe@yourcompany.com". The second input field is labeled "Password:" and contains a series of eight asterisks ("\*\*\*\*\*").

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