English

Magyar

Română

Français

Deutsch

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BTE Products

User Guide • Használati utasítás • Instrucțiuni
de utilizare • Manuel d'utilisation • Bedienungs-
anleitung • تعليمات الاستخدام

SIEMENS

English	3
Magyar	31
Română	57
Français	85
Deutsch	115
عربي	168

Content

Your BTE hearing instruments	4
Intended use	9
General notes on safety	10
Handling batteries	13
Locking and unlocking the battery compartment	14
Inserting and removing the hearing instrument	15
Turning your hearing instrument on and off	16
Changing the hearing program	18
Adjusting the volume	19
Audio input	20
Functionalities	21
Maintenance and care	25
Troubleshooting	26
Disposal information	27
Technical information	28

EN

HU

RO

FR

DE

AR

Your BTE hearing instruments

Life

**optional with
earhook**
(battery type: 312)


Life

**optional with
earhook**
(battery type:
312)

S

**optional with
LifeTube**
(battery type: 13)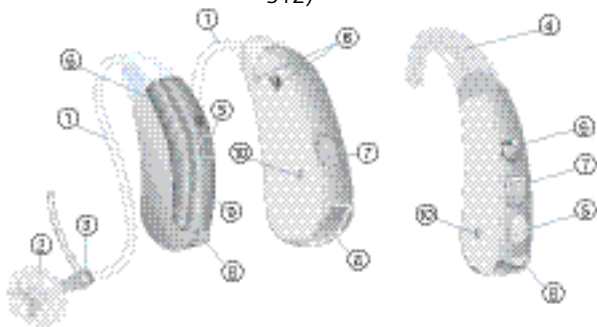

① LifeTube

② LifeTip

③ LifeTip adapter

④ Earhook

⑤ Push button
(optional)*

⑥ Microphone openings

⑦ Cover for programming socket

⑧ Battery compartment

⑨ Microphone cover

⑩ Charging contacts (recharge-
able hearing instruments only)

* Feature availability depending on your hearing instrument model.

EN

HU

RO

FR

DE

AR

S/M/DM

optional with
S-LifeTube
(battery type: 13)


S VC/ M VC/
DM VC

optional with
S-LifeTube
(battery type: 13)

MX

optional with
S-LifeTube
(battery type: 13)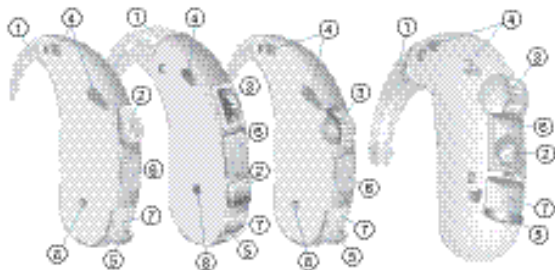

① Earhook

② Push button

③ Volume control

④ Microphone
openings

⑤ Battery compartment

⑥ Cover for programming socket


⑦ Audio input

⑧ Charging contacts (recharge-
able hearing instruments only)


P/HP/DP

(battery type: 13)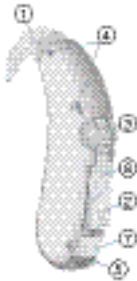

SP

(battery type: 675)

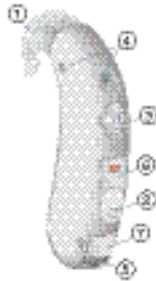
- ① Earhook
- ② Push button
- ③ Volume control
- ④ Microphone openings

- ⑤ Battery compartment
- ⑥ Cover for programming socket
- ⑦ Audio input


P

(battery type:
13)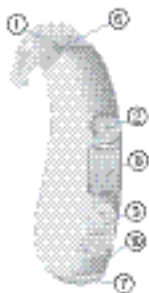

SP

(battery type:
675)

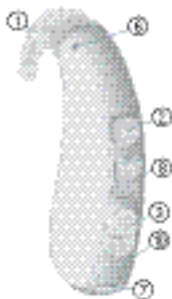Phoenix

(battery type:
13/675*)

- | | |
|------------------|--------------------------------|
| ① Earhook | ⑥ Microphone openings |
| ② Push button | ⑦ Battery compartment |
| ③ Rocker switch | ⑧ Cover for programming socket |
| ④ MTO switch | ⑨ Cover for fitting controls |
| ⑤ Volume control | ⑩ Audio input |

* Ask your Hearing Care Professional.


Explorer 500 P
optional with
S-LifeTube
(battery type: 13)

Explorer 500 M
optional with
S-LifeTube
(battery type: 13)

Prisma 2K
(battery type:
312)

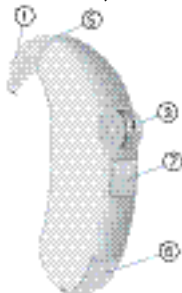- ① Earhook
- ② Push button
- ③ Volume control
- ④ LED (optional)*
- ⑤ Microphone openings

- ⑥ Battery compartment
- ⑦ Cover for programming socket
- ⑧ Audio input
- ⑨ Battery compartment lock

* Feature availability depending on your hearing instrument model.

Intended use

Hearing instruments are intended to help people with hearing loss to hear better. Diagnosis and prescription of a hearing instrument must be performed by hearing health specialists, e.g. audiologists, ENT doctors or acousticians.

Read and follow the instructions of this user guide to avoid injuries or damage to the hearing instruments.

Symbols:

Be aware to information marked with the warning symbol **WARNING**, **CAUTION** or **NOTICE**!

WARNING points out a situation that could lead to serious injuries, **CAUTION** indicates a situation that could lead to minor and moderate injuries.

NOTICE

NOTICE indicates possible property damage.

Advice and tips on how to handle your device better.

Indicates information and warnings especially for children with hearing instruments.

Instruction. Indicates that something has to be done.

Indicates an option that not every hearing instrument has. Ask your Hearing Care Professional to check off the relevant information.

General notes on safety

NOTICE

Your hearing instruments are sensitive to extreme heat, high humidity, strong magnetic fields (> 0.1T), X-rays and mechanical stress.

- ▶ Do not expose your hearing instruments to extreme temperature or high humidity.
- ▶ Do not leave them in direct sunlight.
- ▶ Do not wear them in the shower or when you apply make-up, perfume, aftershave, hairspray or suntan lotion.
- ▶ Do not wear your hearing instruments when you are exposed to short-waves, a strong magnetic field, a high frequency field or X-rays.
- ▶ Do not place your hearing instruments in a microwave oven.

WARNING

Hazard of explosion!

- ▶ Do not use your hearing instruments in areas where there is a danger of explosions (e.g. mining).

In some countries restrictions for the usage of wireless equipment exist.

- ▶ Refer to local authorities for further information.

**WARNING**

Choking hazard posed by small parts.

- ▶ If infants, small children or mentally disabled persons need to wear hearing instruments ensure adequate supervision.
- ▶ Check the integrity of the hearing instruments regularly.
- ▶ Ensure that your child does not detach the hearing instrument from the earmold.
- ▶ Consult your Hearing Care Professional if the housing is deformed.
- ▶ Keep the battery compartment locked. Verify the proper function of the locking mechanism.
- ▶ Keep batteries and accessories out of children's reach.
- ▶ If swallowed consult a physician or a hospital immediately.

NOTICE

Leaking batteries damage the hearing instruments.

- ▶ Turn the hearing instruments off when not in use to preserve the battery.
- ▶ Remove batteries when the instruments are not in use for a prolonged period of time.

NOTICE

Batteries contain harmful substances that pollute the environment.

- ▶ Do not throw used batteries into household trash.
- ▶ Dispose batteries according to national regulations or return them to your Hearing Care Professional.


CAUTION

Risk of injury caused by incorrect use of LifeTip.

- ▶ Always wear the LifeTube with the LifeTip.
- ▶ Make sure that the LifeTip is completely attached to the LifeTip adapter.
- ▶ Change the LifeTube or LifeTip when they are cracked.
- ▶ Put the LifeTube with the LifeTip carefully and not too deep into the ear.
- ▶ Use the LifeTube and LifeTip only in the way described in this user guide.
- ▶ If the LifeTip remains in the ear, have it removed by a medical professional.

Handling batteries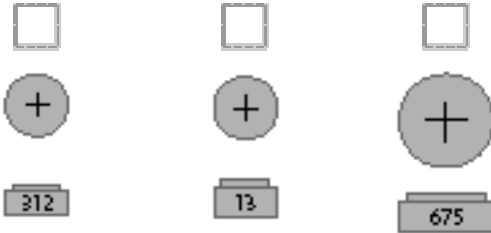

original size


- ▶ Always use the right size of batteries.
- ▶ Charge the rechargeable batteries before first use.
- ▶ Remove the tab on standard batteries only when you are ready to use them.
- ▶ The positive (+) side of the batteries is always the smooth side.
- ▶ Always keep spare batteries with you.

- ▶ Open the battery compartment completely.
- ▶ Remove the empty battery.
- ▶ Insert the proper size battery.

Make sure that the "+" symbol on the battery points up as shown in the picture.

- ▶ Close the battery compartment.

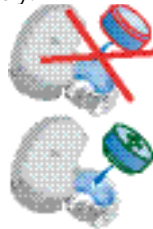Locking and unlocking the battery compartment

Option 1

Locking

- ▶ Push the small slide to the right.

Unlocking

- ▶ Push the small slide to the left.

Option 2

Locking

- ▶ Push the small slide to the bottom.

Unlocking

- ▶ Push the small slide to the top.

Option 3

Locking

- ▶ Push the slide to the right with a suitable pointed tool.

Unlocking

- ▶ Push the slide to the left with a suitable pointed tool.
- ▶ Close the battery compartment completely.

Inserting and removing the hearing instrument

EN

- ▶ Handle the hearing instrument by holding the earmold. Do not pull the Tube.

HU

Inserting

- ▶ Insert the LifeTube/earmold.
- ▶ Place the hearing instrument behind the ear.


RO

FR

Removing

- ▶ Remove the hearing instrument.
- ▶ Pull the LifeTube/earmold carefully out.

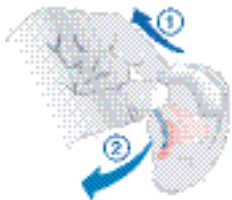DE

AR

Turning your hearing instrument on and off

Via battery compartment

On

- ▶ Close the battery compartment.

Off

- ▶ Open the battery compartment.

Via push button

On

- ▶ Press the push button for 2 seconds.

Off

- ▶ Press the push button for 2 seconds.

Via MTO switch

On

- ▶ Switch to M (microphone) or T (telecoil).

Off

- ▶ Switch to O (off).

Via volume control

On

- ▶ Rotate the volume control upwards.

Off

- ▶ Rotate the volume control completely down.

Via remote control

- ▶ Refer to the remote control user guide.

Changing the hearing program

Via push button

- ▶ Press the push button to switch to the next program.

Via MTO switch

- ▶ Switch to M (microphone) or T (telecoil).

Via remote control

- ▶ Refer to the remote control user guide.

Hearing program	Description of the hearing program
1	
2	
3	
4	
5	

Adjusting the volume


Volume control

- ▶ Rotate the volume control up to increase the volume.

- ▶ Rotate the volume control down to decrease the volume.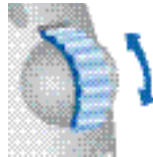

Rocker switch

- ▶ Press up to increase the volume.
- ▶ Press down to decrease the volume.

Via remote control

- ▶ Refer to the remote control user guide.

Audio input

An audio input enables the connection between the hearing instrument and an external device e.g. an FM system, mp3 player, television or an infrared device.

Audio input via an audio shoe

- ▶ Refer to the audio shoe user guide on how to attach and remove the audio shoe correctly.

Audio input integrated in the battery compartment

- ▶ Ask your Hearing Care Professional to replace the battery compartment.

- ▶ Use only the Siemens audio shoe which is designed exclusively for your hearing instrument.

WARNING

Risk of injuries caused by electric shock!

- ▶ Connect audio input only to a device conforming to IEC 60065 (IEC-Norm for audio, video and other electronic devices).

Functionalities

Your Hearing Care Professional can activate the following functionalities.

e2e wireless

- Allows the use of a remote control.
- Automatically adjusts both hearing instruments simultaneously to change the instrument settings, e.g. volume or program.

The performance of the e2e wireless function may be affected by electromagnetic interference – e.g. from a computer monitor or fluorescent lamp system.

- ▶ Move away from the source of interference if you experience difficulty.

Power-on delay

This functionality allows you to turn the hearing instrument on with an automatic delay of 6/12/18 seconds to avoid feedback (whistling).

EN

HU

RO

FR

DE

AR

Checking the hearing instruments' status

- The LED is continuously lit.
The hearing instruments are switched ON.
- The LED is off.
The hearing instruments are switched OFF.

Rechargeable hearing instruments

Rechargeable hearing instruments allow the use of standard or rechargeable batteries.

- ▶ Charge rechargeable batteries before first use.
- ▶ Do not charge standard batteries.
- ▶ Refer to the charger user guide for further information.

Automatic telephone program

- ▶ Hold a telephone near your hearing instrument (*).
The hearing instrument automatically switches to the telephone program.
- ▶ Move the telephone away from your hearing instrument.
The hearing instrument switches back to the previously used program.

- ▶ You can also select the telephone program manually by using the program button.
- ▶ When operating the hearing instrument in telecoil mode, the wireless function may cause a pulsing noise. This is due to the pick-up of the transmission signal. When your hearing instrument is in telecoil mode please use the remote control only with a distance greater than 10 cm (4 inches).

(* AutoPhone magnet accessory

Not all telephones produce a magnetic field strong enough to activate the AutoPhone function. In this case, use the Siemens AutoPhone magnet, which is available as accessory, and attach it to the telephone receiver to activate the AutoPhone function.

- ▶ See the AutoPhone magnet user guide for instructions on how to position the magnet.
- ▶ Use only the Siemens magnet. Ask your Hearing Care Professional for this accessory.

NOTICE

Magnets can disturb electrical devices and delete stored data.

- ▶ Keep magnets away from computers, color monitors, television sets, floppy discs, video- and audio discs and other electronic equipment/ devices.

WARNING

Magnets may interfere with the operation of life supporting systems (e. g. pacemakers, magnetic valves).

- ▶ Use a magnet only if not in doubt about the safe distance. The safe distance between pacemaker and magnet should be at least 10 cm (4 inches).

Maintenance and care

Your hearing instruments are highly sensitive systems. Proper care and handling will ensure the best possible performance.

Clean instruments regularly to prevent damage to the hearing instruments and to avoid health problems.

NOTICE

- ▶ Do not put the hearing instruments in water.

- ▶ Clean your hearing instruments with a soft, dry tissue, as necessary.
- ▶ If the hearing instruments sound distorted or weak ask your Hearing Care Professional to exchange the microphone cover.
- ▶ Place your hearing instruments in a "dry aid kit" with a special moisture-attracting capsule overnight.
- ▶ Ask your Hearing Care Professional for special care sets and further information on how to keep your hearing instruments in good condition.

For instruments with LifeTube

- ▶ Regularly clean LifeTubes with a cleaning wire.
- ▶ Ask your Hearing Care Professional to exchange LifeTubes and LifeTips approximately every 3 to 6 months or as needed.

For informatin on hearing, hearing loss and hearing instruments for children, please visit our Pediatric Website at:<http://www.siemens.com/hearing>

Troubleshooting

Use these tips to resolve simple issues.

Problem	Possible solution
Sound is weak.	Replace empty battery. Clean earmold. Increase volume.
Hearing instrument emits whistling sounds.	Reinsert the earmold until it fits securely. Decrease volume. Clean earmold.
Sound is distorted.	Decrease volume. Replace empty battery. Clean earmold.
Hearing instrument emits signal tones.	Gently close battery compartment completely. Replace empty battery.
Hearing instrument does not work.	Turn hearing instrument on. Gently close battery compartment completely. Replace empty battery. Make sure battery is inserted correctly. Power-on delay is active. Wait 18 seconds and check again.

Consult your Hearing Care Professional if you encounter further problems.

EN

HU

RO

FR

DE

AR

Disposal information

Within the European Union, the marked equipment is covered by "Directive 2002/96/EC of the European Parliament and of the Council of 27 January 2003 on waste electrical and electronic equipment."

Amended by "Directive 2003/108/EC" (WEEE).

- ▶ Recycle hearing instruments, batteries and accessories according to national regulations.

With the CE marking Siemens confirms compliance with the European Directive 93/42/EEC concerning medical devices.

For the products with e2e wireless 2.0 Siemens additionally confirms compliance with the European Directive 99/5/EC (R&TTE) concerning radio and telecommunications terminal equipment.

Technical information

Siemens Acuris, Artis 2, Centra, Cielo 2
IC: 267AB-WL001

Siemens Explorer 500 P,
Motion 300 BTE, Motion 500 BTE, Motion 700 BTE,
Motion 501 DP, Motion 701 DP, Motion 701 DM,
Motion 501 DM VC, Motion 701 DM VC,
Nitro 301 SP, Nitro 701 SP
Life 300, Life 500, Life 700,
Life 301, Life 501, Life 701
FCC ID: SGI-WL002BTE
IC: 267AB-WL002
Operating frequencies: $F_c=3.28$ MHz

N14203

This Class B digital apparatus complies with Canadian ICES-003.

Changes or modifications not expressly approved by SAT GmbH for compliance could void the users authority to operate the equipment.

FCC compliance

This device complies with Part 15 of the FCC Rules and with RSS-210 of Industry Canada.

Operation is subject to the following conditions:

- this device may not cause harmful interference, and
- this device must accept any interference received,

EN

including interference that may cause undesired operation.

HU

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

RO

FR

DE

AR

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment to an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

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© Siemens AG, 03.2011 · ANR
Siemens Audiologische Technik GmbH