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Adjustable lateral control headrest pad

Owners manual

This manual contains important information about this product. Please pass it to the final user upon delivery.

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ENGLISH

Physipro Inc. is delighted to have you among its clients. We sincerely wish to thank you for the trust you have shown in our Company by choosing one of our products. This user manual has been designed to allow the client to use this product, a Physipro Inc. product, safely in an optimal way. For all adjustments and settings, Physipro Inc. strongly recommends you consult a qualified professional.

Physipro Inc. provides you with specific verifications you should perform on a regular basis in order to optimize the performance of your headrest pad and extend its operational life. Physipro Inc. relieves itself from any liability should any bodily injury or property damage result from a lack of care or misuse or from any modification made to the product without prior written consent of the Company.

For Physipro Inc., your satisfaction remains a priority.

1. Recommendations

Do not use this equipment without having read and understood this user manual in its entirety. This manual includes critical information intended to ensure the safety of the user and the individuals who handle this product.

1.1 Basic verification

At the time of delivery and receipt check all parts for shipping damage before using.

1.2 Safety rules

Several safety measures must be applied to ensure the safety of the user. Note that the following list is not comprehensive. Therefore, the people who use the headrest pad have the responsibility to use it cautiously:

- ✓ Do not use the headrest pad to push or lift wheelchair.
- ✓ Installation and adjustment of your headrest pad should be performed only by an authorized dealer or qualified therapist.
- ✓ Make sure that all screws and adjustments are tightened properly, otherwise serious injury or damage may result.

2. Installation

1. Insert the headrest mounting ball **D** in the attaching collar **B**.
2. Insert bolts **C** in the attaching collar **B**.
3. Screw **B**, **C** and **D** on the headrest support plate **A**.
4. Adjust device to the desired position and tighten bolts **C**.

3. Side panel adjustment

1. Loosen bolts **H** and **J** with a 4 mm Allen key.
2. Lengthen or shorten the side panels **G** to obtain the desired position and retighten bolts **H** and **J**.

4. Angle adjustment

1. Loosen bolts **I** with a 4 mm Allen Key.
2. Adjust the side panels **G** in the desired angle and retighten bolts **I**.
3. Fasten zip fastener **F** of each side of the cover **E**.

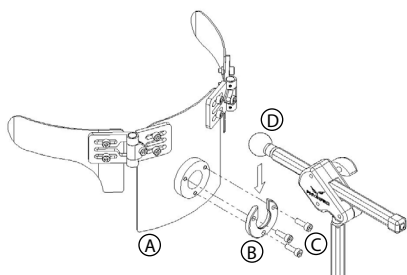
Figure 1

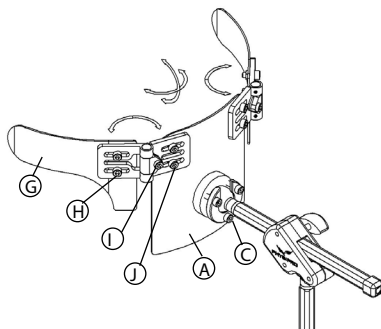
Figure 2

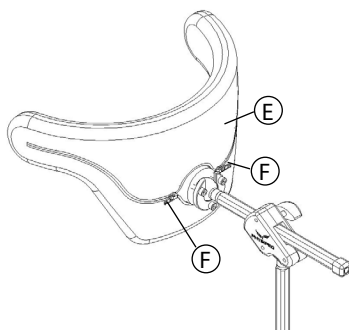
Figure 3

ENGLISH

Maintenance & cleaning

Regular cleaning and maintenance may help extend the life of the headrest pad.

To clean cover:

- 1- Remove the cover from the foam base;
- 2- Machine wash in warm water.

Hang to dry. After washing, replace the cover on the foam base.

To clean foam base:

- 1- Remove the cover;
- 2- Wipe lightly with a damp cloth;
- 3- Wipe off with a clean cloth;
- 4- Let it completely air-dry before reassembly.

Do not use soap. Do not submerge in water.

Disposal of Medical Devices

Do not burn and do not throw away this product and its protective cover outside of designated areas.

Warranty

This product is guaranteed to be free from defects in materials and workmanship for a period of 12 months from the date of purchase. This warranty covers original defects in material and workmanship of the headrest pad components, optional components and new parts that have been replaced by the supplier or by an authorized retailer, with effect from the delivery date or, if applicable, from the date of replacement. This warranty does not apply to punctures, tears or burns, nor to the removable cover. The cover is guaranteed to be free from defects in materials and workmanship for a period of six (6) months from the date of purchase, provided normal use.

Physipro Inc. agrees to repair or replace any inoperative or defective part for the duration of the warranty period. To enjoy this warranty service, please contact Physipro Inc. or an authorized retailer. Do not return this product without prior approval. If you are dissatisfied with this product, send us your comments at the address provided therein, and include the name and address of the supplier and the delivery date of the product.

Exclusions and limitations

This warranty does not apply to products that were intentionally or accidentally altered, misused, neglected, did not have the proper maintenance or had the serial number removed or erased. Furthermore, this warranty does not apply if repairs or modifications were done without written consent from Physipro Inc., or an authorized dealer. The same restrictions apply if any other kind of damages were done without Physipro Inc. involvement. This warranty will not apply for everyday use or failure to respect the rules in this manual.

Physipro Inc. will not take any responsibility for damages incurred during shipping.

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