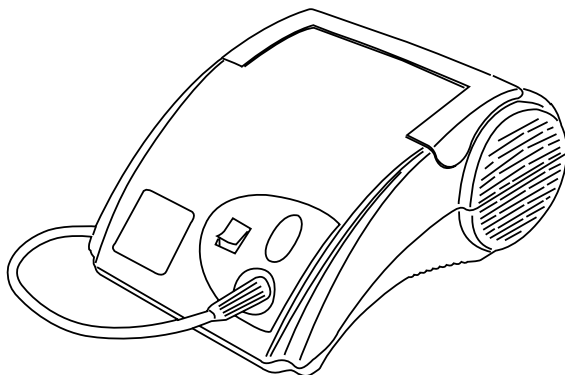
Operator's Manual

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Stratos™ Pro

Compressor Nebulizer System
Model IRC 1700

Dealer: This manual **MUST** be given to the end user.

User: **BEFORE** using this product, read this manual and save for future reference.

**For more information regarding
Invacare products, parts, and services,
please visit www.invacare.com**

Yes, you can.

 WARNING

DO NOT OPERATE THIS EQUIPMENT WITHOUT FIRST READING AND UNDERSTANDING THIS MANUAL. IF YOU ARE UNABLE TO UNDERSTAND THE WARNINGS, CAUTIONS, AND INSTRUCTIONS, CONTACT INVACARE TECHNICAL SERVICES BEFORE ATTEMPTING TO USE THIS EQUIPMENT - OTHERWISE SERIOUS INJURY OR PROPERTY DAMAGE MAY RESULT.

TABLE OF CONTENTS

SPECIAL NOTES	5
INTRODUCTION	7
TYPICAL PRODUCT PARAMETERS	8
SUPPLEMENTAL INFORMATION	10
Compressor.....	10
Extension Cords.....	10
HANDLING	11
Shipping Instructions.....	11
Unpacking.....	11
Inspection.....	11
SECTION 1—IMPORTANT SAFEGUARDS	12
To Reduce the Risk of Electrocution:	13
To reduce the risk of burns, electrocution, fire and injury to Persons:.....	14
SECTION 2—OPERATING INSTRUCTIONS	17
Set Up	17
Treatment Instructions	19
SECTION 3—CLEANING AND DISINFECTING ...	22
Cleaning the Outer Case.....	22

TABLE OF CONTENTS

TABLE OF CONTENTS

Cleaning the Nebulizer	23
Cleaning the Compressor	26
SECTION 4—MAINTENANCE	27
Routine Care	27
Storage	27
Filter Change	28
SECTION 5—TROUBLESHOOTING	29
LIMITED WARRANTY	30

SPECIAL NOTES

Signal words are used in this manual and apply to hazards or unsafe practices which could result in personal injury or property damage. Refer to the table below for definitions of the signal words.

SIGNAL WORD	MEANING
DANGER	Danger indicates an imminently hazardous situation which, if not avoided, will result in death or serious injury.
WARNING	Warning indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.
CAUTION	Caution indicates a potentially hazardous situation which, if not avoided, may result in property damage.

NOTICE

The information contained in this document is subject to change without notice.

 WARNING

Invacare products are specifically designed and manufactured for use in conjunction with Invacare accessories. Accessories designed by other manufacturers have not been tested by Invacare and are not recommended for use with Invacare products.

INTRODUCTION

The Invacare Stratos Pro Compressor Nebulizer System is a portable medical device designed to efficiently deliver physician-prescribed medication to the bronchial lung passages. The nebulizer system consists of the following components:

- Compressor
- Nebulizer
- Air tubing
- Mouthpiece
- One extra filter

TYPICAL PRODUCT PARAMETERS

TYPICAL PRODUCT PARAMETERS

Electrical Requirements:	115 VAC, 60 Hz, Single Phase
Running Current:	1.5 @ 10 psig @ 115 volts (approx.)
Sound Level:	48-62 dBA
Ambient Temperature Range:	50°F - 104°F (10°C to 40°C); 10 to 95% RH
Power Wattage:	72 Watts @ 10 psig
Electric Motor:	Class B in compliance with UL 2111
Foam Intake Filter:	Porex PE Filter.
Safety Systems:	High temperature compressor shutdown.
Width:	7.8 inches
Height:	4.2 inches
Depth:	9.2 inches
Weight:	4.2 lbs
Shipping Weight:	5.2 lbs
Regulatory Listing:	In compliance with UL 1431 Personal Hygiene and Healthcare Appliances and CSA 22.2 No. 68-M92 Motor Operated Appliances.
Maximum Pressure:	39 psi
Maximum Flow Rate:	8.5 L/min.
Operating Flow Rate:	4.8 L/min.@16 psig

TYPICAL PRODUCT PARAMETERS

Storage Conditions:	-13°F to 158°F (-25° to 70°C); 10 to 95% RH
Mode of Operation:	Intermittent Use (Thirty (30) min. ON / Thirty (30) min. OFF)
Altitude:	0 to 6500 ft above sea level

NOTE: Specifications and appearance are subject to change without notice for purposes of improvement.

NOTE: Nebulization rate and particle size are dependent on the characteristics of the nebulizer chosen.

SUPPLEMENTAL INFORMATION

Compressor

If the unit shuts down frequently, you may have a low voltage situation.

Low voltage can also be suspected when:

- The motor does not get up to full power or speed.
- Home fuses or circuit breakers activate when starting compressor.
- Lights dim or remain dim when compressor is started.
- Other motor operated appliances fail to operate properly or too many motor operated appliances are on same circuit.

Extension Cords

Make certain extension cord is in good condition. Make certain extension cord wire is NOT LESS than 18 gauge for 25 feet.

HANDLING

Shipping Instructions

The shipping container has been designed to assure protection of this device.

If this device is to be reshipped by common carrier, it should be packed in the same carton.

Unpacking

1. Check for any obvious damage to the carton or its contents. If damage is evident, DO NOT use. Notify Carrier/Invacare or your local Invacare Dealer.
2. Remove all the loose packing from the carton.
3. Carefully remove all the components from the carton.

NOTE: Retain carton and packing materials for use in shipping or storing the unit until use is required.

Inspection

1. Examine exterior for nicks, dents, scratches or other damage. Inspect all components.

SECTION I —IMPORTANT SAFEGUARDS

WARNING

SECTION I - IMPORTANT SAFEGUARDS contains important information for the safe operation and use of this product. **DO NOT** use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as **Owner's Manuals, Service Manuals or Instruction Sheets** supplied with this product or optional equipment. If you are unable to understand the **Dangers, Warnings, Cautions or Instructions**, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment - otherwise, **injury or damage may occur.**

Federal law restricts this device to sale by or on the order of a physician.

When using electrical products, especially when children are present, basic safety precautions should always be followed, including the following.

SECTION I—IMPORTANT SAFEGUARDS

READ ALL INSTRUCTIONS BEFORE USING

DANGER

Risk of electric shock. DO NOT disassemble. Refer servicing to qualified service personnel.

FIRE RISK - DO NOT use this product in an oxygen enriched environment (i.e – oxygen tent).

To Reduce the Risk of Electrocution:

ALWAYS unplug this product IMMEDIATELY after using.

DO NOT use while bathing.

DO NOT place or store product where it can fall or be pulled into a tub or sink.

DO NOT reach for product that has fallen into water. Unplug IMMEDIATELY.

DO NOT place in or drop into water or other liquid.

DO NOT reach for a product that has fallen into water. Unplug IMMEDIATELY.

WARNING

To reduce the risk of burns, electrocution, fire and injury to Persons:

This product should NEVER be left unattended when plugged in.

Close supervision is necessary when this product is used by, on, or near children and physically challenged individuals.

Use this product only for its intended use as described in this manual and only with the prescribed medication. Use this product only under physician's direction.

Invacare products are specifically designed and manufactured for use in conjunction with Invacare accessories. Accessories designed by other manufacturers have not been tested by Invacare and are not recommended for use with Invacare products.

NEVER operate this product with a damaged cord or plug, if it is not working properly, if it was dropped or damaged, or if it was dropped into water. Call qualified service personnel for examination and repair.

WARNING

Keep the nebulizer tubing, cord, and unit away from heated or hot surfaces, including space heaters, electric blankets, stoves and similar electrical appliances.

NEVER drop or insert objects into any opening or hose.

NEVER use while sleeping or drowsy.

NEVER block the air openings of the product or place it on a soft surface, such as a bed or couch, where the air openings may be blocked. Keep the air openings free from lint, hair and the like. Blocked openings may cause the unit to shut down. Contact dealer immediately.

The Stratos compressor nebulizer system uses the surrounding air to effectively administer the prescribed medication through the nebulizer. The user and administrator should be aware of surroundings so as not to have contaminated air filtered through the compressor nebulizer system. (Examples can be air contaminated with paint or cleaning fumes, weed killer, car exhaust, etc.)

DO NOT use the Stratos compressor nebulizer system in these environments. Otherwise, serious bodily injury may occur.

This unit is oil-less. DO NOT lubricate.

SECTION I —IMPORTANT SAFEGUARDS

WARNING

This equipment is NOT suitable for use in the presence of a flammable anesthetic mixture with air or with nitrous oxide.

DO NOT operate the compressor while it is still in the carrying case.

When not administering treatment, keep out of the reach of small children. Some of the parts of the nebulizer are small enough for children to swallow. The tubing may present a strangulation hazard.

NEVER touch the unit with wet hands.

DO NOT pull on the power cord to unplug the unit from an electrical outlet.

DO NOT overload wall outlets.

Pump maintenance MUST be performed by a qualified service technician.

SAVE THESE INSTRUCTIONS

SECTION 2—OPERATING INSTRUCTIONS

Your nebulizer package includes a nebulizer mouthpiece, nebulizer and air tubing. These items are sold as a package only.

WARNING

Remove the compressor from the carrying case before use.

DO NOT operate unit without filter.

Avoid operating this product in a dusty environment. Otherwise, premature compressor wear could occur.

Clean and disinfect the nebulizer and mouthpiece before using the device for the first time. Refer to Cleaning and Disinfecting on page 22.

Set Up

NOTE: For this procedure, refer to FIGURE 2.1 on page 19.

1. Place the device on a table so that the controls can be reached when seated.

SECTION 2—OPERATING INSTRUCTIONS

CAUTION

Make sure the power switch is in the off position.

2. Remove nebulizer, nebulizer mouthpiece and air tubing from the carrying case.
 3. Secure one end of the tubing to the compressor air outlet.
 4. Connect the other end to the nebulizer.
-

CAUTION

DO NOT add over six ml of medication to the medication cup. The maximum capacity of the nebulizer medication cup is 6 ml.

WARNING

In the case of high humidity, condensation may form inside the internal tubing. Run the unit for two minutes before connecting the nebulizer to the tubing.

5. Add liquid medication to the nebulizer according to your physician's prescription.
-

SECTION 2—OPERATING INSTRUCTIONS

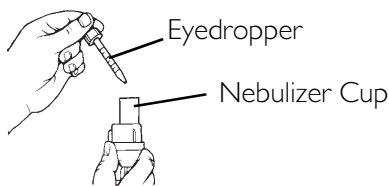
FIGURE 2.1 Adding Medication

⚠ WARNING

Before plugging in the line cord, make sure that the power switch is in the OFF (O) position.

6. Remove the line cord from the storage bin in the rear of the unit.
7. Plug into a 115V/60Hz wall outlet.
8. Insert the nebulizer mouthpiece into the top of the nebulizer.

NOTE: Before attempting to start a cold unit, let the unit reach room temperature in a heated area.

Treatment Instructions

NOTE: For this procedure, refer to FIGURE 2.2 on page 20.

1. Press the power switch ON (-) to start the compressor.

SECTION 2—OPERATING INSTRUCTIONS

2. Place mouthpiece between your teeth and close your mouth.

CAUTION

This device is for intermittent use: 30 min. ON/30 min. OFF. Failure to operate within these limits could damage the device beyond repair.

FIGURE 2.2 Using the Nebulizer

3. As aerosol begins to flow, inhale deeply, slowly, and hold breath for ten seconds. Then exhale slowly through the mouthpiece.

NOTE: If you need to rest during a treatment, place the nebulizer in the round holder located within the handle opening.

NOTE: Begin treatment according to your physician's and/or respiratory therapist's instructions. Medication is dispensed as you tilt the nebulizer towards your mouth.

SECTION 2—OPERATING INSTRUCTIONS

NOTE: Tighten caps on medication jars.

4. When treatment is complete, turn the unit off by setting the switch to the “O” position and unplug the unit from the electrical outlet.
5. Before storing supplies between treatments, clean and dry nebulizer. Wash the nebulizer and accessories. Refer to Cleaning and Disinfecting on page 22.

SECTION 3—CLEANING AND DISINFECTING

Following the cleaning instructions after each use will prevent any remaining medication in the bottle from drying and causing the device to nebulize ineffectively.

It is recommended that the disposable nebulizer and the mouthpiece be thoroughly rinsed with hot tap water after each use and cleaned with a mild detergent after the last treatment of the day. The same pieces **MUST** also be disinfected daily by soaking in a vinegar/water solution or a commercially available disinfectant. If your physician or respiratory therapist specifies a different cleaning procedure, follow their instructions.

Cleaning the Outer Case

WARNING

Unplug unit before cleaning.

DO NOT submerge in water to clean. Clean outer case with damp cloth.

DANGER

DO NOT remove outer case of this unit. Removing the case creates a serious electrical shock hazard. All disassembly and maintenance of this unit **MUST** be done by a qualified technician.

Cleaning the Nebulizer

WARNING

To prevent possible risk of infection due to contamination, follow these instructions:

- Clean the nebulizer before initial operations.
 - Clean the nebulizer after each aerosol treatment.
 - **ALWAYS** use fresh cleaning solution.
 - **DO NOT** towel dry the nebulizer components after cleaning.
-

NOTE: Your physician and/or a Invacare Dealer may specify certain cleaning procedures. If so, follow their recommendations. If not, clean according to the following instructions:

SECTION 3—CLEANING AND DISINFECTING

Vinegar Disinfecting Method

1. Remove the nebulizer and mouthpiece from the tubing.
2. Disassemble nebulizer chamber by turning counterclockwise and separating.
3. Fill two plastic containers or bowls:
 - A. Washing: Hot water/detergent solution.
 - B. Soaking: Hot water/vinegar solution (one part vinegar to three parts water).
4. Thoroughly clean the nebulizer and mouthpiece in hot water/detergent solution.
5. Remove from the water/detergent solution and rinse with clear hot tap water.
6. Soak in hot water/vinegar solution for 30 minutes.
7. Rinse with hot tap water again and air dry thoroughly.

NOTE: If using medical disinfectant cleaners, follow manufacturer's instructions carefully.

8. Keep the outer surface of the tubing dust-free by wiping regularly.

NOTE: The nebulizer air tubing does not have to be washed internally because only filtered air passes through the tubing.

Medical Disinfectant Method

NOTE: If using medical disinfectant cleaners, follow manufacturer's instructions carefully.

Consult your physician or respiratory therapist regarding use of medical disinfectant cleaners and carefully follow the instructions supplied by the manufacturer of the medical disinfectant.

Changing the Nebulizer

The nebulizer is designed for single patient use.

The nebulizer **MUST** be replaced after a long period of inactivity, or when it is obstructed by dry medication, dust, etc. The reusable nebulizer should last from six months to one year with general use. The disposable nebulizer should last 30 days.

Cleaning the Compressor

CAUTION

Unplug unit before cleaning.

NEVER submerge the nebulizer compressor in water, hold under running water or spray with water or other liquids. This will result in compressor damage.

If fluid gets into the area around the compressor outlet, wipe the area off with a dry cloth and let the nebulizer sit on its front end for five minutes. Then run the compressor for two minutes without the nebulizer or tubing adapter in place.

DO NOT clean using abrasive cleaners.

1. Clean the casing of the main unit daily by using a soft cloth moistened with water or a mild detergent.
2. Wipe the casing and immediately dry using a soft clean cloth.

SECTION 4—MAINTENANCE

Routine Care

To keep your device in the best condition and protect the unit from damage, follow these directions:

- DO NOT subject the device or the parts to any strong shocks such as dropping the device on the floor.
- DO NOT store the device in extreme hot or cold temperatures, high humidity or under direct sunlight.
- Use only Invacare parts and accessories.

Storage

1. Put the nebulizer and mouthpiece in the carrying case.
2. Coil the air tubing.
3. Wrap the AC cord in a coil.
4. Zip the bag closed.

NOTE: Store the repackaged device in a dry area. DO NOT place other objects on top of the repackaged device.

Filter Change

CAUTION

DO NOT use cotton or any other material.

DO NOT wash or clean the filter.

Use only Invacare filters.

DO NOT operate without a filter.

The filter must be replaced after approximately 30 hours of use or when it turns GREY. Open the filter holder to replace the filter with a new one.

SECTION 5—TROUBLESHOOTING

PROBLEM	SOLUTION
Device does not switch ON.	1. Make sure the plug is firmly fitted to the wall socket. 2. Try plugging into another wall socket that is known to work. 3. Contact your Invacare representative.
Device does not nebulize, or it nebulizes weakly.	1. Make sure that the ends of the air tubing are fitted tightly onto the main unit and onto the nebulizer. 2. Check whether the nebulizer is empty of medication or has too much medication (max. six ml). 3. Check whether the nebulizer nozzle is obstructed; 4. Check whether the funnel is inserted. If these possible solutions DO NOT work, contact your medical dealer.

LIMITED WARRANTY

PLEASE NOTE: THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.

This warranty is extended only to the original purchaser who purchases this product when new and unused from Invacare Corporation or a dealer. This warranty is not extended to any other person or entity and is not transferable or assignable to any subsequent purchaser or owner. Coverage under this warranty will end upon any such subsequent sale or other transfer of title to any other person. This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state.

Invacare warrants this product when purchased new and unused to be free from defects in materials and workmanship for a period of five years from date of purchase from Invacare or a dealer, with a copy of the seller's invoice required for coverage under this warranty. If within such warranty period any such product shall be proven to Invacare's satisfaction to be defective, such product shall be repaired or replaced, at Invacare's option. This warranty does not apply to normal wear and tear or failure to adhere to the product instructions, nor does it include shipping charges incurred in replacement part installation or repair of any such product. Invacare's sole obligation and your exclusive remedy under this warranty shall be limited to such repair and/or replacement.

For warranty service, please contact the Invacare service department during normal business hours at the toll-free telephone number identified on the back cover of this manual.

LIMITED WARRANTY

Upon receiving notice of an alleged defect in a product, Invacare will issue a serialized return authorization. It shall be the responsibility of the purchaser to pack the product(s) or part(s) in an appropriate manner to avoid shipping damage and return the product(s) or part(s), at the purchaser's expense, to either Invacare's plant or service center as specified by Invacare in advance. The defective product(s) or component part(s) must be returned for warranty inspection using the serial number as identification within 30 days of the return authorization date. **DO NOT** return any product(s) or part(s) to our factory or service center without prior consent. **C.O.D.** shipments will be refused – please pre-pay shipping charges.

LIMITATIONS AND EXCLUSIONS: THE FOREGOING WARRANTY SHALL NOT APPLY TO SERIAL NUMBERED PRODUCTS IF THE SERIAL NUMBER HAS BEEN REMOVED OR DEFACED; PRODUCTS SUBJECTED TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE, OR STORAGE; PRODUCTS MODIFIED WITHOUT INVACARE'S EXPRESS WRITTEN CONSENT (INCLUDING, BUT NOT LIMITED TO, MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS); PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE; PRODUCTS DAMAGED BY CIRCUMSTANCES BEYOND INVACARE'S CONTROL; OR PRODUCTS REPAIRED BY ANYONE OTHER THAN INVACARE OR AN AUTHORIZED INVACARE SERVICE CENTER. SUCH EVALUATION SHALL BE SOLELY DETERMINED BY INVACARE.

LIMITED WARRANTY

THE FOREGOING EXPRESS WARRANTY IS EXCLUSIVE AND IN LIEU OF ANY OTHER WARRANTIES WHATSOEVER, WHETHER EXPRESSED OR IMPLIED, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, AND THE SOLE REMEDY FOR VIOLATIONS OF ANY WARRANTY WHATSOEVER, SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. THE APPLICATION OF ANY IMPLIED WARRANTY WHATSOEVER SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESS WARRANTY PROVIDED HEREIN. INVACARE SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

SOME STATES DO NOT ALLOW THE EXCLUSION OR LIMITATION OF INCIDENTAL OR CONSEQUENTIAL DAMAGE, OR LIMITATION OF HOW LONG AN IMPLIED WARRANTY LASTS, SO THE ABOVE EXCLUSION AND LIMITATION MAY NOT BE APPLICABLE.

THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE/PROVINCIAL LAWS AND REQUIREMENTS.

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© 2006 Invacare Corporation
trademarks of Invacare Corporation.
Stratos is a trademark of Invacare