Assist Rails/Rampes de lit

EN **Model 6632**
 User Manual 1

FR-CA
 Manuel d'utilisation 5

I General

1.1 Symbols

Signal words are used in this manual and apply to hazards or unsafe practices which could result in personal injury or property damage. See the information below for definitions of the signal words.

⚠ DANGER!
 - Danger! indicates an imminently hazardous situation which, if not avoided, will result in death or serious injury.

⚠ WARNING!
 - Warning indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

⚠ CAUTION!
 - Caution indicates a potentially hazardous situation which, if not avoided, may result in property damage or minor injury or both.

ℹ Gives useful tips, recommendations and information for efficient, trouble-free use.

Yes, you can.

This manual **MUST** be given to the user of the product.
 BEFORE using this product, read this manual and save for future reference.

www.invacare.com

Invacare Corporation
USA and Canada
 One Invacare Way
 Elyria, Ohio USA
 44036-2125
 800-333-6900
 1130185-D 2014-6-27

Designed by Invacare Corporation
Manufactured in Mexico for Invacare Corporation

© 2014 Invacare Corporation. All rights reserved. Reproduction, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by "™" and "®". All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.

Making Life's Experiences Possible™

Yes, you can.

NOTES

1. Document size: 8.5 x 11 or A4
2. Document dimension: Portrait
3. Page count: 12 pages
4. Color: Black and white
5. **DO NOT** print this drawing. For informational purposes only. See attached document.

APPROVED ORIGINAL

Tara K Pennington

 5/20/14

TITLE: **MANUAL, 6632 Assist Rails**

PART NUMBER: 1130185	REV: D	SHEET: 1 OF 1
--------------------------------	------------------	-------------------------

ALL DIMENSIONS ARE IN INCHES UNLESS OTHERWISE SPECIFIED
 TOLERANCES (UNLESS OTHERWISE SPECIFIED):

<u>INCHES</u>	<u>METRIC</u>
DECIMALS: ____ X = N/A	X = ± 3mm
.X = ± .1	.X = ± .5mm
.XX = ± .02	.XX = ± .13mm
.XXX = ± .005	.XXX = ± N/A
FRACTIONS: _____ ± 1/64	± 1/64
ANGLES: _____ ± .5°	± .5°

**NATIVE FILES CONTROLLED BY INVACARE
 TECHNICAL WRITING ONLY**

Assist Rails/Rampes de lit

EN Model 6632	
User Manual	1
FR-CA	
Manuel utilisation	5

EN

I General

I.1 Symbols

Signal words are used in this manual and apply to hazards or unsafe practices which could result in personal injury or property damage. See the information below for definitions of the signal words.

DANGER!

– Danger indicates an imminently hazardous situation which, if not avoided, will result in death or serious injury.

WARNING!

– Warning indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

CAUTION!

– Caution indicates a potentially hazardous situation which, if not avoided, may result in property damage or minor injury or both.

Gives useful tips, recommendations and information for efficient, trouble-free use.

This manual **MUST** be given to the user of the product.
BEFORE using this product, read this manual and save for future reference.

Yes, you can.

2 Safety

2.1 General Guidelines

WARNING!

Risk Of Injury Or Damage

Improper use of this product may cause injury or damage.

- If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment
- Do not use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as user manuals, service manuals or instruction sheets supplied with this product or optional equipment.

Continued use of the bed with damaged parts could lead to the bed malfunctioning, causing injury to the user and/or caregiver.

- Check ALL bed components and carton for damage, and test components before use. In case of damage or if the bed is not working properly, contact a technician or Invacare for repair.

To avoid injury or product damage:

- Set-up and Assembly instructions **MUST** be performed by qualified technicians only. “Qualified Technician Only” sections are marked accordingly.

DANGER!

Risk Of Injury Or Death

Bed accessories designed by other manufacturers have **NOT** been tested by Invacare. Use of **NON-Invacare** bed accessories may result in injury or death.

- Use **ONLY** Invacare rails, mattresses, bed extenders and other accessories with Invacare bed products.

DANGER!

Risk Of Death, Injury, Or Damage

Conditions such as restlessness, mental deterioration and dementia or seizure disorders (uncontrolled body movement), sleeping problems, and incontinence can significantly impact a patient's risk of entrapment. Pediatric patients or patients with small body size may also have an increased risk of entrapment.

- Monitor patients with these conditions frequently.

DANGER!

Risk Of Death, Injury, Or Damage

Patient entrapment from the use of bed side rails may cause injury or death. To avoid patient entrapment:

- The Invacare mattress **MUST** fit firmly against the bed frame **AND** bed side rails to prevent patient entrapment. Follow the manufacturer's instructions. Monitor patient frequently. Read and understand the User Manual prior to using the Invacare bed and bed rails. User manuals are available at www.invacare.com or your dealer.
- After any adjustments, repair or service and before use, make sure all attaching hardware is tightened securely.

DANGER!

Risk Of Death, Injury, Or Damage

Proper patient assessment and monitoring, and proper maintenance and use of equipment is required to reduce the risk of entrapment. Variations in bed rail dimensions, and mattress thickness, size or density could increase the risk of entrapment.

- Visit the FDA website at <http://www.fda.gov> to learn about the risks of entrapment. Review “A Guide to Bed Safety”, published by the Hospital Bed Safety Workgroup, located at www.invacare.com. Use the link located under each bed rail product entry to access this bed's safety guide.

DANGER!

Risk Of Death, Injury, Or Damage

To avoid entrapment, product damage, and/or personal injury:

- When used with a manual/electric bed, the bed rails **DO NOT** fall under any weight limitations. Bed rails can be deformed or broken if excessive side pressure is exerted on the bed rails. These bed rails are used for the purpose of preventing an individual from inadvertently rolling out of bed. The bed rails are not intended nor may be used for restraint purposes. If an individual is capable of injuring himself/herself, a physician or a healthcare professional should be consulted for alternative means of safe restraint.
- After raising/lowering the head/foot end of the electric bed, check the distance between the bottom of the bed rail and the mattress. If there is excessive distance between the bottom of the bed rail and the mattress in which individuals may become entangled, adjust the height of the bed rail (if applicable), or provide alternative means of patient protection.
- **DO NOT** use the side rails as push handles for moving the electric bed.
- Replacement mattresses and bed side rails with dimensions different from the original equipment supplied or specified by the bed frame manufacturer are not interchangeable. Variations in bed side rail design, width and thickness or firmness of the mattress could cause/contribute to entrapment.

WARNING!

Risk of Injury or Damage

To avoid injury or damage from instability and misuse:

- Install assist rails only in the pre-punched mounting holes in the bed frame. When mounted correctly in the minimum distance between the assist rail in the assist (up) position and the head board will be approximately 17 inches.

**WARNING!****Risk of Injury or Damage**

To avoid injury from bed side rails or damaging the product:

- These bed rails are intended to prevent an individual from inadvertently rolling out of bed. Do not use for restraint purposes.
- Invacare recommends the use of two sets of half length bed rails: one set at the head end of the bed, and one set at the foot end of the bed where permitted.
- Although bed rails are not rated to any specific weight limitation, the bed rails may become deformed or broken if excessive side pressure is exerted on the bed rails. This bed rail is not an assist rail for getting into or out of bed. Do not use the bed rails as push handles when moving the bed.

**WARNING!****Risk of Injury or Adverse Health Consequences**

Fall hazard exists due to incorrect installation of bed rails.

- To avoid injury, ensure that the Invacare bed rails are installed correctly and securely attached to the bed while entering or exiting.

Fall hazard exists due to use of NON-Invacare bed rails. Non-Invacare bed rails are potentially incompatible with Invacare beds.

- To avoid injury, ensure that only Invacare bed rails are used with Invacare beds at all times.

Fall hazard exists due to use of NON-Invacare mattresses. Non-Invacare mattresses are potentially incompatible with Invacare beds.

- To avoid injury, ensure that only Invacare mattresses are used with Invacare beds at all times.

3 Installation

3.1 Installation

Tools required:

- 1/2 inch deep socket or use extension

For this procedure, refer to FIGURE 1.

Assist rails are shipped in pairs, one right and one left.

Right and left are determined by the user's position as if in the bed.

1. Elevate the head section to allow access to the bed frame.
2. On the left side of the bed, align the two top threaded bushings of the assist rail with the bed frame mounting holes.
3. Secure the assist rail to the bed frame using two mounting bolts and locking washers.

**WARNING!****Risk of Injury or Damage**

To avoid injury or damage from instability:

- DO NOT use assist rails without crossbars installed.

**CAUTION!****Risk of Damage**

To avoid damage from pull tube interference:

- Install the crossbar as shown.


4. Align the crossbar with the lower threaded bushing of the left assist rail.
5. Secure the crossbar to the left assist rail using one mounting bolt and a locking washer in two locations.
6. Repeat STEPS 1 and 2 for the right assist rail on the right side of the bed.
7. Align the crossbar with the lower threaded bushing of the right assist rail.
8. Secure the crossbar to the left assist rail using one mounting bolt and a locking washer in two locations.

Make sure that tube is positioned above the mounting holes before tightening.

9. Torque the bolts from STEPS 2 and 7 to 14 to 16 inch-pounds.

Figure 1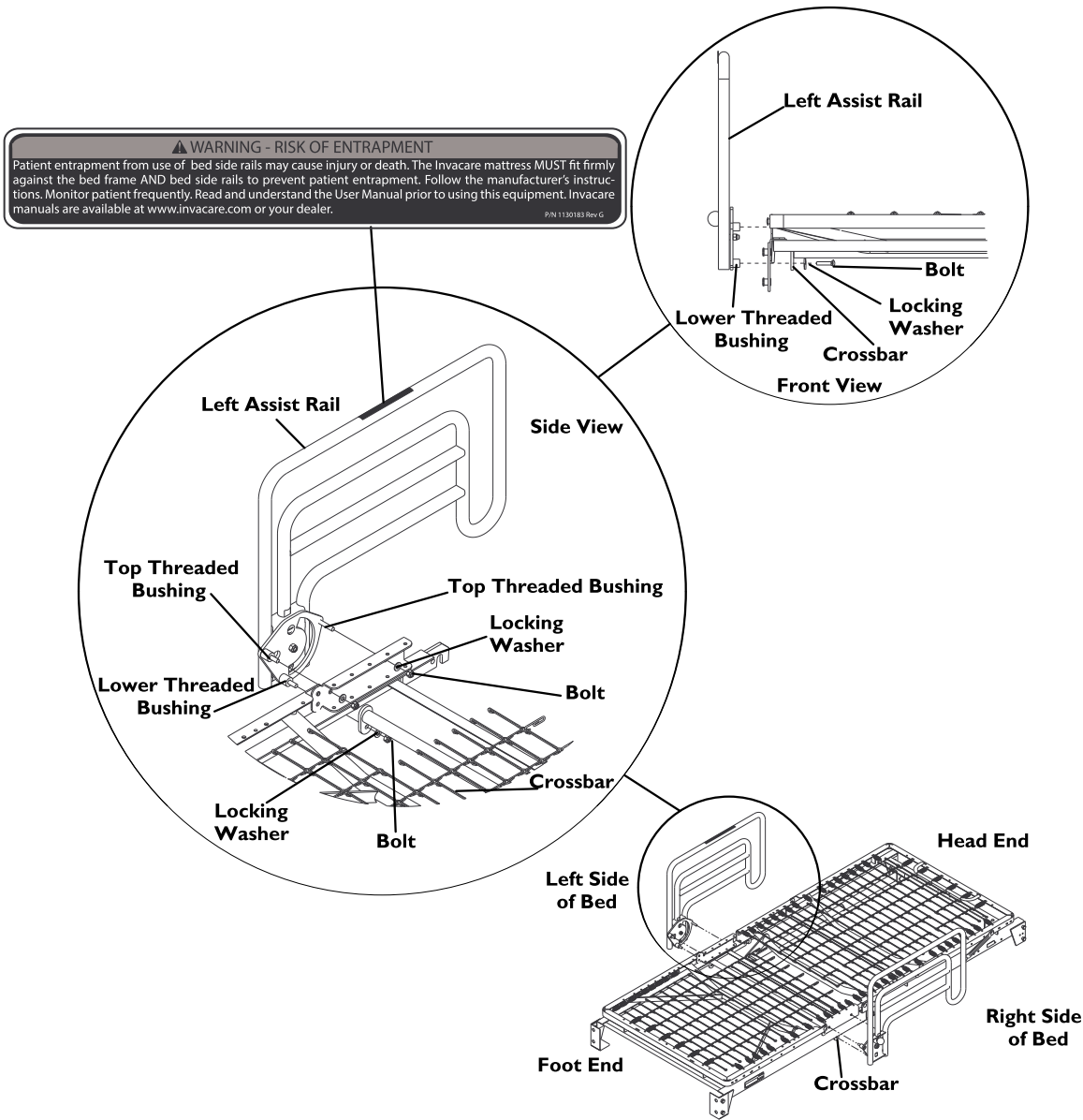

4 Usage

4.1 Usage

 For this procedure, refer to FIGURE 2.

The assist rails have three positions:

- Guard (Down): This position is intended to prevent an individual from inadvertently rolling out of bed.
- Assist (Up): Assists the user in standing or sitting on bed.
- Access (Back): Allows unimpeded access to user.

When changing position of the assist rail, hold and raise the rail with one hand while pulling the adjustment pin outward with the other hand.


1. To change position of the assist rail perform one of the following:

- a. Guard (DOWN) Position: Pull the adjustment pin and lower to assist rail down to the guard position. There will be an audible “Click” when the assist rail locks into position.
- b. Assist (UP) Position: Pull the adjustment pin and lift the assist rail up to the assist position. There will be an audible “Click” when the assist rail locks into position.

 The rail does not lock in the Access Position.

- c. Access (BACK) Position: Pull the adjustment pin and tilt the assist rail back towards the foot end of the bed until the assist rail stops.

Figure 2

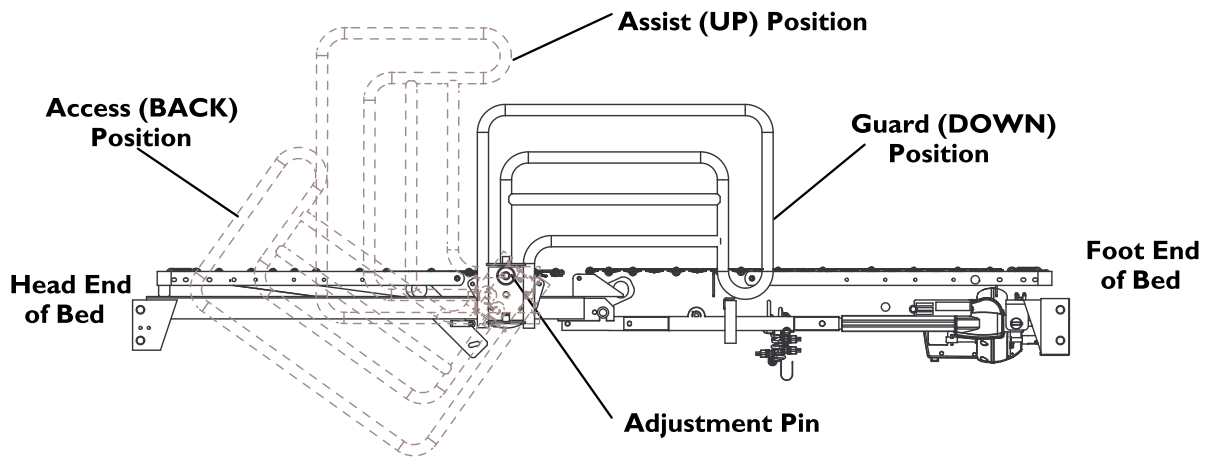5 Warranty

5.1 Limited Warranty

PLEASE NOTE: THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.

This warranty is extended only to the original purchaser/user of our products.

This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state.

Invacare warrants its product to be free from defects in materials and workmanship for two years for the original purchaser. If within such warranty period any such product shall be proven to be defective, such product shall be repaired or replaced, at Invacare's option. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of any such product. Invacare's sole obligation and your exclusive remedy under this warranty shall be limited to such repair and/or replacement.

For warranty service, please contact the dealer from whom you purchased your Invacare product. In the event you do not receive satisfactory warranty service, please write directly to Invacare at the address below. Provide dealer's name, address, model number, date of purchase, indicate nature of the defect and, if the product is serialized, indicate the serial number.

Invacare Corporation will issue a return authorization. The defective unit or parts must be returned for warranty inspection using the serial number, when applicable, as identification within thirty days of return authorization date. DO NOT return products to our factory without our prior consent. C.O.D. shipments will be refused; please prepay shipping charges.

LIMITATIONS AND EXCLUSIONS: THE WARRANTY SHALL NOT APPLY TO PROBLEMS ARISING FROM NORMAL WEAR OR FAILURE TO ADHERE TO THE ENCLOSED INSTRUCTIONS. IN ADDITION, THE FOREGOING WARRANTY SHALL NOT APPLY TO SERIAL NUMBERED PRODUCTS IF THE SERIAL NUMBER HAS BEEN REMOVED OR DEFACED; PRODUCTS SUBJECT TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE; OR PRODUCTS MODIFIED WITHOUT INVACARE'S EXPRESS WRITTEN CONSENT INCLUDING, BUT NOT LIMITED TO: MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS; PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE; PRODUCTS DAMAGED BY CIRCUMSTANCES BEYOND INVACARE'S CONTROL; PRODUCTS REPAIRED BY ANYONE OTHER THAN AN INVACARE DEALER, SUCH EVALUATION SHALL BE SOLELY DETERMINED BY INVACARE.

THE FOREGOING EXPRESS WARRANTY IS EXCLUSIVE AND IN LIEU OF ANY OTHER WARRANTIES WHATSOEVER, WHETHER EXPRESS OR IMPLIED, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, AND THE SOLE REMEDY FOR VIOLATIONS OF ANY WARRANTY WHATSOEVER, SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. THE APPLICATION OF ANY IMPLIED WARRANTY WHATSOEVER SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESS WARRANTY PROVIDED HEREIN. THE MANUFACTURER SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE/PROVINCIAL LAWS AND REQUIREMENTS.

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
Figure 1

⚠ WARNING - RISK OF ENTRAPMENT
 Patient entrapment from use of bed side rails may cause injury or death. The Invacare mattress MUST fit firmly against the bed frame AND bed side rails to prevent patient entrapment. Follow the manufacturer's instructions. Monitor patient frequently. Read and understand the User Manual prior to using this equipment. Invacare manuals are available at www.invacare.com or your dealer.
P/N 1130183 Rev G

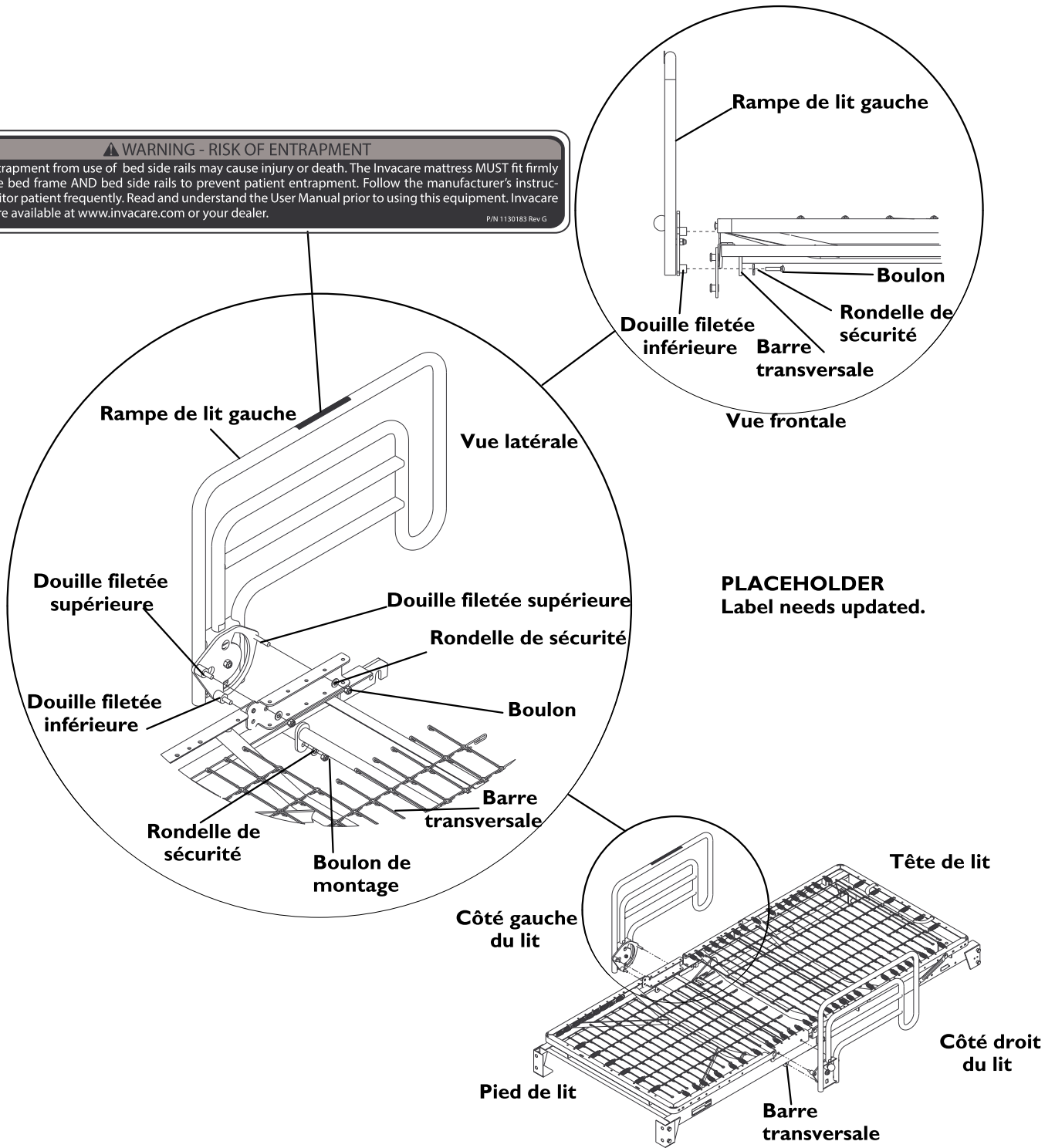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www.invacare.com

Designed by Invacare Corporation

Manufactured in Mexico for Invacare Corporation

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© 2014 Invacare Corporation. All rights reserved. Republication, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by ™ and ®. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.

Yes, you can.®