## **Premium**

PHYSIPRO

Adjustable tension backrest

Owners manual

This manual contains important informations about this product.  
Please pass it to the final user upon delivery.

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**ENGLISH**

Physipro inc. is delighted to have you among its clients. We sincerely wish to thank you for the trust you have shown in our Company by choosing one of our products.

This user manual has been designed to allow the client to use the PREMIUM Back, a Physipro inc. product, safely in an optimal way. For all adjustments and settings, Physipro inc. strongly recommends you consult a qualified professional.

Physipro inc. provides you with specific verifications you should perform on a regular basis in order to optimize the performance of your PREMIUM Back and extend its operational life.

Physipro inc. relieves itself from any liability should any bodily injury or property damage result from a lack of care or misuse or from any modification made to the product without prior written consent of the Company.

For Physipro inc., your satisfaction remains a priority.

## **Table of contents**


<b>1. PREMIUM BACK</b> .....	4
<b>2. COMPONENTS</b> .....	4
<b>3. SAFETY RULES</b> .....	5
<b>4. LATERALS INSTALLATION</b> .....	6
<b>5. PREMIUM BACK INSTALLATION</b> .....	9
<b>6. PREMIUM BACK WITH BACK CANES COVER</b> .....	11
<b>7. ADJUSTMENT</b> .....	12
<b>8. CARE AND MAINTENANCE</b> .....	13
<b>9. WARRANTY</b> .....	14

**ENGLISH****1. PREMIUM BACK**


This adjustable-tension back has been designed to provide additional comfort and adjustability for clients with spinal deformities such as kyphosis (outward curvature of the upper back). The 1" bilateral straps can be individually tightened or loosened as needed.

**2. COMPONENTS**


Backrest ( 1 pce )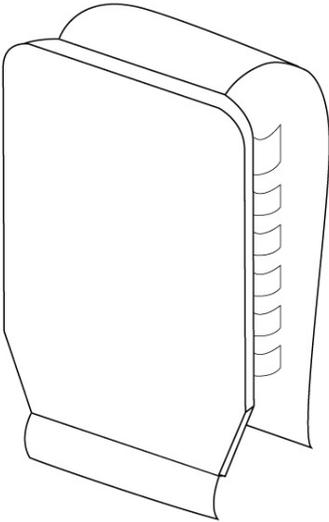

Laterals ( 2 pces )

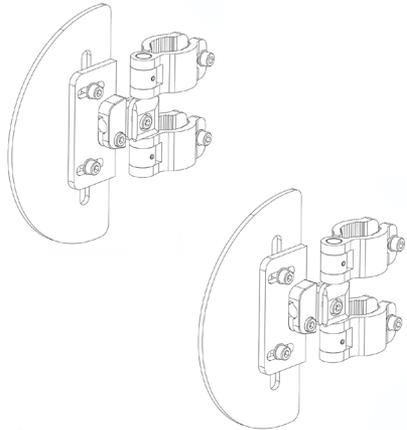Pads ( 2 pces )
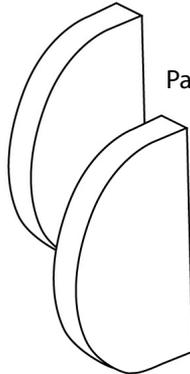
### 3. SAFETY RULES

Do not use this equipment without having read and understood this user manual in its entirety. This manual includes critical information intended to ensure the safety of the user and the individuals who handle the wheelchair.

Installing a back support on a wheelchair may affect the center of gravity of the wheelchair and may cause the wheelchair to tip backwards, potentially resulting in injury. Always assess for the potential need for antitippers or amputee axle adapter brackets to be added to the wheelchair to help increase stability


Do not use Premium Back to push or lift wheelchair. Improper use may cause the back to unexpectedly detach from the wheelchair.

Do not use Premium Back cover or straps to lift wheelchair

**ENGLISH****4. LATERALS INSTALLATION***(for backrest with laterals)***ADJUSTING THE WIDTH BETWEEN THE LATERALS**

1. Determine the desired location of the laterals.
2. Attach and align receiver.
3. Repeat the process for the second lateral

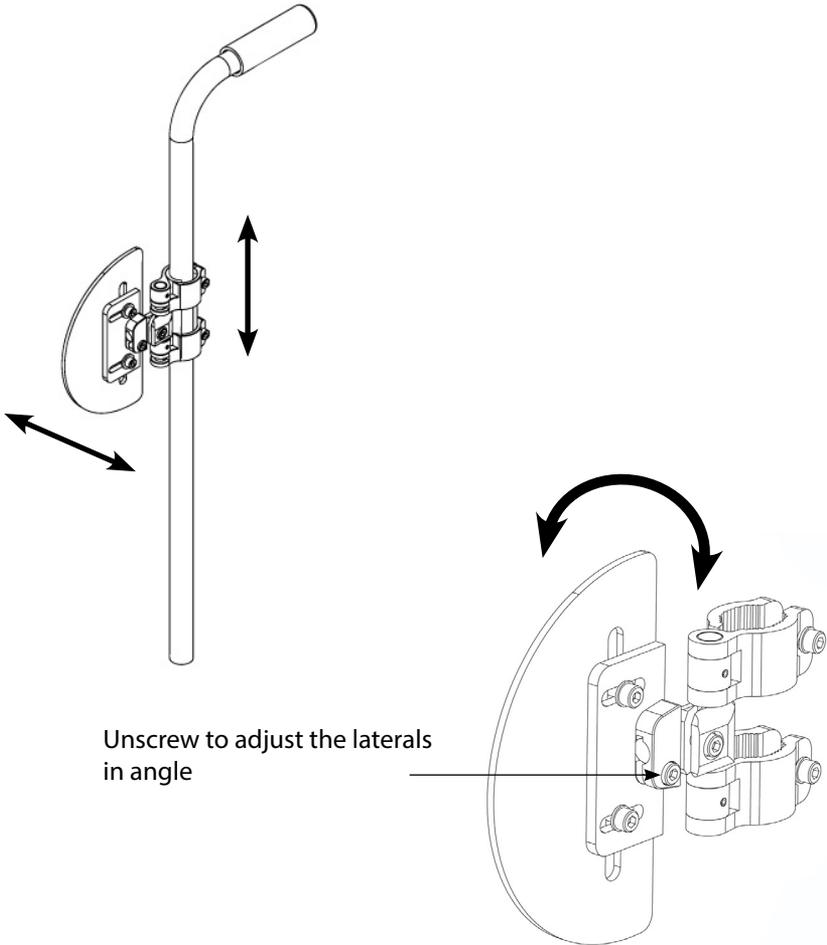### 4.1 Installing the contoured laterals

Make sure a Velcro strip is fixed on the lateral plate facing the inside. (Figure 6.1)

Insert your hand in the cushion through the cover opening to prevent the Velcro strips from touching each other while you place the cushion. (Figure 6.2)

Start inserting the cushion by the top of the lateral. (Figure 6.3)

Push down while making sure the cover opening passes around the plate by making a downward rotating movement. (Figure 6.4)

Place the cushion, remove your hand from the cover and push a few times on the front of the cushion to make sure the Velcro strips are hooked. (Figure 6.5)

Figure 6.1

Figure 6.2

Figure 6.3

Figure 6.4

Figure 6.5

**ENGLISH****4.2 Removing the contoured cushion**

Insert your hand inside the contoured cushion between the plate and the cushion to unhook the Velcro strips.

Move the cover opening down the lateral plate and remove the cushion with an upward rotating movement.

**4.3 Removing the foam from the contoured cushion**

Fold the cushion in the cover. (Figure 6.7)

Carefully and slowly take the cushion out of the cover opening. (Figure 6.8)

Completely remove the cushion. (Figure 6.9)

Figure 6.6

Figure 6.7


Figure 6.8

Figure 6.9

## 5. PREMIUM BACK INSTALLATION


1. Install the two top horizontal adjustment straps to the back canes using the self-tap screws.
2. Insert each strap through the locking buckles
3. Adjust each strap individually to the desired tension
4. Install the Premium back between the laterals
5. Install the pad on the laterals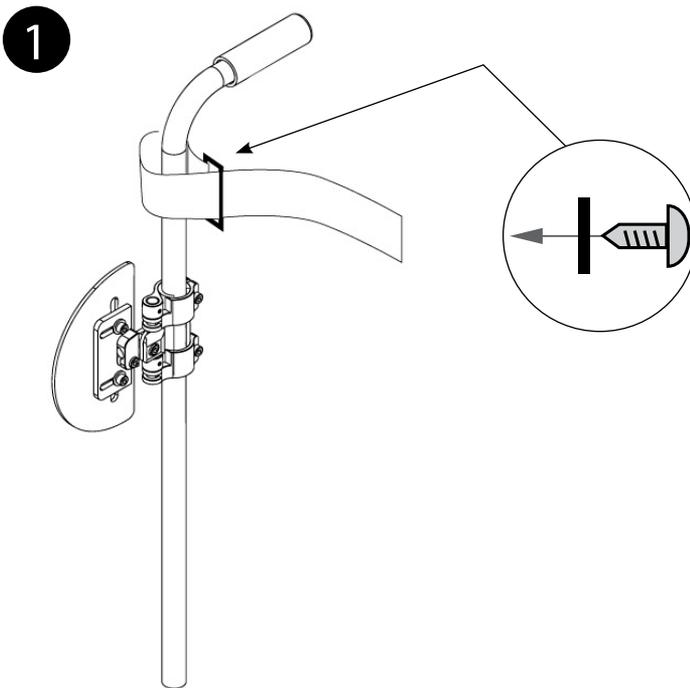


Over-tightening the strap assembly may cause damage to the integrity of the back canes.


## ENGLISH

2

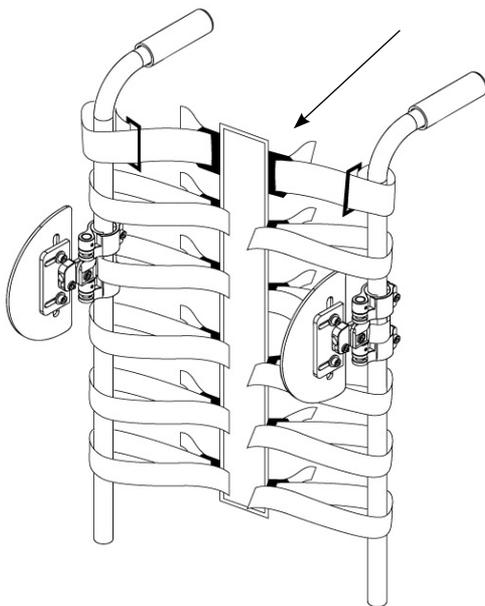
3
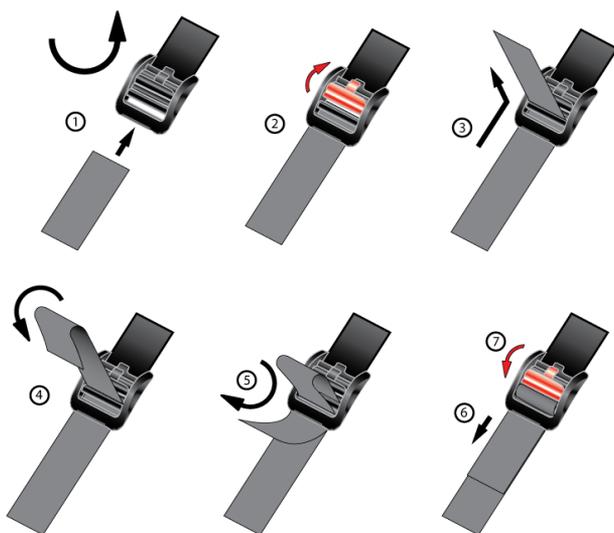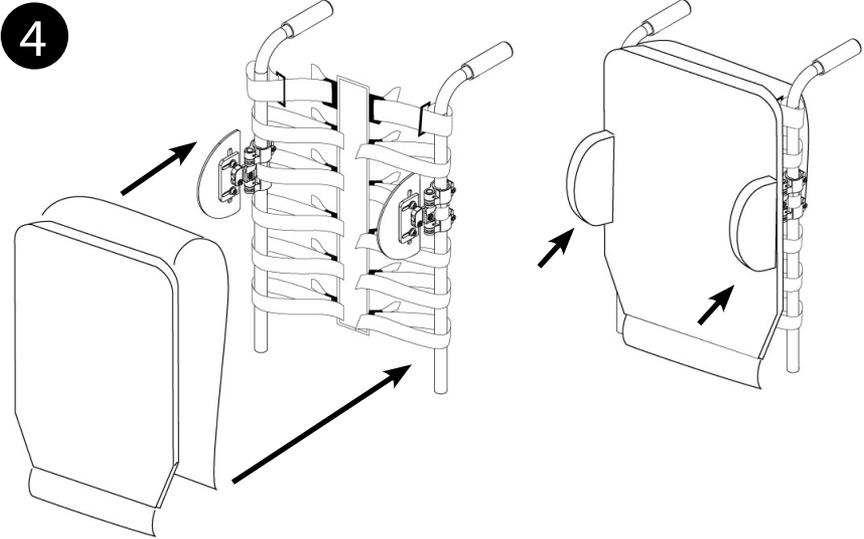
## 6. PREMIUM BACK WITH BACK CANES COVER

1. Install the cover to the back canes with the self tap screws

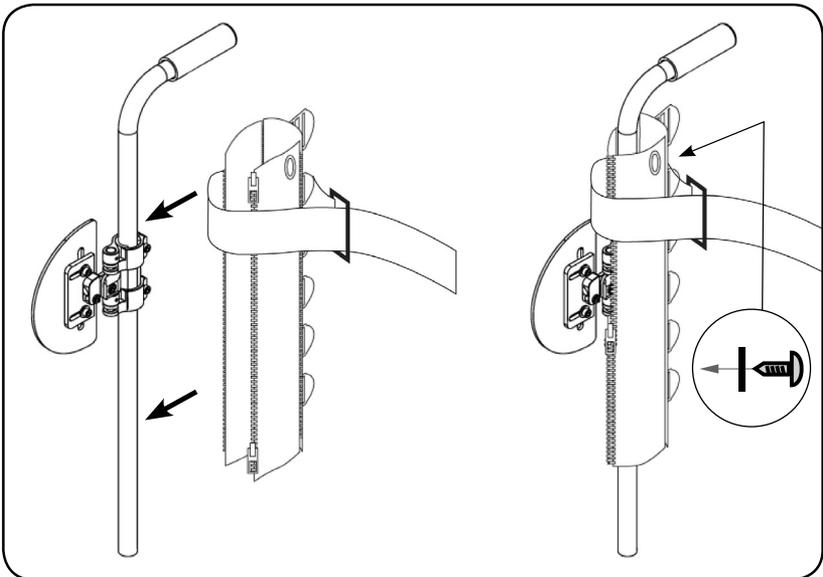**ENGLISH****7. ADJUSTMENT**

1. Lift up the back cover flap to expose the horizontal straps.
2. Adjust each strap individually to the desired tension.
3. Fold down the back cover flap over the horizontal adjustment straps to complete your adjustment.

Ensure that the strap assembly is not over-tighten.  
Over-tightening the strap assembly may cause damage to the integrity of the back canes.

## 8. CARE AND MAINTENANCE

The maintenance of the PREMIUM Back is essential; therefore you will extend its lifetime and ensure its safety. Physipro recommends that the PREMIUM Back be checked for wear and tear every 6 months. Make sure that the lateral are fastened properly when receiving and verified it each week. Do not continue to use the system after identifying loose or broken components.

### CLEANING

#### Outer cover

Wipe down the outer cover with a soft cloth, warm soap and water. The outer cover can be machine wash in warm water on a gentle cycle. Air dry the outer cover.

#### NOTE

- ✓ Remove the foam from inside the cover prior to machine washing
- ✓ The foam core may be washed by hand with light soap and water

#### Strap Assembly

Wipe down strap assembly as needed using light soap and warm water.

#### NOTE

- ✓ Do not machine wash the strap assembly.

machine wash  
maximum temperature 50 C

do not iron

do not use chlorine-based bleach

do not dry clean

**ENGLISH****9. WARRANTY**

The PREMIUM back, a product of Physipro inc., is covered with our twenty-four (24) months manufacturer's warranty. It includes parts and labor on manufacturer's defects, optional parts, and all new parts replaced by your dealer or an authorized establishment from the date of delivery or replacement.

Physipro inc. will repair or replace the defective parts as long as the warranty is valid. You will have to contact Physipro inc. or an authorized dealer to obtain your warranty services. Do not return your backrest without prior consent. If you are not satisfied with the repairs, please send us a written complaint with your dealer's name, address and delivery date to the address on this manual cover.

**Exclusions and limitations**

This warranty does not apply to products that were intentionally or accidentally altered, misused, neglected or did not have the proper maintenance. Furthermore, this warranty does not apply if repairs or modifications were done without a written consent from Physipro inc., or an authorized dealer. The same restrictions apply if any other kind of damages were done without Physipro inc. involvement. This warranty will not apply for everyday use or failure to respect the rules in this manual.

Physipro inc. will not take any responsibilities for damages incurred during shipping.

**Terms and Conditions**

The seat cushions and backrests are reserved exclusively for use on mobility aid (manual or power wheelchairs) Physipro will not be held responsible for any misuse of the product. Crash tests have not been conducted on these products and even if the mobility aid in which it is installed has obtained the ISO-717619, it will not be maintained.

The respect of the safety instructions mentioned in this manual is essential to avoid injury. Improper or inadequate adjustment increases the risk of accident. Before making adjustments, it is important to read the entire contents of this manual. Physipro will not be held responsible for misuse or non professional adjustment.

The product presented in this manual may not be an exact representation of the product you have on hand. The instructions in this manual are still valid for your product. Physipro reserves the right to modify or change this manual without notice.

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