Distributed by:

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Distribué par:

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HENRY SCHEIN INC.
Melville, NY 11747 USA

EC REP

Henry Schein U.K. Holdings Ltd.
Gillingham ME8 0SB U.K.

Irritant
Irritante
Irritant
Reizend
Irritante

Toxic
Tóxico
Toxique
Giftig
Tossico

DESENSITISING GEL

INSTRUCTIONS FOR USE

Gel desensibilizante
INSTRUCCIONES DE USO

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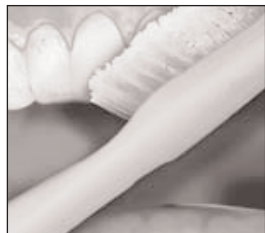
Desensitising Gel

INSTRUCTIONS FOR USE

Description: Henry Schein Desensitising Gel is a unique, sustained release, 3% potassium nitrate and 0.11% weight by weight fluoride ion, sticky, viscous gel. Henry Schein Desensitising Gel is designed to be used in a custom-fabricated tray.

Indications: Henry Schein Desensitising Gel provides a film-like varnish for sensitive teeth, sealing dentinal tubules in exposed dentin and other areas where post-operative sensitivity is a concern. The product is used either by a dental professional in the office or provided to the patient for home treatment of dentin sensitivity.

Procedure:

- Fabricate tray to meet patient's needs. See laboratory instructions. Henry Schein Desensitising Gel may be applied in the same tray and manner as bleaching gel or into a bleaching tray if available.

Note: *If large retention areas exist from prior periodontal surgery that might present problematic impression removal from the teeth, block out embrasures with Block out material.*

- Henry Schein Desensitising Gel is dispensed in 1.2ml unit dose syringes. A complete arch should use most of the contents of a syringe. If sensitivity is localised, place gel in affected area(s) and a small amount in the molar region to hold tray in place.
- The length and number of times a tray is worn depends on condition, patient and clinician. Recommended treatment times can range from 15 minutes to 1 hour.
- Tray should be removed at meal times. Immediately following tray removal, patient should brush and rinse teeth with water to remove all remaining gel.
- DO NOT swallow solution during rinsing. Wash tray and store in a tray storage case. Fresh gel should be used with each application.
- Gel may be warmed before placing in tray to avoid thermal sensitivity.

Laboratory Instructions:

- Pour and trim cast as you would for a standard bleaching tray. Apply 0.5-1.0mm of block-out resin to a clean, dry cast on areas in which patient is experiencing sensitivity (see Fig. 1 & 2). Also apply resin to areas where there are open retention areas, etc. For periodontal patients, clinician may determine that pre-operative impressions are indicated. In such cases, reservoir spacing will be created post-op because of recontoured and apically positioned gingiva.
- Cure block out for about 2 minutes in light curing unit. A normal curing light can be used instead (according to instructions). Wipe off oxygen inhibition layer.

- Prepare a soft splint acc. to manufacturer's instructions.
- Trim excess tray material, allowing 4-6mm to overlap onto gingival tissue. Contour tray margin to avoid anatomical structures such as frenum attachments, bone growth, etc.
- Return tray to model; check tray extensions. Gently flame polish edges, one quadrant at a time.
- While still warm, hold periphery of each segment firmly against model for three seconds with water-moistened finger. If an area is short of the desired length, gently heat and push tray material to the desired location. If this thins the material too much, a new tray should be fabricated.

Precautions:

- Keep Henry Schein desensitisation product away from small children.
- Not for use overnight.
- Do not use smoke or eat while using this product.
- Not intended for use by pregnant women.
- If any problems occur, discontinue treatment and notify your dentist.
- Keep out of heat and/or sunlight at all times. It should be stored at room temperature.
- Patients should not swallow large volumes of Henry Schein Desensitising Gel. Excess material at insertion and residue upon removal should be brushed off and rinsed: DO NOT swallow.

DENTIST INSTRUCTIONS & LAB INSTRUCTIONS

- Construct and dry the stone model, cutting palate and tongue areas out (Fig. 3).
- Vacuum adapt soft tray sheet (0.5 mm) to model. Cool and separate (Fig.4).
- For sensitive roots use LC block out resin on gingival margin to provide a reservoir space for Henry Schein Desensitising Gel. It is recommended to lap tray 2-3mm onto gingiva (Fig. 5).

General Precautions:

- For professional use only
- Review instructions, precautions, and MSDS before beginning treatment. Use only as directed.
- Keep products out of heat/sunlight.
- Confirm that patient has no known allergies to treatment materials.

PATIENT INSTRUCTIONS

- Follow your dentist's instructions on how to load gel into your custom tray. Use no more than 1/3 to 1/2 of syringe (Fig. 6).
- Brush teeth, then insert tray. Lightly tap tray to adapt tray sides to teeth. Remove tray after 15 minutes to 1 hour or as directed by your dentist (Fig. 7).
- Remove excess gel with clean finger or soft toothbrush. Rinse twice; do not swallow rinses (Fig. 8).
- Clean tray with soft brush and cool tap water. Store tray in case provided (Fig. 9).

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