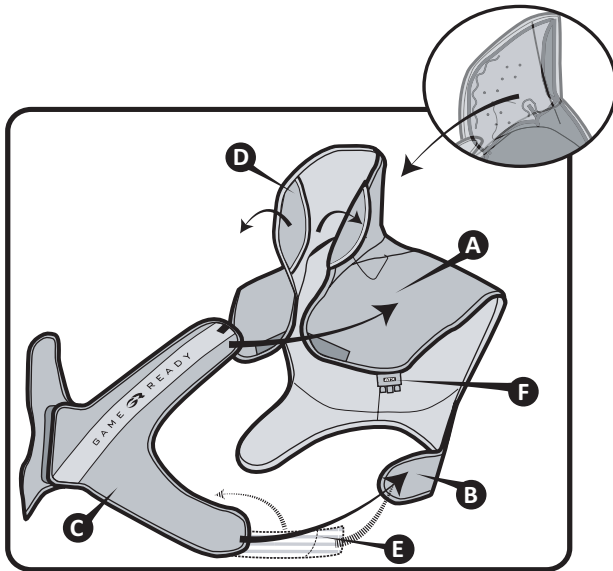
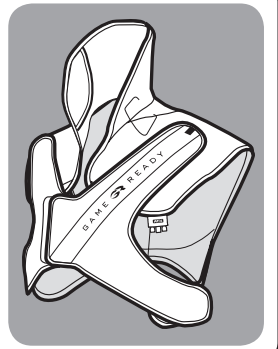
C-T Spine

1

2

3

4


**Application**

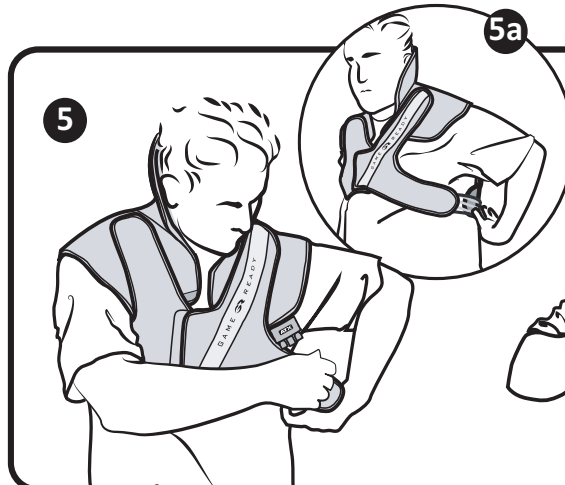**English**

1. Slide one arm through the Wrap.
  2. Slide the other arm through the other side of Wrap.
  3. Take hold of the shoulder wings (A) and pull the Wrap down gently to ensure good contact with the back of the neck.
  4. Fasten the hook and loop closure at the center of the chest.
  5. Adjust the two side straps (B & C) equally on both sides of the body.
  - 5a. Elastic extension bands (E) on each side of the chest straps allow the Wrap to accommodate various body sizes. Unfasten the side straps; detach the elastic strap on the inside and reattach the loop towards the back of the Wrap. Repeat for the other side.
  6. Adjust the two shoulder straps (A & C) by completely unhooking and reapplying them over the chest and shoulders – Take care to keep Connector (F) unobstructed.
- There is an insulation flap (D) at the top of the neck collar. If the collar fits below your ears, flip the insulation flap outward extending the cooling area. If the collar covers your ears, keep the insulation flap tucked inside the Wrap.
  - After fitting the Wrap properly, you should not have to readjust the chest straps for subsequent treatments. Unhook the closure at the center of the chest and remove the Wrap. To reapply, slide the Wrap on like a vest and fasten the hook closure at the center of the chest.


5

5a

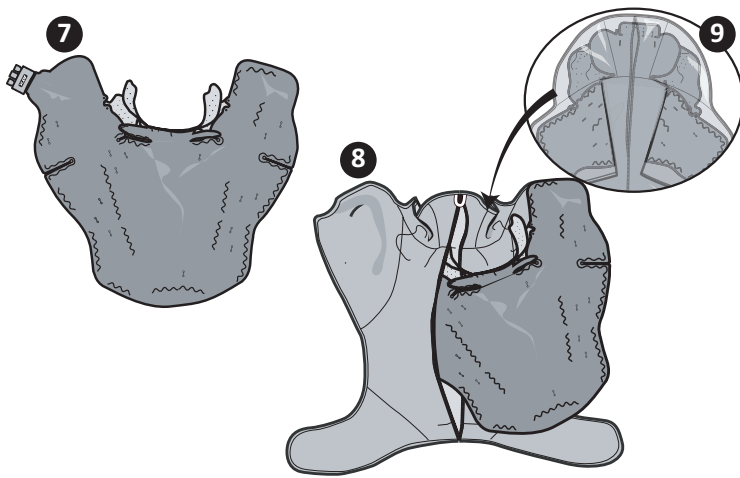6

### Assembly

- Lay the Sleeve with the zipper side down on a flat surface and remove the chest straps (C), then turn the Sleeve over so the zipper is facing up
- Lay the Heat Exchanger out flat, separate from the outer sleeve.
  - Insert the Heat Exchanger into the Sleeve by feeding the connector through the opening (F) on the left shoulder. The blue side of the Heat Exchanger should touch the blue side of the Sleeve.
  - Be sure the Heat Exchanger is flat inside the sleeve on the back and shoulder areas. In order for the collar to lay correctly, rotate the shoulder wings back (A), then smooth the heat exchanger inside the collar. Zip the sleeve closed, and flip the Wrap over with the shoulder wings facing forward. The Wrap should resemble a vest with a high collar. Attach the chest straps (C) to complete assembly, orienting the logo on the chest straps as shown.

### Important

This wrap is intended for use with low-pressure setting. Pressure can be turned off as desired.

Read this complete user manual and Game Ready System user manual including indications, contraindications, cautions and warnings before using this product.

### Contraindications

Compression Therapy using the Game Ready System or any compression therapy device should not be used in patients:

- Who are in the acute stages of inflammatory phlebitis in the affected region.
- Who have any current clinical signs suggestive of deep vein thrombosis in the affected region.
- Who have significant arteriosclerosis or other vascular ischemic disease in the affected region.
- Who have any significant risk factors or current clinical signs of embolism (e.g. pulmonary embolus, cerebral infarction, atrial fibrillation, endocarditis, myocardial infarction, or arteromatous embolic plaque).
- Who have a condition in which increased venous or lymphatic return is not desired in the affected extremity (e.g., carcinoma).
- Who have decompensated hypertension in the affected region.

Cryotherapy using the Game Ready System or any cryotherapy device should not be used in patients:

- Who have significant vascular impairment in the affected region (e.g., from prior frostbite, diabetes, arteriosclerosis or ischemia).
- Who have known hematological dyscrasias which affect thrombosis (e.g., paroxysmal cold hemoglobinuria, cryoglobulinemia, sickle-cell disease, serum cold agglutinins).

### Warnings

Follow the treatment recommendations of your health care practitioner for duration and frequency of use for this device.

Improper placement or prolonged use of the Game Ready System could result in tissue damage.

During the course of therapy, patients should monitor the skin surrounding the treated region and the digits of the extremities of the treated limb (if applicable) for any burning, itching, increased swelling, or pain. If any of these signs are present, or any changes in skin appearance occur (such as blisters, increased redness, discoloration, or other noticeable skin changes), patients are advised to discontinue use and consult a physician.

Game Ready Wraps are not sterile; do not place directly against open wounds, sores, rashes, infections, or stitches. The Wrap may be applied over clothing or dressing.

Game Ready Wraps are available in multiple configurations but are not intended for all possible physiologic uses. For example, the Ankle Wrap is not designed for use on the toes and the Back Wrap is not designed for use in the abdominal region. Not to be used continuously for greater than 20 minutes per treatment session.

This Wrap is supplied with an insulating flap that can be positioned as desired to prevent excessive cooling of the ears. If you feel excessive cooling or discomfort, use additional insulation where needed or adjust the control unit temperature.

Follow recommendation of physician for appropriate use of insulating layer.

Compression Therapy with the Game Ready System should be used only under the supervision of a licensed healthcare practitioner in patients:

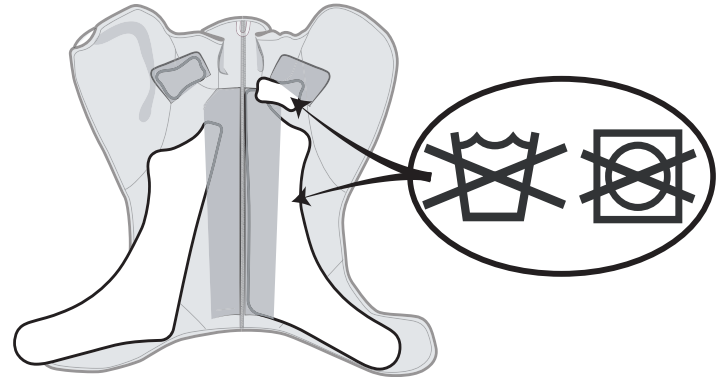
- Who have an open wound in the affected region (the wound must be dressed prior to use of Game Ready).
- Who have an acute, unstable (untreated) fracture in the affected region.
- Who are children under 18 years old or patients who have cognitive disabilities or communication barriers, whether temporary (due to medication) or permanent.
- Who have a cardiac insufficiency or congestive heart failure (with associated edema in the extremities or lungs).

- Who have a localized unstable skin condition (e.g., dermatitis, vein ligation, gangrene, or recent skin graft).
- Who have erysipelas or other active infection in the affected region. Cryotherapy with the Game Ready System should be used only under the supervision of a licensed healthcare practitioner in patients:
  - Who have Raynaud's disease or cold hypersensitivity (cold urticaria)
  - Who have hypertension or extreme low blood pressure.
  - Who have diabetes.
  - Who have compromised local circulation or neurologic impairment (including paralysis or localized compromise due to multiple surgical procedures) in the affected region.
  - Who have a localized unstable skin condition (e.g., dermatitis, vein ligation, gangrene, or recent skin graft).
  - Who have rheumatoid arthritis in the affected region.

### Notes

Apply Wrap with a uniformly close fit, ensuring there are no kinks which may impede water flow.

Ensure connector hose is placed to prevent the wrap from folding or kinking at the hose inlet location of the wrap.

### Care and Cleaning

For daily care and to minimize formation of mildew, remove heat exchanger from sleeve and wipe with dry towel to remove any condensation that may form. Turn sleeve inside out and hang both the sleeve and heat exchanger to release excess moisture.

For extended care, carefully remove the Heat Exchanger and all four plastic support pieces (2 neck, 2 back) from the sleeve and turn the Sleeve inside out. Hand or machine wash the sleeve in cold water and mild detergent, or antibacterial soap. Hang to dry. Heat Exchanger should be wiped with warm water and mild detergent-do not machine wash or place in a dryer. Hang to dry. Return plastic support pieces to Sleeve before using the Wrap.

For multi-patient use, Sterifab® may be used according to manufacturer's instructions on the heat exchanger and/or sleeve to minimize microbe transfer.

### Average Life Expectancy

The life expectancy of sleeves and heat exchangers will vary widely depending on frequency of use. Please reference chart below to determine when to replace product.

#### Sleeve

Light Use (Personal)	12 months
Medium Use	6 months
Heavy Use (Clinic or training facility)	3 months

#### Heat Exchanger

Light Use (Personal)	24 months
Medium Use	18 months
Heavy Use (Clinic or training facility)	12 months

#### Warranty Information

Sleeve: In case of manufacturer defect, sleeve may be returned within 7 days of purchase.

Heat exchanger: 1 year from date of purchase. See warranty card included with Heat Exchanger.

#### Contact Us

In the U.S. call Game Ready Customer Service at 1.888.426.3732 (+1.510.868.2100). From outside of the U.S. please refer to [www.gameready.com](http://www.gameready.com) to find your local distributor's contact information.

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©2014 CoolSystems, Inc. All Rights Reserved