Suprasson P5 Newtron®

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ENGLISH

CONTENTS

1 INTRODUCTION	6	7 MAINTENANCE	
2 WARNINGS	7	Water filter replacement or maintenance	13
3 PRESENTATION		Replacing the handpiece support	14
Presentation	8	Troubleshooting	15
Technical description	8-9	8 DISPOSAL AND RECYCLING	16
4 INSTALLATION		9 LIABILITY	16
Unpacking the device	10	10 CUSTOMER RELATIONS	
Recommendations	10	Manufacturer's details	16
Installation	10	Subsidiaries' address	16-17
Commissioning	10	11 SYMBOLS	18
5 USE	11		
6 MAINTENANCE AND STERILIZATION			
Suprasson® Newtron handpiece maintenance	12		
Tip maintenance	12		
Device maintenance	13		

The list of accessories referred to in this manual is not exhaustive. Consult your local dealer for further information.

I - INTRODUCTION

Congratulations, you have just taken possession of your SUPRASSON P5 NEWTRON® device.

Designed by SATELEC®, the SUPRASSON P5 NEWTRON® is a piezo-electric ultrasonic generator that uses ultrasonic tips to carry out dental treatment.

In order to get the most out of this high technology product, and prolong its life, we ask you to read these instructions carefully before using the device for the first time.

II - WARNINGS

WARNING: To reduce the risk of accident, the following precautions should be taken:

- The SUPRASSON P5 NEWTRON® should only be used by trained and qualified practitioners (dentists, hygienists).
- The electrical supply to which the device is connected should comply with standards in force.
- The water supply system should satisfy the quality criteria required for the practice of dental treatment. It should be equipped with a supply cut-off system.
- Do not cover the device or obstruct the air vents.
- Do not use the device if it appears to be damaged or defective.
- Do not immerse in liquid or use outdoors.
- Do not place the device near a heat source.
- Make sure that the cords do not hinder the free passage of people.
- After use, turn off the water supply.
- Before unplugging the mains cord, shut off the water supply and switch the device off.
- **The SUPRASSON P5 NEWTRON® must not be used when the patient and/or the operator has a pacemaker**
- When unplugging the mains cord, hold both the cord plug and the power socket in the wall.
- The device should be stored in its original packaging, in an appropriate place, where it will not cause any danger to people.
- Do not carry out repairs or modifications to the device without prior authorization by SATELEC®.
- If a fault occurs, contact your supplier rather than have it repaired by an unauthorized person.
- The device is designed only to be used with SATELEC® accessories which are fully compatible with it. In case of doubt, contact an approved dealer or Customer Service (www.acteongroup.com or email satelec@acteongroup.com).
- The device complies with Electromagnetic Compatibility standards (CEI 601-1-2), however, the user should ensure that any possible electromagnetic interference does not present an additional risk.

III - PRESENTATION

3. 1 PRESENTATION

The SUPRASSON P5 NEWTRON® is composed of three main parts:

- a control unit (Figure 1, ref. 1)
- a footswitch (Figure 1, ref. 2)
- a cord with a connector for the handpiece (Figure 1, ref. 3)

The SUPRASSON P5 NEWTRON® control unit has a rotary potentiometer on the top with the marking OI (On/Off) and marked 1 to 20 (power levels).

Various connections are to be found at the rear of the unit:

- 1 mains cord (fixed) with earth pin (Figure 2, ref. 1)
- 1 footswitch cord (fixed)(Figure 2, ref. 2)
- 1 tube (Figure 2, ref. 3), equipped with a filter system (Figure 2, ref. 4), for connecting to the water supply.
- 1 Team-up socket (optional equipment) (Figure 2, ref. 5). This equipment comes with its own instructions.
- Tap for adjusting the water flow (Figure 2, ref. 6).

On the right is the scaler cord (fixed) with a connector for the handpiece (Figure 5, ref. 4).

There is a support located above the cord for the handpiece connector.

There are air vents on the rear and lower panels of the device.

In order to maximize air flow, the device rests on 4 rubber feet, keeping the lower air vents clear.

Accessories: the SUPRASSON NEWTRON® handpiece and tips come with their own instructions.

3. 2 TECHNICAL DESCRIPTION

a) Upper panel of the control unit

The rotary potentiometer (Figure 3, ref. 1) on the top panel is used for:

- starting and stopping the device: OI (Figure 3, ref. 2)
- setting the ultrasonic power: 1 to 20

Its display is divided into 4 coloured sectors each having 5 positions.

The power on indicator (Figure 3, ref. 3) shows the colour and no. of the chosen power setting:

- **green: 1 to 5:** very low to low power
- **yellow: 6 to 10:** medium power
- **blue: 11 to 15:** high power
- **orange: 16 to 20:** very high power

The technology built into this device enables the practitioner to work at a constant amplitude, depending on the power level selected and compensating for the effort applied.

Note: the power level for using tips should be selected in accordance with the instructions in the Tipbook and the tip colour coding system (CCS tips).

b) Rear panel of the control unit

The SUPRASSON P5 NEWTRON® is connected to the electrical supply using the mains cord and earthed plug.

The tube used for connecting the device to the water supply includes a filter (requires regular maintenance; see section 7.2).

Of the "ON/OFF" type, the footswitch is used for operating the SUPRASSON P5 NEWTRON® by the practitioner. Its weight and its non-slip base ensure good stability.

c) Right side panel of the control box

The cord (blue) linking the device and the handpiece connector provides for:

- the circulation of fluid for the spray
- the electrical connection between the SUPRASSON P5 NEWTRON® and the handpiece.

The handpiece support can be replaced in case of wear or breakage (see section 7/7.3).

d) Technical specifications

Mains power supply: 115-230 V- 50 / 60 Hz.

Note: the voltage used is factory set. The device is protected by fuses (not accessible by the user).

Class 2 BF type equipment.

Intermittent operation: 10 mins. operation / 5 mins. rest

Maximum power consumption: 30 VA.

Minimum frequency of vibrations: 27 KHz.

Water pressure at inlet: 1 to 5 bar (14 to 72 PSI).

Water filter: 60 µm.

Turn half way to **adjust water flow** 0-90 ml/min

Overall dimensions:

Control unit:

Width: 150 mm Height: 55 mm

Depth: 170 mm Weight: 900 gr

Footswitch:

Width: 70 mm Height: 30 mm

Depth: 95 mm Weight: 150 gr

Cord: 2000 mm

Handpiece cord: 2000 mm

Operating temperature:

+10 to +40°C at a maximum of 95% relative humidity without condensation.

Storage temperature:

-20 to +70°C at a maximum of 95% relative humidity without condensation.

This equipment is manufactured according to current C.E.I. 601-1, C.E.I. 601-1-2 (certification O.C.CB scheme) regulations and standards and according to the EN ISO 13485 quality control certification system.

IV - INSTALLATION

4. 1 UNPACKING THE DEVICE

On receipt of the device, check for any damage caused in transit.

If necessary, contact your supplier.

4. 2 RECOMMENDATIONS

Have connections to the electrical and water supply carried out by an approved dental installation engineer.

The SUPRASSON P5 NEWTRON® should be connected to the electrical supply in accordance with current standards. Use of a mains plug with connection to the earth is compulsory.

The water supply system should satisfy the quality criteria required for the practice of dental treatment. In addition, it should be equipped with a water supply cut-off system.

4. 3 INSTALLATION

- Set up the control unit in the desired position.
- Check that the control potentiometer is set to position OI (off).
- Connect the SUPRASSON P5 NEWTRON®'s water tube with its filter to the water supply.
- Connect the mains plug to an electrical socket.

- Place the footswitch so as to make it easy to operate with the foot.
- Check there is no trace of damp in the handpiece connection, otherwise dry it (wipe it and blow on it with the multifunction syringe).
- Connect the handpiece simply by plugging it into the connector cord, aligning the electrical contacts.
- Place the handpiece on the stand.

IMPORTANT: do not twist the handpiece connector on its cord.

4. 4 COMMISSIONING

Before using the SUPRASSON P5 NEWTRON® for the first time, we recommend following the instructions below:

- Before the first use, it is vital to clean, decontaminate and sterilize the accessories (cf section 6).
- After installation, operate the spray for a few minutes in order to rinse the device's internal water circuits: with the device operating at the lowest power (1) and with pressure being applied to the footswitch.
- Disconnect the handpiece.
- Carry out maintenance and/or sterilization of all equipment according to the procedures defined in chapter 6.

V - USE

The SUPRASSON P5 NEWTRON® is operated in the following manner:

- Check that the device is plugged in and insulated correctly.
- Check there is no trace of damp in the handpiece connection, otherwise dry it (wipe it and blow on it with the multifunction syringe).
- Plug in the handpiece.
- Gently operate the spray: operation should be at the lowest power (1) and with pressure being applied to the footswitch.
- Install the required tip.
- Tighten it carefully with the wrench to ensure optimum operation of the ultrasound (an "Easytorque F01400" torque wrench is available as an option).
- Adjust to the desired power (according to the colour and no.) using the control button. Refer to the Tipbook to find out the power settings for SATELEC® tips.

important:

- Do not disconnect the cord handpiece when power to the device is switched on and the footswitch depressed.
- The SUPRASSON P5 NEWTRON® should not be used with accessories other than those supplied by SATELEC®.
- Correct vibration of a tip depends on correct tightening which should not be forced beyond its stop point. In addition, to avoid self-locking, the tip should be removed after each use.
- Over-tightening the tip may lead to either it or the handpiece breaking.
- At the end of the working day:
 - the water supply should be switched off;
 - the device should be switched off (OI);
 - the mains plug should be unplugged.

VI - MAINTENANCE AND STERILIZATION

The following maintenance and/or sterilization instructions should be carried out before using the accessories supplied with the device for the first time.

Avoid using cleaning agents and disinfectants containing inflammable substances. If this is unavoidable, ensure that the product has evaporated and there is no combustible product on the device and accessories before operation.

6. 1 SUPRASSON NEWTRON® HANDPIECE MAINTENANCE

After each use, the water circuits of the whole unit comprising the SUPRASSON NEWTRON® and the tip should be rinsed with clean water for 20 to 30 seconds.

For sterilization of the handpiece, please refer to the specific instructions for the product.

6. 2 TIP MAINTENANCE

a) Life

The shape and the mass of the tips are the determining factors for obtaining the best results from the ultrasonic generator. Vigilance on the part of the user concerning these two characteristics ensures the device's consistent performance.

In consequence, we strongly advise against modifying the structure of the tips by filing or bending them.

Similarly, the ageing of a tip through normal wear leads to changes in its characteristics.

Always replace a tip that is worn or which has suffered an accidental impact (fall etc.).

Use the TIPCARD to check the wear of scaling tips. Renew the most frequently used tips a minimum of one to three times a year.

b) Sterilizing tips

All tips should be cleaned before sterilization with cotton wool soaked in alcohol, wipes or by immersion in an ultrasonic tank.

Sterilizing tips can be carried out by all conventional heat processes, the sterilization conditions and cycles depending on the equipment.

Sterilizing equipment: autoclave

Sterilization under wet heat - instrument wrapped: 134°C at 2 bar (200 KPa) for 18 minutes.

WARNING: during sterilization, metal parts of different types must not touch each other. Any contact will lead to the formation of electrolytic couples causing localized deterioration. In order to avoid this happening, place the objects to be sterilized in a cloth or sterilizing bag.

6. 3 SUPRASSON P5 NEWTRON® DEVICE MAINTENANCE

a) Maintenance

The SUPRASSON P5 NEWTRON® control unit, as well as the handpiece cord, should be cleaned and disinfected (using alcohol and/or disinfectant dental wipes) on a daily basis.

b) Inspection

The accessories and cords should be inspected regularly for insulation faults and replaced if necessary.

It is important to check the state of cleanliness of the air vents of the control unit in order to avoid overheating.

Check there is no trace of damp in the handpiece connection, otherwise dry it (wipe it and blow on it with the multifunction syringe).

WARNING: Do not use the device if it appears to be defective.

Note : If a fault occurs, it is advisable to contact the supplier of your SUPRASSON P5 NEWTRON® rather than have it repaired by an unauthorized person.

VII - MAINTENANCE

Note: for carrying out repairs to defective items where such repairs can be carried out, SATELEC® makes all information necessary available to technical staff of the SATELEC® approved dealers network on request.

7. 1 WATER FILTER REPLACEMENT OR MAINTENANCE

The water filter should be cleaned regularly and should be changed every 6 months.

This is done in the following manner (Figure 4):

- Switch off the water supply, switch off the device (OI position) then unplug the mains plug (Figure 4, diag. 1).
- Unscrew the two parts of the filter (Figure 4, diag. 2) using two 10mm open-ended wrenches.
- Withdraw the filter cartridge (Figure 4, diag. 3) to replace (ref: F 10389) or clean it under running water.
- Proceed in the same manner for the seal.
- Carry out the procedure in reverse to reassemble (Figure 4, diags. 4,5 & 6).
- Check that the spray operates correctly and that there are no leaks (Figure 4, diag. 7).

Warning: a worn or damaged cartridge should be replaced.

7. 2 REPLACING THE HANDPIECE SUPPORT

In case of accidental breakage (fall, etc.), the device is designed to permit the replacement of the handpiece support (ref: E 58365).

To replace the handpiece support lay the control unit on the opposite side to the support.

A screwdriver of the Pozidrive type is recommended for this operation.

Proceed as follows:

- Switch off the water supply, switch off the device (OI position) then unplug the mains plug (Figure 4, diag. 1).
- On the lower panel of the SUPRASSON NEWTRON® control unit, unscrew the fixing screw (Figure 5, ref. 1) of the handpiece support (Figure 5, ref 2).
- Release the support from the centring lug (Figure 5, ref. 3) and the two support lugs on the upper part of the unit.
- Slide the support along the cord (Figure 5, ref 4) then release it, noting its direction.
- Insert the handpiece cord into the new support in the direction you noted.
- Slide the support along the cord.
- Put the support in place on the base of the device, positioning the centring lug and the supporting lugs on the upper part of the unit correctly, and aligning the holes for the fixing screws.
- Insert the screw and tighten (be careful not to over-tighten).
- Clean the device.

7. 3 TROUBLESHOOTING

Please refer to the table on the following page

FAULT	POSSIBLE CAUSE	REMEDY
No operation	• Defective mains cord connection • Presence of fluid or moisture between handpiece and cord • Blown fuse	• Check mains socket • Dry any moisture, especially on electrical contacts • Return to SATELEC After Sales Service
No spray	• Defective water nose connection • No water pressure • Clogged filter or defective solenoid valve	• Check water supply • Check pressure of mains • Clean or replace filter • Return to SATELEC After Sales Service
No water at tip but vibration present	• Obstructed tip or file • Wrong tip selection • Incorrect spray adjustment	• Remove obstruction for tips or file • Check tip • Adjust spray
Insufficient power Low vibration	• Worn or deformed tip • Incorrect use : wrong angle of application or inadequate pressure on tooth • Presence of fluid or moisture between handpiece and cord	• Replace tip • See tip utilization method in Tipbook • Thoroughly dry electrical contacts
No ultrasonic output	• Incorrect tip tightening • Defective connector contact • Cut wire in handpiece cord	• Tighten with wrench • Clean cord contacts • Return to SATELEC After Sales Service for cord replacement
Water leak between handpiece and its base or at junction between handpiece or cord	• Worn sealing gasket (1,15 x 1) in handpiece	• Replace gasket (kit F00070)

VIII - DISPOSAL AND RECYCLING

Disposal of the device and its accessories should be via a collection centre for electronic apparatus or by returning them to SATELEC® After Sales Service.

In no circumstances should the device and its accessories be disposed of by the user.

At the end of their life, tips should be discarded in safety containers designed for this purpose.

IX - LIABILITY

There shall be no liability on the part of the manufacturer in cases of:

- non-compliance with the manufacturer's installation instructions (water pressure, mains voltage);
- maintenance or repairs carried out by persons not authorized to do so by the manufacturer;
- use with an electrical system that does not comply with regulations in force;
- uses other than those specified in this manual;
- non-compliance with instructions contained in this manual.

Note: the manufacturer reserves the right to modify the device and/or the instructions manual without prior notice.

X - CUSTOMER RELATIONS

10. 1 MANUFACTURER'S DETAILS

SATELEC

17, avenue Gustave Eiffel

BP 216

33708 MERIGNAC cedex - France

Tel. +33 (0) 556.34.06.07

Fax. +33 (0) 556.34.92.92

E.mail : satelec@acteongroup.com

www.acteongroup.com

10. 2 SUBSIDIARIES' ADDRESS

FRANCE

SATELEC ACTEON GROUP

17, avenue Gustave Eiffel - BP 216

33708 MERIGNAC cedex - France

Tel. +33 (0) 556.34.06.07

Fax. +33 (0) 556.34.92.92

E.mail : satelec@acteongroup.com

U.S.A.

ACTEON NORTH AMERICA

130 Gaither Drive - Suite 100

MOUNT LAUREL NJ 08034 - U.S.A.

Tel. +1 856 222 9988

Fax. +1 856 222 4726

E.mail : info@us.acteongroup.com

GERMANY

ACTEON GERMANY

Industriestrasse 9 - D-40822 METTMANN

Tel. +49 21 04 95 65 10

Fax. +49 21 04 95 65 11

E.mail : info@de.acteongroup.com

SPAIN

ACTEON IBERICA
Avda Principal n°11 H
Poligono Industrial Can Clapers
E-08181 SENTMENAT (BARCELONA)
Tel. +34 93 715 33 66
Fax. +34 93 715 32 29
E.mail : info@es.acteongroup.com

U.K.

SATELEC (U.K.) LIMITED
Unit 1B - Steel Close - Eaton Cocon, St Neots
GB-CAMBS PE19 8TT
Tel. +44 1480 477 307
Fax. +44 1480 477 381
E.mail : info@uk.acteongroup.com

MIDDLE EAST

ACTEON MIDDLE EAST
Numan Center - 1st Floor N°111
Gardens Street
PO Box 468 - AMMAN 11953 - JORDAN
Tel. +962 6 553 4401
Fax. +962 6 553 7833
E.mail : info@me.acteongroup.com

CHINA

ACTEON CHINA
Office 401 - 12 Xinyuanxili Zhong Street
Chaoyang District - BEIJING 100027 - CHINA
Tel. +86 10 646 570 11/2/3
Fax. +86 10 646 580 15
E.mail : info@cn.acteongroup.com

PHILIPPINES

ACTEON PHILIPPINES
3F King's Court II Bldg. - Corner Pasong Tamo &
de la Rosa Street
Makati City - 1200 METRO MANILA - PHILIPPINES
Tel. +632 811 2487
Fax. +632 811 2488
E.mail : info@ph.acteongroup.com

THAILAND

ACTEON (THAILAND) LTD
23/81 Sorachai Bldg 20th floor
Sukumvit 63 Road - Klongtan Nua Wattana
10110 BANGKOK - THAILAND
Tel. +66 2 714 3295
Fax. +66 2 714 3296
E.mail : info@th.acteongroup.com

KOREA

ACTEON KOREA
8F Hanil B/D - 132-4 1Ga Bongrae-dong
JOONG-GU - SEOUL - KOREA
Tel. +82 2 753 41 91
Fax. +82 2 753 41 93
E.mail : info@kr.acteongroup.com

INDIA

ACTEON INDIA
E-91, G.I.D.C. Electronic Estate - Sector 26
GANDHINAGAR 382044 (Gujarat)
Tel. +91 79 2646 2040
Fax. +91 79 2646 2041
E.mail : info@in.acteongroup.com

XI - SYMBOLS

 Alternating current

 Type BF device

 Warning, please refer to the accompanying documentation.

Marking: please refer to the label stuck to the device.

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