**OPERATOR'S MANUAL - ENGLISH  
MANUAL DE INSTRUCCIONES - SPANISH  
MANUEL D'UTILISATION - FRENCH  
BENUTZERHANDBUCH - GERMAN  
MANUAL DO OPERADOR - BRAZILIAN PORTUGUESE  
MANUALE D'USO - ITALIAN**

## **PLATINUM™ 5**

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## SPECIAL NOTES

---

Invacare 5 <sup>with</sup>/SensO<sub>2</sub>®  
Model IRC5LXO2AW

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Invacare 5  
Modelo IRC5LXAW

## SPECIAL NOTES

**WARNING/CAUTION** notices used in this manual apply to hazards or unsafe practices which could result in personal injury or property damage.

### NOTICE

**THE INFORMATION CONTAINED IN THIS DOCUMENT IS SUBJECT TO CHANGE WITHOUT NOTICE.**

## WARNING

**DO NOT OPERATE THIS EQUIPMENT WITHOUT FIRST READING AND UNDERSTANDING THIS MANUAL. IF YOU ARE UNABLE TO UNDERSTAND THE WARNINGS, CAUTIONS, AND INSTRUCTIONS, CONTACT INVACARE TECHNICAL SERVICES BEFORE ATTEMPTING TO USE THIS EQUIPMENT - OTHERWISE SERIOUS INJURY OR PROPERTY DAMAGE MAY RESULT.**

## SAVE THESE INSTRUCTIONS

## DANGER

**Users MUST NOT SMOKE while using this device. Keep all matches, lighted cigarettes or other sources of ignition out of the room in which this product is located. NO SMOKING signs should be prominently displayed. Textiles and other materials that normally would not burn are easily ignited and burn with great intensity in oxygen enriched air. Failure to observe this warning can result in severe fire, property damage and cause physical injury or DEATH.**

---

TABLE OF CONTENTS

**CONCENTRATOR  
TABLE OF CONTENTS**

<b>SPECIAL NOTES .....</b>	<b>2</b>
<b>SAFETY SUMMARY .....</b>	<b>5</b>
<b>FEATURES .....</b>	<b>8</b>
<b>SPECIFICATIONS .....</b>	<b>9</b>
<b>HANDLING .....</b>	<b>12</b>
Unpacking .....	12
Inspection .....	12
Storage .....	12
<b>INTRODUCTION .....</b>	<b>13</b>
<b>OPERATING INSTRUCTIONS .....</b>	<b>13</b>
Select a Location .....	13
Recommended End-User Guidelines for Optimum Performance .....	13
Plug in Power Cord .....	14
Connect Humidifier (if so prescribed) .....	14
Power Switch .....	16
Flowrate .....	16
SensO <sub>2</sub> <sup>®</sup> Oxygen Purity Indicator (if the unit features the O <sub>2</sub> Sensor) .....	17
Initial Startup of the Concentrator .....	17
Explanation of Oxygen Purity Indicator Lights .....	18
Indicators (if the unit does not feature the O <sub>2</sub> Sensor) .....	18
<b>CONCENTRATOR MAINTENANCE .....</b>	<b>19</b>
Cleaning the Cabinet Filters .....	19
Cabinet .....	20
Humidifier .....	20
<b>TROUBLESHOOTING GUIDE .....</b>	<b>20</b>
<b>PREVENTIVE MAINTENANCE RECORD .....</b>	<b>22</b>
<b>WARRANTY .....</b>	<b>23</b>

## **TABLE OF CONTENTS**

---

<b>SPANISH OWNER'S MANUAL (MANUAL DEL USARIO) .....</b>	<b>24</b>
<b>FRENCH OWNER'S MANUAL (MANUEL D'UTILISATION) .....</b>	<b>49</b>
<b>GERMAN OWNER'S MANUAL (BENUTZERHANDBUCH) .....</b>	<b>73</b>
<b>BRAZILIAN PORTUGUESE OWNER'S MANUAL (MANUAL DO OPERADOR) .....</b>	<b>97</b>
<b>ITALIAN OWNER'S MANUAL (MANUALE D'USO) .....</b>	<b>123</b>

---

## SAFETY SUMMARY

---

### SAFETY SUMMARY

#### CAUTION

Risk of electric shock. **DO NOT** disassemble. Refer servicing to qualified service personnel.

**READ THE FOLLOWING INFORMATION BEFORE OPERATING THIS PRODUCT.**

#### WARNING

The use of oxygen therapy requires that special care be taken to reduce the risk of fire. Any materials that will burn in air, and some that will not, are easily ignited and burn rapidly in high concentrations of oxygen. For safety concerns, it is necessary that all sources of ignition be kept away from the product and preferably out of the room in which it is being used. **NO SMOKING** signs should be prominently displayed.

A spontaneous and violent ignition may occur if oil, grease or greasy substances come in contact with oxygen under pressure. These substances **MUST** be kept away from the oxygen concentrator, tubing and connections, and all other oxygen equipment. **DO NOT** use any lubricants unless expressly recommended by Invacare.

For optimum performance, Invacare recommends that each concentrator be on and running for a minimum of 30 minutes at a time. Shorter periods of operation may reduce maximum product life.

If the concentrator has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water, call **Qualified Service Personnel** for examination and repair.

Keep the cord away from **HEATED** or **HOT** surfaces.

Do not move or relocate concentrator by pulling on the cord.

**NEVER** drop or insert any object into any opening.

**NEVER** block the air openings of the product or place it on a soft surface, such as a bed or couch, where the air opening may be blocked. Keep the openings free from lint, hair and the like.

---

## **SAFETY SUMMARY**

---

### **WARNING (CONTINUED)**

Fill humidifier with water to the level shown by the manufacturer. **DO NOT** overfill.

Keep unit at least 7.5 cm (3 in) away from walls, draperies, furniture, and the like.

Invacare recommends that Crush-Proof oxygen tubing (supplied by Invacare) be used with this product and **NOT** exceed 15.2 m (50 ft) in length.

### **MAINTENANCE**

The Invacare Oxygen Concentrator was specifically designed to minimize routine preventive maintenance at intervals of once per year. Only professionals of the healthcare field or persons fully conversant with this process such as authorized or factory trained personnel should perform preventive maintenance or performance adjustments on the oxygen concentrator.

### **RADIO FREQUENCY INTERFERENCE**

Most electronic equipment is influenced by Radio Frequency Interference (RFI). **CAUTION** should be exercised with regard to the use of portable communications equipment in the area around such equipment.

### **TO REDUCE THE RISK OF BURNS, ELECTROCUTION, FIRE OR INJURY TO PERSONS.**

Avoid using while bathing. If continuous usage is required by the physician's prescription:

The concentrator must be located in another room at least 2.1 m (7 ft) from the bath.

**DO NOT** come in contact with the concentrator while wet.

**DO NOT** place or store product where it can drop into water or other liquid.

**DO NOT** reach for product that has fallen into water.  
**UNPLUG IMMEDIATELY.**

**A product should NEVER be left unattended when plugged in.**

---

---

## SAFETY SUMMARY

### **WARNING (CONTINUED)**

This device is to be used only in accordance with the prescription of a physician and this Owners Manual. If at any time the patient or attendant conclude that the patient is receiving an insufficient amount of oxygen, the supplier and/or physician should be contacted immediately. No adjustments should be made to the flowrate unless prescribed by a physician.

Close supervision is necessary when this product is used near children or physically-challenged individuals.

Use this product for only intended use as described in this manual.

**DO NOT** use parts, accessories or adapters other than those authorized by Invacare.

**DO NOT** connect the concentrator in parallel or series with other oxygen concentrators or oxygen therapy devices.

Use of some administration accessories or certain humidifiers not specified for use with oxygen concentrator may impair the performance.

In certain circumstances oxygen therapy can be hazardous. Invacare recommends that you seek medical advice before using this machine.

This equipment is to be used as an oxygen supplement and is not considered life supporting or life sustaining.

Avoid creation of any spark near medical oxygen equipment. This includes sparks from static electricity created by any type of friction.

### **OXYGEN CONCENTRATION**

#### **WARNING**

Take care not to set the flow above RED ring. An oxygen flow greater than 5 L/min will decrease the oxygen concentration.


---

## FEATURES


---

### FEATURES

#### FRONT VIEW

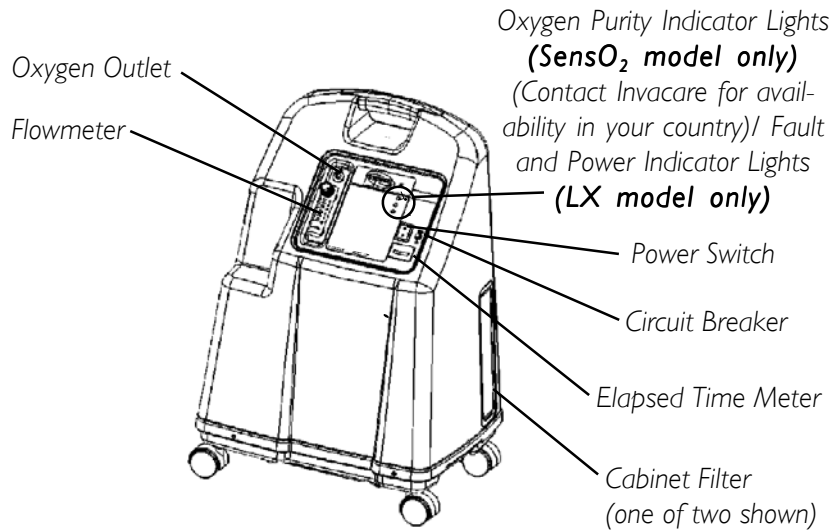#### REAR VIEW
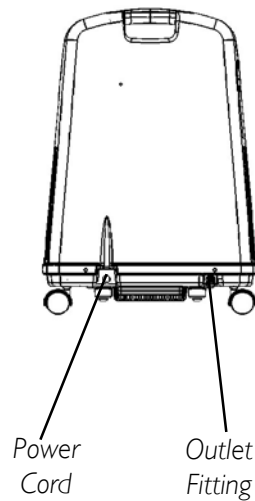
#### ACCESSORIES (NOT SHOWN):

HomeFill® home oxygen compressor - IOH200AW

Contact local Invacare representative for available accessory information.

**NOTE:** This outlet fitting is to be used only for filling oxygen cylinders with the HomeFill home oxygen compressor. The outlet fitting does not affect concentrator performance. Refer to the HomeFill owner's manual, for connection and operating instructions. When not in use, the plug provided with the concentrator should be inserted into the outlet fitting. For more information about the HomeFill, contact your Invacare dealer.

## SPECIFICATIONS

# SPECIFICATIONS

-  Alternating Current
-  Type BF Equipment
-  Running
-  NOT Running
-  Attention-Consider Accompanying Documents
-  DO NOT Smoke
-  Class II

**Electrical Requirements:** 230 VAC  $\pm$  10% (253 VAC/207 VAC), 50 Hz

**Rated Current Input:** 2.0 A

**Sound Level:** 44 dB Average

**Altitude:** Up to 1828 m(6,000 ft) above sea level without degradation of concentration levels. **NOT RECOMMENDED FOR USE ABOVE 1828 M (6000 FT).**

**\*Oxygen Output**

**Concentration Levels:** ALL 5LXA/5LXO2A MODELS  
 93% minimum at 1 to 3 L/min  
 91% minimum at 4 L/min  
 87% minimum at 5 L/min (maximum recommended flow)  
**\* Stated concentration levels achieved after initial warm-up period (approximately 30 minutes).**

**Maximum Outlet Pressure:** 34.5  $\pm$  3.45 kPa  
 (5  $\pm$  0.5 psi)

### 5LXO<sub>2</sub> INDICATORS:

Label Symbol	Status	Indicator Lights (LED)
	SYSTEM FAILURE Below 73%	<b>RED</b> Continuous Audible Alarm Sieve-GARD™ Compressor Shutdown
	73% to 85%	<b>YELLOW</b>
O <sub>2</sub>	SYSTEM OKAY Over 85%	<b>GREEN</b>

**SPECIFICATIONS**

**SPECIFICATIONS (CONTINUED)**

**5LX INDICATORS:**

Label Symbol	Status	Indicator Lights (LED)
I/O	SYSTEM OKAY	GREEN
	SYSTEM FAILURE	RED

<b>Flow Range:</b>	0.5 - 5 L/min. Flowrates less than 1 L/min., are not recommended.
<b>Low Flow Alarm:</b>	0 L/min. to 0.5 L/min., Rapid Audible Alarm Beeping (No Accessories Connected)
<b>Average Power Consumption:</b>	395W
<b>Pressure Relief Mechanism Operational at:</b>	241 kPa ± 24.1 kPa (35 psi ± 3.5 psi)
<b>Change in maximum recommended flow when back pressure of 7kPa is applied:</b>	0.7 L/min.
<b>Filters:</b>	Cabinet (2), Outlet HEPA and Compressor Inlet.
<b>Safety System:</b>	Current overload or line surge shutdown. High temperature compressor shutdown. High Pressure Alarm w/compressor shutdown. Low Pressure Alarm w/compressor shutdown. Battery Free Power Loss Alarm. Sen <sub>2</sub> Oxygen System ( <b>SensO<sub>2</sub> Model</b> ) Low Flow Alarm.
<b>Width:</b>	46.7 cm ± 1 cm (18 3/8" ± 3/8")
<b>Height:</b>	67.0 cm ± 1 cm (26 3/8" ± 3/8")
<b>Depth:</b>	36.5 cm ± 1 cm (14 3/8" ± 3/8")
<b>Weight:</b>	23.2 kg ± 0.5 kg (51 lbs. ± 1 lb)
<b>Shipping Weight:</b>	26.3 kg ± 0.5 kg (58 lbs. ± 1 lb)

---

## SPECIFICATIONS

### **SPECIFICATIONS (CONTINUED)**

<b>Operating Ambient: Temperature:</b>	10°C - 35°C (50°F - 95°F) at 20-60% relative humidity
<b>Exhaust:</b>	Less than Ambient + 19°C (+ 35°F).
<b>Oxygen Output:</b>	Less than Ambient + 2°C (+ 4°F).
<b>Cabinet:</b>	Impact resistant flame-retardant plastic cabinet that conforms to UL 94-V0.
<b>Regulatory Listing:</b>	<b>5LXA/5LXO2A Models</b> ETL certified complying with: EN 55011 EN 61000-3-2 EN 61000-3-3 IEC 60601-1, A1, A2 IEC 60601-1-2 <b>5LXO2A Model only</b> ISO 8359 MDD
<b>Electrical:</b>	No extension cords.
<b>Placement:</b>	No closer than 7.5 cm (3 in) from any wall, furniture, draperies, or similar surfaces.
<b>Tubing:</b>	Crush-Proof Tubing - 2.1 m (7 ft) cannula with a maximum 15.2 m (50 ft) extension <b>(DO NOT PINCH)</b> .
<b>Relative Humidity:</b>	20 to 60%
<b>Time of Operation:</b>	Up to 24 hours per day.
<b>Recommended Storage and Shipping Temperature:</b>	-29°C to 65°C (-20°F to 150°F), at 15-95% relative humidity

## HANDLING


---

### HANDLING

The concentrator should always be kept in the upright position to prevent cabinet damage while being transported. The shipping container has been designed to assure maximum protection of the concentrator. If the concentrator is to be reshipped by common carrier, additional cartons are available from Invacare.

### UNPACKING

1. Check for any obvious damage to the carton or its contents. If damage is evident, notify the carrier, or your local dealer.
2. Remove all loose packing from the carton.
3. Carefully remove all the components from the carton.

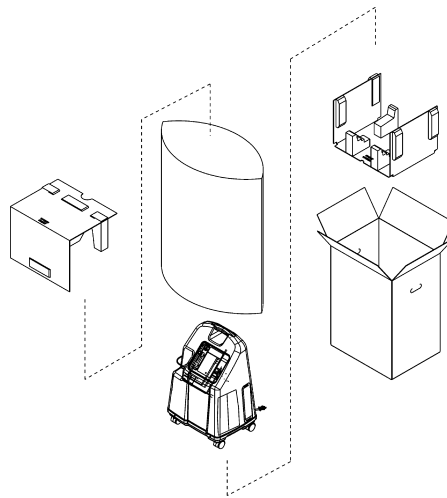**FIGURE I - PACKAGING**

*NOTE: Unless the oxygen concentrator is to be used immediately, retain containers and packing materials for storage until use of the concentrator is required.*

### INSPECTION

1. Examine exterior of the oxygen concentrator for nicks, dents, scratches or other damages. Inspect all components.

### STORAGE

1. Store the repackaged oxygen concentrator in a dry area.
2. **DO NOT** place other objects on top of the repackaged concentrator.

---

## INTRODUCTION/ OPERATING INSTRUCTIONS

---

### INTRODUCTION

Your oxygen concentrator is intended for individual use in the home. It is an electronically operated device that separates oxygen from room air. It provides high concentration of oxygen directly to you through a nasal cannula. Clinical studies have documented that oxygen concentrators are therapeutically equivalent to other types of oxygen delivery systems.

Your provider will show you how to use your oxygen concentrator. He/She should be contacted with any questions or problems regarding your oxygen concentrator. This owner's manual will tell you about your concentrator and will serve as a reference as you use your concentrator.

### OPERATING INSTRUCTIONS

#### SELECT A LOCATION

You may select a room in your house where using your oxygen concentrator would be most convenient. Your concentrator can be easily rolled from room to room on its casters.

Your oxygen concentrator will perform best when operated under the following conditions. Usage in environments other than those described may result in the need for increased equipment maintenance. The air intake of the unit should be located in a well ventilated area to avoid airborne pollutants and/or fumes.

#### RECOMMENDED END-USER GUIDELINES FOR OPTIMUM PERFORMANCE:

<b>Temperature:</b>	10°C - 35°C (50°F - 95°F)
<b>Electrical:</b>	No extension cords.
<b>Placement:</b>	No closer than 7.5 cm (3 in) from the wall, furniture, draperies, or similar surfaces.
<b>Tubing and Cannula:</b>	2.1 m (7 ft) cannula with a maximum 15.2 m (50 ft) of <b>Crush-Proof</b> Tubing ( <b>DO NOT PINCH</b> ).
<b>Environment:</b>	Smoke and soot-free. No confined spaces ( <b>Example: NO closets</b> ).
<b>Relative Humidity:</b>	20 to 60%
<b>Time of Operation:</b>	Up to 24 hours per day.
<b>Flow Rate</b>	<b>From 0.5 L/min. to 5 L/min.</b> Flowrates less than 1 L/min., are not recommended.
<b>Minimum Operating Time:</b>	30 Minutes

## **OPERATING INSTRUCTIONS**

---

*NOTE: Ensure that your concentrator is at least 7.5 cm (3 in) away from walls, draperies or furniture to assure sufficient air flow. Avoid deep pile carpets and heaters, radiators or hot air registers.*

### **PLUG IN POWER CORD**

1. Plug in power cord to an electrical outlet.

### **CONNECT HUMIDIFIER (IF SO PRESCRIBED)**

1. Fill humidifier with distilled water to the level indicated by the manufacturer. (Recommended humidifiers: Invacare 3260, 003-40, 006-40, or M5120).


### **WARNING**

**DO NOT** overfill humidifier.

Replace the humidifier cap and securely tighten.


**DO NOT** reverse the oxygen input and output connections. **Water from the humidifier bottle WILL** travel through the cannula back to the patient.

---

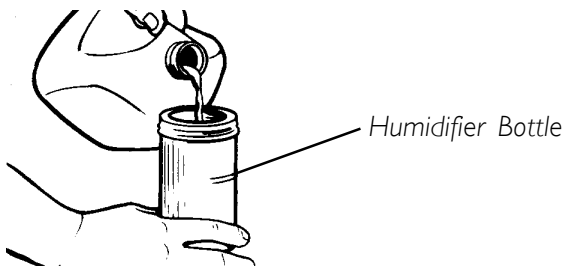**FIGURE 2 - FILLING THE HUMIDIFIER**

2. Place the humidifier in the humidifier compartment.

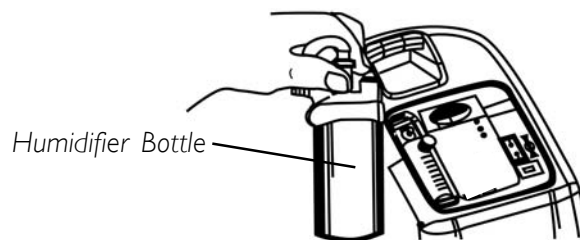
**FIGURE 3 - HUMIDIFIER COMPARTMENT**


## OPERATING INSTRUCTIONS

3. Remove the Filter Access Door located on the side of the concentrator. Insert a flathead screwdriver in the plate groove on the top edge of the Filter Access Door and gently pry the Filter Access Door open (**FIGURE 4, DETAIL "A"**).
4. The humidifier adapter is next to the inlet filter. Pull up and remove the humidifier bottle adapter (**FIGURE 4, DETAIL "B"**).
5. Attach it to the humidifier by turning the wing nut on the humidifier bottle counterclockwise until it is securely fastened (**FIGURE 4, DETAIL "C"**).


### DETAIL "A"

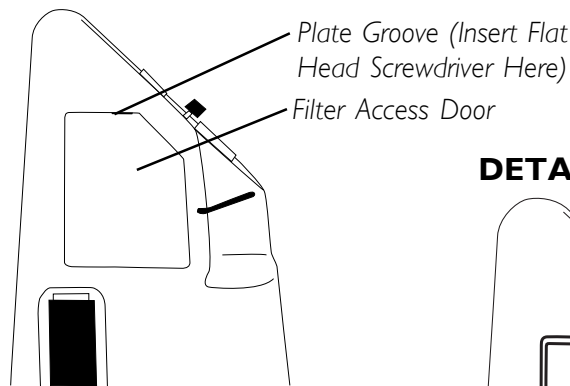### DETAIL "B"
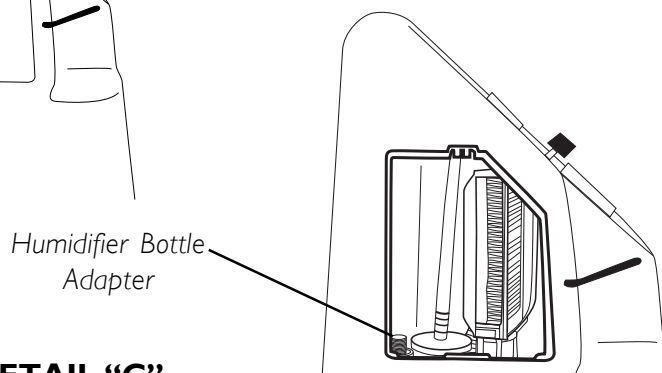
### DETAIL "C"

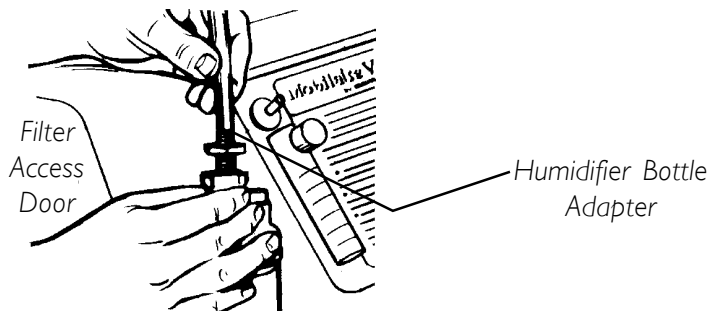
**FIGURE 4 - ATTACHING THE HUMIDIFIER BOTTLE ADAPTER**

## OPERATING INSTRUCTIONS

---

6. Attach oxygen tubing from the humidifier bottle to oxygen outlet connector on the oxygen concentrator.

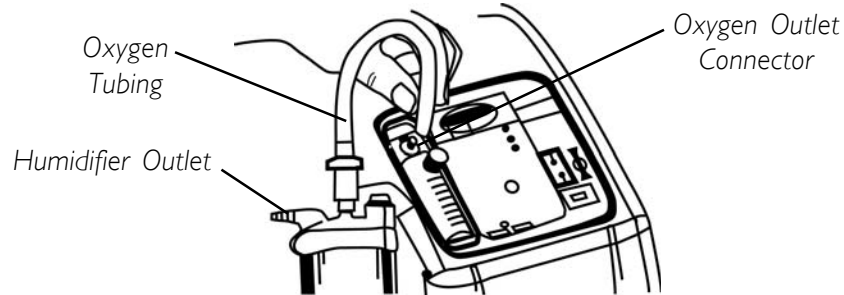**FIGURE 5 - ATTACHING THE OXYGEN TUBING**


7. Attach the cannula/patient supply tubing to the humidifier bottle outlet.
8. After assembly, ensure that oxygen is flowing through the cannula.
9. Replace the filter access door on the side of concentrator by snapping it back into the plate groove.

### WARNING

**For optimum performance, Invacare recommends that the concentrator be operated a minimum of 30 minutes during any use. Shorter periods of operation may adversely affect the life of the product.**

### POWER SWITCH

1. Press power switch to the **ON** position. All the panel lights and the audible alarm will come on for one second, indicating that the unit is functioning properly.

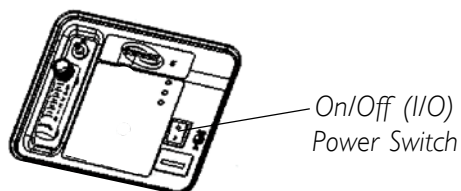### FLOWRATE FIGURE 6 - POWER SWITCH

1. Turn the flowrate knob to the setting prescribed by your physician or therapist.


## OPERATING INSTRUCTIONS

### WARNING

**DO NOT** change the L/min. setting on the flowmeter unless a change has been prescribed by your physician or therapist.

*NOTE: To properly read the flowmeter, locate the prescribed flowrate line on the flowmeter. Next, turn the flow knob until the ball rises to the line. Now, center the ball on the L/min. line prescribed (FIGURE 7).*

2. If the flowrate on the flowmeter ever falls below 0.5 L/min. for more than about one minute, the **LOW FLOW ALARM** will be triggered. This is a rapid beeping of the audible alarm. Check your tubing or accessories for blocked or kinked tubing or a defective humidifier bottle. After rated flow is restored, more than 0.5 L/min., the Low Flow audible alarm will go off.

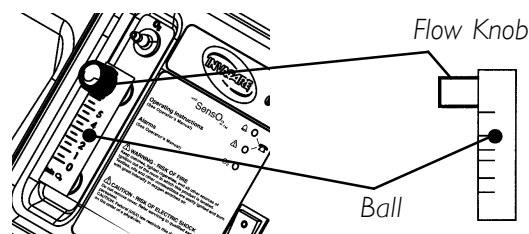**FIGURE 7 - ADJUSTING THE FLOWRATE-  
PLATINUM 5 WITH SENSO<sub>2</sub> SHOWN**

*NOTE: The use of some accessories such as the Homefill compressor will deactivate the Low Flow Alarm.*

### **SENSO<sub>2</sub> OXYGEN PURITY INDICATOR (IF YOUR UNIT FEATURES THE O<sub>2</sub> SENSOR FEATURE)**

This feature monitors the purity of oxygen generated by the oxygen concentrator. If purity falls below factory preset standards, indicator lights on the control panel will illuminate.

### **INITIAL STARTUP OF THE CONCENTRATOR**

*NOTE: Concentrator may be used during the initial start warm-up time (approximately 30 min.) while waiting for the O<sub>2</sub> purity to reach maximum.*

When the unit is turned on, the green light will come on (SYSTEM OK/O<sub>2</sub> greater than 85%). After 5 minutes, the oxygen sensor will be operating normally and will control the indicator lights depending on oxygen concentration values. The explanation of the indicator light functions are as follows.

## OPERATING INSTRUCTIONS

### EXPLANATION OF OXYGEN PURITY INDICATOR LIGHTS (FIGURE 8):

**GREEN LIGHT (O<sub>2</sub>)** - Normal Operation

**YELLOW LIGHT (⚠)** - Call Supplier **IMMEDIATELY**. You may continue to use the concentrator unless instructed otherwise by your supplier. Be certain that **BACKUP OXYGEN** is nearby.

**RED LIGHT (🔔)** - Total unit shutdown. Switch **IMMEDIATELY** to **BACKUP OXYGEN** supply. Call Supplier **IMMEDIATELY**.

**GREEN LIGHT** - with **YELLOW LIGHT FLASHING** - Call Supplier **IMMEDIATELY**. Oxygen sensor malfunctioning; you may continue to use the concentrator.

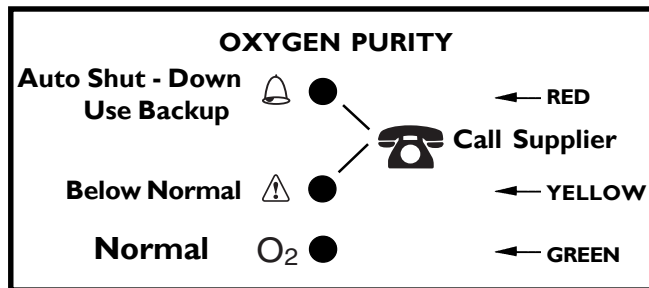
FIGURE 8 - O<sub>2</sub> DISPLAY CONTROL PANEL

### INDICATORS - IF YOUR UNIT DOES NOT FEATURE THE O<sub>2</sub> SENSOR

#### LX MODELS INDICATOR LIGHT EXPLANATION (FIGURE 9):

**RED LIGHT (🔔)** - Total Unit Shut-Down. Switch **IMMEDIATELY** to a **BACK-UP OXYGEN** supply. Call Supplier **IMMEDIATELY**.

**GREEN LIGHT (I/O)** - On/Off. System okay.

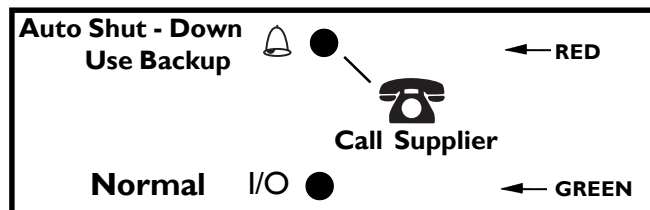
FIGURE 9 - DISPLAY CONTROL PANEL

## CONCENTRATOR MAINTENANCE


*NOTE: Normal maintenance is to be performed by the user or caregiver. The following outlines normal maintenance procedures.*

*NOTE: The Invacare Concentrators are specifically designed to minimize routine preventive maintenance at intervals of once per year. In places with high dust or soot levels, maintenance may need to be performed more often. Preventive maintenance must be performed at a minimum of one (1) year in service to assure years of additional reliability. Only authorized or factory-trained personnel should perform preventive maintenance on the concentrator. Power should be disconnected before beginning preventive maintenance on the Invacare concentrator.*

### A. CLEANING THE CABINET FILTERS

*NOTE: There are two (2) cabinet filters. One (1) is located on each side of the cabinet (FIGURE 1).*

1. Remove each filter and **CLEAN AT LEAST ONCE A WEEK** depending on environmental conditions (FIGURE 2).

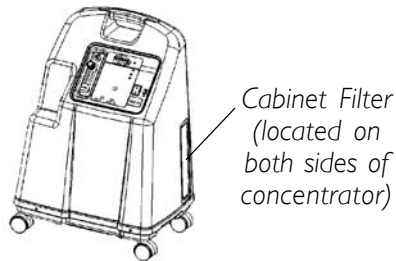
**FIGURE 1 - CABINET FILTER LOCATION**

*NOTE: Environmental conditions that may require more frequent cleaning of the filters include but are not limited to: high dust, smoking, air pollutants, etc.*

2. Clean the cabinet filters with a vacuum cleaner or wash in warm soapy water and rinse thoroughly (FIGURE 2).
3. Dry the filters thoroughly before reinstallation.

### CAUTION

**DO NOT operate the concentrator without the filters installed.**

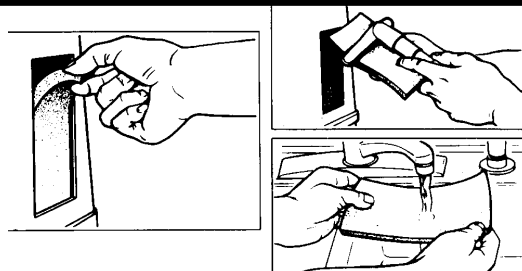**FIGURE 2 - CLEANING THE CABINET FILTERS**

## MAINTENANCE/TROUBLESHOOTING

### B. CABINET

#### WARNING

**UNPLUG** the concentrator when cleaning. To avoid electrical shock, **DO NOT** remove cabinet.

1. Clean the cabinet with a mild household cleaner and non-abrasive cloth or sponge.

### C. HUMIDIFIER

*NOTE: To clean the oxygen humidifier, follow the instructions provided by the manufacturer. If none are provided, follow these steps:*

1. Clean the humidifier every day.
2. Wash it in soapy water and rinse it with a solution of ten (10) parts water and one (1) part vinegar.
3. Rinse thoroughly with hot water and refill with distilled water to the level shown on the humidifier.

#### WARNING

**DO NOT** overfill Humidifier Bottle.

## CONCENTRATOR TROUBLESHOOTING GUIDE

SYMPTOM:	PROBABLE CAUSE:	SOLUTION:
<b>Alarm:</b> <b>SHORT BEEPS,</b> <b>LONG PAUSE</b>  <b>Concentrator</b> <b>NOT operating,</b> <b>power switch ON.</b> <b>Beep.....Beep.....</b>	<b>Main Power Loss:</b> 1. Power cord NOT plugged In.  2. No power at outlet.  3. Tripped circuit breaker.	1. Insert plug into outlet.  2. Inspect house circuit breakers or fuses. If problem recurs, use a different outlet.  3. <b>Call Service Provider.</b>
<b>Alarm:</b> <b>CONTINUOUS</b>  <b>Concentrator</b> <b>NOT operating,</b> <b>Power Switch ON.</b>  <b>Beeeeeeep....</b>	<b>System Failure:</b> 1. Unit overheating due to blocked air intake.      2. Insufficient power at outlet.   3. Internal repairs required.	1a. Remove and clean cabinet filters.  1b. Move oxygen concentrator at least 7.5 cm (3 in) away from walls, draperies or furniture.  2. Do not use extension cords. Move to another electrical outlet or circuit.  3. <b>Call Service Provider.</b>

## TROUBLESHOOTING

SYMPTOM:	PROBABLE CAUSE:	SOLUTION:
<p><b>YELLOW or RED LIGHT Illuminated.</b></p> <p><b>Alarm:</b> <b>CONTINUOUS</b> <b>On red light only.</b></p> <p><b>* Only applies to SensO<sub>2</sub> models.</b></p>	<ol style="list-style-type: none"> <li>1. Low oxygen purity.*</li> <li>2. Kinked or blocked tubing, cannula or humidifier.*</li> <li>3. Flowmeter set at 1/2 L/min. or less.*</li> <li>4. Unit overheating due to blocked air intake.</li> <li>5. Insufficient power at outlet.</li> <li>6. Internal repairs required.</li> </ol>	<ol style="list-style-type: none"> <li>1. Clean or replace filters.</li> <li>2. Inspect for kinks or blockages. Correct, clean or replace item. Once corrected, turn power OFF for 60 seconds and then turn power back ON.</li> <li>3. Check flowmeter is set to 1.0 L/min. or more. Refer to Recommended End-User Guidelines in the Operating Instructions of this manual.</li> <li>4a. Remove and clean cabinet filters.</li> <li>4b. Move concentrator at least 7.5 cm (3 in) from walls, draperies, and furniture.</li> <li>5. Do not use extension cords. Move to another electrical outlet or circuit.</li> <li>6. <b>Call Service Provider.</b></li> </ol>
<p><b>GREEN LIGHT with YELLOW Light Flashing.</b></p>	<ol style="list-style-type: none"> <li>1. Internal repairs required.</li> </ol>	<ol style="list-style-type: none"> <li>1. <b>Call Service Provider.</b></li> </ol>
<p><b>Alarm:</b> <b>RAPID</b> <b>Beep....Beep...Beep...Beep</b></p>	<p><b>LOW FLOW ALARM</b></p> <ol style="list-style-type: none"> <li>1a. Kinked or blocked tubing, cannula or humidifier.</li> <li>1b. Flowmeter set at 0.5 L/min. or less.</li> </ol>	<ol style="list-style-type: none"> <li>1a. Inspect for kinks or blockages. Correct, clean or replace item. Once corrected, turn power OFF for 60 seconds and then turn power back ON.</li> <li>1b. Reset flowmeter to prescribed flowrate. Flowrates less than 1L/min. are not recommended.</li> </ol>

# PREVENTIVE MAINTENANCE RECORD

## PREVENTIVE MAINTENANCE RECORD INVACARE PLATINUM OXYGEN CONCENTRATOR

MODEL NO. IRC \_\_\_\_\_ SERIAL NO. \_\_\_\_\_

Yes, you can:

ON EACH INSPECTION																				
Record Date of Service																				
Record Elapsed Hours On Hour Meter																				
Clean Cabinet Filter(s)																				
Check Prescribed L/min. Flow Rate																				
<b>EVERY 90 DAYS</b>																				
Check Oxygen Concentration (UNITS WITHOUT SensO <sub>2</sub> )																				
<b>ANNUALLY OR DURING PREVENTIVE MAINTENANCE SCHEDULE</b>																				
Clean/Replace Cabinet Filter(s)																				
Check Outlet HEPA Filter*																				
Check Compressor Inlet Filter*																				
Check Oxygen Concentration (SensO <sub>2</sub> MODELS)																				
Check Power Loss Alarm																				
<b>AS REQUIRED</b>																				
Clean Heat Exchanger																				
Replace Exhaust Muffler																				
Rebuild Top End of Compressor																				

\*Refer to Preventive Maintenance section of Service Manual.

## **LIMITED WARRANTY**

This warranty is extended only to the original purchaser of our products and does not affect statutory rights.

Invacare warrants the Oxygen Concentrators IRC5LXAW and IRC5LXO2AW to be free from defects in materials and workmanship for a period of five (5) years from date of purchase. The 4-way valve body carries a lifetime warranty from date of purchase. **DO NOT** open or attempt to service valve body. This will void any and all warranty applicable to the valve body. Routine maintenance items, such as filters, etc..., are excluded from cover under this warranty. If within such warranty period the product shall be proven to Invacare's reasonable satisfaction to be defective, it shall be repaired or replaced, at Invacare's option; Invacare's sole obligation, and your exclusive remedy under this warranty being limited to such repair or replacement.

For warranty service, please contact Invacare Service Department during normal working hours. Upon receiving notice of an alleged defect in a product, Invacare will issue a serialised return authorisation. The product shall then be returned to either Invacare's plant or service center as specified by Invacare. Defective component part(s) must be returned for warranty inspection using the serial number as identification within thirty (30) days of return authorization date. **DO NOT** return products to our factory without prior consent.

**LIMITATIONS AND EXCLUSIONS: THE FOREGOING WARRANTY DOES NOT COVER NORMAL WEAR AND TEAR AND SHALL NOT APPLY TO PRODUCTS SUBJECTED TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE, COMMERCIAL OR INSTITUTIONAL USE, PRODUCTS MODIFIED WITHOUT INVACARE'S EXPRESS WRITTEN CONSENT (INCLUDING, BUT NOT LIMITED TO, MODIFICATION THROUGH THE USE OF UNAUTHORISED PARTS OR ATTACHMENTS) OR TO PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE.**

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© 2009 Invacare Corporation
**Invacare Corporation**