PHYSIPRO

Seating and mobility solutions

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HP2 back Dossier HP2

This manual contains important informations about this product.

Please pass it to the final user upon delivery.

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ENGLISH

Physipro inc. is delighted to have you among its clients. We sincerely wish to thank you for the trust you have shown in our Company by choosing one of our products. This user manual has been designed to allow the client to use the HP2 back, a Physipro inc. product, safely in an optimal way. For all adjustments and settings, Physipro inc. strongly recommends you consult a qualified professional. Physipro inc. provides you with specific verifications you should perform on a regular basis in order to optimize the performance of your HP2 back and extend its operational life.

Physipro inc. relieves itself from any liability should any bodily injury or property damage result from a lack of care or misuse or from any modification made to the product without prior written consent of the Company. *For Physipro inc., your satisfaction remains a priority.*

Table of contents

1. HP2 models	3
2. Components	3
3. Recommendations	4
3.1 Basic verification	4
3.2 Safety rules	4
4. Specifications	5
5. Back installation	6
5.1 Installing the mount hardware	6
5.2 Hardware installation instructions	7
5.3 Attaching the HP2 back shell	9
6. Adjusting HP2 back height, depth and angle	10
6.1 Height Adjustment	10
6.2 Tightening the receptacle anchors	10
6.3 Depth Adjustment	11
6.4 Tilt Adjustment	13
6.5 Adjustment of the extension option	14
7. Laterals	15
7.1 Laterals Installation	15
7.1.1 Installing the profiled cushion	16
7.1.2 Removing the profiled cushion	17
7.1.3 Removing the foam from the profiled cushion	17
7.2 Height Adjustment of Laterals	18
7.3 Depth Adjustment of the Laterals	19
7.4 Additional Depth Adjustments for the Laterals	20
7.5 Angle Adjustment of the Laterals	20
7.6 Optional HP2 back Swing-Away lateral	20
7.7 Contour Laterals Installation Instructions	21
7.8 Adjustment of the Laterals	21
8. Removing and inserting the HP2 back	22
9. Care and Maintenance	22
10. Warranty	23

1. HP2 models

Three models are available:


A **contour model** designed to provide overall back support.

A **profiled model** designed for active users, which permits upper body freedom of movement and provides greater ease of propulsion.

A **sport model** designed for rigid structure chairs. It provides enhanced lumbar support and greater freedom of movement.

2. Components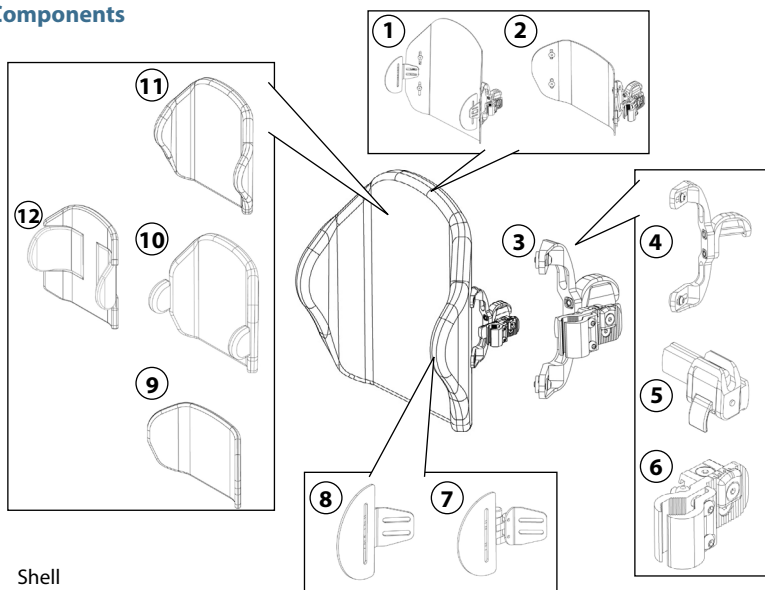

1. Shell
2. Shell of the sport model
3. Anchoring system
4. Anchor receptacle
5. Anchor slide
6. Universal clamp
7. Swing away lateral
8. Fixed lateral
9. Sport cushion
10. Cushion with external lateral support
11. Cushion with integrated lateral support
12. Cushion with integrated lateral contour support

ENGLISH

3. Recommendations

Do not use this equipment without having read and understood this user manual in its entirety. This manual includes critical information intended to ensure the safety of the user and the individuals who handle the wheelchair.

3.1 Basic verification

At the time of delivery and receipt of the HP2 back, the following verifications must be performed to ensure the safety of the user:

- ✓ Make sure the universal clamp is solid;
- ✓ Make sure the the back's anchor receptacle is solid;
- ✓ Make sure the protective cover and the cushion are in good shape;
- ✓ Make sure the lateral supports are correctly attached to the back.

3.2 Safety rules

Several safety measures must be applied to ensure the safety of the user. Note that the following list is not comprehensive. Therefore, the people who use the HP2 back have the responsibility to use it cautiously:

- ✓ Always use the anchors made for the back;
- ✓ Installing the HP2 back on a wheelchair may affect the center of gravity of the wheelchair and may cause the wheelchair to tip backwards potentially resulting in injury.
- ✓ Always judge for the potential need for anti-tippers or amputee axle adapter brackets to be added to help increase stability.
- ✓ Do not use the HP2 back to push or lift wheelchair.
- ✓ Installation and adjustment of your HP2 back should be performed only by an authorized dealer or qualified therapist.
- ✓ Do not use the HP2 back handle to lift wheelchair.

4. Specifications

Technical specifications	
Nominal widths*	12, 14, 16, 18 and 20 in (30, 35, 40, 45 and 50 cm)
Nominal heights**	12, 14, 16, 18 and 20 in (30, 35, 40, 45 and 50 cm)
Thickness	0.094 in (2.4 mm)
Cushion	Sunmate soft ½ in (12 mm) and Viscose ½ in (12mm)
Material	Aluminum
Weight	Less than 4 lbs (Aluminum)
Hardware	Standard hardware
Chest panel width	4 or 4½ in (10 or 11.5 cm) – with the foam base
Chest panel	4, 6, 7, 8 and 10 in (10, 15, 18, 20 and 25 cm) with the foam base

*The actual width of the HP2 back plate is one (1) inch smaller (2.5 cm) than the nominal value.

**The back height may be customized.

Each HP2 back size can be installed on different wheelchair widths. To choose correctly the size of your back, it is important to consider the widths of the wheelchair and of the user's thorax. See the dimensions tables below:

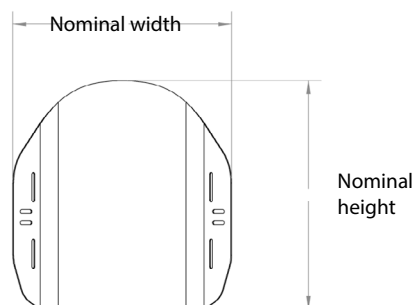
Table 2 – Dimensions of the HP2 back, Profiled and Contour models

Actual Plate Width	Nominal Width	Nominal Height Available for the Back	Thorax Width HP2 back	Associated Wheelchair Width	Associated Wheelchair Width With Extension Option
11" (28 cm)	12" (30 cm)	12", 14" (30, 35 cm)	7" to 9 ½" (18 to 24 cm)	12" to 13" (30 to 32.5 cm)	14" (35.5 cm)
13" (33 cm)	14" (35 cm)	14", 16" (35, 40 cm)	9" to 11 ½" (23 to 29 cm)	14" to 15" (35 to 37.5 cm)	16" (40.5 cm)
15" (38 cm)	16" (40 cm)	16", 18" and 20" (40, 45, 50 cm)	11" to 13 ½" (28 to 34.5 cm)	16" to 17" (40 to 42.5 cm)	18" (46 cm)
17" (43 cm)	18" (45 cm)	16", 18" and 20" (40, 45, 50 cm)	13" to 15 ½" (33 to 39 cm)	18" to 19" (45 to 47.5 cm)	20" (51 cm)
19" (48 cm)	20" (50 cm)	16", 18" and 20" (40, 45, 50 cm)	15" to 17 ½" (38 to 44.5 cm)	20" to 21" (50 to 52.5 cm)	22" (56 cm)

ENGLISH

Table 3 – Sizes associated to the HP2 back Sport model

Actual Plate Width	Nominal Width	Nominal Height	Associated Wheelchair Width	Associated Wheelchair Width With Extension Option
11" (27.5 cm)	12" (30 cm)	9" (23 cm)	12 to 13" (30 to 32.5 cm)	14" (35.5 cm)
13" (33 cm)	14" (35 cm)	9" (23 cm)	14" to 15" (35 to 38 cm)	16" (40.5 cm)
15" (38 cm)	16" (40 cm)	9" (23 cm)	16" to 17" (40 to 43 cm)	18" (46 cm)
17" (43 cm)	18" (45 cm)	9" (23 cm)	18" to 19" (45 to 48 cm)	20" (51 cm)

5. Back installation

Please read the following instructions before beginning installation.

The following steps explain how to fully install the HP2 back. The installer should follow the steps in the prescribed order.


Tools required *4 mm Allen key*


5.1 Installing the mount hardware

When you receive your back, this step will already be done by our technicians. However, this procedure will help you put it together again.


1. Insert bolt **A** in the dedicated hole and make sure the washer **B** is inserted between the bolt and the back.
2. Put the nut **C** in place, but do not tighten it, because the receptacle needs to move easily in the groove.
3. Repeat steps 1 and 2 with bolts **D**, **E**, **F** and **G**.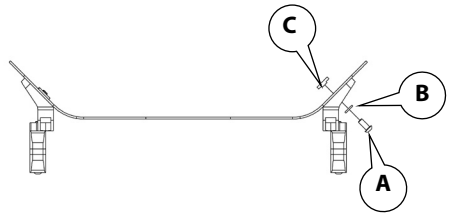

Here is what it should look like:

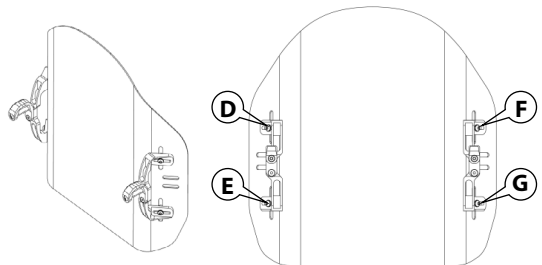
Note that, at this moment, the receptacle is not tightly fastened to the back. It can move up and down as well as sideways. It will be securely fastened in a subsequent step (5.2) when the back will be positioned correctly in the anchors.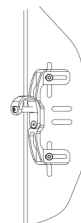


5.2 Hardware installation instructions

Note - The following steps must be done before the client is in the wheelchair.

1. Loosen the clamp screws **A** with the enclosed 4 mm Allen key

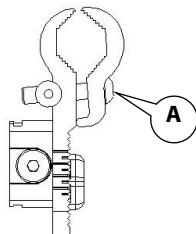
2. Open the clamp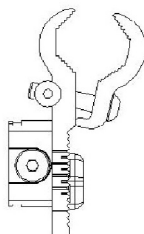

3. Determine the desired location of the receivers on the wheelchair back canes. Attach and align receiver to the wheelchair back canes. Tighten both clamp screws **A**.

4. Repeat the process for the second anchoring point.

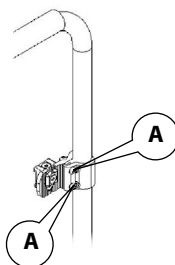


5. Make sure the mounting hardware is set at equal heights on each back canes and parallel to the seat frame.

The head of screws **A** must always point towards the outside of the wheelchair.

ENGLISH

6. Loosen the screws **E** with the enclosed 4 mm Allen key so that it slides fore and aft.
7. Insert the assembly **B** into the assembly **D**.
8. Tighten securely the screws **E**.
9. Make sure the mounting hardware **B** and **C** are parallel by aligning both anchors.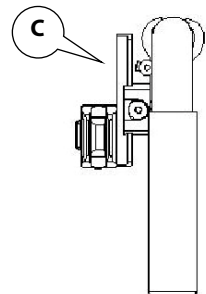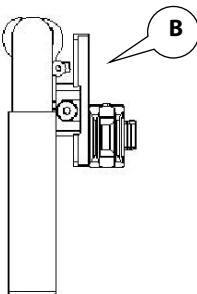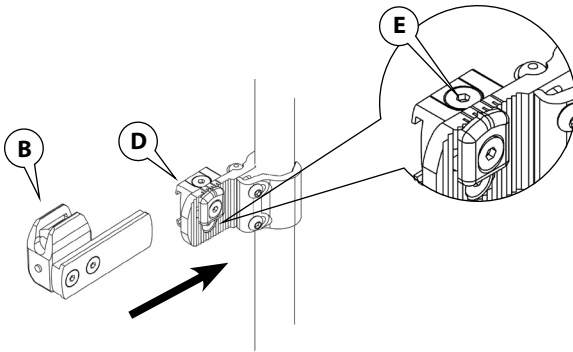


5.3 Attaching the HP2 back shell


The HP2 back is really easy to install with the two-point anchoring system. It allows for a quick and efficient insertion of the back in one simple move.

Steps to follow:


1. Open the anchor mechanism **B** by pressing on the latch.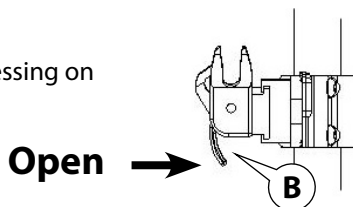

2. Put the HP2 back **F** into the anchors **B**.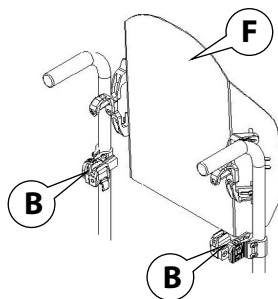

3. Once the HP2 back is in both anchors **B**, it is important to **pull on both latches until you hear a "click"**. This means the back is locked in place.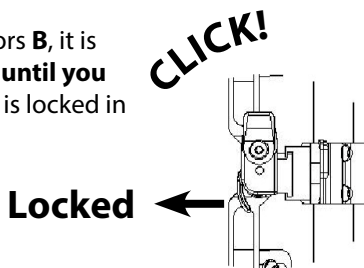

It is essential to make sure the back is correctly fastened to the wheelchair. The latches must have made a sound confirming that they are locked in place.


ENGLISH

6. Adjusting HP2 back height, depth and angle

6.1 Height Adjustment

1. Loosen screws **A** and adjust the height of the back shell to required position.
2. Securely tighten screws **A**.


Note – After height adjustment is complete, make sure the width was not inadvertently changed.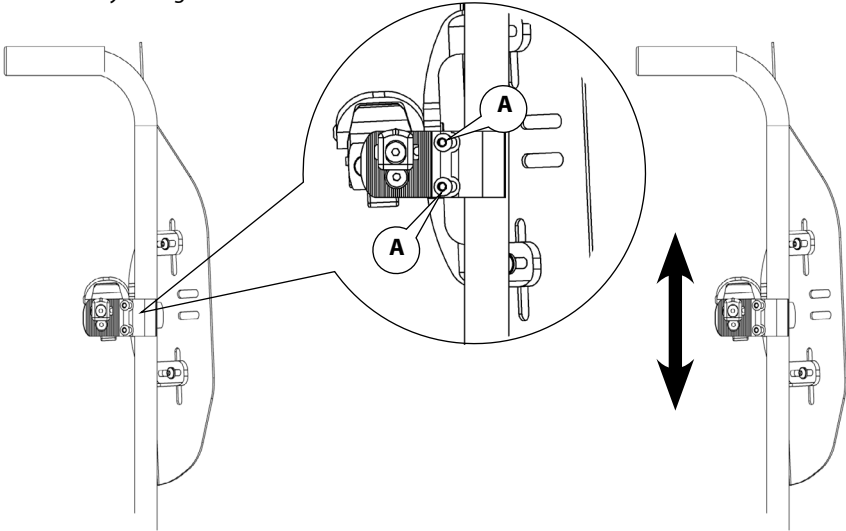


6.2 Tightening the receptacle anchors

The height of the receptacle anchors has been identified. Those anchors should now be solidly fastened to the back. (It is important to first make sure the back is placed into its anchors.)

1. Slide the receptacle into the grooves until the correct height has been reached. Make sure it is the same height on both sides.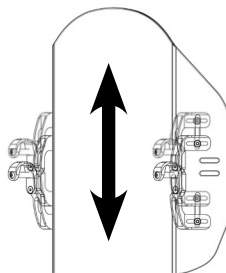

- 2. Make sure the back's interface does not touch the backs mounts. (If it does, slightly move forward the back plate so that the receptacles slide into the grooves.)
- 3. Tighten securely all four screws **A**.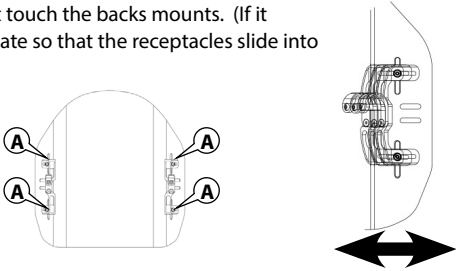


6.3 Depth Adjustment

The selection of the back format is measured according to the width of the wheelchair. The back can be installed on a wheelchair that has the same width or a wheelchair with 2 larger width formats.

Depth Chart

* 12" and 20" are unavailable for the sport version. These dimensions may vary of $\pm 1/16$ in (0.16 cm). The measures in table 3 include the Sunmate soft cushion of 1 inch. The depth measure is taken from the front of the back mounts. ** OPTION, Adjustment Extension.


HP2 back width					
	12*	14	16	18	20*
External width of the wheelchair	12	-1 to +2 ½ in			
	13	-1 ½ to +2 in			
	14	** -2 to +1 ½ in	-1 to +2 ½ in		
	15		-1 ½ to +2 in		
	16		** -2 to +1 ½ in	-1 to +2 ½ in	
	17		-1 ½ to +2 in		
	18		** -2 to +1 ½ in	-1 to +2 ½ in	
	19			-1 ½ to +2 in	
	20			** -2 to +1 ½ in	-1 to +2 ½ in
	21				-1 ½ to +2 in

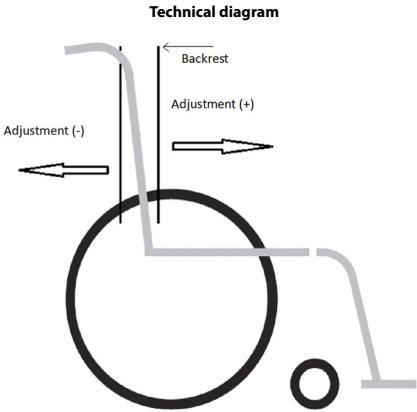ENGLISH

Depth Adjustments

1. Loosen screws **E** on each hardware to let the back slide freely. (It is important to loosen both sides to adjust easily the depth).
2. Slide the back fore and aft until the desired depth has been reached and securely tighten screws **E**.

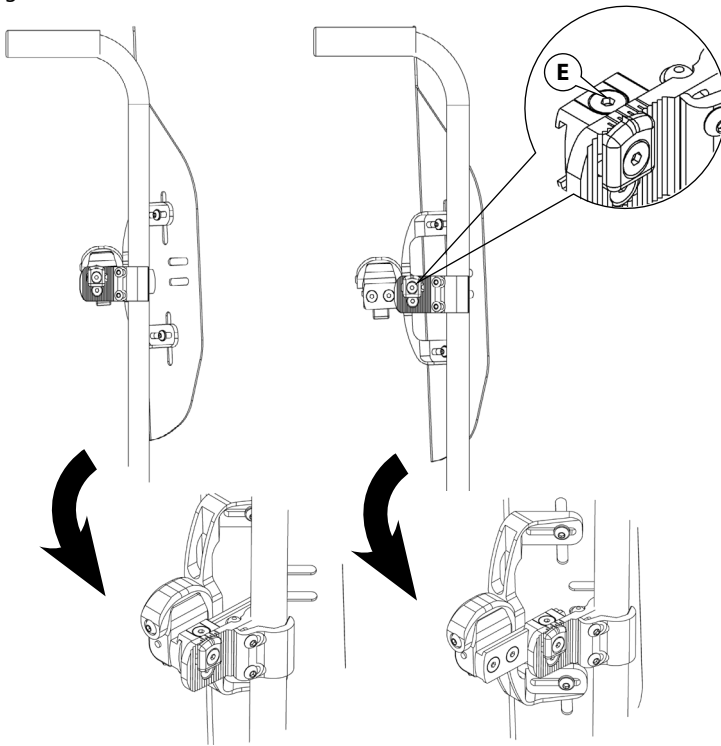It is important to note that if the depth of the HP2 back is not equal to zero, then the seat depth will be modified. If the depth of the back is negative, the seat will seem too far back, and if it is positive, it will seem too far out.

A security mechanism has been installed on the sliding fasteners. It is therefore impossible for the back installed on a wheelchair to come out of its anchors, even in the event of a violent spasm.




6.4 Tilt Adjustment

It is really easy to give a positive or a negative angle to the HP2 back:

1. Loosen the screw **G** on each of the anchors.
2. Tilt the back to the desired angle.
3. Tighten securely the screws **G**.

The back tilts back up to 20° , which represents an angle of 110° between the seat and the back.

The maximum front tilt is -14° , which represents an angle of 76° between the seat and the back.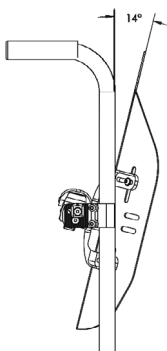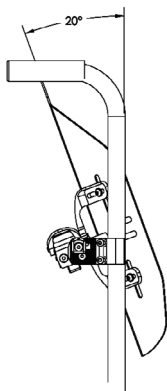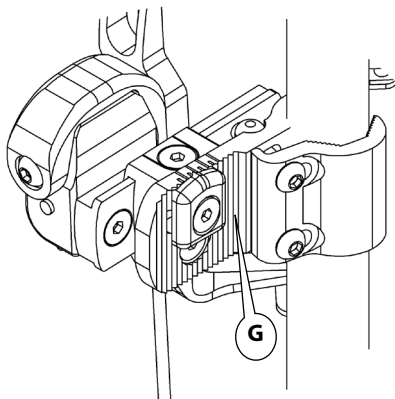


It is important to note that, in most cases, the depth of the seat will necessarily be modified after adjusting the angle of the back. The depth will, in fact, be increase or decrease according to the angle.

ENGLISH


6.5 Extension option adjustment

All adjustments can be made with a 4mm Allen wrench or a 10mm straight wrench.


1. Use the original anchor support system to add the extension plate (left side shown).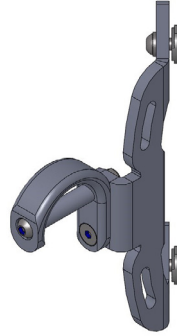


2. Unscrew the 2 bolts to remove the U-bolt.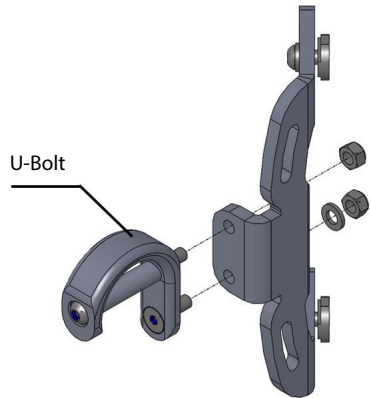

3. Insert the extension plate between the U-bolt and the fastening system.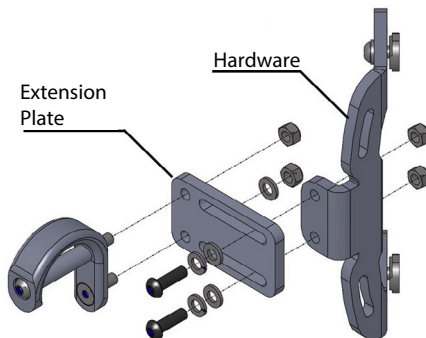

4. Be aware of the plate position and temporarily fasten it on both sides.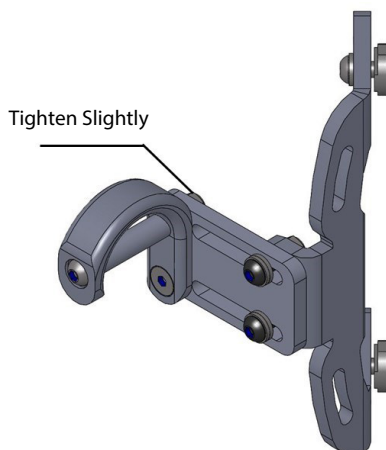


5. Insert the backrest with the anchor support system in the anchor slide (see section 5.3)
6. When everything is in place and well adjusted, tighten all the bolts, but don't overtighten the screw and the bolt holding the U-bolt.

7. Laterals

(optional, except for the HP2 sport back)

7.1 Laterals Installation

1. Slide the bolts **A** in the support as shown here and insert the washers **B** between the bolts and the back.
2. Put the nuts **C** in place.
3. Do not tighten the bolts right away, you need to be able to adjust the height and the depth of the laterals.

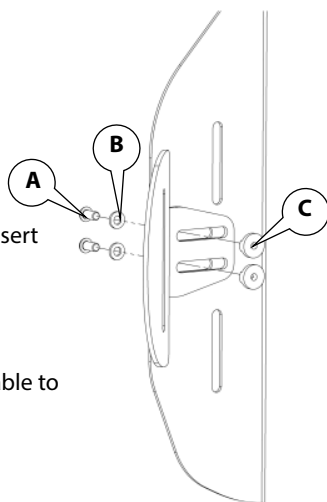ENGLISH

7.1.1 Installing the profiled laterals

Make sure a Velcro strip is fixed on the lateral plate facing the inside. (Figure 6.1)

Insert your hand in the cushion through the cover opening to prevent the Velcro strips from touching each other while you place the cushion. (Figure 6.2)

Start inserting the cushion by the top of the lateral. (Figure 6.3)

Push down while making sure the cover opening passes around the plate by making a downward rotating movement. (Figure 6.4)

Place the cushion, remove your hand from the cover and push a few times on the front of the cushion to make sure the Velcro strips are hooked. (Figure 6.5)

Figure 6.1

Figure 6.2

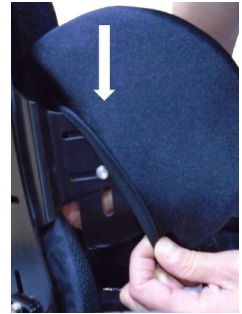
Figure 6.3

Figure 6.4

Figure 6.5

7.1.2 Removing the profiled cushion

Insert your hand inside the profiled cushion between the plate and the cushion to unhook the Velcro strips.

Move the cover opening down the lateral plate and remove the cushion with an upward rotating movement.

7.1.3 Removing the foam from the profiled cushion

Fold the cushion in the cover. (Figure 6.7)

Carefully and slowly take the cushion out of the cover opening. (Figure 6.8)

Completely remove the cushion. (Figure 6.9)

Figure 6.6

Figure 6.7

Figure 6.8

Figure 6.9

ENGLISH

7.2 Height Adjustment of Laterals

It is possible to adjust the height of the laterals according to the size chosen and the height of the HP2 back.

Note that the apex measure will depend on the height to which you installed the HP2 back on the mounts of the back.







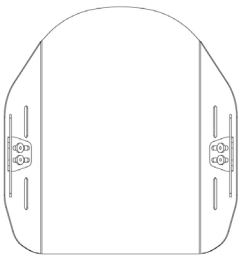
Table 5 - Height of the Lateral Plates
(cushions not included)

Standard Sizes of the Lateral Plates				
Pediatric	Ultra small	Small	Medium	Large
4" (10.1 cm)	6" (15.25 cm)	7" (17.8 cm)	8" (20.3 cm)	10" (25.4 cm)
				
N/A	Adjustable in height by 1" (2.5 cm)	Adjustable in height by 1" (2.5 cm)	Adjustable in height by 1½" (3.8 cm)	Adjustable in height by 3" (7.6 cm)

Follow the steps below to adjust the height:

1. Loosen screws **A** and **B** on the outside of the lateral.
2. Slide up and down the lateral using the vertical groove of the plate to adjust the height.
3. Tighten screws **A** and **B**.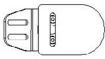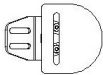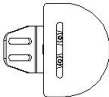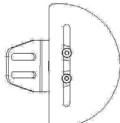
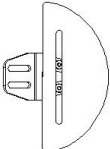
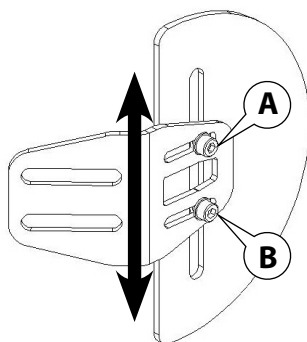


7.3 Depth Adjustment of the Laterals

It is possible to adjust the depth of the laterals. This allows you to keep an effective depth of 7 inches (17.5 cm) all the time. If you need an effective depth superior to 7 inches (17.5 cm), you need to order laterals that are custom-made.

Table 6 – Laterals Depth


Position	Actual Depth
A	7 in (18 cm)
B	6 ½ in à 7 in (16.5 to 18 cm)
C	6 in à 7 in (15 to 18 cm)

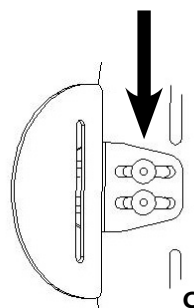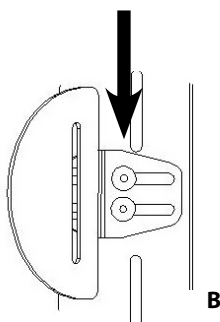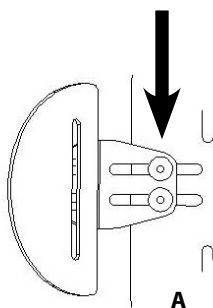ENGLISH



7.4 Additional Depth Adjustments for the Laterals

1. Loosen screws **A** and **B** on the outside of the lateral.
2. Slide the lateral plate using the horizontal groove of the anchor.
3. Tighten screws **A** and **B**.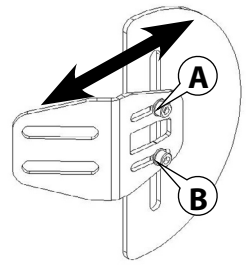

7.5 Angle Adjustment of the Laterals


1. Loosen screws **A** and **B** behind the HP2 back.
2. Pivot the lateral until you reach the desired angle.
3. Tighten screws **A** and **B**.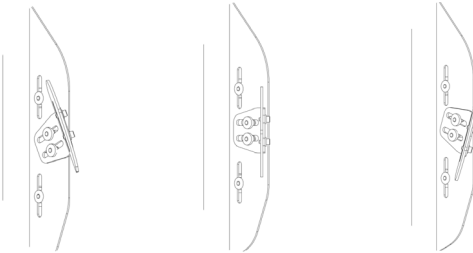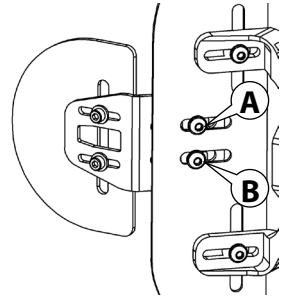

7.6 Optional HP2 back Swing-Away lateral


The swing-away laterals are adjusted exactly the same way.

To push them out, just push the central part **A** of the mechanism upwards and push the lateral out.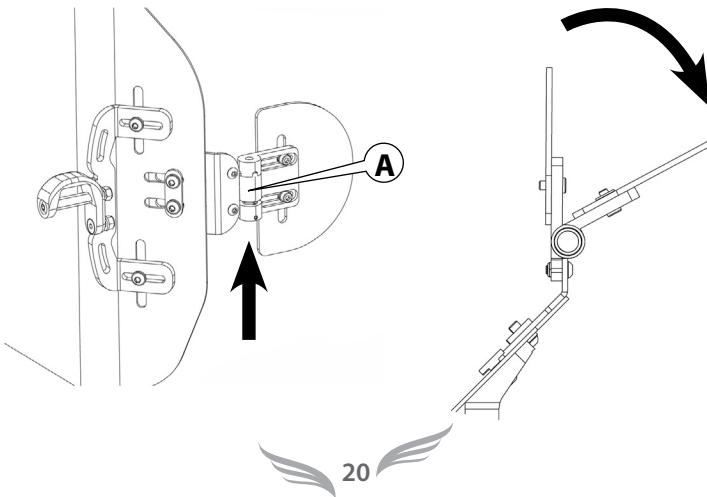

7.7 Contour Laterals Installation Instructions



- 1. Slide the bolts **A** in the support as shown here and insert the washers **B** between the bolts and the back.
- 2. Position the nuts **C** in place.
- 3. Do not tighten the bolts right away, you need to be able to adjust the height and the depth of the laterals.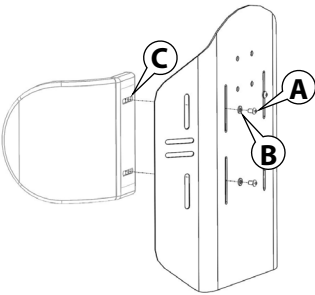

7.8 Adjustment of the Laterals

Follow the steps below to adjust the height:

- 1. Loosen screws **A** on the outside of the laterals.
- 2. Slide up and down the lateral using the groove of the plate.
- 3. Tighten screws **A**.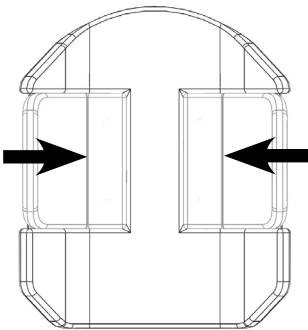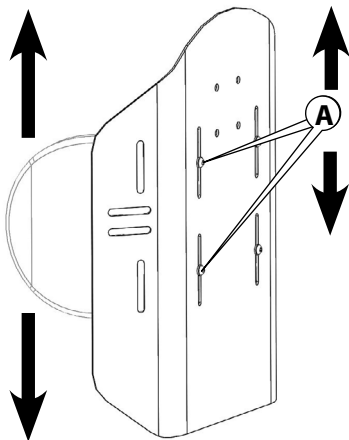

Width Between the Laterals	
Back	Laterals
16"	12" to 13"
18"	14" to 15"
20"	16" to 17"


ENGLISH

8. Removing and inserting the HP2 back

Removing the HP2 back

The HP2 back can be easily removed from the wheelchair if necessary. Move the slide release lever **A** to unlock.

Once both levers are in the forward position, simply lift the back straight up using the handle located on the top of the back.

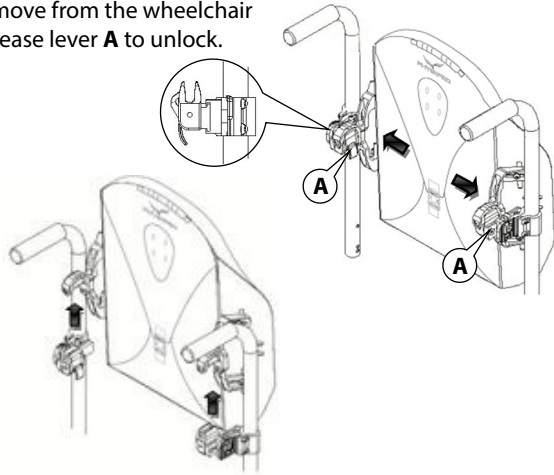Inserting the HP2 back

Hold the back so the pins are lined up to the receivers and push into place. If properly inserted, the pins will “click” to a locked position.

If you are not sure if you have heard the locking “click”, pull up on the back using the handle to ensure that it has been securely locked.

9. Care and Maintenance

The maintenance of the HP2 back is essential, therefore you will extend its lifetime and ensure its safety. Physipro recommends that a clinician such as a doctor or therapist experienced in seating and positioning inspect your back.

Table 7 – Care and maintenance

Checklist	When receiving	Each week	Each month	Every six months
Back				
Observe the condition of the aluminum	x		x	
Observe the condition of the back's folds	x		x	
Anchors				
Make sure the receptacles and the universal clamps of the back are solid;	x	x		
Make sure the anchor slides and latches work well.	x	x		
Laterals (HP2 back)				
Make sure that the laterals are fastened properly;	x	x		
Observe the condition of the aluminum.	x			x

Cleaning**To clean the back cover**

- ✓ Locate the zipper on the top of the cover and unzip
- ✓ Remove the cover from back shell
- ✓ Remove foam and turn cover inside out
- ✓ Machine wash in warm water (50° C)
- ✓ Air dry

machine wash
maximum temperature 50C

do not use chlorine-based bleach

do not iron

do not dry clean

To clean the back shell

- ✓ The aluminum shell can be cleaned with soap and warm water.
No other maintenance should be required.

Note – Never store your back support in conditions of extreme heat or exposure to direct sunlight for long periods of time. Never store heavy or sharp objects on top of your back cushion.

To clean the positioning components

- ✓ Positioning components can be wiped clean with soap and water.
Do not immerse positioning components under water.

10. Warranty

The HP2 back, a product of Physipro inc., is covered with our twenty-four (24) months manufacturer's warranty. It includes parts and labor on manufacturer's defects and optional parts, and all new parts replaced by your dealer or an authorized establishment from the date of delivery or replacement. Physipro inc. will repair or replace the defective parts as long as the warranty is valid. You will have to contact Physipro inc. or an authorized dealer to obtain your warranty services. Do not return your back without prior consent. If you are not satisfied with the repairs, please send us a written complaint with your name, address and delivery date to the address on this manual cover.

Exclusions and limitations

This warranty does not apply to products that were intentionally or accidentally altered, misused, neglected or did not have the proper maintenance. Furthermore, this warranty does not apply if repairs or modifications were done without a written consent from Physipro inc., or an authorized dealer. The same restrictions apply if any other kind of damages were done without Physipro inc. involvement. This warranty will not apply for everyday use or failure to respect the rules in this manual. Physipro inc. will not take any responsibilities for damages incurred during shipping.

Terms and Conditions

The seat cushions and backs are reserved exclusively for use on mobility aid (manual or power wheelchairs) Physipro will not be held responsible for any misuse of the product. Crash tests have not been conducted on these products and even if the mobility aid in which it is installed has obtained the ISO-717619, it will not be maintained. The respect of the safety instructions mentioned in this manual is essential to avoid injury. Improper or inadequate adjustment increases the risk of accident. Before making adjustments, it is important to read the entire contents of this manual. Physipro will not be held responsible for misuse or non professional adjustment. The product presented in this manual may not be an exact representation of the product you have on hand. The instructions in this manual are still valid for your product. Physipro reserves the right to modify or change this manual without notice.

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