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CONTENTS

	Page
1. Checking on delivery	15
2. Description of the Kelvin^{Comfort}	15
3. Maintaining your Kelvin^{Comfort}	23
4. Guarantee	24
Checklist	25

1. CHECKING ON DELIVERY

Check whether the modular seat element delivered corresponds to the packing note. If your modular seat element is incomplete or damaged on delivery, please contact your dealer. Your dealer must notify the manufacturer or supplier of the incomplete delivery or damage within two weeks.

2. DESCRIPTION OF THE KELVIN^{Comfort}

The Kelvin^{Comfort} seat element is a patented product and complies with the Essential Requirements for Medical Devices Directive 93/42/EEC. The Kelvin^{Comfort} seat element has a CE mark.

2.1 FEATURES

The Kelvin^{Comfort} is a modular seat element with a dynamic design. Its stable frame has a completely freely suspended cover which adapts to the basic contours of the body. The cover is made up of unzippable cushions with a granule and latex filling. Because the granules can move freely and the latex is soft and springy, the seat is dynamic whilst always enabling the user to sit comfortably even with many changes of posture.

2.2 TARGET GROUP

The target group for the Kelvin^{Comfort} has grown dramatically over the years. The Kelvin^{Comfort} is ideal for passive wheelchair users, for whom posture adjustments are not a top priority. This target group benefits from a comfortable, secure feeling without compromising on mobility, so that interaction with the environment is ensured.

The Kelvin^{Comfort} is a modular seat element which enables the problems associated with sitting to be approached individually. Characteristic of this target group are the functional restrictions and curvature resulting from different neurological and other conditions. By adjusting the shape and amount of filling in the cushions, an individual shape can be created which offers effective support. The dynamic character gives the user controlled freedom of movement. The Kelvin^{Comfort} offers this target group maximum support while allowing the user to sit comfortably and in a relaxed position.

2.3 DIMENSIONS


	Seath with	Seat Depth	Backrest height
No 1	34 cm	48 cm	55 cm
No 2	40 cm	48 cm	55 cm
No 3	46 cm	48 cm	55 cm

BECAUSE OF THE FLEXIBLE NATURE OF THE KELVIN^{COMFORT}, ALL DIMENSIONS ARE APPROXIMATE.


2.4 CUSHIONS

The Kelvin^{Comfort} is made up of loose cushions. All cushions except the seat cushion are filled with a combination of granules (2.5 mm) and latex strings. Each cushion has a different number which corresponds to the number on the cover of the seat system. The cushion arrangements and options can be seen below.


Backrest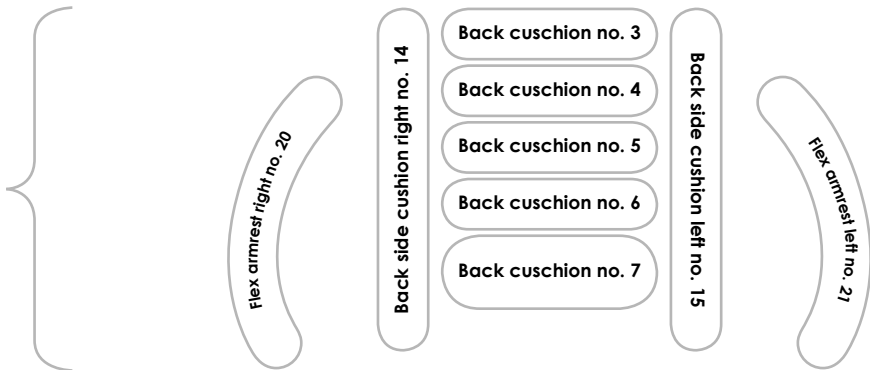

Seat

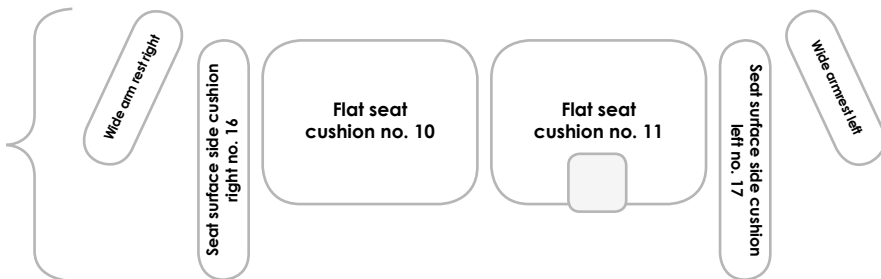Leg support

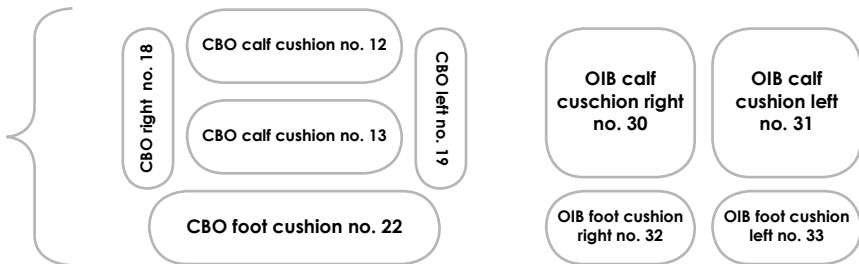Seat cushion

The Kelvin^{Comfort} seat cushion is made up of two components: a flexible, soft layer of latex strings and a durable, pressure-distributing foam part. This combination ensures optimum sitting comfort. The seat cushion is fitted with an incontinence cover made of waterproof material.

■ **Flat seat cushion (no.10)**

This is the standard seat cushion. The seat cushion is secured with a zip along the back edge and two Velcro strips underneath.

■ **Seat cushion with abduction pad (no.11)**

If required, a seat cushion with an abduction pad is also available. The abduction pad is positioned between the knees and prevents them from touching. The seat cushion is attached in the same way as the standard seat cushion.

2.5 LEG SUPPORT

The Kelvin^{Comfort} can be fitted with different types of leg support. There is a choice of four different types which can be easily interchanged.

■ **Classic leg support (CBO)**

This is the standard leg support which is fitted with granule cushions. The knee angle and the ankle angle can be adjusted by means of toothed hinges. The length of the leg support can be adjusted to the user's calf length.

■ Separately adjustable leg support (OIB)

This is made up of two separately adjustable leg supports for the left and right legs. The knee angle, leg angle and calf length on each leg support can be adjusted separately.

THE CLASSIC LEG SUPPORT AND THE SEPARATELY ADJUSTABLE LEG SUPPORT ARE BOTH ATTACHED TO THE FRAME IN THE SAME WAY AND ARE THEREFORE EASILY INTERCHANGEABLE.

■ Comfort legrest

■ Profix legrest

TO FIT THE COMFORT LEGREST OR THE PROFIX LEGREST YOU WILL NEED AN ADAPTER WHICH IS EASY TO ATTACH TO THE FRAME.

2.6 ARMREST

■ Flex arm support

The Flex arm support is secured to the seat element with a zip at shoulder height. The Flex arm support has a granule and latex filling.

■ Wide armrests

The wide armrests are fitted with insertion tubes which are inserted into the tubes on the seat element.

2.7 HEADRESTS

Depending on the position of the head, you can choose between four different types of headrests. All cushions on Kelvin head supports have a granule and latex filling.

■ Kelvin headrest, small

This is a flexible headrest with a cushion.

■ Kelvin headrest, small, with Edison lever

This is a flexible headrest which can be very easily fixed in many different positions. It has a granule cushion. It is used for users who sit off-balance, i.e. whose head is not in a central position or needs more support from the side.

■ Kelvin headrest, large

The width of the headrest matches the width of the seat element. It has a granule cushion. Designed for users who sit 'actively' and whose head needs support in several different positions.

■ Kelvin head cushion

A cushion to support the head which is zipped on to the top of the seat element. For use by smaller users whose head does not project above the backrest.

THERE IS A PLASTIC HEADREST ATTACHMENT ON THE SEAT ELEMENT FOR FITTING THE HEADREST.

2.8 COVERS

There are different covers available for the Kelvin^{Comfort} seat element.

- Terry cover with openings and incontinence material for the seat element. The terry cover is made of 80% cotton for added absorbency.
- Comfort cover with openings and incontinence material for the seat element. The Comfort cover is made of Dacron with a terry top layer. The Dacron optimises contact with the body, giving the user an extremely comfortable feeling. The terry top layer is made of 80% cotton for added absorbency.
- Terry cover for the Flex arm support and headrests.

Attaching the covers to the seat element

There are two tapes on the inside of the cover which are threaded through the seat to the outside in the two corners of the seat. There are also two stretch tapes on the edge of the cover along the seat surface which go round the outside instead of through the seat. All tapes must be tied to the tiltable top part of the frame.

Attaching the covers to the Flex arm support

Once the terry covers have been fitted over the Flex arm support, you can attach them to the back tube to which the push handles are attached.

2.9 ADJUSTMENT OPTIONS

■ Backrest adjustment

The KelvinComfort has an adjustable backrest. To adjust the angle of the backrest, unzip the cover at the side and unscrew the cross-shaped block for fastening the back of the Kelvin^{Comfort} and the backrest angle adjustment (and the seat depth adjustment if applicable) on the frame.

Use a size 8 Allen key to loosen and tighten the toothed hinge.

N.B.

- ONLY ADJUST THE BACKREST WHEN THERE IS NO-ONE SITTING IN THE CHAIR.
- MAKE SURE YOU DO NOT COMPLETELY REMOVE THE BOLT IN THE TOOTHED HINGE.
- WHEN ADJUSTING THE BACKREST ANGLE, MAKE SURE THAT THERE IS A GAP OF AT LEAST 10 CM BETWEEN THE KELVIN SEAT ELEMENT AND THE CANTO SUBFRAME.

3. MAINTAINING THE KelvinCOMFORT

3.1 CLEANING THE CUSHIONS

The cushions are easy to clean with a damp cloth and an all-purpose cleaner that does not contain any aggressive substances and that is suitable for textiles and plastics.

3.2 WASHING THE COVERS

- The terry cover and the Comfort cover for the seat element can be washed at up to 60°.
- The terry covers for the headrest and the Flex arm support can be washed at up to 60°.
- The Flex arm support can be washed at up to 60°.

Tip: Use liquid laundry detergent.

N.B.

- THE FLEX ARM SUPPORTS MUST BE COMPLETELY EMPTIED.
- READ THE LAUNDRY LABEL ON THE COVERS FOR CARE INSTRUCTIONS.

3.3 CUSHIONS

- All cushions except the seat cushion are filled with a combination of granules (2.5 mm) and latex strings.
- Filling level: The cushions are filled to a specific standard. No liability can be accepted if their contents are changed by third parties without consulting the supplier or manufacturer.
- Zips: The zips on the cushions must be completely closed.

4. GUARANTEE

The manufacturer guarantees the frame of the Kelvin^{Comfort} against frame breakage for a period of one year. The manufacturer also guarantees the quality of the sewn seams on the outer cover and the granule cushions for one year.

The guarantee covers manufacturing defects only and excludes: abnormal or improper use, negligence, overloading or accidents. This guarantee supersedes any other guarantee laid down in law or verbally notified, with the exception of any written guarantee provided by the manufacturer.

Guarantees are only valid within Europe.

Defects must be reported to the manufacturer or supplier within 4 weeks.

Please contact your dealer if components need replacing.

Checklist

✓ SITTING POSITION

Seat the user as far back in the seat element as possible so as to prevent them from slumping back in the chair.

✓ BACKREST ANGLE

The wrong backrest angle can affect the sliding force which can result in an increased risk of decubitus. N.B.: Always adjust the backrest angle when there is no-one sitting in the chair. There must be a gap of at least 10 cm between the Kelvin^{Comfort} seat element and the frame.

✓ TILTING

Altering the seat angle regularly reduces the risk of decubitus. The seat angle can be altered using the tilt adjustment on the frame.

✓ TRANSFER

Transfer the user in and out of the chair with a passive lift; position the user on the tilted seat element from the side.

✓ LIFT MAT

Make sure there are no folds, edges or fastening clips of the lift mat between the user and the seat element as they can increase the risk of decubitus. Remove the lift mat after the transfer if possible.

✓ COVERS

The covers must be positioned in such a way that the laundry label is on the top. Secure the tapes to the tiltable part of the frame. When fitting the covers, make sure they are as smooth as possible with no folds.

✓ CLEANING

The cushions are easy to clean with a damp cloth and an all-purpose cleaner that does not contain any aggressive substances and that is suitable for textiles and plastics.

✓ WASHING INSTRUCTIONS

The Comfort cover, the terry covers and the Flex arm support can be washed at up to 60°. N.B.: The Flex arm supports must be completely emptied before washing. Tip: Use liquid laundry detergent.

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