# Aquatec® Dot

EN	<b>Shower stool</b> User Manual . . . . .	1
DE	<b>Duschhocker</b> Gebrauchsanweisung . . . . .	4
FR	<b>Tabouret de douche</b> Manuel d'utilisation . . . . .	5
ES	<b>Taburete de ducha</b> Manual del usuario . . . . .	7
IT	<b>Sgabello doccia</b> Manuale d'uso . . . . .	8
PT	<b>Banco de duche</b> Manual de utilização . . . . .	10
NL	<b>Douchekruk</b> Gebruiksaanwijzing . . . . .	11
SV	<b>Duschstol</b> Bruksanvisning . . . . .	13
NO	<b>Dusjkrakk</b> Bruksanvisning . . . . .	14
DA	<b>Badestol</b> Brugsanvisning . . . . .	16
FI	<b>Suihkujakkara</b> Käyttöohje . . . . .	17

EN

## I General

### I.1 Images

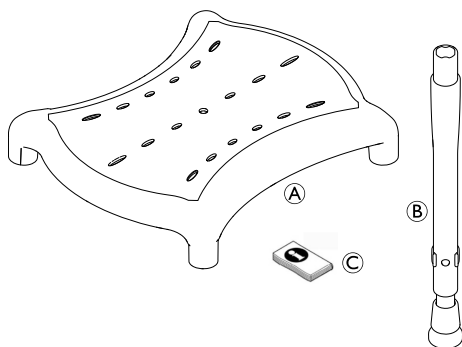
Fig. 1

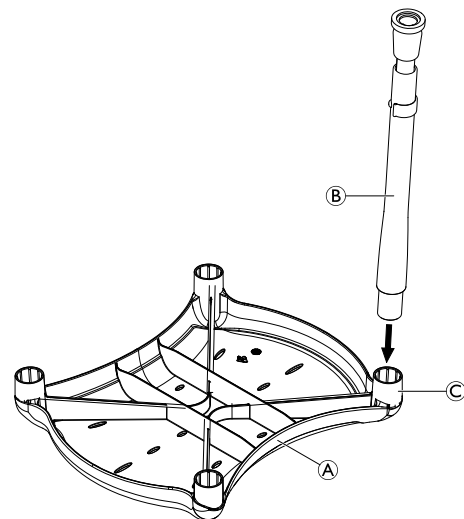
Fig. 2

This manual MUST be given to the user of the product.  
BEFORE using this product, read this manual and save for future reference.

**AQUATEC®**

Yes, you can.®

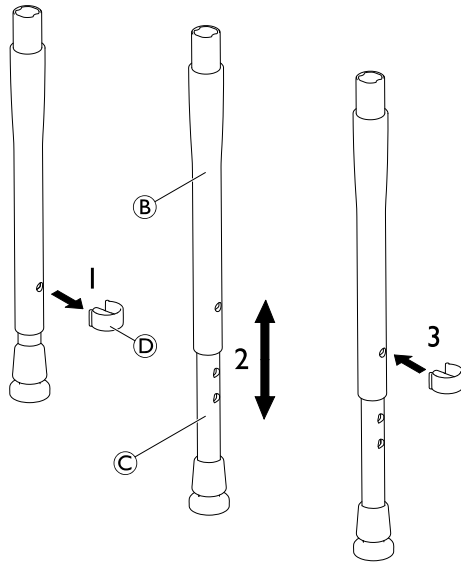
Fig. 3

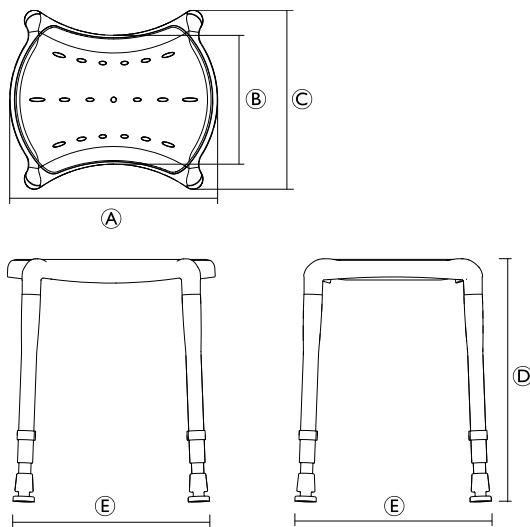
Fig. 5

### 1.2 Symbols in this user manual

In this user manual, warnings are marked with symbols. The title next to the warning symbol indicates the level of danger.

**WARNING**  
Indicates a dangerous situation which could lead to death or serious injury if not avoided.

**CAUTION**  
Indicates a dangerous situation which could lead to minor injuries if not avoided.

**IMPORTANT**  
Indicates a dangerous situation which could lead to damage if not avoided.

Useful tips, recommendations and information to ensure efficient and smooth operation.

This product complies with the directive 93/42/EEC for medical products. The launch date for this product is specified in the CE declaration of conformity.

Manufacturer

### 1.3 Intended use

The shower stool is to be used solely as an aid for sitting in the bathroom or in the shower for people with sufficient torso stability. Any other use is prohibited.

Operating temperature: 10 – 40 °C

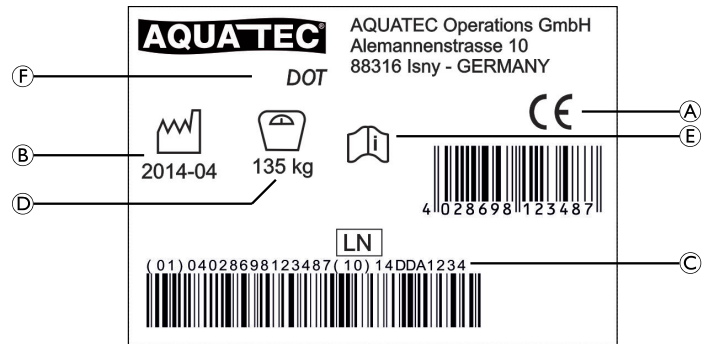
Fig. 4

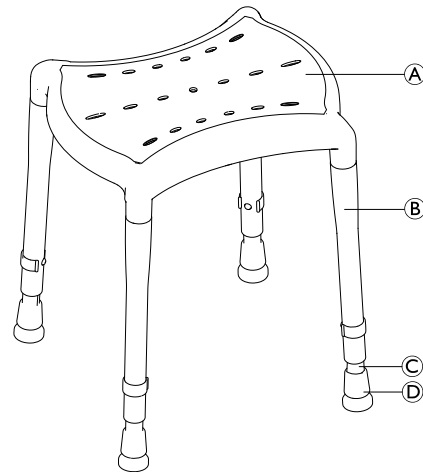
Fig. 6

### 1.4 Scope of delivery

The following components are included in the scope of delivery (Fig. 1):

(A)	Seat plate
(B)	Height adjustable legs (4x) composed of two into each other moveable tubes, clips and rubber tips
(C)	User manual

### 1.5 Warranty

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries. Warranty claims can only be made through the provider from whom the appliance was obtained.

### Quality

Quality is of the utmost importance for our company; all processes are based on the ISO 9001 and ISO 13485 standards.

This product features the CE mark, in compliance with the Medical Device Directive 93/42/EEC Class I.

The product meets the requirements of the standards DIN EN 12182 (Technical aids for disabled persons - general requirements and test methods), DIN EN ISO 14971 (Medical devices - Application of risk management to medical devices) and DIN EN ISO 10993-1 (Biological evaluation of medical devices - Part 1: Evaluation and testing within a risk management system).

Invacare® is continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum. It is Invacare's goal to ensure that we, at the widest possible range, use REACH compliant materials and components.

For further information please contact Invacare® in your country (for addresses, see back page of this manual).

## 1.6 Service life

The expected service life of this product is three years when used in accordance with safety instructions and intended use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

## 2 Safety

### 2.1 Safety information

**WARNING!**  
**Risk of falling!**

- Do not stand on this product.
- Do not use as a climbing aid.
- Do not use for transfer from the side or partial load.
- Do not sit on the product if not all four legs have full contact to the ground.

**WARNING!**  
**Risk of injury!**

- Do not use if defective.
- Do not make any unauthorized alterations or modifications to the product.

### 2.2 Labels and symbols on the product (Fig. 4)

Ⓐ	Conformity mark
Ⓑ	Product time frame
Ⓒ	Serial number
Ⓓ	Load capacity
Ⓔ	Note (observe accompanying documents)
Ⓕ	Product designation

The identification label is attached to the underside of the seat plate.

## 3 Setup

### 3.1 Assembling the product (Fig. 2)

1. Place the seat plate Ⓐ with the seat surface down on a flat table.
2. Attach the leg Ⓑ into the seat plate by pressing it into the support Ⓒ.
3. Repeat the same procedure for the other three legs.

**WARNING!**  
**Risk of injury**

- Make sure that all four legs are completely inserted in the supports.

### 3.2 Adjusting the seat height (Fig. 3)

1. Remove the safety clip Ⓓ from the leg.
2. Adjust the desired height by moving the adjustment tube Ⓒ into or out of leg Ⓑ.
3. Re-attach the safety clips through the desired holes of the leg and the adjustment tube.
4. Repeat the same procedure for the other three legs.

**WARNING!**  
**Risk of falling!**

- Make sure that all legs are adjusted to the same height.
- Make sure that the safety clips are fully engaged in the holes of all legs.

## 4 Maintenance

### 4.1 Maintenance and periodic inspection

The product is maintenance-free, provided the cleaning and safety instructions outlined in this user manual are adhered to.

**WARNING!**

- Before first use, clean the product thoroughly.
- Before every use, check the product for damage and ensure that it is securely assembled.

### 4.2 Cleaning and disinfection

**IMPORTANT!**

- All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean. For further information on decontamination in Health Care Environments, please refer to 'The National Institute for Clinical Excellence' guidelines on Infection Control [www.nice.org.uk/CG139](http://www.nice.org.uk/CG139) and your local infection control policy.
- Clean by hand.
  - Do not use any aggressive (pH value under 4.5 or over 8.5) or abrasive cleaning agents.
  - Max. cleaning temperature 60 °C.

The product can be cleaned and disinfected using commercially available agents.

1. Disinfect the product by wiping down all generally accessible surfaces with disinfectant (after disassembling the product, if possible).

### 4.3 Re-use

The product is suitable for repeated use. The number of times it can be used depends on how often and in which way the product is used. Before reuse, clean and disinfect the product thoroughly (® 4.2 Cleaning and disinfection, page 3), check the safety functions and reset if necessary.

### 4.4 Disposal

The disposal and recycling of used devices and packaging must comply with the applicable local legal regulations.

## 5 Technical data

### 5.1 Dimensions and weight (Fig. 5)

Ⓐ	343 mm
Ⓑ	213 mm
Ⓒ	295 mm
Ⓓ	400 – 550 mm, in steps of 25 mm
Ⓔ	365 mm, in highest position
Product weight	1.65 kg
Max. user weight	135 kg

### 5.2 Materials (Fig. 6)

Ⓐ	PP LGF
Ⓑ	PA6 GF

Ⓒ	PA6 GF
Ⓓ	TPE-O

All product components are corrosion free and latex free.

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Het product is geschikt voor herhaald gebruik. Hoe vaak het product kan worden hergebruikt, hangt af van de frequentie en gebruikwijze. Vóór hergebruik moet het product grondig worden gereinigd en gedesinfecteerd (© 4.2 Reiniging en desinfectie, pagina 13). Controleer de veiligheidsfuncties en reset indien nodig.
Invacare AB
Algol-Trehab Oy
Aquatec Operations GmbH