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millflash (T)

DATA RECORDING DEVICE OPERATOR'S MANUAL

ENGLISH

REVISIONS

The following table lists subsequent editions/revisions of the manual. The "Description" field brief explains the subject of the latest revision.

Edition	Rev.	Date	Description
1	0	06-2006	First issue

TABLE OF CONTENTS

- INTRODUCTION** 1
 - USAGE 1
 - GENERAL WARNINGS 1
 - MANUAL PURPOSE 1
- CONTENTS OF THE PACKAGE** 2
 - DIMENSIONS AND WEIGHT 2
 - DESCRIPTION OF CONTENTS 2
- INFORMATION ABOUT THE PRODUCT** 3
 - TECHNICAL FEATURES 3
- INSTALLATION** 4
 - CONNECTING THE MILLFLASH TO MILLENNIUM 4
 - PROPER POSITIONING OF MILLFLASH 6
 - PROPER POSITIONING OF MILLFLASH 6
 - INSTALLING THE SW ON PC 6
- USE OF THE MILLFLASH** 7
 - ON "START CYCLE" COMMAND 7
 - IN ORDER NOT TO LOSE DATA 7
 - FILE STORAGE 7
 - CONNECTING THE MILLFLASH TO PC 8
 - MANAGING THE FILES BY MILLFLASH SW 9
 - Launching the program 9
 - Dialogue with the device 9
 - Saving the Report file 10
 - Report file management 10
 - File name 11
 - Files visualization 11
- MAINTENANCE** 12
 - DIAGNOSTICS 12
 - EXTERNAL CLEANING 12
- CUSTOMER SUPPORT** 12

INTRODUCTION

Dear Customer

Thank you for choosing a product from M.O.COM. Srl. We hope that you will find it completely satisfactory.

This manual describes all procedures for the correct installation and use of the device.

Should you have any questions or suggestions concerning this product or our support, do not hesitate to contact us.

Note and warning symbols used through the manual

NOTE	
	PAY SPECIAL ATTENTION; NOTES FOR PROPER INSTALLATION AND MAINTENANCE PROCEDURES AND USE METHODS.

WARNING	
	THIS SYMBOL INDICATES A POTENTIAL DANGER OF PROPERTY DAMAGE. FOLLOWS THE INSTRUCTIONS IN THE MANUAL TO PREVENT POTENTIAL DAMAGE TO MATERIALS, EQUIPMENT OR OTHER PROPERTY.

USAGE

millflash is designed for recording the sterilization and test cycle data performed on steam sterilizers of the **Millennium** series. The recorded data can be downloaded and managed on a PC.

GENERAL WARNINGS

The data recording device must be used according to the procedures detailed in this manual

- **Do not** place the units on warm or hot surfaces.
- **Do not** pour water or other liquids

NOTE	
	THE INFORMATION IN THIS MANUAL IS SUBJECT TO CHANGE WITHOUT NOTICE.
	M.O.COM. SRL SHALL NOT BE RESPONSIBLE FOR DIRECT, INDIRECT, ACCIDENTAL OR CONSEQUENTIAL DAMAGE OR OTHER DAMAGES RELATED TO THE SUPPLY/USE OF THIS INFORMATION.
	THIS DOCUMENT MAY NOT BE REPRODUCED, ADAPTED OR TRANSLATED, IN WHOLE OR IN PART, WITHOUT THE PRIOR, WRITTEN AUTHORIZATION OF M.O.COM. SRL
	 IS REGISTERED TRADEMARKS OF M.O.COM. SRL

MANUAL PURPOSE

The purpose of this manual is to provide instructions for:



- becoming generally familiar with the product;
- its correct installation;
- its safe and efficient use.

CONTENTS OF THE PACKAGE

DIMENSIONS AND WEIGHT


Package:

Total weight about 0,65 Kg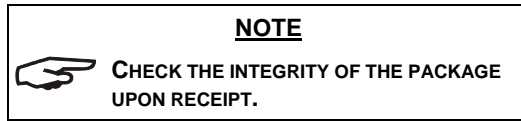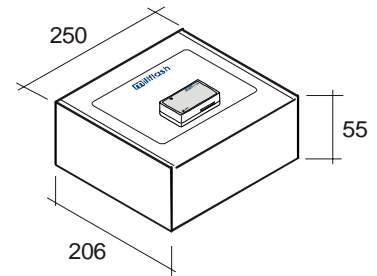


Once the package is opened, check that:


- the supply matches the specifications of the order (see the accompanying document);
- that there is no obvious product damage.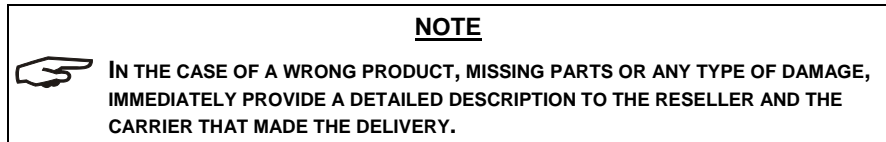

DESCRIPTION OF CONTENTS

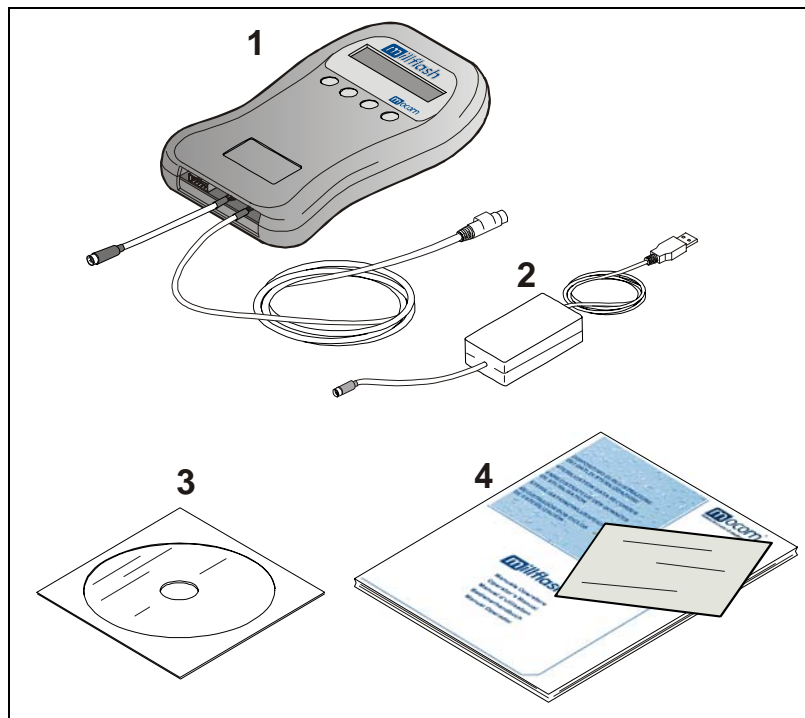- * Recording device and interface cables (Ref. 1)
- * USB interface cable for Millflash-PC connection (Ref. 2)
- * CD with SW installation on PC (Ref. 3)
- * Operator's Manual and warranty certificate (Ref. 4)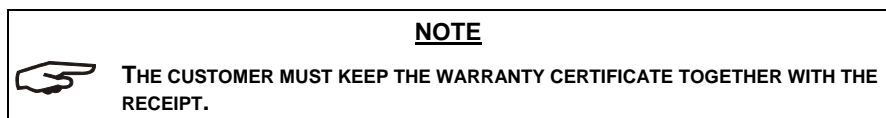


INFORMATION ABOUT THE PRODUCT

millflash is a specially designed system to interface to the sterilizers of the **Millennium** series and automatically record on its internal memory the printout file generated by the sterilizer and containing the data of the performed cycle.

Later, connecting the Millflash to a USB port of a standard PC, it will be possible to download and manage every stored file through the software provided with the unit.

Thanks to its high memory capacity, simplicity and efficiency of use, **millflash** represents a valid solution for the management of sterilization cycle reports.

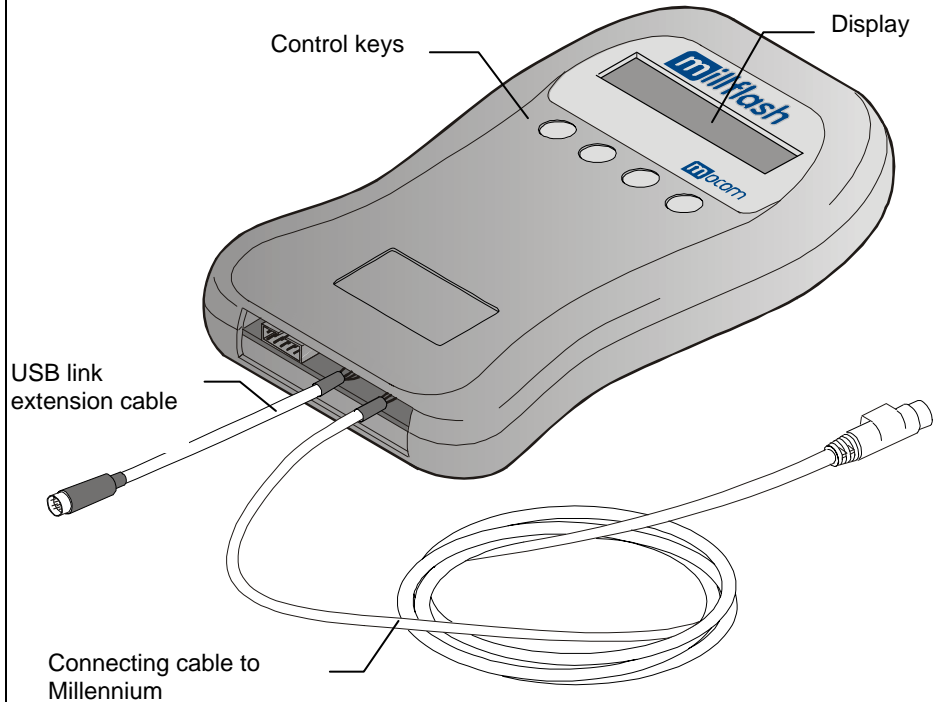ENGLISH

TECHNICAL FEATURES

Data recording unit

Operating temperature	+0°C ÷ +40°C <i>(without direct sun light, or proximity of other heat sources or excessive humidity)</i>
Storage temperature	0°C ÷ +40°C <i>(protected from humidity or other heat sources)</i>
Size <i>(without cables)</i>	108 x 165 x 20 mm (WxDxH)
Weight <i>(without cables)</i>	155 grams
Usage	Internal
Memory capacity	Equivalent to 100 sterilization cycles (average value)

Minimal requirements for PC hardware

1 free USB1.1 port (or higher)
CD-ROM reader
Operating system: Windows 98SE, XP or higher

Compatibility with sterilizers

Millflash is fully compatible with the new series of sterilizers **Millennium** equipped with PS/2 connector (6-pin) providing the power supply and interface of the recording device.

INSTALLATION

The installation consists in the following operations:

- Connecting the Millflash to the sterilizer;
- Proper positioning of the Millflash
- Installing the management SW on PC

CONNECTING THE MILLFLASH TO MILLENNIUM

WARNING


SWITCH OFF THE STERILIZER.

1. Open the service door and connect the PS2 cable of Millflash to the sterilizer's PS2 connector;
2. Switch on the sterilizer.

NOTE

NO EXTRA PARAMETER MUST BE CONFIGURED ON THE STERILIZER. THE RECORDING DEVICE IS FULLY COMPATIBLE WITH ANY PRINTING MODE SET ON THE STERILIZER.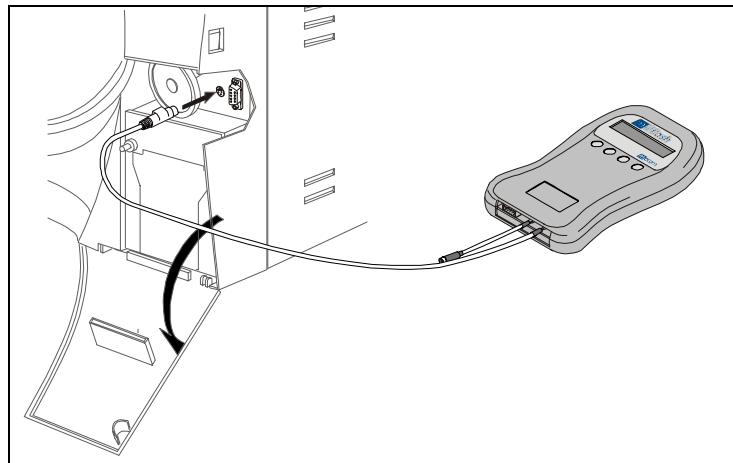

During the initial test, the LCD display of Millflash should show:

D a t a L o g g e r			
E s c			*

flashing (stand-by status)

Warning: If on first connection the Millflash display shows:

P r o g r a m m e r		O N	
M e n u		I n f o	

Press **Menu**

D a t a L o g g e r			
E n t		E s c	


Press **Ent**

D a t a L o g g e r			
E s c		D W N L d	


Press **DWNLd**

Millflash switches to **Stand-by** status.


After connecting Millflash to the unit, the size of available memory and the number of stored files can be checked in the following way: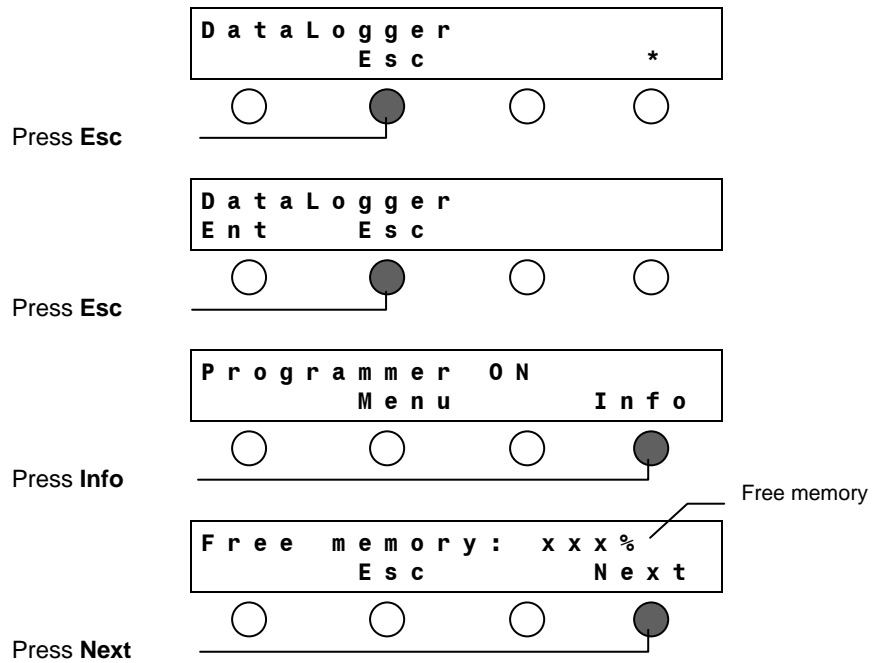


The following screens appear, alternating with one another: _____ Number of stored files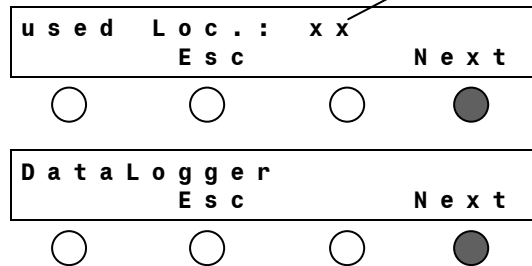

To return to stand-by, press **Esc** to go back to the previous screens: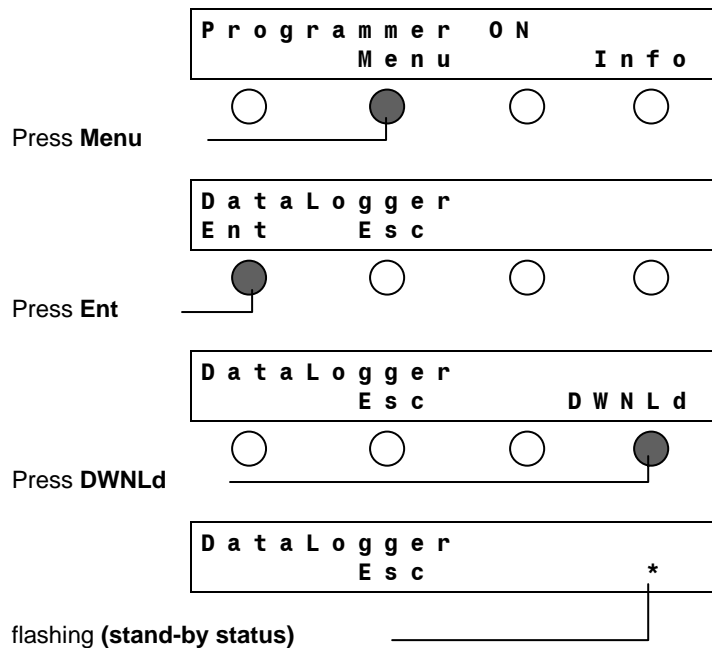

PROPER POSITIONING OF MILLFLASH

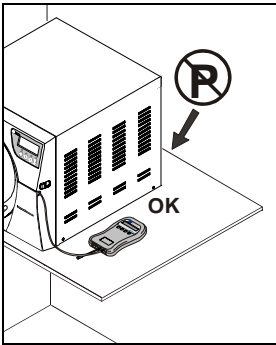The recording device can be installed and works correctly with any orientation.

NOTE

HOWEVER, IT IS SUGGESTED NOT PLACING THE RECORDING DEVICE WITH ITS SLOT UPWARDS, TO AVOID THAT EXTRANEIOUS MATTER COULD FALL INSIDE.

INSTALL MILLFLASH SO AS TO LEAVE THE DISPLAY CLEARLY VISIBLE IN ORDER TO BE ABLE TO IDENTIFY A CONDITION OF ANOMALY IMMEDIATELY.

ATTENTION

DO NOT POSITION THE RECORDING DEVICE NEAR HEAT SOURCES (NOT BEHIND THE STERILIZER).

INSTALLING THE SW ON PC

The stored data can be copied, read and printed through the Millflash software installed on an IBM-compatible personal computer equipped with an USB port.

Installation of the Millflash SW included in the supplied CD:

- Insert the CD in the CDROM drive, copy the file "**Setup_Millflash revision].exe**" on the PC and launch it.
- Follow the instructions for the installation procedure that appear on the screen. During installation the directory "Millflash" will be created, containing any required files.
- Moreover, the program icon is created on the PC desktop.

USE OF THE MILLFLASH

During the sterilizer's normal cycle, the recording unit does not require any actions by the user.


WARNING

THE STERILIZER DOES NOT DETECT THE PRESENCE OF MILLFLASH. THE OPERATOR HAS TO CHECK THE CORRECT CONNECTION TO THE UNIT.

The recording device operation is independent of the presence or not of the internal printer of the sterilizer.

ON "START CYCLE" COMMAND

If the Millflash memory is full or there is not enough free space to store a new cycle's data file, at the **Start** command the Millennium display will show the following message: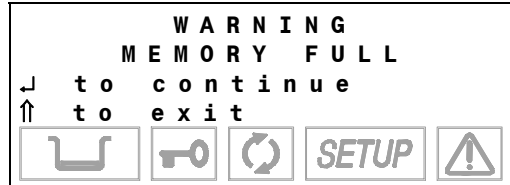

In this case press key ↑ on the sterilizer's command panel to cancel the command, then download the memory to a PC, connect Millflash back to the unit and repeat the **Start** command.

Press key ↓ if you want to continue.

WARNING

IF A NEW CYCLE IS STARTED WITHOUT STORING THE DATA FILE OF THE PERFORMED CYCLE, THIS WILL BE IRRETRIEVABLY LOST.

IN ORDER NOT TO LOSE DATA

NOTE


IT IS RECOMMENDED TO SAVE THE DATA IN MEMORY PERIODICALLY AND TO CHECK ITS CORRECT CONTENT ON THE PC, THROUGH THE MANAGEMENT SW .

If at the end of a sterilization cycle the Millflash is disconnected, the printout file will not be stored and if the operator starts a new cycle, **the file will be lost and will not be recoverable.**

If the operator notes the problem **before** starting a new cycle, he/she only has to connect Millflash to the equipment and the file will be stored automatically.

FILE STORAGE

At the end of the sterilization cycle and after the release of the door blocking mechanism, the sterilizer sends Millflash the cycle data file (identical to the one printed on paper if present). The recording phase of the data file is shown on the Millflash display:

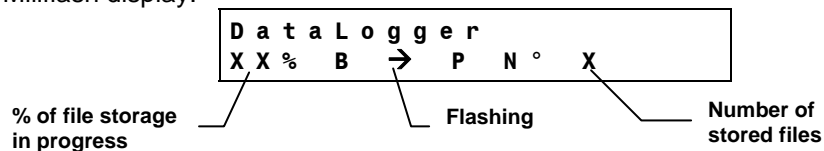WARNING

DURING THE MEMORIZATION PHASE DO NOT SWITCH OFF THE UNIT AND DO NOT DISCONNECT THE CABLE.

Millflash always reads a single file even when the option "Printout copies" on the sterilizer is configured for a higher value.

After being read on the unit, the cycle data file is deleted and is no longer available on the equipment.

CONNECTING THE MILLFLASH TO PC

1. Disconnect the connecting cable of Millflash from the sterilizer;


WARNING

 **TO AVOID POSSIBLE DAMAGE TO THE MEMORY OR DATA LOSS, BEFORE DISCONNECTING THE CABLE WAIT UNTIL ALL OPERATIONS IN PROGRESS ARE FINISHED.**

D a t a L o g g e r
X X % B → P N ° X

Wait until it is fixed (flashing during recording)

2. Connect the supplied Flash Serial USB to a free USB port of the PC and the MiniDIN connector to the connector of the Millflash extension cable;

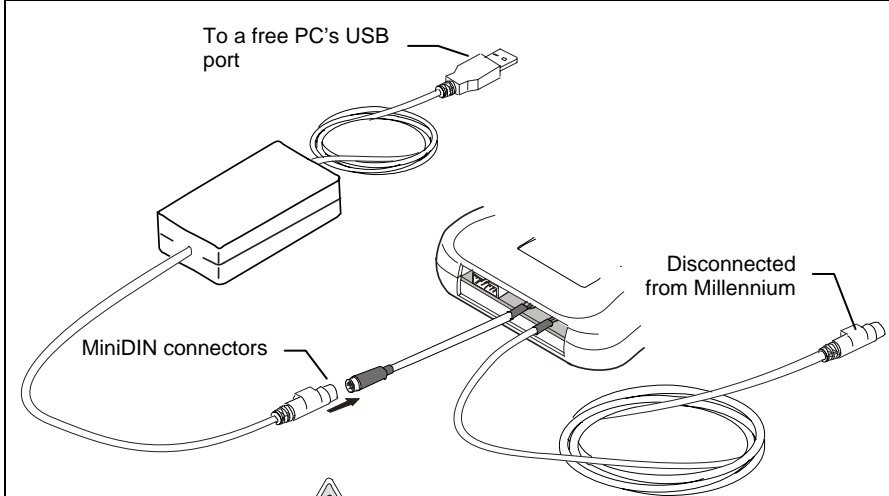 **WARNING**

Do not insert the device's PS2 et MiniDIN connectors on a PS2 port of the PC in order to avoid possible breakdowns of the internal components.

NOTE

 **MOST OPERATING SYSTEMS RECOGNIZE THE PRESENCE OF NEW HARDWARE AND PROCEED AUTOMATICALLY WITH THE INSTALLATION. IN A FEW CASES THE OPERATING SYSTEM COULD ASK TO INSERT THE INSTALLATION DISK; USE THE CD SUPPLIED AND FOLLOW THE DISPLAYED INSTRUCTIONS.**

Ended the file downloading from Millflash to PC, disjoin the MiniDIN connectors.

WARNING

 **BEFORE REMOVING THE MILLFLASH-PC LINK, MAKE SURE THAT THE ONGOING OPERATIONS ARE COMPLETED AND, WHERE PRESENT, ACTIVATE THE SAFE REMOVAL FACILITY.**

WARNING

 **BEFORE RECONNECTING THE MILLFLASH TO PC AND RECORDING NEW FILE REPORTS OF THE STERILIZER, REMEMBER TO DISJOIN THE MINIDIN CONNECTORS OF THE USB CABLE CONNECTION.**

MANAGING THE FILES BY MILLFLASH SW

Launching the program

Dialogue with the device

The Millflash software is a Windows (version 98 and XP) program that allows to interface the PC to the data recording device through a USB port, to download on PC the files in the Millflash memory and to manage them.


Launch the Millflash program from its desktop icon, or select the executable program file.

NOTE

BEFORE LAUNCHING THE PROGRAM, BE SURE THAT THE MILLFLASH-PC-CONNECTION IS ALREADY MADE , OTHERWISE THE DISPLAY WILL SHOW AN ERROR MESSAGE.

After launching the program, a window appears. containing the file reports folder (on the first launch it will be empty). Click on the **"Connect"** button to enable the connection to Millflash.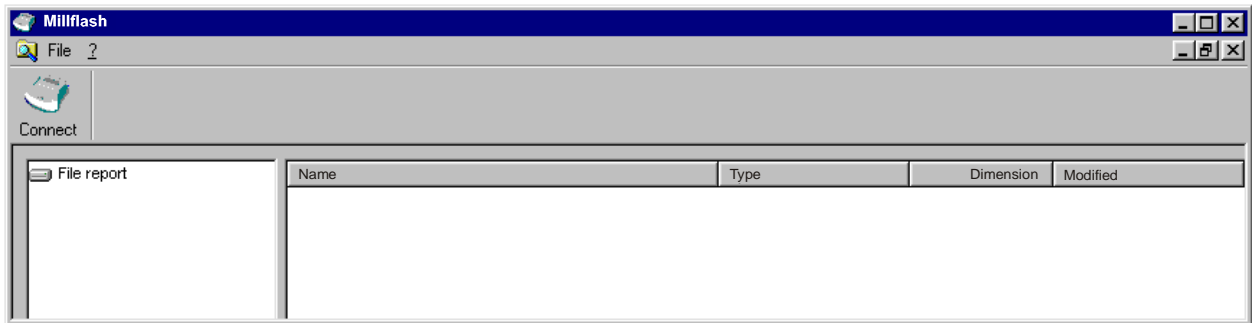

A second window appears, containing the file list related to the stored sterilization cycles, and the Millflash display will show the message **"PC link..."** ; in case of failed connection, an error message will be shown.

The keys functions are also present as sub-menus in the menu bar


File code	Description	File name	File date
DT0369AA	DataLogger	10BM1234_0001	05-04-06 11.15
DT0369AA	DataLogger	10BM1234_0002	06-04-06 10.45
DT0369AA	DataLogger	10BM1234_0003	10-04-06 11.30
DT0369AA	DataLogger	10BM1234_0004	11-04-06 10.20
DT0369AA	DataLogger	10BM1234_0005	30-04-06 10.24

Memory space [KB] Memory used [%] 100

Ready

Saving the Report file

To save to PC the file contained on Millflash, select the **Save** button (or File-Save from menu). The three keys and the window menu are disabled during the save process; the message **“Ready”** in the status bar shows is replaced by **“Saving...”**, followed by a number and by a progress bar that shows the progress of the save process for the individual files.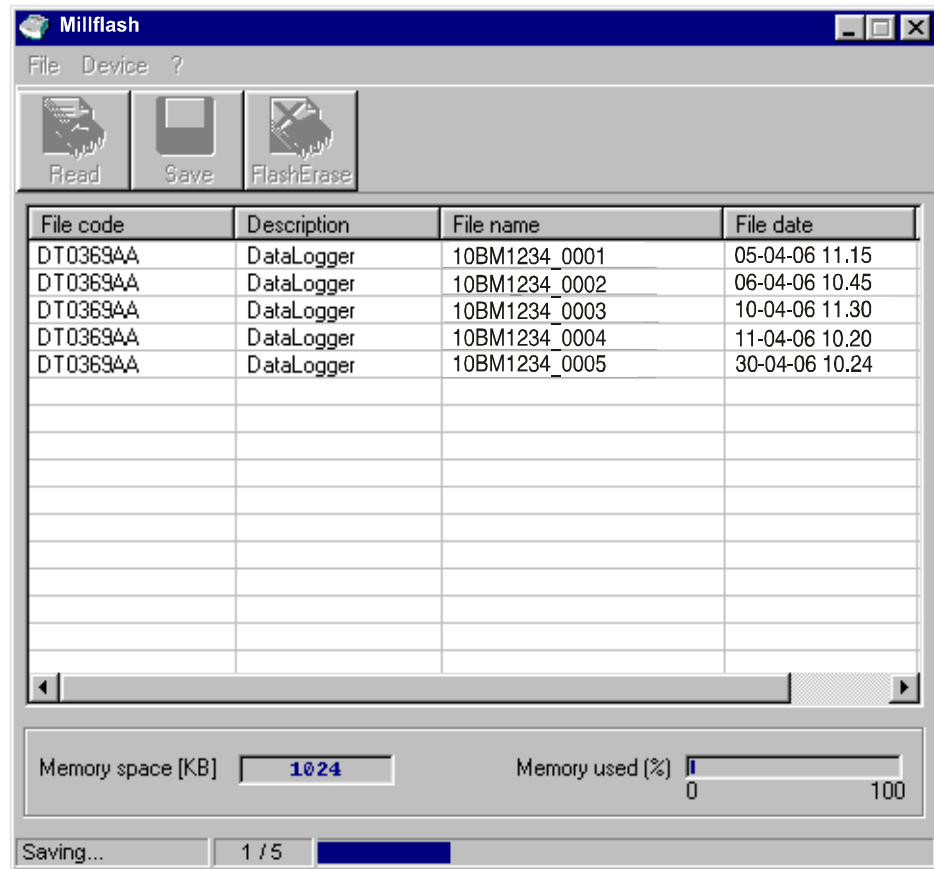


Report file management

At the end of the save process (status **“Ready”** and function keys enabled), close the window for the dialogue with the Millflash device and proceed to the management of the files saved on the PC.


The files are saved according to the cycle date in a directory automatically generated by the program and made up of folders for the years and subfolders for the months.

The files names are assigned on the basis of the cycle data, type, size and date of modification of files are also included.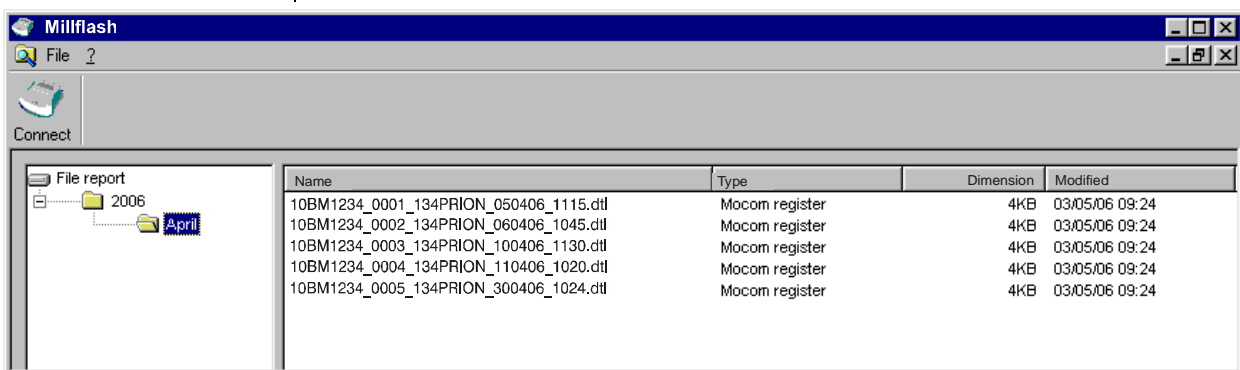

File name

The files saved on the PC are named "Mocom register". Each new file is assigned a default name according to the information included in the original file: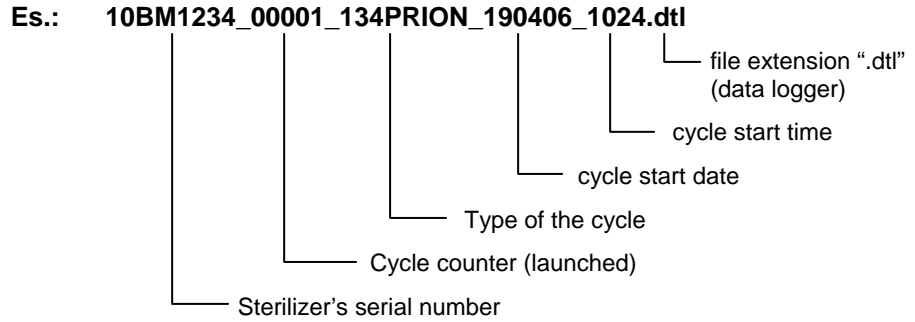

Files visualization

A double click on the file name, will show the window with the file content.

There are two types of visualization:

- reduced - default, shown on file opening
- extended – click the **"Extend view"** button to see the details of the sterilization cycle, with all data omitted in the reduced view.


If the cycle did not completed successfully, the view on opening is the extended one and the reduced view cannot be selected.

To print the displayed file, connect a printer to the PC and click the **"Print"** button.

File name

Extended view

To print the file

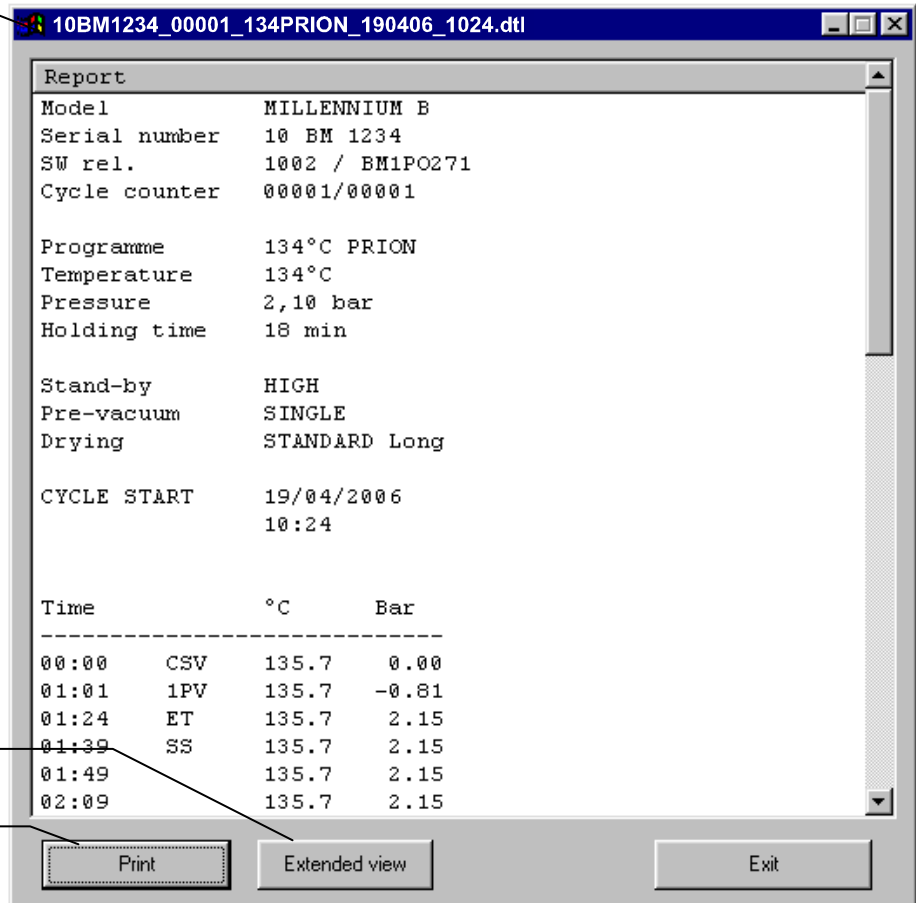MAINTENANCE

DIAGNOSTICS

If the Millflash display shows the message:

D o w n l o a d E r r o r

try to disconnect and connect back the connection cable.

If the same message is shown again, call the technical service.

EXTERNAL CLEANING

Clean with a soft cloth slightly damp with a non-abrasive and not corrosive detergent (neuter pH).

CUSTOMER SUPPORT

**FOR ANY REQUEST FOR
TECHNICAL SERVICE FOR THE PRODUCT,
WHETHER IN OR OUT OF WARRANTY,
DIRECTLY CONTACT THE

TECHNICAL SUPPORT DEPARTMENT

OF THE DEALER OR RESELLER
THAT SUPPLIED THE PRODUCT.**

M.O.COM. Srl is completely available to customers to provide any technical information about the product as well as to offer suggestions and advice on steam sterilization procedures.

In this regard, please refer to the following address:

M.O.COM. Srl
Customer Support
Via delle Azalee, 1
20090 Buccinasco (MI)
ITALY
Tel. (+39) 02-45701505
Fax (+39) 02-45701258
e-mail at@mocom.it
website www.mocom.it

To help us in the indispensable work of improving the quality of our products and service, please send your comments and/or suggestions by **letter** or **fax** or to the following **e-mail** address:

uc@mocom.it *(Marketing/sale department)*

Thank you in advance for your valuable assistance.

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