SIEMENS

Scio Module Reference Manual

for use with **SIEMENS** SC 7000/8000/9000XL Monitor

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**The Scio Module is manufactured by
Dräger Medical AG&Co. KGaA
23542 Lübeck, Germany**

This device bears the **CE** mark in accordance with the provisions of the Directive 93/42/EEC of June 14, 1993 concerning medical devices.

**Distributed by:
Siemens Medical Solutions USA, Inc.
Electromedical Group
16 Electronics Avenue
Danvers, MA 01923**

© Siemens AG 2003. All rights reserved
Printed in Germany

These instructions are intended to supplement the information in the Siemens SC 7000/8000/9000XL operating instructions. See the SC 7000/8000/9000XL User's Guide for complete information.

Reproduction in any manner, in whole or in part, in English or in any other languages, except for brief excerpts in reviews and scientific papers is prohibited without prior written permission of Siemens AG.

Before using all Siemens devices, read all the manuals that are provided with your device carefully. Medical equipment, however sophisticated, should never be used as a substitute for the human care, attention, and critical judgment that only trained health care professionals can provide.

SCIO MODULE

Table of Contents

| | |
|---|----|
| Overview | 2 |
| Hardware Setup | 4 |
| Site of Operation | 4 |
| Installing the Water Trap | 5 |
| Connections | 5 |
| Warm-Up | 6 |
| Calibration | 6 |
| Scio Setup | 8 |
| etCO ₂ * Monitoring | 8 |
| O ₂ /N ₂ O Monitoring | 10 |
| Agent Monitoring | 12 |
| Combined Display (O ₂ /Agent/N ₂ O) | 15 |
| Maintenance and Repair | 16 |
| Status Messages | 20 |
| Accessories | 21 |
| Technical Data | 22 |
| Scio Module | 22 |
| MultiGas | 23 |

Overview

The Scio module is a free-standing unit that samples breathing gases from adults and pediatric patients in non-, partial- and total rebreathing systems. It continuously measures the content of CO₂, N₂O, O₂ and one of the anesthetic agents Halothane, Isoflurane, Enflurane, Sevoflurane and Desflurane in any mixture. It communicates real time and derived gas information to the host system. Except for N₂O, the monitor saves values derived by the modules in its trend storage.

CAUTION: *The gas information is intended to be used by trained and authorized health care professionals only.*

The Scio module calculates both inspiratory and expiratory O₂ values (iO₂ and etO₂). Full technical descriptions of the Scio module is available from your local Siemens representative. The sample flow rate of the Scio module is fixed at 150 ml/min. (±20 ml/min.)

WARNING: **If the 170 ml/min. flow rate is not suitable for patients with compromised respiratory status, the physician should increase the fresh gas supply to satisfy the patient's respiratory requirements.**

The Scio module uses infrared light for the measurement of CO₂ and volatile anaesthetics. A small amount of the patient respiratory gas is drawn through a measuring chamber. An infrared light is shone through the chamber and the gas sample absorbs different amounts of light. A paramagnetic cell is used for the measurement of O₂. This cell uses a physical reaction that is proportional to the O₂ concentration. Mechanical shocks during measurement or the presence of other paramagnetic agents can distort the measurement of oxygen concentration.

Siemens disclaims all liability for the consequences of product alterations or modifications, as well as for the consequences which might result from the combination of this product with other products, whether supplied by Siemens or by other manufacturers, if such a combination is not endorsed by Siemens.


CAUTION: *The Scio module purges and zeroes itself approximately once every two hours. The purging/zeroing cycle lasts no longer than 25 seconds. Waveforms flatline and P-box values blank from the screen during this cycle. **Multigas Zero in Progress** appears in the message box.*

NOTES:

- The Scio module requires 510(k) review and is not yet commercially available in the US.
- The Scio module is not yet licensed in accordance with the Canadian Medical Device Regulations.
- The presence of organic cleaning solutions or gases containing freon may impact the accuracy of the Scio module.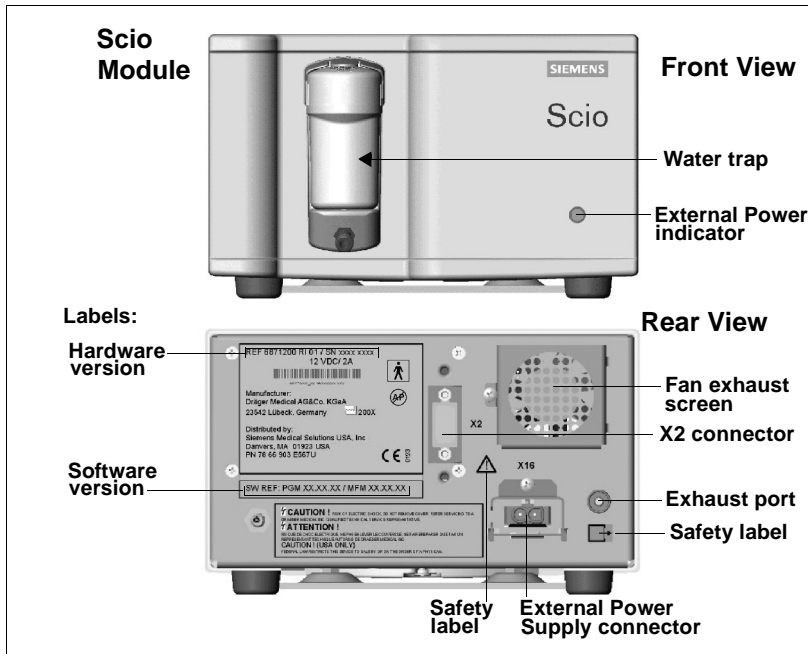

Hardware Setup

Site of Operation

The site of operation must meet the temperature, humidity, and atmospheric pressure requirements listed on page 22. In addition, observe the following guidelines:

- Make sure that the platform which supports the module is large enough, level, and stable.
- Make sure that the fan exhaust screen at the rear of the module and the ventilation holes on the underside are not obstructed.
- Place the module at least 25 cm (10 inches) from any possible source of ignition, such as sparking.
- Place the module close enough to the patient so that the sampling tubing can reach the airway T-connector and the exhaust tubing the hospital's exhaust gas scavenging system without stretching.


WARNINGS:

- **Do not use mobile phones within 33 feet (10 m) of the Scio module. Wireless phones may cause failure.**
- **Do not expose the Scio Module to mechanical vibrations or shock during measurement. Mechanical vibrations or shock can have adverse effects on gas measurement values.**
- **The operation of the Scio module in magnetic resonance imaging environments (MRI) is not supported.**
- **Do not use a Scio module near devices with microwave or other high-frequency emissions. These emissions may interfere with the modules' operation.**
- **SC 7000/8000/9000XL and the Scio Module must both be powered from hospital grade connectors within the same medically used room.**
- **When the SC 7000/8000/9000XL is used with the Scio it meets the Class A limits of CISPR11. The system is not intended for connection to public mains.**

Installing the Water Trap

Install the water trap into its receptacle on the Scio module by pushing against the Scio. A click will indicate that it is seated properly. Confirm that the water trap is empty. (For information on replacing the water trap, see page 18.)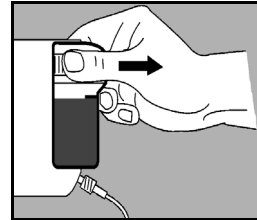

Connections

The Scio module connects to the SC 7000/8000/9000XL monitor via an INFINITY DOCKING STATION (IDS) or an INFINITY device Communication Power Supply (CPS). Scio functions are not available in DirectNet mode.


WARNINGS:

- **Do not use the module without a Watertrap.**
- **Sampling tubing should be kept as short as possible (but not stretched) to minimize dead space and optimize response time. Long sampling lines degrade the performance of side stream measurements, may affect accuracy, and result in slower response times.**
- **Always use Siemens-approved Scio sampling tubing (polypropylene). Never use standard pressure-sensor tubing (PVC). PVC tubing absorbs anesthetic agents, which it later releases (degassing). The use of standard PVC tubing can result in erroneous agent concentration readings.**
- **To avoid the risk of explosion, do not use flammable anesthetic agents such as ether and cyclopropane with the Scio module.**

Connect the module and tubing as follows:

1. Connect one end of the sampling tubing to the water trap, and the other end to the airway T-connector. (For information on changing sampling tubing and T-connector, see page 17).
2. Connect one end of the exhaust tubing to the exhaust port at the rear of the module, and the other end to the hospital's gas-scavenging system.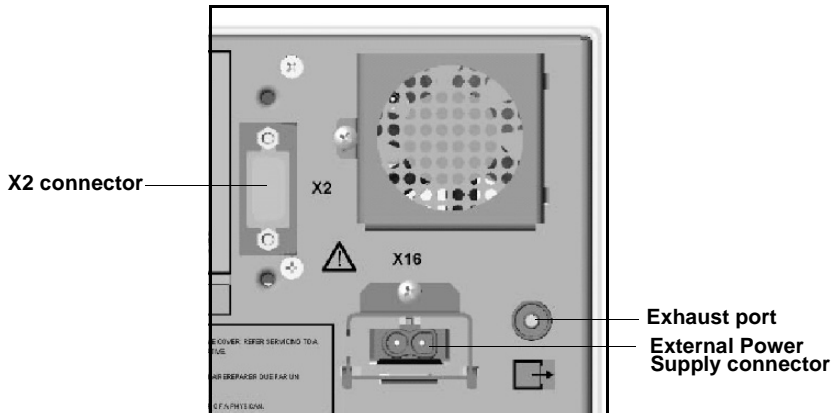

3. Connect one end of the monitor/module intermediate cable to the X2 connector at the rear of the module and the other end to the CPS/IDS connector marked X12 . If you are using a breakout box, connect the end to the connector on the breakout box.
4. Connect the power supply to the external power supply connector at the rear of the module.
5. Connect the power supply cord to a hospital-grade outlet.

Warm-Up

Upon start-up, the Scio module passes through an initialization and warm-up period. During this time, the etCO₂*, O₂, and/or Agent parameter boxes display a question mark (?) and the monitor does not transmit Scio data over the network, nor is this data trended.

WARNING: During warm-up, reported values might not be accurate. To achieve full accuracy, a typical warm-up period of 5 minutes is recommended. Refer to the *Technical Data* appendix for a detailed description of Scio accuracy.

Calibration

The Scio module is self-zeroing and does not need calibration by the clinical staff.

The O₂ cell in the Scio does not require routine calibration, however, the user may choose to perform a 1-point or 2-point calibration in the event that the measured values are not as expected. A 1-point calibration is performed against room air (typically 21% ± 1% O₂) and a 2-point calibration is performed against 100% O₂ as well as room air.

A yearly check of the Scio calibration components should be performed by authorized technical personnel.

1 Point Calibration

1. Access the O₂ menu (see page 11) or the combined MULTIGAS menu (see page 15).
2. Click on **1 Point Cal.**

During calibration, the monitor displays the message MULTIGAS Oxygen Cal. in Progress. When the calibration is complete, the monitor displays MULTIGAS Oxygen Cal. Accepted and the current date and time appear next to the menu field *Last O₂ Cal.* If the calibration fails, the monitor displays an error message (see page 20 for status messages).

2 Point Cal.

1. Make sure a medical-grade O₂ source is available.
2. Access the O₂ menu (see page 11) or the combined MULTIGAS menu (see page 15).
3. Click on **2 Point Cal.** A popup screen appears with the message Supply 100% O₂, then Press Continue.
4. Connect the O₂ source to the water trap on the module and supply 100% oxygen. Wait until the waveform levels off at a maximum display height and the numerical data in the O₂ parameter box is stable. This may take several minutes on older O₂ cells.

WARNING: The O₂ value in the MULTIGAS parameter box must be stable before calibration continues, otherwise the monitor cannot obtain a reliable calibration constant, which results in inaccurate measurements.

5. Press **Continue**. The monitor displays MULTIGAS Oxygen Cal. in Progress.
6. When the 100%-O₂ calibration is complete, the monitor displays the message Remove 100% O₂, then Press Continue. Wait for the value in the parameter box to stabilize and then remove the O₂ source.
7. Click on **Continue**.

When the calibration is complete, the monitor displays MULTIGAS Oxygen Cal. Accepted and the current date and time appear next to the menu field *Last O₂ Cal.* If the calibration fails, the monitor displays an error message (see page 20 for status messages).

Scio Setup

Scio parameters are displayed in the etCO₂*, O₂/N₂O and Anesthetic Agent and parameter boxes. Each has its own setup menu, described in the following pages.

NOTES:

- Some Scio parameter labels are marked by an asterisk (*) to distinguish them from parameters monitored by the etCO₂ pod or module.
- You can display the O₂/N₂O and agent parameters in a single combined parameter box (see page 15).


etCO₂* Monitoring

The etCO₂* waveform displays the instantaneous CO₂ measurements calculated by the Scio module. The etCO₂* parameter box displays the current values for:

Inspired CO₂ (iCO₂*) — The level of CO₂ in the airway during the inspiration phase.

End-tidal CO₂ (etCO₂*) — The level of CO₂ in the airway at the end of expiration.

Respiration Rate (RRc*) — The patient's respiration rate, derived from the etCO₂* signal by calculating an average rate over the two most recent breaths.

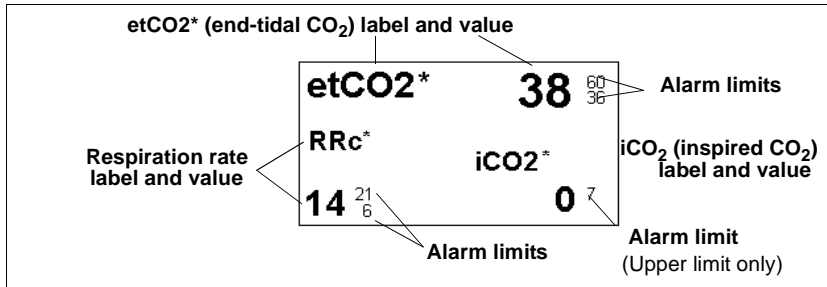NOTE: The monitor does not alarm for etCO₂ or inspiratory and expiratory agent limit violations until it has established a valid respiratory rate.

To access the Scio etCO₂* setup menu:

- Click on the etCO₂* parameter box

or

 1. Press the **Menu** fixed key.
 2. Click on **Patient Setup**.
 3. Click on **Parameters** to display a list of available parameters.
 4. Click on **etCO₂***.

Quick Reference Table -- etCO₂* Setup

Click on the following items to execute etCO₂* setup functions:

| <i>The etCO₂* Setup Menu</i> | | |
|---|--|---|
| Menu Item | Description | Settings |
| Scale | Sets etCO ₂ waveform scale | • 40, 60, 80 |
| Agent Display | Displays a separate Agent parameter box;. Ghosted if the monitor is displaying combined MultiGas parameter box (page 15) | • ON, OFF |
| Pressure Comp. | Sets compensation for ambient atmospheric pressure | • Auto |
| RRc* Apnea Time | Sets time that the monitor waits before reporting a cessation of breathing as an apnea event | • OFF, 10, 15, 20, 25, 30 s |
| Apnea Archive | Allows you to store and/or record automatically an alarm event for apnea. You can later review stored alarms on the Event Recall screen. | • OFF, Record, Store (default), Str./Rec. |

The etCO₂* Setup Menu

| Menu Item | Description | Settings |
|----------------------------|---|----------|
| Auto Zero Delay | <p>Delays automatic zeroing for 5 minutes for uninterrupted monitoring.</p> <p> WARNING: Delaying the auto zero may impact the accuracy of the device.</p> <p>Note: Gas sensors in the Scio module are automatically zeroed and calibrated against room air. During zeroing, the monitor temporarily blanks Scio parameter values. One minute before automatic zeroing, the monitor sounds an attention tone and displays the message Auto zero in 1 minute.</p> | N/A |
| etCO ₂ * Alarms | Accesses etCO ₂ * alarms in Alarm Limits table (see SC 7000/8000/9000XL User's Guide Chapter 5) | |

O₂/N₂O Monitoring

The O₂ waveform indicates O₂ concentrations calculated by the Scio module. The O₂ parameter box can display the current concentration values for the following parameters:



Inspired O₂ (iO₂) — The level of O₂ in the airway during the inspiration phase

Expired O₂ (etO₂) — The level of O₂ in the airway during the expiration phase

N₂O — The concentration of N₂O in the patient's airway

The appearance of the O₂ parameter box varies depending on whether or not the N₂O display is turned on in the O₂ menu.

Typical O₂/N₂O parameter box displays are shown below.
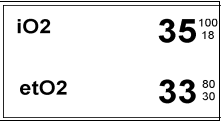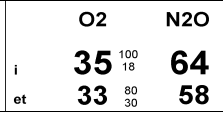
| Parameter Box | Description |
|---|---|
|  | <p>ScioModule: N₂O display turned off. This module calculates both iO₂ and etO₂ values.</p> |
|  | <p>Scio Module: N₂O display turned on. This module calculates both iO₂ and etO₂ values.</p> |

| Parameter Box | Description |
|---|-------------|
| <p>Notes: The Δ symbol in the parameter box indicates that the O₂ lower alarm limit has been set to a value less than 21% (the percentage of O₂ in room air). The O₂ parameter box does not show N₂O alarm limits because N₂O does not alarm.</p> | |

To access the O₂/(N₂O) setup menu:

- Click on the O₂/N₂O parameter box
- or*
- 1. Press the **Menu** fixed key.
- 2. Click on **Patient Setup**.
- 3. Click on **Parameters** to display a list of available parameters.
- 4. Click on **O2**.

Quick Reference Table -- O2/N2O Setup

Click on the following items to execute O₂/N₂O setup functions

| The O ₂ /(N ₂ O) Setup Menu | | |
|---|--|----------------------|
| Menu Item | Description | Settings |
| MultiGas Parameter | Enables combined MultiGas parameter box display (see SC 7000/8000/9000XL User's Guide Chapter 23) | • ON, OFF |
| O2 Scale | Sets O ₂ waveform scale | • 50 %, 100 % |
| N2O Display | Displays N ₂ O values Notes: This selection is ghosted and not available if the monitor is configured to display a combined Scio parameter box (see page -15). There is no alarm function for N ₂ O and the parameter box does not show N ₂ O alarm limits. | • ON, OFF |
| Auto Zero Delay | Delays automatic zeroing for 5 minutes for uninterrupted monitoring (see page 9 for information) | N/A |
| 1 Point Cal. | See page 6 for detailed information on calibration functions | |
| 2 Point Cal. | | |
| Last O2 Cal. | | |
| O2 Alarms | Accesses Limits table (see SC 7000/8000/9000XL User's Guide Chapter 5) | N/A |

Agent Monitoring

You can identify agent waveforms and parameters by color (*Halothane*--Red; *Desflurane*--Light blue; *Enflurane*--Orange; *Sevoflurane*--Yellow; *Isoflurane*--Purple).

The Agent parameter box shows the inspiratory and end-tidal values for the currently monitored agent. If the module is in automatic agent identification mode (ID Override OFF) and has not yet identified or cannot identify the agent, the Agent parameter box displays **Agent?**. A question mark also appears next to the identified agent's label (e.g., **etHal?**), if the calculated agent concentration value exceeds the module's accuracy range for agent monitoring (see *Technical Data* appendix).

WARNING: If the monitor displays a question mark next to an agent's parameter label, use the displayed agent concentration values with caution.

When the Scio module detects a mix of two or more anesthetic agents, the displayed agent waveform reflects the combined agent concentrations. The color of the waveform represents the agent with the highest level of concentration. If there is a change of the anesthetic agent during monitoring, the parameter box displays a MIXED label and replaces the concentration values with asterisks (**). The mixed label (e.g. HAL-HAL) indicates the agent that has the highest level of concentration. This label will switch to that of the second administered agent when its concentration exceeds that of the first agent.

MAC Values

When the monitor, operating in adult mode, has identified an agent, the parameter box shows a value for the minimum alveolar concentration of the agent (MAC value).





WARNINGS:

- **MAC values are merely a guideline based on the anesthetic requirements of an *average* adult patient. Age and other factors are not taken into account.**
- **The monitor's MAC values cannot be applied to children or neonates.**

1 **MAC** (minimum alveolar concentration) is equal to the alveolar anesthetic concentration at one atmosphere (760 mmHg) at which 50% of all patients no longer respond to noxious stimuli, and corresponds to the following expiratory agent concentrations.

| Agent Concentrations (MAC Values) | | | | | |
|-----------------------------------|------|-------|------|-----|------------------|
| HAL | ENF | ISO | SEV | DES | N ₂ O |
| 0.75% | 1.7% | 1.15% | 1.7% | 6% | 100% |

The table below shows typical Agent parameter box displays.
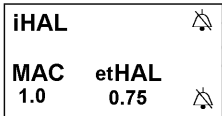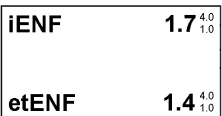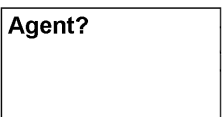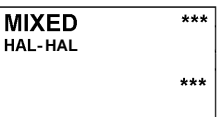
| Parameter Box | Description |
|---|---|
|  | Agent identified. The monitor is in adult monitoring mode (MAC values displayed). |
|  | Agent identified. The monitor is in pediatric or neonatal monitoring mode (no display of MAC values). |
|  | Agent not yet identified in automatic identification mode (ID Override OFF). |
|  | More than one agent identified. The mixed label (ex. HAL-HAL) indicates the agent that has the highest level of concentration. This label will switch to that of the second administered agent when its concentration exceeds that of the first agent.. |

To access the Agent setup menu:

- Click on any Agent parameter box if displayed
- or*
- 1. Press the **Menu** fixed key.
- 2. Click on **Patient Setup**.
- 3. Click on **Parameters** to display a list of available parameters.

4. Click on **AGENT** to display the Agent menu.

Quick Reference Table -- Agent Setup

Click on the following items to execute Agent setup functions:

| <i>The Agent Setup Menu</i> | | |
|-----------------------------|--|---|
| Menu Item | Description | Settings |
| Agent Scale | Sets Agent waveform scale | • 1, 2, 3, 5, 10, or 20% |
| Agent Display | Displays a separate Agent parameter box Note: Ghosted if the monitor is displaying combined MultiGas parameter box (page 15) | • ON, OFF |
| ID Override | Configures Scio module to identify an agent automatically or to measure the concentration levels of a user-specified anesthetic agent. Note: The presence of organic cleaning solutions or gases containing freon may impact the accuracy of the Scio module. | • OFF -- Module automatically identifies agent and displays its concentration levels <i>Manual override selections:</i> • HAL • ISO • ENF • SEV • DES Note: If you select an agent from the ID Override menu that is different from the one actually administered, the monitor displays a message indicating the type of agent it actually detects (i.e. <i>Hal detected</i>). |
| Auto Zero Delay | Delays automatic zeroing for 5 minutes for uninterrupted monitoring.  WARNING: Delaying the auto zero may impact the accuracy of the device. Note: Gas sensors in the Scio module are automatically zeroed and calibrated against room air. During zeroing, the monitor temporarily blanks Scio parameter values. One minute before automatic zeroing, the monitor sounds an attention tone and displays the message Auto zero in 1 minute. | N/A |
| Agent Alarms | Accesses Agent alarms in Alarm Limits table (see SC 7000/8000/9000XL User's Guide Chapter 5) | |

Combined Display (O₂/Agent/N₂O)

The O₂/N₂O and Agent parameters can be combined to share a single waveform channel and MultiGas parameter box. Typical combined MultiGas parameter boxes are shown below.

| Parameter Box | Description | | | | | | | | | | | | |
|---|----------------|----------------|------------------|------------------|---|----|-----|----|----|----|-----|----|--|
| <table border="1"> <tr> <td></td> <td>O₂</td> <td>ISO</td> <td>N₂O</td> </tr> <tr> <td>i</td> <td>35</td> <td>1.5</td> <td>64</td> </tr> <tr> <td>et</td> <td>33</td> <td>1.3</td> <td>58</td> </tr> </table> | | O ₂ | ISO | N ₂ O | i | 35 | 1.5 | 64 | et | 33 | 1.3 | 58 | The Scio module has identified an agent and displays concentration levels for O ₂ , isoflurane, and N ₂ O. |
| | O ₂ | ISO | N ₂ O | | | | | | | | | | |
| i | 35 | 1.5 | 64 | | | | | | | | | | |
| et | 33 | 1.3 | 58 | | | | | | | | | | |
| <table border="1"> <tr> <td></td> <td>O₂</td> <td>AA?</td> <td>N₂O</td> </tr> <tr> <td>i</td> <td>35</td> <td></td> <td>64</td> </tr> <tr> <td>et</td> <td>33</td> <td></td> <td>58</td> </tr> </table> | | O ₂ | AA? | N ₂ O | i | 35 | | 64 | et | 33 | | 58 | The Scio module is in automatic agent identification mode (ID Override OFF), but has not yet identified the agent. |
| | O ₂ | AA? | N ₂ O | | | | | | | | | | |
| i | 35 | | 64 | | | | | | | | | | |
| et | 33 | | 58 | | | | | | | | | | |

To enable the combined MULTIGAS parameter box display:

1. Open the Main Screen setup menu (see SC 7000/8000/9000XL User’s Guide page 2-4).
2. Click on **More** to go to the Main Screen menu’s second page.
3. Select **MultiGas Parameter**; click the knob to select **ON**.

NOTES:

- You can also enable MultiGas Parameter from the O₂ menu (see page 11) or the O₂/N₂O menu (page 11).
- The combined MULTIGAS parameter box takes the place of the O₂ parameter box on the Main Screen. Make sure that the O₂ or the MULTIGAS parameter is appropriately assigned on the parameter priority list (see SC 7000/8000/9000XL User’s Guide Chapter 2).
- When you select the combined MULTIGAS parameter box, N₂O values are automatically enabled for display.

To access the combined (O₂/Agent/N₂O) setup menu:

1. Ensure that **MultiGas Parameter** is **ON**.
2. Return to the Main Screen.
3. Click on the O₂/Agent/N₂O combined parameter box.

Quick Reference Table -- Combined Display Setup

Click on the following items to execute setup functions for Combined (O₂/Agent/N₂O) MultiGas monitoring and display.

| The Combined (O ₂ /Agent/N ₂ O) Setup Menu | | |
|---|---|---|
| Menu Item | Description | Settings |
| Waveform | Selects waveform for display | • O ₂ , Agent |
| MultiGas Parameter | Enables combined MultiGas parameter box display (see SC 7000/8000/9000XL User's Guide Chapter 23) | • ON, OFF |
| O ₂ Scale | Sets O ₂ waveform scale (see page 9) | • 50 %, 100 % |
| N ₂ O Display | Displays N ₂ O values in O ₂ /N ₂ O parameter box (see page 11) | • ON, OFF |
| Agent Display | Displays a separate Agent parameter box (see page 14) | • ON, OFF |
| <p>Note: During the combined display, the N₂O and Agent displays are automatically turned on and their selections are ghosted on the combined menu. You can only access these selections and turn the displays off, once you have turned off the combined MultiGas parameter display.</p> | | |
| Agent Scale | Sets Agent waveform scale | • 1-20 % in increments of 1 |
| ID Override | Configures Scio module to identify an agent automatically as specified by the user (see page 14) | <ul style="list-style-type: none"> • OFF -- Identifies agent and displays its concentration levels • Override options: <div style="margin-left: 20px;"> HAL ISO ENF SEV DES </div> |
| Auto Zero Delay | Delays automatic zeroing for 5 minutes (see page 9 for detailed information) | N/A |
| 1 Point Cal. | See page 6 for detailed information on calibration functions. | |
| 2 Point Cal. | | |
| Last O ₂ Cal. | | |
| O ₂ Alarms | Accesses O ₂ alarms in the Alarm Limits table (see SC 7000/8000/9000XL User's Guide Chapter 5) | |

Maintenance and Repair

To ensure safety, the Scio module requires routine cleaning (see SC 7000/8000/9000XL User's Guide Chapter 26, *Cleaning, Disinfecting, and Sterilizing*). Monitoring accessories such as sam-

pling tubing, T-connectors, water traps, and fan filters are not reusable and need to be replaced at regular intervals.

General equipment inspection and maintenance is required. Once a year, check all devices, accessories, and cables for damage and test the ground resistance, chassis and patient leakage currents as well as all alarm functions. Make sure that all safety labels are legible. Maintain a record of these safety inspections.

WARNINGS:

- **Occupational safety: Used sampling tubing, T-connectors and water traps could be contaminated and must be handled and disposed of with care. Infection hazard may be present. Dispose of these items in accordance with local regulations.**
- **Because of the danger of electric shock, never remove the cover of any device while it is in operation or connected to a power outlet.**

Changing the Sampling Tubing and T-Connector

The sampling tubing and T-connector that connect the Scio module to the patient's airway are not reusable. They must be replaced under the following conditions:

- A new patient is connected to the module
- The cleanliness of the tubing or connector is suspect or compromised

NOTE: Use only Siemens-approved sidestream sampling lines. Siemens does not assume responsibility for the reliability and safety of Scio measurements, if non-approved tubing is used.

Emptying the Water Trap



The water trap should be emptied if the contents have reached the 'full' mark.

NOTE: If water trap has been in use for 1 month, it should be replaced. (See "Replacing the Water Trap" below.)

To empty the water trap:

1. Disconnect the sampling tubing.
2. Remove the trap from its connector by holding it firmly on the ridged surfaces and pulling it out from the Scio module.
3. Connect an empty syringe (size > 20 ml and without a needle) to the port on the back of the water trap.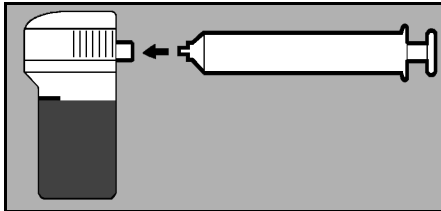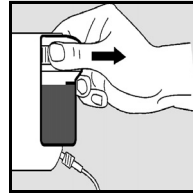

4. Aspirate water trap contents into syringe.
5. Remove syringe and discard.


NOTE: Do not attempt to clean water trap.

6. Reseat water trap by pressing it against the Scio. A click will indicate that it is seated properly.
7. Reconnect the sampling tubing. (Before monitoring a new patient, exchange the tubing and T-connector, see page 17.)

Replacing the Water Trap

To replace the water trap:

1. Disconnect the sampling tubing.
2. Remove the trap from its connector by holding it firmly on the ridged surfaces and pulling it out from the Scio module
3. Insert the new trap by pressing it against the Scio. A click will indicate that it is seated properly.
4. Connect the sampling tubing. (Before monitoring a new patient, exchange the tubing and T-connector, see page 17.)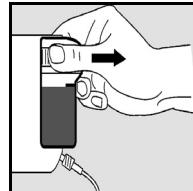

Cleaning the Fan Filter


The fan filter should be cleaned 1 time per month.

NOTE: If fan filter has been in use for 1 year, it should be replaced. (See “Replacing the Fan Filter” below.)

To clean the fan filter:

1. Locate the fan on the rear of the module.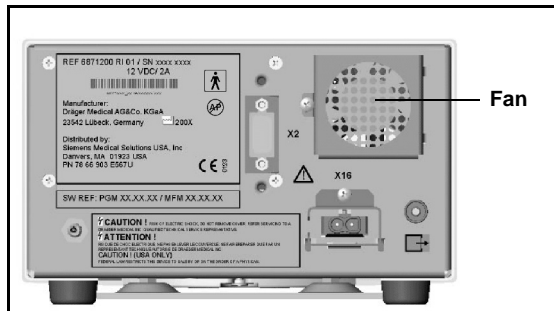

2. Grasp the plastic fan filter and remove it from its holding slots.
3. Vacuum up any accumulation of dust at the fan port and inside the filter.
4. Reinsert fan filter.

Replacing the Fan Filter

To change the fan filter:

1. Locate the fan on the rear of the module.
2. Grasp the plastic fan filter and remove it from its holding slots.
3. Vacuum up any accumulation of dust at the fan port.
4. Insert a new filter.

Status Messages

| Message | Condition | Suggested Action |
|---|--|---|
| Agent? | The module has not yet identified or cannot identify agent because : <ul style="list-style-type: none"> • Agent is unknown (ie., not HAL, DES, ISO, SEV or ENF) • Agent concentration is too low • Vaporizer is leaking • Traces of disinfectant are present | <ul style="list-style-type: none"> • Select agent manually (see page 14). • Check/replace vaporizer. |
| MultiGas Too Warm | Fan port blocked Hardware problem | <ul style="list-style-type: none"> • Clear/unblock port. • Call Manufacturer. |
| i [parameter] > # et [parameter] < # | The inspired or expired concentrations of the parameter fall outside the current upper or lower alarm limits for that parameter | <ul style="list-style-type: none"> • Observe the patient and treat if necessary. • Adjust the alarm limits for that parameter. |
| i [parameter] out of range (high) i [parameter] out of range (low) | The inspired concentrations of the parameter fall outside the monitor's measuring range | <ul style="list-style-type: none"> • Observe the patient and treat if necessary. • Check connections. • Unplug and re-plug Scio module. |
| et [parameter] out of range (high) et [parameter] out of range (low) | The expired concentrations of the parameter fall outside the monitor's measuring range | <ul style="list-style-type: none"> • Power-cycle monitor or un-dock and re-dock monitor. • Call Manufacturer. |
| MultiGas Data Invalid | Communication problem | <ul style="list-style-type: none"> • Unplug and re-plug Scio module. • Power-cycle monitor or un-dock and re-dock monitor. • If message persists, call Manufacturer. |
| Mixed Agents | Scio Module agent identifier detected the presence of more than one anesthetic agent in the patient circuit. | <ul style="list-style-type: none"> • If unintentional, check anesthesia delivery system. • After a change of agent, wait until all traces of the first agent completely clear out of the breathing circuit. |
| MultiGas Zero Accepted | Zero successful | <ul style="list-style-type: none"> • None |
| MultiGas Zero Failed | Zero performed using the wrong gas Occlusion or leak present HW problem | <ul style="list-style-type: none"> • Verify surrounding environmental atmosphere is not contaminated. • Check for leaks and occlusions. • Call Manufacturer. |
| MultiGas Warming Up (? in parameter box) | Scio module is warming up and is operating at reduced accuracy | <ul style="list-style-type: none"> • Do not rely on Scio Module values. |

| Message | Condition | Suggested Action |
|----------------------------------|---|--|
| MultiGas H/W failure | Loss of communication or hardware problem detected | <ul style="list-style-type: none"> • Check connections. • Unplug and re-plug Scio module. • Power-cycle monitor or un-dock and re-dock monitor. • Call Manufacturer. |
| MultiGas Incompatible | Scio H/W or S/W incompatibility | <ul style="list-style-type: none"> • Check version numbers. • Call Manufacturer. |
| MultiGas Initialization | Device initializing | <ul style="list-style-type: none"> • Wait |
| MultiGas Oxygen Cal. Accepted | Oxygen calibration successful | <ul style="list-style-type: none"> • None |
| MultiGas Oxygen Cal. in Progress | Oxygen calibration in progress | <ul style="list-style-type: none"> • Wait |
| MultiGas Sample Line Occlusion | Scio module sample line occluded Watertrap full, defective, or not installed. | <ul style="list-style-type: none"> • Check sample line and replace if necessary. • Check watertrap, replace or install, if necessary. |
| MultiGas Unplugged | Scio module has become disconnected | <ul style="list-style-type: none"> • Check the connection and reconnect if necessary. • If message persists, call Manufacturer. |
| MultiGas Zero in Progress | Scio module zero in progress | <ul style="list-style-type: none"> • Wait |

Accessories

| | |
|------------------------------|-----------------|
| Scio Water Trap (12 pieces) | 78 68 123 E530U |
| Scio Sample line (Set of 10) | 78 68 115 E530U |
| Power supply | 59 53 539 E530U |

Technical Data

Scio Module

| Physical Attributes | |
|--|---|
| Size (H x W x D) <i>with watertrap:</i> | 122 x 222 x 300 mm (4.80 x 8.70 x 11.7 in) |
| Weight: | 3.0 Kg (6.6 lbs) |
| Cooling: | Fan |
| Mains frequency: | 50/60 Hz |
| Power requirement: | < 0.8 A at 100-120 Vac; <0.4 A at 200-240 Vac |
| Chassis Leakage Current: | ≤ 300 μA (per UL 544) ≤ 500μA (per IEC 60601-1) |
| Electric Shock Protection: | Type BF |
| Protection Class: | Class 1 |
| Mode of operation: | Continuous |
| Power: | from specified power supply |
| Sound Pressure level | ≤ 45 dB(A) |
| Air Ingression, leakage | < 45 ml during zeroing, < 10 ml/min leakage |
| Sample Flow rate | 150 ml/min. ± 20 ml/min. |
| Environmental Requirements | |
| Temperature range: | Operating: 10°C to 40°C (50°F to 104°F) Storage: -20°C to 70°C (-4°F to +158°F) |
| Relative humidity: | Operating: 5% to 90% Storage: 5% to 95% |
| Atmospheric pressure | Operating: 525 to 795.1 mmHg (70 to 106 kPa) Storage: 375 to 795.1 mmHg (50 to 106 kPa) |
| Notes: | <ul style="list-style-type: none">• Readings comply to ATPS conditions.• This device is not intended for use in the presence of flammable gases. |

MULTIGAS

| | |
|--|---|
| Parameter display: | etCO ₂ *, iCO ₂ *, RRc*, et O ₂ , iO ₂ , O ₂ , N ₂ O, etHAL, iHAL, etISO, iISO, etENF, iENF, etSEV, iSEV, etDES, iDES Note: Asterisks distinguish Scio module parameter labels from those of the etCO ₂ module. |
| Agents measured: | Halothane, Isoflurane, Enflurane, Sevoflurane, Desflurane |
| Display Values: | Agents, CO ₂ , O ₂ : Inspired and expired concentrations (%), plus trend, and waveform N ₂ O: Inspired and expired concentrations (%) |
| Measuring methods: | CO ₂ , Agents, N ₂ O: Infrared O ₂ : Paramagnetic sensor |
| Display Range: | CO ₂ : 0% to 10% (resolution 0.1%) O ₂ : 0% to 100% (resolution 1.0%) N ₂ O: 0% to 100% (resolution 1.0%) Agents: Halothane: 0% to 10% (resolution 0.1%) Isoflurane: 0% to 10% (resolution 0.1%) Enflurane: 0% to 10% (resolution 0.1%) Sevoflurane: 0% to 11% (resolution 0.1%) Desflurane: 0% to 24% (resolution 0.1%) |
| Full Accuracy after warm-up period complete - Agents: (when RR ≤ 40) | Halothane (up to 8.5%vol): ±(0.15 vol % + 15% rel.) Isoflurane (up to 8.5%vol): ±(0.15 vol % + 15% rel.) Enflurane (up to 10%vol): ±(0.15 vol % + 15% rel.) Sevoflurane (up to 10%vol): ±(0.15 vol % + 15% rel.) Desflurane (up to 20%vol): ±(0.15 vol % + 15% rel.) |
| Full Accuracy after warm-up period complete - N₂O: (when RR ≤ 40) | ±(2 vol % + 8% rel.) |
| Full Accuracy after warm-up period complete - O₂: (when RR ≤ 40) | ±3 vol % |
| Full Accuracy after warm-up period complete - CO₂: (when RR ≤ 40) | ±0.5 vol % or ±12% rel., whichever is greater |
| Rise Time: (10% to 90% at 150 ml/min. flow rate) | CO ₂ : <500 ms O ₂ : <650 ms N ₂ O: <500 ms Agents: <500 ms |
| Delay Time with water trap and 2.5m sample line: | < 4 sec. |

| | |
|--------------------------------|--|
| Respiration Rate (RRc): | Display: Respiration Rate (rpm) Measuring Range: 0 to 90 rpm Resolution: 1 rpm Accuracy: 0 to 60 rpm: ± 1 rpm >60 rpm: not specified |
|--------------------------------|--|

Siemens reserves the right to make modifications to equipment without prior notice.

EM

Dräger PartNo. 90 37 868

**Distributed By:
In the United States:**

All Others:

ASK No. L836-02-7B00

**Siemens Medical Solutions USA, Inc.
Electromedical
Systems Group, PCS
16 Electronics Avenue
Danvers, MA 01923
USA**

**Siemens Medical Solutions, Inc.
Electromedical Systems
Division
Röntgenvägen 2
S-171-95 Solna
Sweden**

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