
HEINE QUALITY  
MADE IN GERMANY

## HEINE K 180® Ophthalmoscope

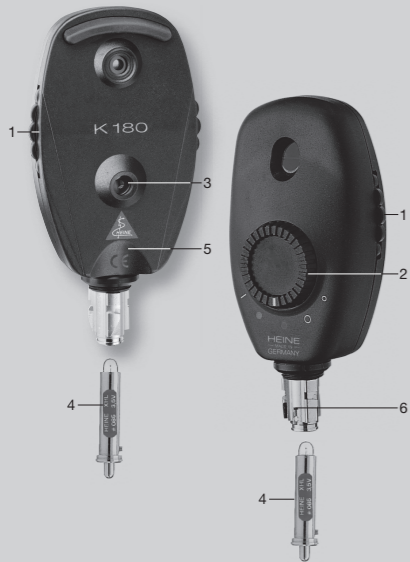CE

HEINE Optotechnik GmbH & Co. KG  
Kientalstr. 7 · 82211 Herrsching · Germany  
Tel. +49(0)81 52 / 38 - 0  
Fax +49(0)81 52 / 38 - 2 02  
E-Mail: info@heine.com · www.heine.com  
med 7896    2014-01-13

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DEUTSCH

### HEINE K 180® Ophthalmoskop

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ENGLISH

### HEINE K 180® Ophthalmoscope

 Before using the Ophthalmoscope read these instructions carefully and keep them in a safe place for future reference.

 In accordance with local regulations this product should be disposed of as an electronic device separately.

#### Safety information

**Range of applications:** The K 180 ophthalmoscope is designed solely for examination of the eye.

**Safety in use:** Use only handles or devices which are specifically designed for medical use as a power supply. The supply voltage must be the same as bulb voltage.

#### Using the instrument

For maximum comfort during prolonged use, hold the instrument so that the thumb coincides with the groove in the instrument body (5). In this position, the lens (1) and filter wheel (2) can be operated comfortably with the forefinger of the same hand. The value of the chosen correction can be seen in the window (3). Positive values are green, negative values red.

#### Cleaning and Sterilisation

The ophthalmoscope body can be wiped clean with a soft cloth soaked in alcohol. The glass discs in the viewing and light-exit windows can be cleaned by using a cotton wool swab in a spiral motion from the centre of the disc working towards the edge.

#### Changing the bulb

Disconnect the ophthalmoscope from the handle and pull out the bulb (4). Wipe clean the glass of the new bulb and push the bulb all the way in. The projection (7) must locate in the slit (6).

**Please note:** The performance of this unit can only be guaranteed if genuine HEINE XHL Xenon Halogen spare bulbs are used.

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#### General Conditions of Warranty

Instead of the statutory warranty time period of 2 years, HEINE will grant a guarantee of 5 years from the date of the consignment of the goods ex works, concerning its equipment (excluding disposables, e.g. bulbs, single-use articles, and rechargeable batteries). Date of consignment means that HEINE hands the goods over to the transport carrier, freight forwarder or any other person designated by the Customer for the transport of the goods without loading the collecting vehicle.

The guarantee covers irrevocable workmanship, on condition of the proper use of the equipment and the observation of the operating instructions. During the warranty- and guarantee time period, errors and deficiencies arising on the equipment will be rectified free of charge, in so far as such are evidenced by defective materials, processing and/or constructional errors. Should buyer complain of a material deficiency during the warranty time period, then the onus of proof is always to be on the orderer, that the product was defective already upon receipt of the goods. The statutory warranty and the guarantee do not apply to loss or damage caused by wear and tear, negligent use, the non-employment of original HEINE components and/or spares (in particular bulbs, as these have been especially developed for HEINE instruments in accordance with the following criteria: colour temperature, useful service life, safety, optical quality and performance. The statutory warranty and the guarantee do not apply to interventions by persons not authorised by HEINE or when the operating instructions are not observed by the customer. Any modification of a HEINE product with parts or additional parts which do not conform to the original HEINE specification will invalidate the warranty for the correct function of the product and further invalidate any warranty claims which result from such a change or modification. Further claims, in particular claims for replacement of loss or damage, which are experienced otherwise than directly on the HEINE product itself, are hereby excluded.

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