Invacare® **Softform Active™ 2**  
Hybrid Therapeutic Support Surface  
**Model SRS3080**

**SRS3080M (Mattress)**  
**SRS3080P (Power Unit)**  
**SRS3080PS (Power Supply)**

## User Manual

EN

For more information regarding bed rail safety, refer to the Bed Rail Entrapment Risk Notification Guide at [www.invacare.com](http://www.invacare.com).

This manual **MUST** be given to the user of the product.

**BEFORE** using this product, read this manual and save for future reference.

**Yes, you can.®**

© 2012 Invacare Corporation. All rights reserved. Reproduction, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by ™ and ®. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.

<b>1 GENERAL</b>	<b>4</b>
Symbols .....	4
Limited Warranty .....	5
<b>2 OVERVIEW</b>	<b>6</b>
Label Location.....	6
Typical Product Parameters .....	7
General Description.....	9
<b>3 SAFETY</b>	<b>10</b>
General Guidelines .....	10
<b>4 OPERATION</b>	<b>13</b>
Installing the Side Rails.....	13
Installing the System .....	13
Identifying the Front Panel.....	16
Operating the System .....	17
Language Selection.....	19
Power Fail .....	19
Low Pressure Display.....	19
Preparing for CPR Procedure.....	19
<b>5 MAINTENANCE AND TROUBLESHOOTING</b>	<b>21</b>
Cleaning the System and Infection Control/Decontamination.....	21
Troubleshooting.....	24

### I General

---

#### I.1 Symbols

##### Warnings

Signal words are used in this manual and apply to hazards or unsafe practices which could result in personal injury or property damage. See the information below for definitions of the signal words.

#### **DANGER**

Danger indicates an imminently hazardous situation which, if not avoided, will result in death or serious injury.

#### **WARNING**

Warning indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

#### **CAUTION**

Caution indicates a potentially hazardous situation which, if not avoided, may result in property damage or minor injury or both.

#### **IMPORTANT**

Indicates a hazardous situation that could result in damage to property if it is not avoided.

Gives useful tips, recommendations and information for efficient, trouble-free use.

## 1.2 Limited Warranty

PLEASE NOTE: THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.

This warranty is extended only to the original purchaser who purchases this product when new and unused from Invacare or a dealer. This warranty is not extended to any other person or entity and is not transferable or assignable to any subsequent purchaser or owner. Coverage under this warranty will end upon any such subsequent sale or other transfer of title to any other person.

This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state.

Invacare warrants the cover, foam, power unit and underlay when purchased new and unused to be free from defects in materials and workmanship for the following periods: Cover - 4 years, Foam - 8 years, Power unit - 2 years, and Underlay - 2 years from the date of purchase from Invacare or a dealer, with a copy of the seller's invoice required for coverage under this warranty. Invacare warrants the power unit when purchased new and unused to be free from defects in materials and workmanship for a period of two years from the date of purchase from Invacare or a dealer, with a copy of the seller's invoice required for coverage under this warranty. If within such warranty period any such product shall be proven to be defective, such product shall be repaired or replaced, at Invacare's option, with refurbished or new parts. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of any such product. Product repairs shall not extend this warranty - coverage for repaired product shall end when this limited warranty terminates. Invacare's sole obligation and your exclusive remedy under this warranty shall be limited to such repair and/or replacement.

For warranty service, please contact the dealer from whom you purchased your Invacare product. In the event you do not receive satisfactory warranty service, please write directly to Invacare at the address on the back cover. Provide dealer's name, address, model number, and the date of purchase, indicate nature of the defect and, if the product is serialized, indicate the serial number.

Invacare Corporation will issue a return authorization. The defective unit or parts must be returned for warranty inspection using the serial number, when applicable, as identification within thirty days of return authorization date. DO NOT return products to our factory without our prior consent. C.O.D. shipments will be refused; please prepay shipping charges.

LIMITATIONS AND EXCLUSIONS: THE WARRANTY SHALL NOT APPLY TO PROBLEMS ARISING FROM NORMAL WEAR OR FAILURE TO ADHERE TO THE ENCLOSED INSTRUCTIONS. IN ADDITION, THE FOREGOING WARRANTY SHALL NOT APPLY TO SERIAL NUMBERED PRODUCTS IF THE SERIAL NUMBER HAS BEEN REMOVED OR DEFACED; PRODUCTS SUBJECTED TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE; OR PRODUCTS MODIFIED WITHOUT INVACARE'S EXPRESS WRITTEN CONSENT INCLUDING, BUT NOT LIMITED TO: MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS; PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE; PRODUCTS DAMAGED BY CIRCUMSTANCES BEYOND INVACARE'S CONTROL; PRODUCTS REPAIRED BY ANYONE OTHER THAN AN INVACARE DEALER, SUCH EVALUATION SHALL BE SOLELY DETERMINED BY INVACARE.

THE FOREGOING EXPRESS WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES WHATSOEVER, WHETHER EXPRESS OR IMPLIED, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE AND THE SOLE REMEDY FOR VIOLATIONS OF ANY WARRANTY WHATSOEVER, SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. THE APPLICATION OF ANY IMPLIED WARRANTY WHATSOEVER SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESS WARRANTY PROVIDED HEREIN. INVACARE SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE/PROVINCIAL LAWS AND REQUIREMENTS.

## 2 OVERVIEW

### 2 Overview

#### 2.1 Label Location

**i** Serial Number Label located on back of Power Unit

MODEL **SRS3080P**

Serial No. **Window Adhesive Free**

**ETL** US  
Intertek  
4004040

CONFORMS TO Only use with power supply SRS3080PS  
UL STD 60601-1  
CERTIFIED TO  
CAN/CSA STD C22.2 NO. 601.1  
Power Consumption 12Vdc, 1.2A, 15W

**INVACARE**   **Made in UK**

Distributed by  
Invacare Corporation  
One Invacare Way, Elyria,  
OH 44036  
Phone (800) 832-4707  
www.invacare.com

Manufactured by  
MJS Healthcare Ltd on behalf of  
Invacare UK Operations Ltd,  
Pencoed Technology Park,  
Pencoed, Bridgend,  
CF35 5AQ, UK.  
www.invacare.com

Refer to Typical Product Parameters - **MATTRESS** on page 8 for icon definitions

**▲ WARNING**  
Patient entrapment with bed side rails may cause injury or death. Mattress **MUST** fit bed frame and side rails snugly to prevent patient entrapment. Follow the manufacturer's instructions. Monitor patient frequently. Read and understand the Owner's/Operator's Manual prior to using this equipment. Invacare manuals are available at [www.invacare.com](http://www.invacare.com) or your dealer.

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## 2.2 Typical Product Parameters

**POWER SUPPLY (MODEL SRS3080PS):**

	Class II, Double
	Attention - Consider Accompanying Documents
<b>INPUT VOLTAGE:</b>	100-240 V AC
<b>OUTPUT VOLTAGE:</b>	12V DC
<b>INPUT FREQUENCY:</b>	50 - 60 Hz
<b>OUTPUT CURRENT:</b>	1.25 A
<b>AC/DC ADAPTER TYPE:</b>	FW7660M
<b>ALTERNATIVE AC/DC ADAPTER TYPES:</b>	UEI5WCP, VEPI5US
	Refer to <u>A 10 ft (3m) power lead</u> . on page 9

**POWER UNIT (MODEL SRS3080P):**

	Type B Equipment
<b>INPUT VOLTAGE:</b>	12V DC 15W
<b>CURRENT:</b>	1.25A
<b>POWER SUPPLY:</b>	USE MODEL SRS3080PS
<b>CYCLE TIME</b>	10 minutes
<b>DIMENSIONS (L X W X H):</b>	9.4" x 6.6" x 3.2" (23.5 cm x 17 cm x 8 cm)
<b>WEIGHT:</b>	4.4 lbs (2 kg)
<b>AIR FLOW:</b>	244 cu in/min (4 L/min)
<b>OPERATING PRESSURE</b>	60 mm/Hg
<b>COMPLIANCE:</b>	Conforms to UL STD 60601-1 ETL certified as complying with CAN/CSA STD C22.2 NO. 601.1

## 2 OVERVIEW

### MATTRESS

<b>PART NUMBER</b>	SRS3080M
<b>COVER:</b>	Two-way stretch, vapor permeable, water resistant
<b>FOAM:</b>	Blue - Gr. Rx 39/200 - Single piece Foam 'U' core; Pink - Gr. RX 39/120 - Deep contoured insert pad
<b>SIZE:</b>	80 inches x 36 inches x 6 inches (203 cm x 91.5 cm x 15.2 cm)
<b>WEIGHT:</b>	28.67 lbs (13 kg)
<b>FIRE RETARDENCY:</b>	16 CFR 1632; 16 CFR 1633; TBI29

	DO NOT pierce or cut		DO NOT bleach
	DO NOT dry clean.		User weight limit of 500 lbs (227 kg)
	DO NOT put near flame.		Tumble dry low heat setting
	Wash at 176° F (80° C)		DO NOT iron

### 2.3 General Description

- The Softform Active 2 Hybrid Support Surface is a pressure relieving static mattress that contains an alternating air cell insert.
- The Softform Active 2 Hybrid Support Surface system acts as a static pressure reducing support surface for patients at high risk, that can by facilitating the air power unit, introduce effective alternating pressure if the patient's condition requires alternating pressure therapy.
- The complete system consists of:
  - A mattress insert comprised of alternating inflatable air cells that connect to a twin air connector hose.
  - A micro processor power unit
  - A 10 ft (3m) power lead.

### 3 Safety

---

#### 3.1 General Guidelines

#### **WARNING**

DO NOT use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as owner's manuals, service manuals or instruction sheets supplied with this product or optional equipment. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment - otherwise, injury or damage may occur.

#### **ACCESSORIES WARNINGS**

Invacare products are specifically designed and manufactured for use in conjunction with Invacare accessories. Accessories designed by other manufacturers have not been tested by Invacare and are not recommended for use with Invacare products.

#### **NOTICE**

THE INFORMATION CONTAINED IN THIS DOCUMENT IS SUBJECT TO CHANGE WITHOUT NOTICE.

Check all parts for shipping damage and test before using. In case of damage, DO NOT use. Contact Invacare/Carrier for further instruction.

#### Contraindications

#### **WARNING**

ALWAYS consult the patient's physician before using the Softform Active 2 Hybrid Support Surface.

**Installation****WARNING**

The Softform Active 2 Hybrid Support System is recommended to be installed on medical bed frames with bed side or assist rails. It is preferred that the rails be in the raised position whenever a patient is on the bed. Health care professionals assigned to each case should make the final determination whether side or assist rails are warranted after assessing patient risks of entrapment and falls in accordance with State patient restraint legislation or facility interpretation of such legislation.

Controls on the footboard may be obstructed by the power unit on a few bed frames. It may be necessary to relocate the power unit. Refer to [Installing the System](#) on page 13.

Check that air hoses and power cord are clear of moving bed components before placing a patient on the bed. Operate all bed frame motorized functions through their full range of motion to be certain that there is no pulling, interference or pinching.

**Electrical****DANGER**

Electrical shock hazard. DO NOT remove cover. Refer to qualified service personnel.

**WARNING**

Before performing any maintenance to the power unit, disconnect the power cord from the wall outlet.

DO NOT insert items into any openings of the power unit. Doing so may cause fire or electric shock by shorting the internal components.

The power unit MUST be kept away from all heat sources and radiators during operation.

## 3 SAFETY

### Entrapment May Occur

#### **WARNING**

Refer to the **Bed Rail Entrapment Risk Notification Guide** at [www.invacare.com](http://www.invacare.com) for additional safety information. Mattress **MUST** fit bed frame and side rails snugly to prevent patient entrapment. After any adjustments, repair or service and before use, make sure all attaching hardware is tightened securely.

### Fire Hazard- Smoking

#### **DANGER**

**DO NOT SMOKE** while using this device. A cigarette can burn a hole in the bed surface and cause damage to the mattress. Also, patient clothing, bed sheets, etc. may be combustible and cause a fire. Failure to observe this warning can result in severe fire, property damage and cause physical injury or death.

Smoking by visitors in the room will contaminate the system. Therefore, visitor smoking is **NOT** permitted.

### Fire Hazard - Anesthesia Equipment

#### **DANGER**

There is an explosion risk if used with flammable anesthetics.

### Fire Hazard - Oxygen

#### **DANGER**

There is a possible fire hazard when used with oxygen administering equipment other than nasal mask or half bed tent type. The oxygen tent should **NOT** extend below mattress support level.

## 4 Operation

### 4.1 Installing the Side Rails

#### **WARNING**

Refer to the **Bed Rail Entrapment Risk Notification Guide** at [www.invacare.com](http://www.invacare.com) for additional safety information. Mattress **MUST** fit bed frame and side rails snugly to prevent patient entrapment.

Refer to the instructions provided with the side rails for the installation procedure.

### 4.2 Installing the System

#### Installing the Power Unit

1. Rotate the bed hooks on the back of the power unit into position.
2. Place the power unit on the footboard.

If the bed does not have a footboard, place the power unit on a flat surface, leaving room for the hose to hang down.

**FIGURE 1** Installing the System

## 4 OPERATION

### Connecting the Hose


#### **CAUTION**

Ensure that the hose connecting the power unit to the mattress is routed such that it cannot be stepped on, kinked, squeezed or otherwise damaged.

1. Connect the “Active” twin hoses to the power unit connectors.
2. Push the hose connectors onto the control connectors until an audible click is heard.

The audible click indicates that the hose connectors are properly engaged with the control connectors.

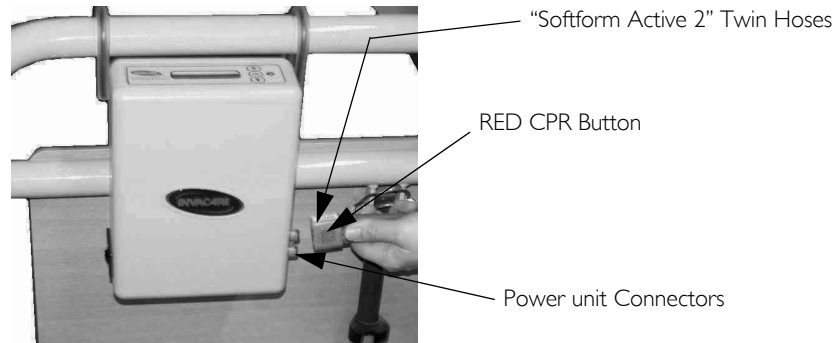**FIGURE 2** Connecting the Hose

## Connecting the Power Supply (SRS3080PS)

### CAUTION

Ensure that the power cord of the power unit is not pinched, or has any objects placed on it, and also ensure it is not located where it can be stepped on or tripped over.

For this procedure, refer to FIGURE 3 on page 16.

1. Examine the power supply (SRS3080PS) cord supplied with the power unit.
2. Perform one of the following:
  - If the power cord is damaged - Call your supplier for a replacement power supply, ref SRS3080PS (UE15WCPC, VEPI5US, FW7660M).
  - If the power cord is not damaged - Plug the end of the supplied power supply into the power outlet on the side of the power unit (Detail “B”).
3. Power supply is 12 volt transformer and does not require ground where it connects to outlet on the wall.
4. Switch on the main power and the power unit (Detail “C”).

Once the power is turned on, the system will start to pressurize, indicated by a green LED (System in Cycle) and three audible beeps. Once operating pressure is reached the green light will be permanently illuminated and the power unit will emit two short beeps.

To greatly reduce shearing and friction and to prolong the life of the mattress, Invacare recommends the use of the auto-contour feature when positioning the patient to a seating position. Also, use the auto-contour feature when repositioning the patient to a lying down position.

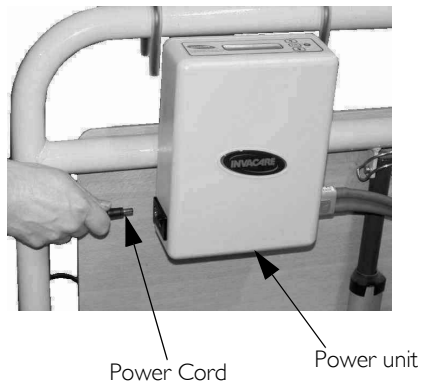
The external power supply or adapter is used as the disconnect device and shall be readily operable.

## 4 OPERATION


### Detail “A”

### Detail “B”


### Detail “C”

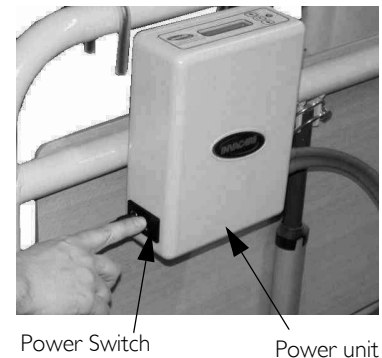**FIGURE 3** Connecting the Power Supply (SRS3080PS)

### 4.3 Identifying the Front Panel
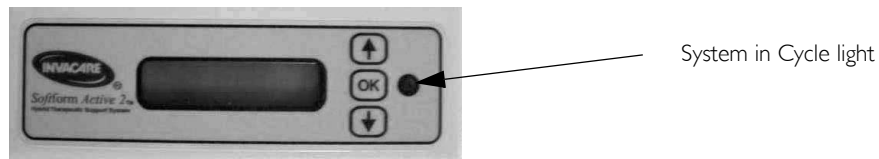
## 4.4 Operating the System

### Powering Up System

- I. Switch power unit on **I/O** to power up system.
  - A. The system will sound three audible beeps and the green light illuminates, LCD illuminates with “System Set Up” displayed.

- B. After five to ten minutes the LCD will display “System Ok”.

No other action is required. System will begin alternating pressure in 60 mm/hg pressure range.

If the user desires to change the alternating pressure range or system language, follow the steps in [Entering Menu Mode](#) below.

### Entering Menu Mode

There is a 3 second delay between each function.

Power unit display will return to “System Ok” if no selection is made within 15 seconds.

The sequence of STEPS 1 and 2 are required to prevent accidental lockout.

1. Press arrow up and display shows “System OK” with “1” underneath.
2. Press arrow down and display shows “System OK” with “2” underneath.

## 4 OPERATION

3. Press OK and display shows “Pressure”.

### Pressure Adjustment

**i** Default setting is at 60mmHg

1. Press OK to select Pressure.
2. Perform one of the following:
  - Press Arrow Up to select pressure (60, 70, 80).
  - Press Arrow Down to select pressure (60, 50, 40).

**i** System will show new pressure setting.

#### 60, 70, 80 Pressure

#### 60, 50, 40 Pressure

#### 4.5 Language Selection

1. Navigate to Pressure selection. Refer to [Entering Menu Mode](#) on page 17.
2. Once at Pressure selection, toggle up or down (using arrow buttons) and “Language” is displayed.
3. Once at “Language” selection, toggle up or down (using arrow buttons) to desired language.

 System settings will be shown in selected language.

#### 4.6 Power Fail

In the event of power outage, the continuous audible alarm sounds. The power unit has internal memory and retains the previous settings during the power outage.

 If at any time there is a power failure, a constant audible beep will sound. Power needs to be re-established for power unit to function correctly and will run through the set up again.  
If power cannot be re-established the alarm may be muted by switching off power switch.

#### 4.7 Low Pressure Display

 In the event of pressure loss, an audible intermittent beep shall sound and the LCD will display low pressure. The alarm may be muted by switching off power switch.

#### 4.8 Preparing for CPR Procedure

 The Softform Active 2 Hybrid Support Surface has been fully tested to comply with the current CPR standard of 4-5 cm compression depth. This was achieved at all stages of inflation/deflation. A report of these findings conducted by the Resuscitation unit at the University Hospital of Wales, Cardiff is available upon request.

## 4 OPERATION

1. Switch off the power unit.
2. In the event of CPR procedures becoming necessary, disconnect the hose connector from the power unit by pressing and holding down the red CPR button then pulling firmly away from the power unit and the mattress will begin to deflate.
3. When CPR is complete, reactivate the system following the instructions in [Installing the System](#) on page 13.

---

## 5 Maintenance and Troubleshooting

---

### 5.1 Cleaning the System and Infection Control/Decontamination

#### **WARNING**

Before cleaning or disassembling the Softform Active 2 Hybrid Support Surface, check the underside of the mattress folds for sharp objects such as scissors, needles, etc. These objects should be removed and discarded before proceeding with further cleaning or disassembly; otherwise injury or damage to the product may occur.

Because of potential risk of infectious exposure, cleaning with the patient on the bed is not recommended.

All equipment should be inspected. Any item that is visibly soiled with the patient's blood or other body fluids should be properly cleaned or removed.

Laundry workers should treat all soiled bedding as if it were contaminated with pathogenic microorganisms.

ALWAYS use universal precautions, especially where the product is known or suspected to be contaminated. ALWAYS refer to your infection control manual for advice.

DO NOT use alcohols or phenols.

ALWAYS ensure the main supply has been disconnected.

Avoid getting the power unit wet.

#### **CAUTION**

To prevent accidental cover damage, DO NOT place hypodermic needles, venflons, scalpels or other similarly sharp objects onto the mattress.

DO NOT use phenol's or alcohols.

1. Remove the bedding.
2. Ensure that the power unit is off.
3. Unplug the power supply from the wall outlet.

## 5 MAINTENANCE AND TROUBLESHOOTING

4. Ensure that the underside of the mattress is clear of all sharp objects.
5. Examine the surface of the power unit and mattress assembly components for visible blood or body fluids.
6. Perform one of the following:
  - If blood is present, decontaminate the product.
    - i. Remove all visible soil with disposable paper towels.
    - ii. Scrub the area with freshly prepared effective detergent disinfectant solution.
  - If blood is not present, remove any soil from the cover with paper towels.
7. Perform the following steps to clean the mattress cover:
  - A. Using a damp single use wipe, scrub the area with a solution of neutral detergent and hot water.
  - B. Rinse cover with clean water using a damp single use wipe.
  - C. Thoroughly dry cover.
  - D. Remove the cover and launder it using the following method:

If soiled, the cover should be removed, cleaned and decontaminated.

### **CAUTION**

Laundry workers should always wash their hands before working with clean bedding.

Do NOT overload the machine.

Do not use chlorine bleach because it may damage the fabric coating.

High air temperatures will damage the fabrics and void the Invacare warranty.

- i. Place the cover in a washing machine.
- ii. Wash with warm water below 176°F (80°C).
- iii. Add diluted detergent and disinfectant according to the manufacturer's instructions.
- iv. Remove excess water.

## 5 MAINTENANCE AND TROUBLESHOOTING

- v. Cover should be thoroughly dried before placing back on the product.
  - vi. Line dry cover completely before using.
8. Perform the following steps to clean the power unit and hose fittings:
- A. Wipe all controls, chassis and hose fittings with a quaternary disinfectant solution.
  - B. Gently clean all crevices as they can harbor microorganisms.
  - C. Air dry all treated surfaces.
9. Perform the following steps to clean the mattress:
- A. Using a clean sponge or paper towel, wipe down the mattress with a dilute detergent solution of quaternary cleaner disinfectant or other germicidal detergent solution.
  - B. Wipe the mattress with a clean dry cloth.
-  The active insert may also be wiped down using a solution of neutral detergent and hot water, applied with a single use wipe. The power unit, tubing and main lead may be wiped over with a neutral detergent.

### Disinfection

Ensure that any residual detergent has been removed prior to disinfection. A 0.1% chloride solution (1,000 ppm) can be used. Alternatively, where required, a 1% chloride solution (10,000 ppm) can be used. When covers are immersed in solutions of hypochloride, contact should be maintained for at least 30 minutes. Ensure that the cover is rinsed with clean water using a damp single use wipe, and thoroughly dried. Where combined cleaning and disinfection products are used, ensure that the cover is rinsed with clean water using a damp single use wipe, and thoroughly dried.

## 5 MAINTENANCE AND TROUBLESHOOTING

### 5.2 Troubleshooting

#### **DANGER**

DO NOT attempt to dismantle the power unit. Failure to comply will invalidate the warranty and may result in death or serious injury.

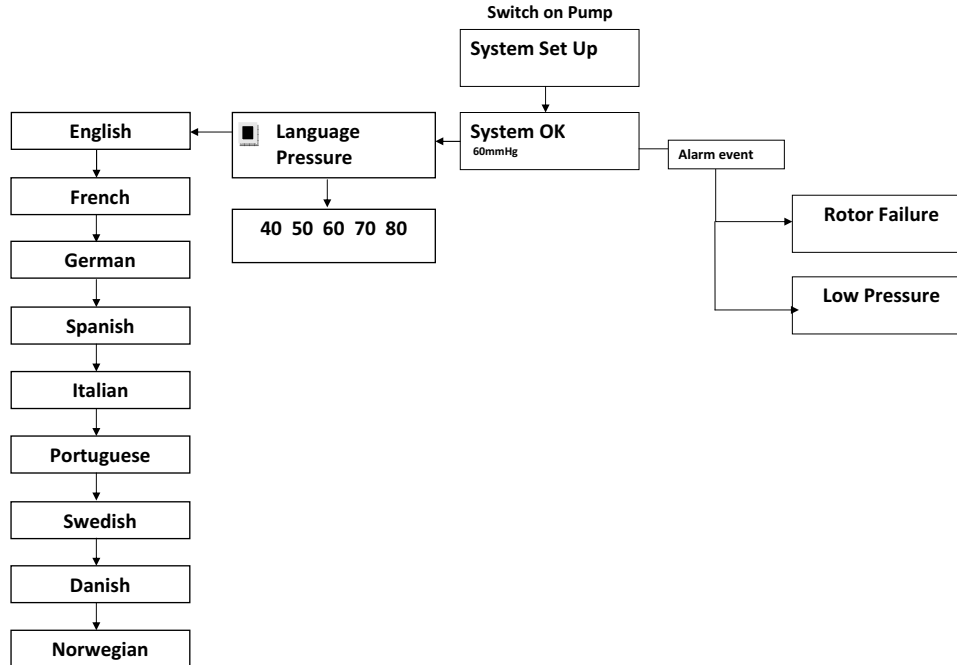
PROBLEM	CAUSE	SOLUTION
Mattress not inflating Not alternating properly	Mattress hose disconnected	Connect hose connectors and lock them in place
	Air hose kinked or split	Unkink hose or replace split hose
	Major leak in the air cushions	Replace leaking air cells.
	Has power, however, power unit does not come on	Send power unit back to factory for repair
	Not alternating, solenoid malfunction	Send power unit back to factory for repair
	No air, power unit malfunction	Send unit for repair
No power (Turn off power switch to stop the beep alarm.)	Power unit off	Check power source and turn on unit
	Power cord disconnected	Connect power cord to the power source
	No power in the power source	Check power source has power and turn on unit.
	Power outage	Wait till the power source has power
No Audible Alarm	Rechargeable Battery Defective	Fit new alarm PCB *NB Battery Not serviceable
Can't Enter Menu	System not ready	Wait until set up is complete and "System Ok" is displayed.
Low Pressure Alarm. "Low Pressure" displayed with audible beeps	Disconnection of CPR or mattress hose disconnected	Ensure connections are made and secured.

## 5 MAINTENANCE AND TROUBLESHOOTING

PROBLEM	CAUSE	SOLUTION
Rotor Failure	Power unit Failure	Return unit to manufacturer

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