This manual contains important information about this product.
Please pass it to the final user upon delivery.

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ENGLISH

Physipro inc. is delighted to have you among its clients. We sincerely wish to thank you for the trust you have shown in our Company by choosing one of our products. This user manual has been designed to allow the client to use the Valeo Backrest, a Physipro inc. product, safely in an optimal way. For all adjustments and settings, Physipro inc. strongly recommends you consult a qualified professional. Physipro inc. provides you with specific verifications you should perform on a regular basis in order to optimize the performance of your Valeo Backrest and extend its operational life. Physipro inc. relieves itself from any liability should any bodily injury or property damage result from a lack of care or misuse or from any modification made to the product without prior written consent of the Company.

For Physipro inc., your satisfaction remains a priority.

Table of content

1. Main components of the Valeo backrest	3
2. Foam composition and optional components	3
3. Specifications	4
4. Recommendations	5
4.1 Usage verification	5
4.2 Safety rules	5
5. Backrest Installation	6
5.1 Pre-assembly	6
5.2 Removable Mounting Hardware (optional)	6
5.3 Backrest Adjustments	6
5.4 Function	7
5.5 Backrest Installation	8
5.6 Backrest Angle Adjustment	9
6. Backrest components installation	11
6.1 Adjustable Laterals	11
6.2 Laterals mount on the backpost	13
6.3 Profiled Adjustable Laterals	14
6.4 Headrest support	16
7. Care and maintenance	17
8. Warranty	18
Clickit Fixit	19

1. Main components of the Valeo backrest

The backrest plate is offered in two different thickness levels: standard (3/32") and heavy duty (1/8"). The mounting hardware is mounted on the backposts which enable a height setting.

Many options are available depending on the customer's needs. Figure 1 shows the backrest's components.

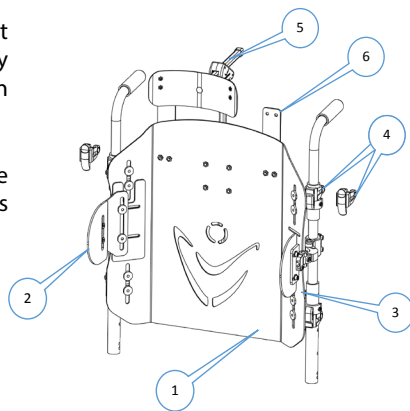
Figure 1

1. Backrest plate (2 thickness choices)
2. Integrated adjustable laterals
3. Adjustable laterals on backposts
4. Stabilizers (fixed and removable)
5. Headrest support
6. Harness adjustment device

2. Foam composition and optional components

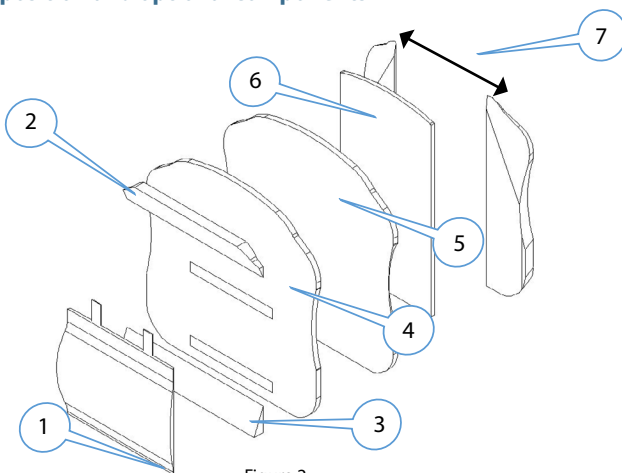
Figure 2

- | | |
|-------------------------------------|-----------------------------------|
| 1. Sacrolumbar corrector (optional) | 5. Urethane (Neocor)* |
| 2. Thoracic wedge (optional) | 6. Aerolite |
| 3. Lumbar wedge (optional) | 7. Thoracolumbar wedge (optional) |

* Depends on the chosen configuration.

ENGLISH

3. Specifications

Description	Values
Nominal widths*	10", 12", 14", 16", 18", 20"
Nominal heights**	10", 12", 14", 16", 18", 20"
Backrest plate material	Aluminum
Backrest plate thickness	Standard : 3/32" Heavy Duty : 1/8"
Mounting Hardware	Fixed or removable. (mounted on the backposts)
Laterals height Cushioning included	4", 6", 7", 8", 10"
Weight	3 lb (16" x 18" - 3/32" thicknesses)

* Real width = Nominal width - 1"

** Real height = Nominal height

Each Valeo backrest size can be installed on two different wheelchair widths. In order to choose the right backrest size, it is important to know the wheelchair width and the user's thoracic width. Dimensions table:

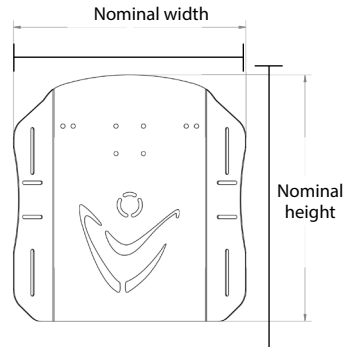
Figure 3

Nominal width available	Nominal height available	Integrated laterals adjustment	Adjustment of the laterals backpost	Associated wheelchair width
10"	10", 12"	5" to 8"	3" to 10"	10" and 11"
12"	12", 14"	7" to 10"	5" to 12"	12" and 13"
14"	14", 16"	9" to 12"	7" to 14"	14" and 15"
16"	16", 18", 20"	11" to 14"	9" to 16"	16" and 17"
18"	16", 18", 20"	13" to 16"	11" to 18"	18" and 19"
20"	16", 18", 20"	15" to 18"	13" to 20"	20" and 21"

4. Recommendations

Do not use this equipment without having read and understood this user manual in its entirety. This manual includes critical information intended to ensure the safety of the user and the individuals who handle the backrest.

4.1 Usage verification

As soon as the client receives the Valeo backrest, verifications have to be made to ensure a safe usage :

- ✓ Check if the laterals are well adjusted and fixed on the backrest;
- ✓ Check if the mounting hardware are adjusted at the right height and fixed on the backposts;
- ✓ Check if any other accessory is suitably adjusted and firmly fixed.

4.2 Safety Rules

Several safety measures must be applied to ensure the safety of the user. Note that the following list is not comprehensive. Therefore, the people who use the Valeo backrest have the responsibility to use it cautiously:

- ✓ Never neglect the mounting hardware, or else, make sure the mounting hardware are closed before using the wheelchair;
- ✓ Do not use the Valeo backrest to push or lift the wheelchair;
- ✓ Do not install the Valeo backrest on a wheelchair that is not built for it;
- ✓ Regularly check the integrity of the backrest, the backposts and the fasteners.

In case of spasticity that could damage the wheelchair and the Valeo hardware, a dynamic backrest is recommended.

ENGLISH

5 Backrest Installation

5.1 Tools necessary for installation

Tool Chart	
C	4 mm Allen Key
J	
M	2x Flat-head screwdriver
N	
O	4 mm and 10 mm Allen Key

5.2 Pre-Assembly

Use screw **C** and nuts **D** to assemble the backrest supports **A** and **E** with the backrest cane holders **B**. At this stage, secure lightly, as the adjustment will be made later. The 5° adjustment angle assembly has to be installed on top of the backrest.

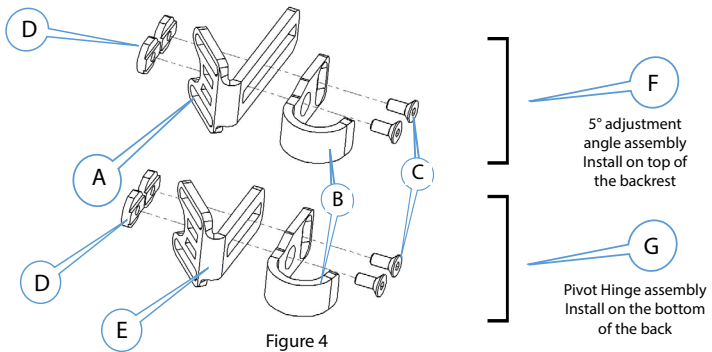
Figure 4

5.3 Hardware installation

Using screws **J**, washers **I** and nuts **D**, install the hardware system **F** and **G** on the back of the backrest. At this stage, secure lightly, as the adjustment will be made later.

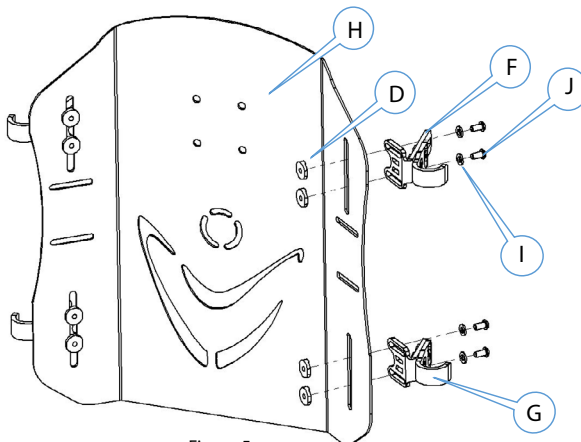
Figure 5

5.4 Stabilizers installation

5.4.1 Removable Stabilizers (optional)

The removable stabilizers are designed to be installed on top of the backrest to facilitate installing and removing the VALEO Backrest.

To close the stabilizer, push in on the release clip 1, to open the stabilizer push in on the release clip 2, as shown in figure 6.

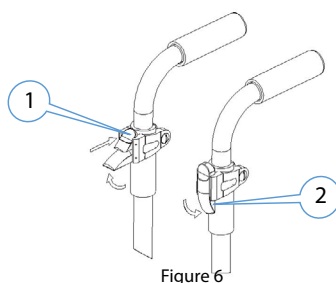
Figure 6

5.4.3 Stabilizer installation

Use the hardware shown in figure 7 to install the stabilizers. Place the plastic spacer **K** (split plastic tube) using the small part of the backpost (Figure 8). Keep it on the top of the backpost for now. Position the stabilizer **M** and **N** on the backposts **L** using the small part of the backpost with the peak pointing down for the high stabilizer and the peak pointing up for the low stabilizer (Figure 9). Slide the high stabilizer **M** on the plastic spacer and the low stabilizer **N** on the lower structure of the backpost (Figure 10). Install the hardware to fix the stabilizer. At this stage, secure lightly, as the adjustment will be made later.

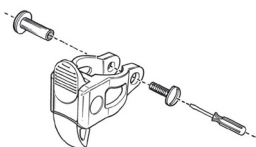
Figure 7

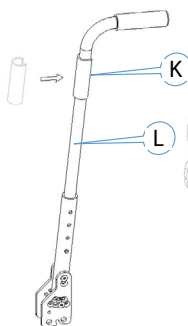
Figure 8

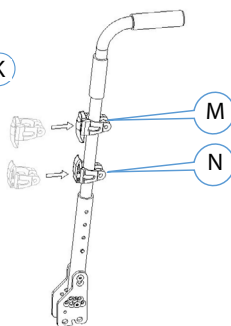
Figure 9

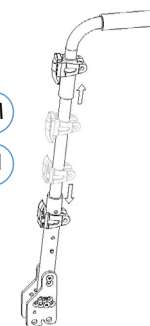
Figure 10

Ensure the mounting hardware are secure when adjustments are made before using the backrest, failure to follow these suggestions may result in serious bodily injury to the user and damage to the wheelchair.

Make sure you hear the locking click sound when you set the backrest in the stabilizers.

ENGLISH

5.5 Backrest installation and adjustment

5.5.1 Backrest installation

Place the backrest with the stabilizer on the backposts. Close the stabilizers. Ensure that the backrest is secure on the backposts.

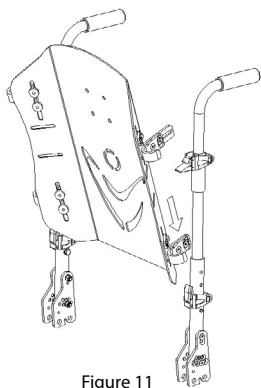
Figure 11


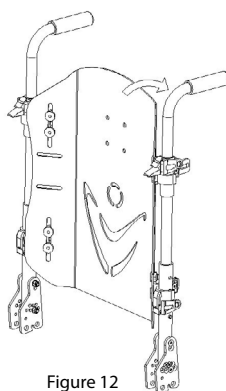
Figure 12


5.5.2 Backrest adjustment

Slide the support **G** under the peak of the stabilizer **N**. Position the supports on each side and slightly secure screws **C** and **J** (the final adjustments will be made later).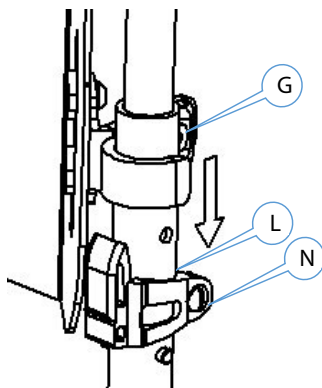

Front view

Figure 13

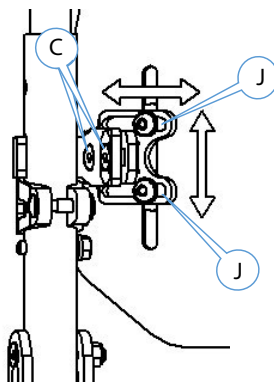Back View

Figure 14

For the user's safety, this height adjustment should not be made when the user is sitting in the wheelchair.

5.5.3 Adjustment of the high stabilizer

Continue the adjustment with the high stabilizer **N** by keeping a space of about 1/8" between **N** and the stabilizer **M**. Position the stabilizer on each side and slightly tighten screws **C** and **J** (The final adjustments will be made later).


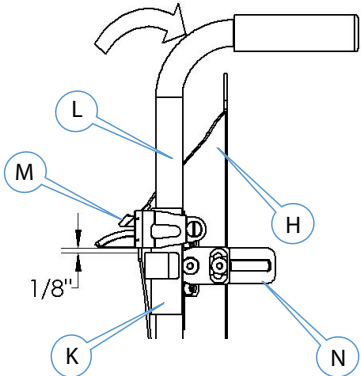
Figure 15

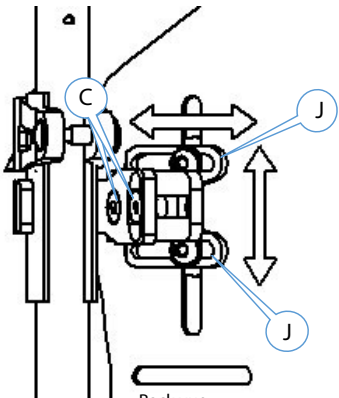Back vue
Figure 16

5.6 Adjustment backrest inclination

5.6.1 To select the desired backposts angle

Determine the desired angle for the user referring to the following chart. The angle is adjustable by 5° increments.

Adjust the backposts to the desired angle (5° increments) then adjust the backrest from 0° to 5° forward. Combined with the backpost, the backrest can be adjusted from -10° to 25° continuously.

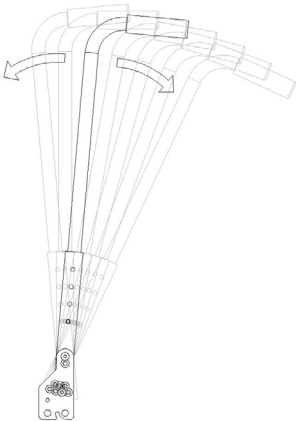
Figure 17

Backrest angle configurations		
Tilt angle		
Backposts	Valeo Backrest	Actual angle
85°	-5° to 0°	80° to 85°
90°	-5° to 0°	85° to 90°
95°	-5° to 0°	90° to 95°
100°	-5° to 0°	95° to 100°
105°	-5° to 0°	100° to 105°
110°	-5° to 0°	105° to 110°
115°	-5° to 0°	110° to 115°

ENGLISH

5.6.2 Position the backposts

Unscrew and remove the adjustment screw **O**. Based on the desired tilt angle and the appropriate hole, tilt the backpost **L**. Reposition screw **O** in the plate and the backpost by tightening on each side.

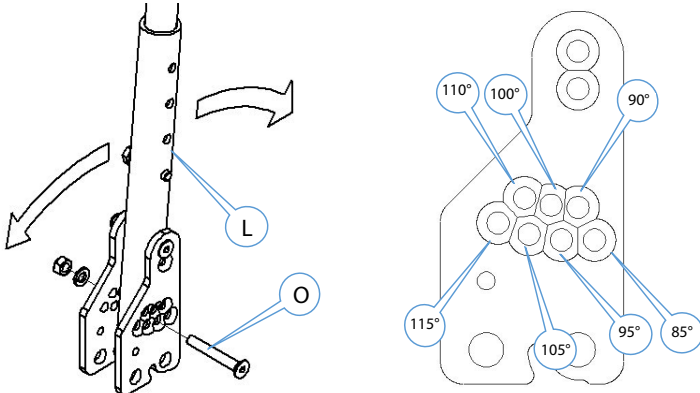
Figure 18

5.6.3 Position the backrest angle

Adjust the backrest angle from 0° to 5° forward as needed in order to obtain a precise degree. Unscrew screw **C** and tighten securely after adjusting the angle.

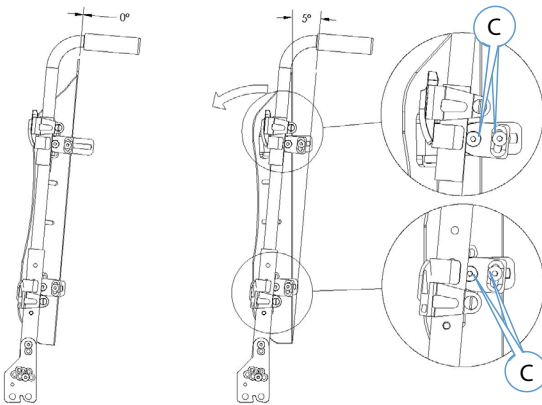
Figure 19

5.6.4 Final adjustment of the backrest

Retighten all screws **J** and the stabilizer screws. For the removable option, make sure it works properly by removing and replacing the backrest. Adjust the stabilizer by loosening and tightening as needed.

At this stage, check if all the bolts are secure.


6 Components Installation

6.1 Adjustable Laterals

Using double sided tape, stick the plate **A** on the backrest **B**. Pay attention to align it with the grooves. Position the laterals as shown in Figure 20.

Position plate **F** and plate **G** as shown. Assemble the laterals using bolts M6 x 4 mm **C**, flat washer **D** and nuts **E** provided.

At this stage, secure lightly, as the adjustment will be made later.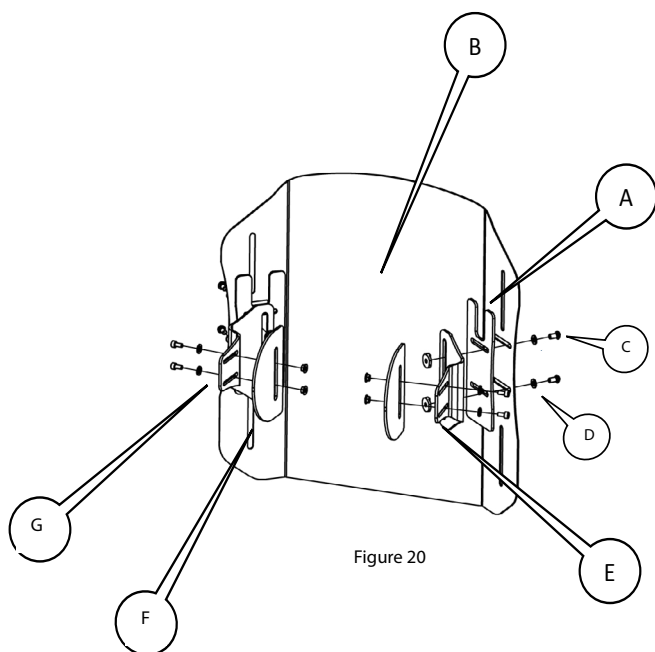

ENGLISH

6.1.1 Adjustments

The distance **(A)** between the laterals can be adjusted as in the following table.:

Dimensions (Backrest size)	Laterals adjustment
10"	3.5" to 6.5"
12"	5.5" to 8.5"
14"	7.5" to 10.5"
16"	9.5" to 12.5"
18"	11.5" to 14.5"
20"	13.5" to 16.5"

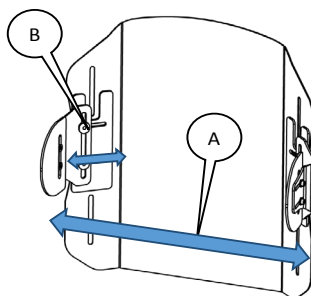
Figure 21

Loosen the bolts **(B)** to adjust the width. Secure when adjustment is completed. Note that this adjustment can also change the lateral height in ± 1 " range.

The bolts **(A)** can adjust the height and depth of the laterals as shown on Figure 22.

Loosen the bolts to change the setting. Make sure to fix the bolts back.


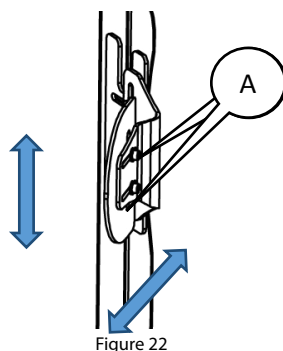
Figure 22

The next table shows possible height setting, depending on the choice.

	Pediatric	X-small	Small	Medium	Large
					
Plate height setting	—	1 ½"	1 ½"	2 ½"	2 ½"
Depth	4"	4 ½"	4 ½"	4 ½" 5 ½"	4 ½" 5 ½"

6.2 Laterals mount on the backpost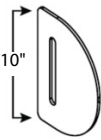

These laterals are not compatible with the detachable option of the Valeo back. This lateral model interferes with the back when it is being detached.

Place the clamps at the desired height on the wheelchair cane.

Assemble the bolts (**A**)*, the half cylinder (**B**) and the column nut (**C**) as shown in Figure 13. Tighten slightly at this stage to make adjustments possible. Make sure the bolts (**D** and **E**) and the lateral plate bolts (**F**) are not too tight to be able to adjust at will. Position the lateral using the 2 pivot points (**G**) and sliding the clamps on the wheelchair cane for the height setting.

Secure the bolts for final adjustments. For the height setting range when using laterals plate, refer to the section 5.1.

Note: 2 screw lengths are included to use with $\frac{3}{4}$ " and 1" tubes.

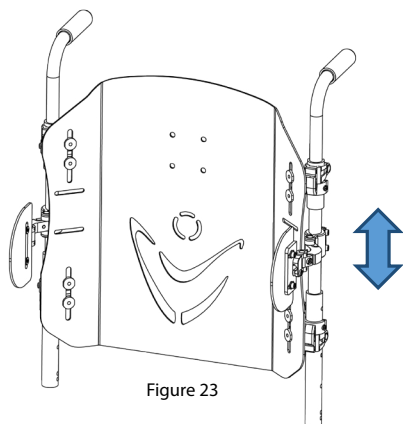
Figure 23

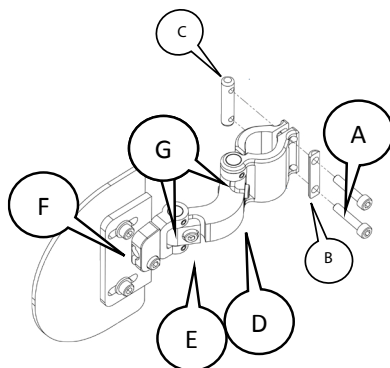
Figure 24

ENGLISH

6.3 Profiled laterals

6.3.1 Installation

Make sure a Velcro strip is fixed on the lateral plate facing the inside. (Figure 25)

Insert your hand in the cushion through the cover opening to prevent the Velcro strips from touching each other while you place the cushion. (Figure 26)

Start inserting the cushion by the top of the lateral. (Figure 27)

Push down while making sure the cover opening passes around the plate by making a downward rotating movement. (Figure 28)

Place the cushion, remove your hand from the cover and push a few times on the front of the cushion to make sure the Velcro strips are hooked. (Figure 29)

Figure 25

Figure 26

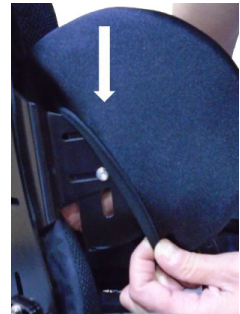
Figure 27

Figure 28

Figure 29

6.3.2 Removing the profiled cushion

Insert your hand inside the profiled cushion between the plate and the cushion to unhook the Velcro strips.

Move the cover opening down the lateral plate and remove the cushion with an upward rotating movement.

6.3.3 Removing the foam from the profiled cushion

Fold the cushion in the cover. (Figure 31)

Carefully and slowly take the cushion out of the cover opening. (Figure 32)

Completely remove the cushion. (Figure 33)

Figure 30

Figure 31

Figure 32

Figure 33

ENGLISH

6.4 Headrest support

Install the fixing block **(A)** with the four bolts **(B)** and nuts directly included in the backrest by inserting a wedge in between **(C)** as shown in figure 25. Secure the bolts well.

Insert the headrest support rod **(D)** in the fixing block. Secure the headrest rod at the desired adjustment with the knob **(E)** and adjust the positioning stop **(F)**.

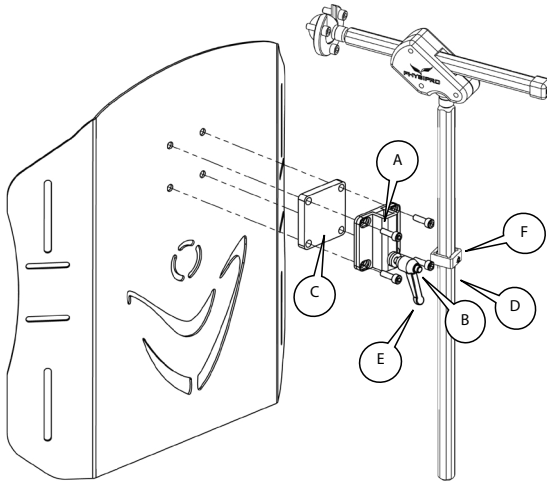
Figure 25

7. Care and maintenance

Valeo backrest maintenance is essential. By performing the maintenance in the recommended way, you will maximize its life cycle and ensure its safety of use. Do not forget to have your backrest checked up by a certified professional at least twice a year. Eliminate all stains (food and/or biological) as fast as possible.

Check list	When receiving	Each week	Each month	Every six month
Backrest Observe the condition of the aluminum;	x		x	
Anchors Make sure the receptacles and the universal clamps of the backrest are solid; Make sure the anchor slides and latches work well.	x x	x x		
Laterals Make sure that the laterals are fastened properly;	x	x		
Clean up Clean up the jointing.				x

ENGLISH

8. Warranty

The Valeo backrest, a product of Physipro inc., is covered with our twenty-four (24) months manufacturer's warranty. It includes parts and labor on manufacturer's defects, optional parts, and all new parts replaced by your dealer or an authorized establishment from the date of delivery or replacement.

Physipro inc. will repair or replace the defective parts as long as the warranty is valid. You will have to contact Physipro inc. or an authorized dealer to obtain your warranty services. Do not return your backrest without prior consent. If you are not satisfied with the repairs, please send us a written complaint with your dealer's name, address and delivery date to the address on this manual cover.

Exclusions and limitations

This warranty does not apply to products that were intentionally or accidentally altered, misused, neglected or did not have the proper maintenance. Furthermore, this warranty does not apply if repairs or modifications were done without a written consent from Physipro inc., or an authorized dealer. The same restrictions apply if any other kind of damages were done without Physipro inc. involvement. This warranty will not apply for everyday use or failure to respect the rules in this manual.

Physipro inc. will not take any responsibilities for damages incurred during shipping.


Terms and Conditions

The seat cushions and backrests are reserved exclusively for use on mobility aid (manual or power wheelchairs) Physipro will not be held responsible for any misuse of the product. Crash tests have not been conducted on these products and even if the mobility aid in which it is installed has obtained the ISO-717619, it will not be maintained.


The respect of the safety instructions mentioned in this manual is essential to avoid injury. Improper or inadequate adjustment increases the risk of accident. Before making adjustments, it is important to read the entire contents of this manual. Physipro will not be held responsible for misuse or non professional adjustment.

The product presented in this manual may not be an exact representation of the product you have on hand. The instructions in this manual are still valid for your product. Physipro reserves the right to modify or change this manual without notice.

CLICKIT® FIXIT™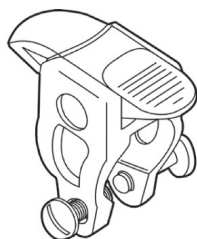

en For more information on Bodypoint products, and a list of distributors outside the USA go to www.bodypoint.com.

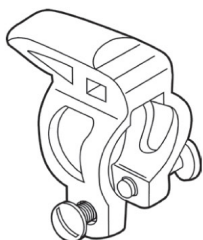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


ENGLISH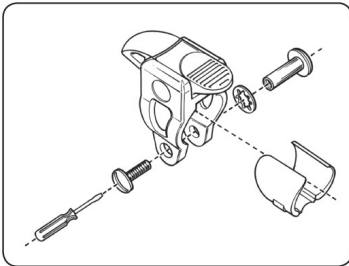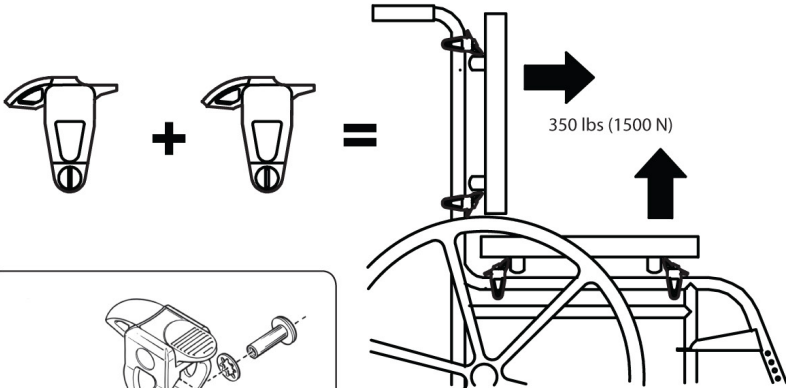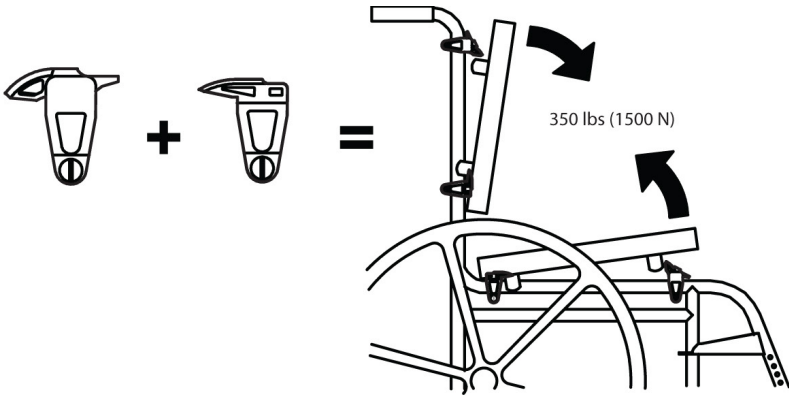

en WARRANTY

This product carries a limited lifetime warranty against defects in workmanship and materials arising under normal use by the original consumer. Contact your supplier or Bodypoint for warranty claims.

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MDSS GmbH