Positioning in bed

Owners manual

This manual contains important informations about this product.
Please pass it to the final user upon delivery.

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

Positioning in bed


Prevention of bedsores on the sacrum, ischium and the heels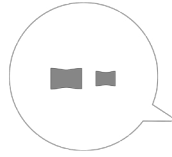


Effective solution for everyday use (All purpose cushions).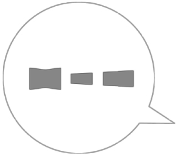


Prevention of bedsores on the elbows and heels.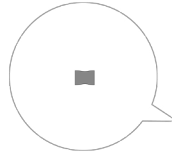

Prevention of bedsores on the sacrum, ischium and heels.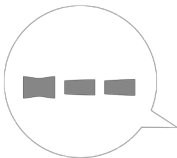

Prevention of bedsores on the heels.

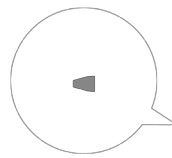Controls lower-limb adduction.

Prevention of bedsores on the sacrum, ischium and the heels.

Prevention of bedsores on the trochanter area.

Prevention of bedsores on the ankles, sacrum and knees.

	LI8410	Circular Cushion
	LI8310	Knee Abduction Cushion
	LI8320	Multi Purpose Cushion (Small)
	LI8330	Multi Purpose Cushion (Medium)
	LI8340	Cylinder Cushion
	LI8350	Positioning Cushion for Heel
	LI8360	Positioning Cushion for Hand
	LI8370	Hip Abduction Cushion
	LI8400	30° Lateral Positioning Cushion < 5,4 pieds (165 cm)
	LI8380	30° Lateral Positioning Cushion > 5,4 pieds (165 cm)
	LI8390	Positioning Cushion
	LI8420	Transport bag

ENGLISH

Maintenance & cleaning

Regular cleaning and maintenance may help extend the life of your cushion.

Components:

To clean, wipe surfaces down with soap and slightly damp cloth only; never use solvents. Do not submerge in water.

Disposal of Medical Devices

Do not burn and do not throw away the cushion and its protective cover outside of designated areas.

Storing

The positioning in bed cushions must be stored on a flat surface. It should, be stored protected from humidity, light and excessive heat.

Warranty

The bed positioning is guaranteed to be free from defects in materials and workmanship for a period of 12 months. This warranty covers original defects in material and workmanship of the cushion components, optional components and new parts that have been replaced by the supplier or by an authorized retailer, with effect from the delivery date or, if applicable, from the date of replacement. This warranty does not apply to punctures, tears or burns, nor to the cover.

Physipro inc. agrees to repair or replace any inoperative or defective part for the duration of the warranty period. To enjoy this warranty service, please contact Physipro inc. or an authorized retailer. Do not return this product without prior approval. If you are dissatisfied with this product, send us your comments at the address provided therein, and include the name and address of the supplier and the delivery date of the product.

Furthermore, this warranty shall not apply to products that have been damaged through repairs or modifications made without the prior written consent of Physipro inc. or of an authorized retailer. The same restrictions shall apply for damages resulting from other circumstances beyond the control of Physipro inc.

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