EP-AMP

Sound Amplifier

The SPYPOINT EP-AMP sound amplifier increases low sounds like animal sounds, leaf noises and voices. It is perfect for hunting, outdoors, sporting events, conferences, theater performances, watching TV and more.

FEATURES

- Amplifies sound up to 50 dB
- Discreet, lightweight and small
- Capture sounds up to 100ft
- Adjustable volume
- Flexible and adjustable to all ears
- Soft tips helps insure comfortable fit
- Included: 3 AG3/LR-41 alkaline button batteries and 3 silicone rubber ear tips with different diameters

COMPONENTS

POWER

The sound amplifier can be powered by an AG3/LR-41 alkaline button battery. The life of this battery is about 10 hours of continuous use. To prolong battery life, turn off your EP-AMP when not in use and for an optimal performance, replace battery on a regular basis. You have the option to use a 312D zinc air hearing aid battery as an alternative power source. This battery may provide longer battery life. Insert the battery and be sure that its positive post (+: flat end) is facing up; it's clearly marked on the bottom of its battery compartment. **DO NOT THROW USED BATTERIES IN GARBAGE.**

INSTALLING THE SILICONE RUBBER EAR TIP

Place a silicone rubber ear tip on the receiver end, point towards the front of the device. (see figure below) If the silicone rubber ear tip you chose is too long, you may slightly cut it down the same angle.

INTERRUPTOR ON/OFF

Allows turning on/off the sound amplifier.

KNOB TO ADJUST THE VOLUME

Allows you to adjust the volume. For best results, we suggest you start using the device with volume on low and gradually increase according to your preference. Do not set the volume too high, may cause excessive feedback and pose health risk.

WEARING THE DEVICE

Hold the device horizontally and insert into your ear canal. Press slightly to obtain a comfortable fit. Do not attempt twisting or wiggling to insert the device into your ear, these motions will dislocate the ear tip to cause poor sound reception. The following image shows the device properly installed in the left ear. It is possible that adjusting volume and sound quality differs depending the ear. First try the device in the left ear and then the right to determine which works better for you.

MAINTENANCE/ USE

To prolong life of your sound amplifier, cleaning your device daily is important:

1. With a soft bristled brush, brush gently the receiver end to remove all the wax and dirt that may have adhered.
2. Wipe the case with damp cloth only. Do not use liquid or aerosol cleaners to clean the device.
3. When your not using your sound amplifier, take the battery out of the device and store it in a dry cool place. **KEEP OUT OF REACH OF CHILDREN.**

⚠ **Do not place the device in contact with water.**

⚠ **Do not use this device as a hearing aid.**

⚠ **Does not decrease violent sounds and harmful noises (like gun shots).**

LIMITED WARRANTY

SPYPOINT EP-AMP designed by GG Telecom, is covered by a one (1) year warranty on material and workmanship starting from its original date of purchase. **The sales receipt is your proof of purchase and must be presented if warranty service is needed.** This warranty does not cover any GG Telecom product which has been subjected to misuse, neglect and accidents or has been improperly used or maintained. Any modification or tampering of the product will affect its operation, performance, durability and void this warranty.

REPAIR SERVICE

Repairs for damages not covered by the warranty will be subject to a reasonable charge. For technical assistance, write to tech@spypoint.com. Give a description of the problem with a phone number where you can be reached. **IMPORTANT:** Under no circumstances GG Telecom will accept returns without a Return Material Authorization number (RMA).

WWW.SPYPPOINT.COM

Note: For the latest update of the user's manual, refer to our website

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EUROHUNT GmbH