English

Magyar

Română

Français

Deutsch

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# BTE Products

User Guide • Használati utasítás • Instrucțiuni  
de utilizare • Manuel d'utilisation •  
Bedienungsanleitung • تعليمات الاستخدام

**REXTON** 

English	3
Magyar	31
Română	57
Français	83
Deutsch	111
عربي	163

# Content

Your BTE hearing instruments	4
Intended use	8
General notes on safety	9
Handling batteries	12
Locking and unlocking the battery compartment	13
Inserting and removing the hearing instrument	15
Turning your hearing instrument on and off	16
Changing the hearing program	18
Adjusting the volume	19
Audio input	20
Functionalities	21
Maintenance and care	25
Troubleshooting	26
Disposal information	27
Technical information	28

EN

HU

RO

FR

DE

AR

## Your BTE hearing instruments

OPEN

**optional with earhook**  
(battery type: 312)


OPEN

FREE

**optional with Free Tube**  
(battery type: 13)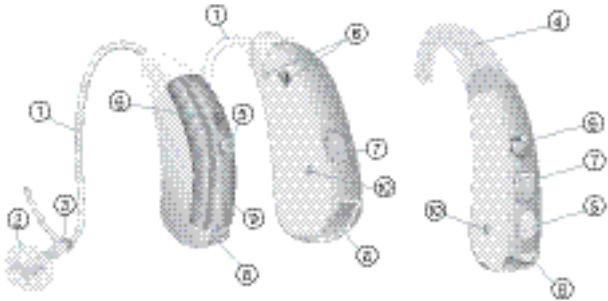

① Free Tube

② Free Tip

③ Free Tip adapter

④ Earhook

⑤ Push button  
(optional)

⑥ Microphone openings

⑦ Cover for programming socket

⑧ Battery compartment

⑨ Microphone cover

⑩ Charging contacts (rechargeable hearing instruments only)

EN

HU

RO

FR

DE

AR



FREE

P

**optional with Free Tube**  
(battery type: 13)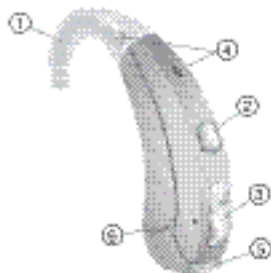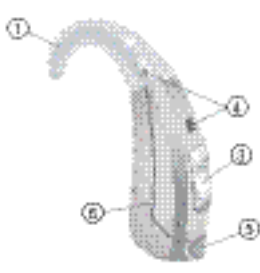

① Earhook

② Push button

③ Rocker switch

④ Microphone openings

⑤ Battery compartment

⑥ Charging contacts

P/HP

P/HP


P/HP

P/HP

(P: battery type: 13)  
(HP: battery type: 675)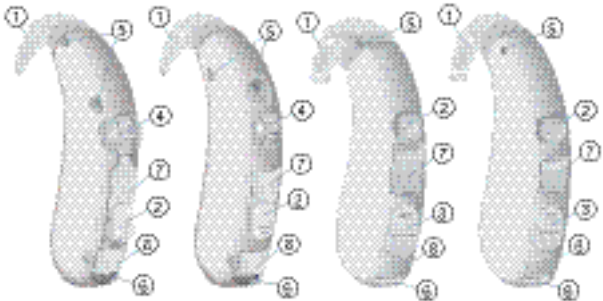

- ① Earhook
- ② Push button
- ③ Rocker switch
- ④ Volume control

- ⑤ Microphone openings
- ⑥ Battery compartment
- ⑦ Cover for programming socket
- ⑧ Audio input

Arena/RX  
(battery type: 312)

① Earhook

② MTO switch

③ Volume control

④ Microphone openings

⑤ Battery compartment

⑥ Cover for fitting controls

⑦ Audio input

## Intended use

Hearing instruments are intended to improve the hearing of hearing impaired persons. Diagnosis and prescription of a hearing instrument must be performed by hearing health specialists, e.g. ENT doctors, audiologists or acousticians.

Read and follow the instructions of this user guide to avoid injuries and to protect the hearing instruments from damage.

Symbols:	
Be aware of information marked with the warning symbol <b>WARNING</b> , <b>CAUTION</b> or <b>NOTICE</b> !	
	<b>WARNING</b> points out a situation that could lead to serious injuries, <b>CAUTION</b> indicates a situation that could lead to minor and moderate injuries.
<b>NOTICE</b>	<b>NOTICE</b> indicates possible property damage.
	Advice and tips on how to handle your device better.
	Instruction. Indicates that something has to be done.
	Indicates an option that not every hearing instrument has. Ask your Hearing Care Professional to check off the relevant information.

# General notes on safety

EN

HU

RO

FR

DE

AR

## NOTICE

Your hearing instruments are sensitive to extreme heat, high humidity, strong magnetic fields ( $> 0.1$  T), X-rays and mechanical stress.

- ▶ Do not expose your hearing instruments to extreme temperature or high humidity.
- ▶ Do not leave them in direct sunlight.
- ▶ Do not wear them in the shower or when you apply make-up, perfume, aftershave, hairspray or suntan lotion.
- ▶ Do not wear your hearing instruments when you are exposed to short-waves, a strong magnetic field, a high frequency field or X-rays.
- ▶ Do not place your hearing instruments in a microwave oven.

## WARNING

Hazard of explosion in explosive atmospheres!

- ▶ Do not use your hearing instruments in areas where there is a danger of explosions (e.g. mining area).

In some countries restrictions for the usage of wireless equipment exist.

- ▶ Refer to local authorities for further information.

### **WARNING**

Choking hazard posed by small parts.

- ▶ Keep hearing instruments, batteries and accessories out of children's reach.
- ▶ If swallowed consult a physician or hospital immediately.
- ▶ These hearing instruments are not designed for children under the age of 36 months or mentally disabled persons.

### **NOTICE**

Leaking batteries damage the hearing instruments.

- ▶ Turn the hearing instruments off when not in use to preserve the battery.
- ▶ Remove batteries when the instruments are not in use for a prolonged period of time.

EN

HU

RO

FR

DE

AR

**NOTICE**

Batteries contain harmful substances that pollute the environment.


- ▶ Do not throw used batteries into household trash.
- ▶ Dispose batteries according to national regulations or return them to your Hearing Care Professional.

**CAUTION**

Risk of injury caused by incorrect use of Free Tip.

- ▶ Always wear the Free Tube with the Free Tip.
- ▶ Make sure that the Free Tip is completely attached to the Free Tip adapter.
- ▶ Change the Free Tube or Free Tip when they are cracked.
- ▶ Put the Free Tube with the Free Tip carefully and not too deep into the ear.
- ▶ Use the Free Tube and Free Tip only in the way described in this user guide.
- ▶ If the Free Tip remains in the ear, have it removed by a medical professional.


## Handling batteries

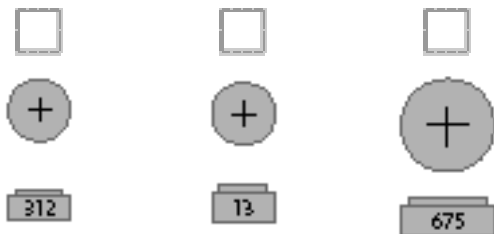original size

- ▶ Always use the right size of batteries.
  - ▶ Charge the rechargeable batteries before first use.
  - ▶ Remove the tab on standard batteries only when you are ready to use them.
  - ▶ The positive (+) side of the batteries is always the smooth side.
  - ▶ Always keep spare batteries with you.
- ▶ Open the battery compartment completely.
  - ▶ Remove the empty battery.
  - ▶ Insert the proper size battery. Make sure that the "+" symbol on the battery points up as shown in the picture.
  - ▶ Close the battery compartment.

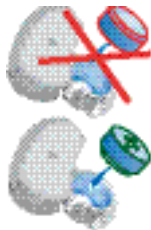## Locking and unlocking the battery compartment

EN

HU

RO

FR

DE

AR


### Option 1

#### Locking

- ▶ Push the small slide to the right.

#### Unlocking

- ▶ Push the small slide to the left.

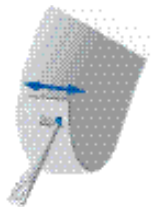### Option 2

#### Locking

- ▶ Push the small slide to the bottom.

#### Unlocking

- ▶ Push the small slide to the top.

### Option 3

#### Locking

- ▶ Push the slide to the right with a suitable pointed tool.

#### Unlocking

- ▶ Push the slide to the left with a suitable pointed tool.
- ▶ Close the battery compartment completely.

## Option 4

- ▶ Use separate lock cover for the on/off switch and battery compartment.

## Option 5


### Locking

- ▶ Close battery compartment.
- ▶ Ensure that the transparent battery compartment lock is pushed in.

### Unlocking

- ▶ Open battery compartment lock with suitable tool. The battery compartment can be opened.

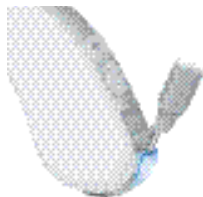## Inserting and removing the hearing instrument

EN

- ▶ Handle the hearing instrument by holding the earmold. Do not pull the Tube.

HU

RO

### Inserting

- ▶ Insert the Free Tube/earmold.
- ▶ Place the hearing instrument behind the ear.


FR

DE

### Removing

- ▶ Remove the hearing instrument.
- ▶ Pull the Free Tube/earmold carefully out.

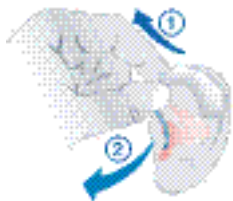AR

## Turning your hearing instrument on and off

Via battery compartment

On

▶ Close the battery compartment.

Off

▶ Open the battery compartment.

Via battery compartment lock

On

and battery compartment locked

Off

and battery compartment unlocked

### Via push button

#### On

- ▶ Press the push button for 2 seconds.

#### Off

- ▶ Press the push button for 2 seconds.

### Via MTO switch

#### On

- ▶ Switch to M (microphone) or T (telecoil).

#### Off

- ▶ Switch to O (off).

### Via remote control

- ▶ Refer to the remote control user guide.

## Changing the hearing program

### Via push button

- ▶ Press the push button to switch to the next program.

### Via MTO switch

- ▶ Switch to M (microphone) or T (telecoil).

### Via remote control

- ▶ Refer to the remote control user guide.

Hearing program	Description of the hearing program
1	
2	
3	
4	
5	

# Adjusting the volume

EN

HU

RO

FR

DE

AR


## Volume control

- ▶ Rotate the volume control up to increase the volume.

- ▶ Rotate the volume control down to decrease the volume.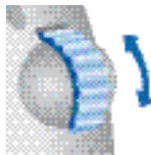

## Rocker switch

- ▶ Press up to increase the volume.
- ▶ Press down to decrease the volume.

## Via remote control

- ▶ Refer to the remote control user guide.

## Audio input

An audio input enables the connection between the hearing instrument and an external device e.g. an FM system, mp3 player, television or an infrared device.

### Audio input via an audio shoe

- ▶ Refer to the audio shoe user guide on how to attach and remove the audio shoe correctly.

### Audio input integrated in the battery compartment

- ▶ Ask your Hearing Care Professional to replace the battery compartment.

- ▶ Use only the Rexton audio shoe which is designed exclusively for your hearing instrument.

### **WARNING**

Risk of injuries caused by electric shock!

- ▶ Connect audio input only to a device conforming to IEC 60065 (IEC-Norm for audio, video and other electronic devices).

# Functionalities

Your Hearing Care Professional can activate the following functionalities.

## wireless

- Allows the use of a remote control.
- Automatically adjusts both hearing instruments simultaneously to change the instrument settings, e.g. volume or program.

The performance of the wireless function may be affected by electromagnetic interference – e.g. from a computer monitor or fluorescent lamp system.

- ▶ Move away from the source of interference if you experience difficulty.

## Power-on delay

Your hearing instrument is equipped with a "power-on delay" feature which can be activated by your Hearing Care Professional. This allows you to turn the hearing instrument on with an automatic delay of 12 seconds, so that you may insert the hearing instrument into your ear without experiencing unpleasant feedback (whistling).

EN

HU

RO

FR

DE

AR

### Rechargeable hearing instruments

Rechargeable hearing instruments allow the use of standard or rechargeable batteries.

- ▶ Charge rechargeable batteries before first use.
- ▶ Do not charge standard batteries.
- ▶ Refer to the charger user guide for further information.

### Automatic telephone program

- ▶ Hold a telephone near your hearing instrument (\*).

The hearing instrument automatically switches to the telephone program.

- ▶ Move the telephone away from your hearing instrument.

The hearing instrument switches back to the previously used program.

- ▶ You can also select the telephone program manually by using the push button.
- ▶ When operating the hearing instrument in telecoil mode, the wireless function may cause a pulsing noise. This is due to the pick-up of the transmission signal. When your hearing instrument is in telecoil mode please use the remote control only with a distance greater than 10 cm (4 inches).

#### (\* AutoPhone magnet accessory

Not all telephones produce a magnetic field strong enough to activate the AutoPhone function. In this case, use the Siemens AutoPhone magnet, which is available as accessory, and attach it to the telephone receiver to activate the AutoPhone function.

- ▶ See the AutoPhone magnet user guide for instructions on how to position the magnet.
- ▶ Use only the Siemens magnet. Ask your Hearing Care Professional for this accessory.

### **NOTICE**

Magnets can disturb electrical devices and delete stored data.

- ▶ Keep magnets away from computers, color monitors, television sets, floppy discs, video- and audio discs and other electronic equipment/ devices.

### **WARNING**

Magnets may interfere with the operation of life supporting systems (e. g. pacemakers, magnetic valves).

- ▶ Use a magnet only if not in doubt about the safe distance. The safe distance between pacemaker and magnet should be at least 10 cm (4 inches).

## Maintenance and care

Your hearing instruments are highly sensitive systems. Proper care and handling will ensure the best possible performance.

Clean instruments regularly to prevent damage to the hearing instruments and to avoid health problems.

### NOTICE

- ▶ Do not put your hearing instruments in water.

- ▶ Clean your hearing instruments with a soft, dry tissue, as necessary.
- ▶ If the hearing instruments sound distorted or weak ask your Hearing Care Professional to exchange the microphone cover.
- ▶ Place your hearing instruments in a "dry aid kit" with a special moisture-attracting capsule overnight.
- ▶ Ask your Hearing Care Professional for special care sets and further information on how to keep your hearing instruments in good condition.

### For instruments with Free Tube

- ▶ Regularly clean Free Tubes with a cleaning wire.
- ▶ Ask your Hearing Care Professional to exchange Free Tubes and Free Tips approximately every 3 to 6 months or as needed.

EN

HU

RO

FR

DE

AR

## Troubleshooting

Use these tips to resolve simple issues.

Problem	Possible solution
Sound is weak.	Replace empty battery. Clean earmold. Increase volume.
Hearing instrument emits whistling sounds..	Reinsert the earmold until it fits securely. Decrease volume. Clean earmold.
Sound is distorted	Decrease volume. Replace empty battery. Clean earmold.
Hearing instrument emits signal tones.	Gently close battery compartment completely. Replace empty battery.
Hearing instrument does not work.	Turn hearing instrument on. Gently close battery compartment completely. Replace empty battery. Make sure battery is inserted correctly. Power-on delay is active. Wait 12 seconds and check again.

Consult your Hearing Care Professional if you encounter further problems.

## Disposal information

Within the European Union, the marked equipment is covered by "Directive 2002/96/EC of the European Parliament and of the Council of 27 January 2003 on waste electrical and electronic equipment."

Amended by "Directive 2003/108/EC" (WEEE).

- ▶ Recycle hearing instruments, batteries and accessories according to national regulations.

With the CE marking Siemens confirms compliance with the European Directive 93/42/EEC concerning medical devices.

For the products with wireless 2.0 Siemens additionally confirms compliance with the European Directive 99/5/EC (R&TTE) concerning radio and telecommunications terminal equipment.

EN

HU

RO

FR

DE

AR

## Technical information

Rexton Revera WL P, Revera WL PD, Revera WL FREE  
Rexton Day BTE  
IC: 267AB-WL001

Rexton Bridge 8 BTE, Bridge 12 BTE, Insite BTE  
Rexton Bridge 8+ BTE, Bridge 12+ BTE, Insite+ BTE  
Rexton Day 4+ BTE, Day 6+ BTE  
FCC ID: SGI-WL002BTE  
IC: 267AB-WL002  
Operating frequencies:  $F_c=3.28$  MHz

N14203

This Class B digital apparatus complies with Canadian ICES-003.

Changes or modifications made to this equipment not expressly approved by SAT GmbH for compliance may void the FCC authorization to operate this equipment.

### FCC compliance

This device complies with Part 15 of the FCC Rules and with RSS-210 of Industry Canada.

Operation is subject to the following conditions:

- this device may not cause harmful interference, and
- this device must accept any interference received, including interference that may cause undesired operation.

EN

HU

RO

FR

DE

AR

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment to an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

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© Siemens AG, 03.2012 · MS
Siemens Audiologische Technik GmbH