APEX POINTER™+

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GB

INTRODUCTION

Congratulations for the purchase of the Apex Pointer  apex locator. Your new instrument is compact, user friendly and provides incomparable precision in determining immediate and exact working length in the root canal. Please read carefully the operation and safety instructions to achieve the best advantages from the unit.

INDICATIONS

Apex Pointer  is an electronic apparatus for determining the apex location.

CONTRAINDICATIONS

Do not use the Apex Pointer  on patients with implanted heart pacemakers or other equipment which have been warned against use of small electrical appliances. Do not connect the Apex Pointer  to other instruments and do not use it in combination with other instruments or systems.

WARNINGS

- Do not use rechargeable batteries. Only AA type non-rechargeable alkaline batteries should be used.
- The file clip and the lip clip should be connected to the measuring cables after connecting the measuring electrodes to the unit connector (marked PROBE).
- It is forbidden to disconnect the measuring electrodes from the unit connector before the file clip and the lip clip disconnection from the measuring cables.

PRECAUTIONS


- To avoid damages to the Apex Pointer  or false display values ensure the following:
 - Prevent contact between the measuring electrode and the dental file with metal crowns, bridges, filling materials or another instrument.
 - Avoid the exposure of the Apex Pointer  to liquids.
 - Avoid exposure of the Apex Pointer  to direct sunlight or heat above 70°C.
 - Do not use the Apex Pointer  in presence of flammable materials.
 - Use only 1.5 Volt AA non-rechargeable alkaline batteries and make sure to insert them in the correct direction. Wrong direction may harm the instrument.

SPECIFICATIONS

- The Apex Pointer + has an automatic internal self test procedure operating when the instrument is turned on.
- The Apex Pointer + does not need any calibration, it is ready to use immediately after self test has elapsed.
- The measurement is independent of canal conditions – dry, humid with blood, Saline, hypochlorite, or other liquids. File type or size might have only a negligible influence on the displayed values.
- Large intuitive liquid crystal display (LCD) provides simple and intuitive data.
- The displayed values are directly related to the file tip location in the canal and to its distance from the apical constriction and the apex.
- Apex Pointer + uses easy to replace 4 standard 1.5 Volt AA batteries.
- Low power consumption and automatic turn off promise long battery life.
- Controlled audible indications with varying tone sequences draw the practitioner's attention when working in the vicinity of the apical region.
- The lip clip and the file clip can be sterilized in an autoclave.

GB

DESCRIPTION

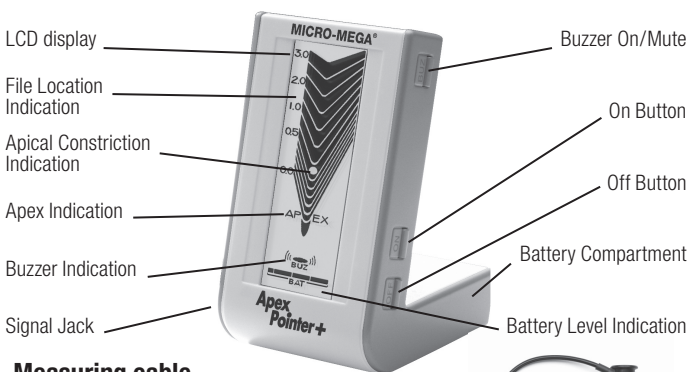Measuring cable

File clip

Touch probe

Lip clip

AA Battery

BATTERY REPLACEMENT


1. Turn off the instrument.
2. Disconnect the measuring wire from the probe connector.
3. Open the battery compartment cover by pushing the locking clip towards the front panel of the Apex Pointer + and lift the cover.
4. Place 4 x 1.5 Volt AA non-rechargeable alkaline batteries into the compartment in the correct directions according to the indication marks.
5. Put back the cover ensuring that the locking clip is fixed properly.
6. Apex Pointer + is now ready for work.

GETTING TO WORK

1. Insert the electric plug of the measuring wire into the signal jack on the left-hand side panel (marked 'PROBE') of the instrument.
2. Turn on the Apex Pointer + by pressing shortly on the "ON" button. Wait until the self test is completed and a short beep is heard. The LCD appearance is the following:
3. Verify that the battery indicator shows a level of at least 2 lines, otherwise replace the batteries.
4. Before applying the electrodes to the patient's body connect the lip clip to the file clip and verify that the display shows "File beyond apex".
5. Connect the lip clip to the patient's lower lip and the file clip or the touch probe to the file that is inserted into the treated root canal.
 - **Important: The file clip and the lip clip should be connected to the measuring cables after connecting the measuring electrodes to the unit connector (marked PROBE).**
 - **It is forbidden to disconnect the measuring electrodes from the unit connector before the file clip and the lip clip disconnection from the measuring cables.**
6. The location of the file tip in reference to the apex will be indicated on the display.



Display after self test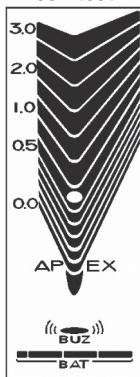

FILE LOCATION INDICATION

■ Pre-apical zone

- Carefully insert the file into the canal and observe the display of the Apex Pointer +. The V shaped scale lines show the location of the file tip – the upper line that can still be seen indicates the location of the file tip with respect to the apical constriction and the apex.

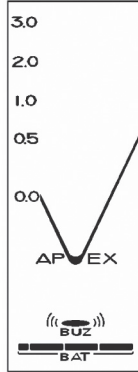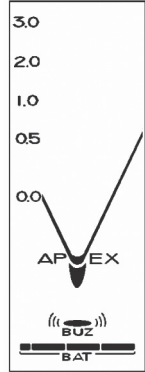File at pre-apical zone**File at apical constriction****File at apex foramen****File beyond apex**


- The line with the bright elliptical area at its bottom associated with the reading 0.5 indicates the apical constriction.
- When the file reaches this position – a beating beep starts to alert the dentist.

■ Apical zone

- Between the apical constriction and the apex the scale is divided into 5 regions. As long as you insert the file deeper into this region the beep tact increases. When the apex is reached the V line marked with 0.0 starts to blink. When the apex is intruded an extra indication below the word apex starts also to blink and the beep becomes continuous.

BUZZER CONTROL

- When the upper button on the right-hand side panel is pressed, the beep is muted. The “BUZ” indicator on the LCD changes from

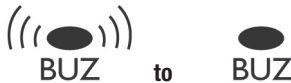- To recover the beep, press the button again.

AUTO-SHUT OFF

- The Apex Pointer + turns itself off after 2 minutes of operation if at least one of the measuring electrodes is not connected to the lip or to the root canal respectively.
- You may also shut off the instrument whenever you wish by pressing shortly on the “OFF” button which is the lowest button on the right-hand side panel of the Apex Pointer +.

USER PRECAUTIONS

- We recommend not using the Apex Pointer + on patients with implanted pacemakers or other electrical implants.
- Prevent contact between the measuring electrode and the dental file with metal crowns, bridges, filling materials or another instrument.
- Avoid high level of liquid in the cavity or in the root canal in order to prevent direct electrical conductance between the file and the soft tissue. In this case dry the canal with air stream.
- Note that canals with open apices may cause erroneous measurements.
- Measurements in canals without patency will prevent proper operation of the instrument.
- Avoid the exposure of the Apex Pointer + to liquids.
- Avoid exposure of the Apex Pointer + to direct sunlight or heat above 70°C.
- Do not use the Apex Pointer + in presence of flammable materials.
- Use only 1.5 Volt alkaline AA batteries and make sure to insert them in the correct direction. Wrong direction may harm the instrument. Do not use rechargeable batteries

MAINTENANCE

- Besides occasional battery replacement, your Apex Pointer + does not contain any user serviceable parts. Any service and repair should be performed by authorized service personnel.
- The Apex Pointer + and its accessories may be cleaned with a cloth impregnated with soap and water. **DO NOT USE CHEMICAL AGENTS OR DETERGENTS.**
- The lip clip and the file clip be sterilized by autoclave (134°C).

GUARANTEE

- Two year for instruments.
- Six months for cables.
- The guarantee is applicable under normal conditions of use and will be deemed void in the event of accidental damage or any unauthorized modification.

TECHNICAL SPECIFICATIONS

- | | |
|-------------------|----------------|
| • Model | Apex Pointer + |
| • Type | CE |
| • Safety Standard | IEC60601-1 |

The system is internally powered continuous operated movable equipment with BF applied part and without signal input/output parts. The unit is ordinary equipment and is not intended for use in the presence of flammable mixtures.

- | | |
|---------------------|--|
| • Power Supply | 6 Volt DC (4x1.5 volt - AA non-rechargeable batteries) |
| • Power Consumption | 0.45 Watt (typical) |
| • Measuring voltage | 60mV (maximum) |
| • Measuring Current | 11µA |
| • Dimensions | 70 x 90 x 120 mm (W x D x H) |
| • Display | Reflective Liquid crystal |
| • Weight | 220 gr (including batteries) |

ENVIRONMENTAL CONDITIONS FOR TRANSPORTATION

- Temperature -10° to +70°C
- Relative Humidity 10% to 90%, non condensing
- Pressure 500 HPA to 1060 HPA.

ENVIRONMENTAL CONDITIONS FOR OPERATION

- Température 10-40°C
- Humidity: 30 – 75% RH (NC)
- Pressure: 10 – 15.5 PSI

EXPLANATION OF SYMBOLS

BF applied parts

Attention, consult accompanying documents

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