LUXITE

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- 1 Overview 27
- 2 List of Part 28

5. Installation of LUXITE 29

- 1 Charger and AC Adapter 29
- 2 Battery 29
- 3 Light Probe 30
- 4 Anti-glare Shield 30
- 5 Charging 31

6. Using of LUXITE 32

- 1 Power On/Off 32
- 2 Control Button 32
- 3 LCD Display 33
- 4 Battery Icon 34
- 5 Sleep Mode 34
- 6 Audible Sound 34
- 7 Setting Memorizing 37

7. Program Modes 35

- 1 Turbo Mode 35
- 2 Normal Mode 35
- 3 High Mode 36
- 4 Soft Start Mode 36
- 5 Pulse Mode 37
- 6 Pulse Soft start Mode 37

8. Maintenance 38

9. Problem Solution 39

10. Specification 40

11. Quality Assurance 40

2. Feature of LUXITE

We appreciate for your purchasing the LUXITE Light of Itena.
You can treat more convenient and safe after reading this user manual.

- High intensity (MAX. 1600mW/cm² ± 10%) to polymerize
- Available various curing program modes and time selecting
- Real Pulse Mode function
- Wireless
- Ergonomic design
- Color LCD display and unique thermal control system
- Interchangeable Battery Pack
- Various Light Probe

3. Caution

1 Safety

- Read this manual thoroughly before using the device.
- Safety certification.

	Type BF Applied Part
	Manufacturer
	Recycle
	Double Insulation
	Use only in Protected Areas
	Observe Operating Instructions

Manufacturer doesn't have liability and it is not covered warranty period as followed cases:

1. Damages from improper using and unauthorized repairing.
2. Modifications that have been carried out by a non-authorized persons or companies.
3. Fire, flood, lightning, and other acts of God.

2 Notice

Please read the following information for safe and proper use of this unit.

Caution

Operating Circumstance:

Temperature	10℃ ~ 40℃
Humidity	30 % ~ 70 %
Air Pressure	700 hPa ~ 1060 hPa

- Ensure the Light Probe is fully inserted prior to use. If the Light Probe is not fully inserted, only partial light intensity will result.
- Do not handle the device by gripping Light Probe. It would be separated and the handpiece will fall down.
- Do not disassemble the product due to potential of electric shock, short-circuit, or fire.
- Do not allow the unit gets wet or immerse in water. Because this unit is not prevent from of water it causes electric-shock or functional error.
- Protect unit from exposure to extreme temperatures. Do not leave the product in direct sunlight or in high humidity areas such as bathroom or in a closed car in summer.
- Do not put your product in a place subject to excessive dust. Protect power cord from heat sources. Keep the minimum required distance between the power cord and heat sources.
- Keep the product out of reach of children.
- Never place your curing light in a microwave oven as it will cause the battery to explode.
- Do not expose the curing light to excessive vibration or sharp impact.

2 Notice

Please read the following information for safe and proper use of this unit.

Caution

- Use only Itena approved batteries, chargers, and adapters. Using other batteries, chargers, and adapters voids your warranty and may cause damage.
- Unplug the power cord and charger during lightning storm to avoid electric shock or fire.
- When using the power plug, ensure that it is firmly connected. If it is not, it may cause excessive heat or fire.
- Do not damage the power cord by bending, twisting, pulling, or heating as it may cause a fire or an electric shock.
- Do not place any heavy items on the power cord. Do not allow the power cord to be crimped as it may cause fire or electric shock.
- Keep the metal contacts of the battery clean.
- Do not put the battery in your mouth. Placing a battery in your mouth may cause serious injury.
- The battery may need recharging if it has not been used for a long period of time.
- Remove the Battery Pack from the handpiece of not in use for a long period of time.
- Never dispose of the battery by incineration. It can be cause fire or explosion.
- The battery of this unit is Lithium-ion battery. Please discard the Battery Pack properly in the appointed place.
- Reset
For solving some problems of programming, you can do Reset. Remove Battery Pack from handpiece and reconnect.
- Do not touch the terminal areas of Charger Base or Battery with fingers. Do not insert conductive objects, such as metal wire or safety-pin. It may cause electric shock or fire.

2 Notice


- 1 This unit should be used only dental curing.
- 2 This unit is intended for use on polymerization resins, 420~490nm. Please get the proper polymerization time and use the curing light.
- 3 Do not expose eyes to light source. Serious injury to eyes may result.
- 4 Do not use the curing light, if patient has suffered photo biological reactions in the past.
- 5 Do not use Light Probe, if it is broken or scratched.
- 6 Must unplug before disinfectants or cleaning.
- 7 Every person who has previously suffered retinopathy, should visit an eye specialist before using the unit.
- 8 Any person who has a cataract treatment should use with caution. The intensity of the light is potentially dangerous. Discuss with eye care specialist before using this light curing.
- 9 Disinfect prior to use.
- 10 Do not irradiate soft tissue which not protected. It could cause injury.
- 11 Light could be reflected from teeth. Use Anti-glare Shield that include in package.

4. Overview of LUXITE

1 Overview


- LUXITE is built with the newest technology.
- LUXITE generates high power light intensity. (1600mW/cm² ±10%)
- Dentists are able to select various program modes and time which they want.

LUXITE Overview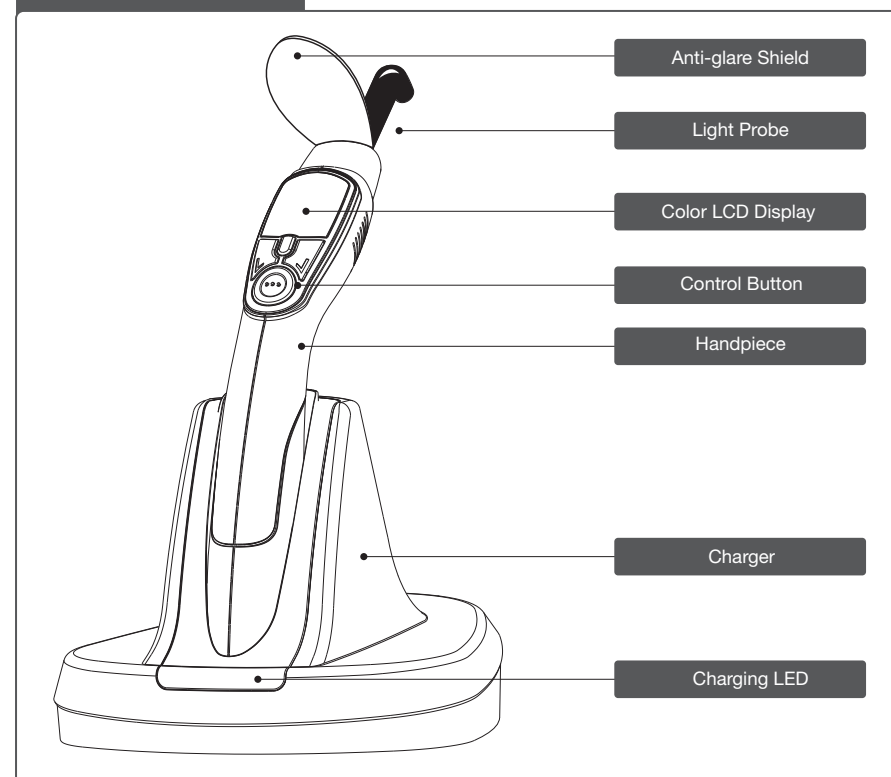


2 List of Part


Please inspect contents of package to ensure all components are included as shown below.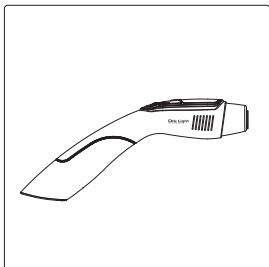


Handpiece(Item No. LU-01)

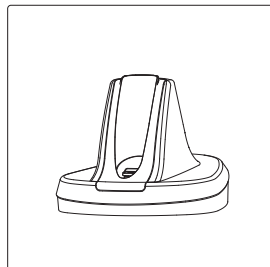
Charger(Item No. LU-02)

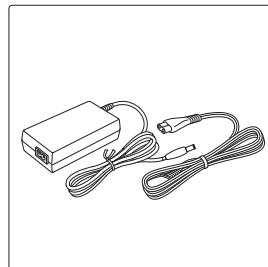
AC Adapter(Item No. LU-03)

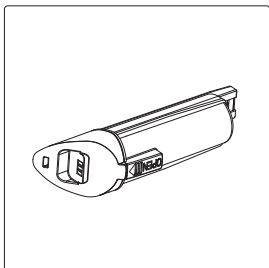
Battery Pack(Item No. LU-04)

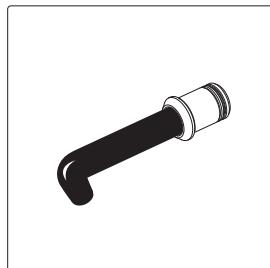Light Probe
(8mm Item No. LX-01)

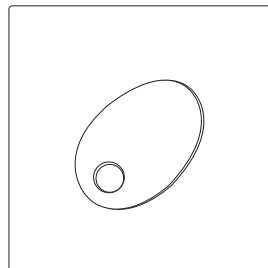Anti-glare Shield
(Item No. LU-05)


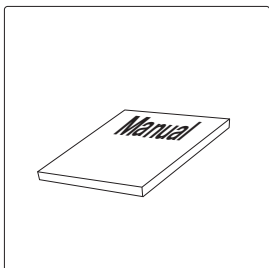
Manual(Item No. LU-09)

5. Installation of LUXITE

1 Charger and AC Adapter


Place the charging base on dry flat surface. AC Adapter is for 100 to 220V. Connect adapter to charger base after connecting power cable as shown.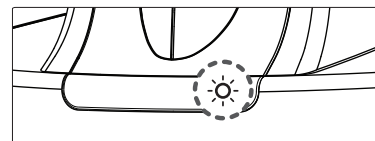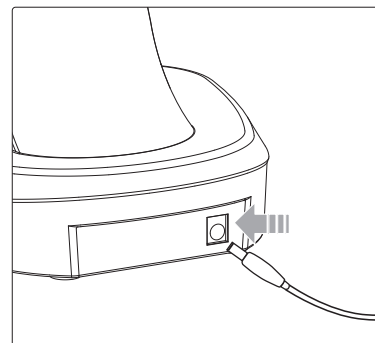

The green light on when the connection is normal.


2 Battery

Insert Battery Pack into handpiece as shown.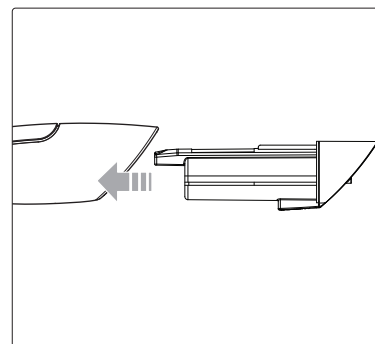

Caution


Please do not force when Battery Pack into the handpiece. It can make wrong action.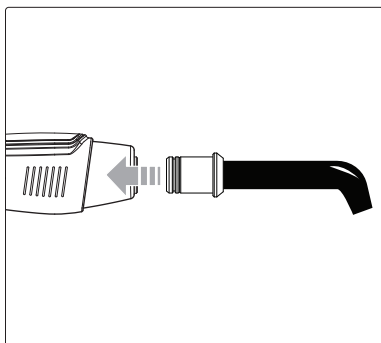

3 Light Probe


Insert the Light Probe into the handpiece carefully. If push as far as no longer putting in, union is completed.

*The basic Probe Light is included. (8mm)
Additional Light Probes are available.
Please contact Itena or local dealer to
obtain other kinds of Light Probes.*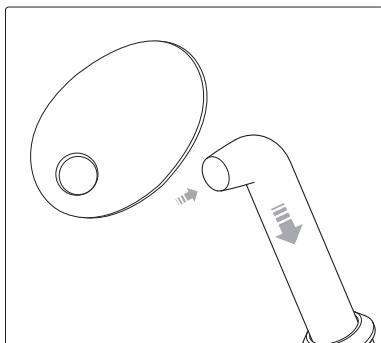

4 Anti-glare Shield

Fit the Anti-glare Shield with a Light Probe as shown.

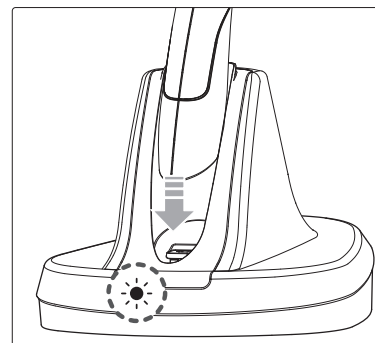5 Charging

Insert the handpiece onto the front side of charger as shown.
In front charger light is ON.



Only use sources of electricity 100 to 220V. When use different power, it would be make a fire and damaged to product.

During charging: Red light is turned on when Battery Pack is charging.

It is normal even if red LED flickers when charging.

Charge complete : Green light ON.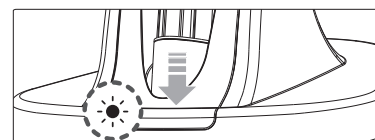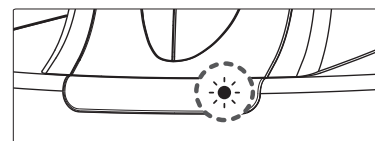

LUXITE's Battery Pack is possible to charge separately.

Insert the Battery Pack onto battery charger's terminal exactly. Please check light is ON.

Caution

Do not touch the terminal areas of Charger Base or Battery with fingers.
Do not insert conductive objects, such as metal wire or safety-pin.
It may cause electric shock or fire.

If you do not want to interrupt for continual use, extra Battery Pack(Item No. DL-04) is needed.
In this case, please contact to Itena or local dealer.

1 Power On/Off

- When pressing the start button, you can standby which last used mode with beep after intro display. (Power On function)
- It can work when connect the Battery Pack is connected to the hadnpiece.
- If press the start button, it is started to exposure from selected mode.
- If you want to turn off, just press the start button about 1.5 seconds.

Intro display


Last used mode

Caution


Press any button if you want to cut off exposure.

2 Control Button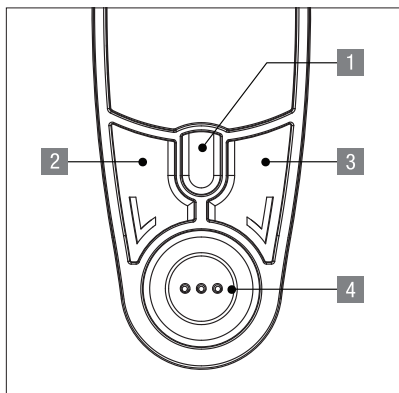

- 1 TURBO MODE Button**
It is button for Turbo Mode.
- 2 TIME Button**
It is button for lighting time select.
- 3 MODE Button**
It is button for mode select.
- 4 START Button(Power ON/OFF Button)**
It is button for starting the exposure and power on / off.

3 LCD Display

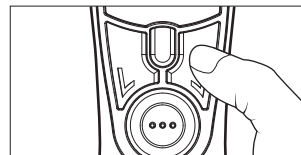
- LUXITE offers display modes that Graph Mode and Text Mode. Press the Mode button for 1.5 seconds it would be changed to graph display and text display.
- A long beeps after converting is completed.

< Graph Mode >

Graph Mode Display

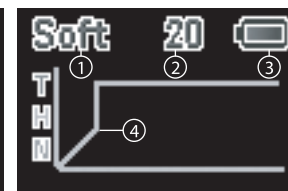- ① Program mode
- ② Time
- ③ Battery
- ④ Output graph

< Text Mode >

Text Mode Display

- ① Program mode
- ② Time
- ③ Battery
- ④ Program mode

① Residual Time

Caution

At the first, setting time is shown, and it is count down during exposure.

4 Battery Icon

This icon shows Battery Pack's residual quantity.

Please charge Battery Pack or change to a charged Battery Pack when warning of Low Battery.

5 Sleep Mode

If you do not any action during 4 minutes, LUXITE Light turns off automatically. Turn off LCD display window with beep.

6 Audible Sound

Following three kind of audible sounds:

- Long beep:
 - When changing display mode. (TEXT MODE <-> GRAPH MODE)
 - When finished each program modes.
- Short beep:
 - When get ready for start.
 - When changing mode.
 - Every 10 seconds passed. (Turbo Mode: Every 5 seconds passed)
 - When turning off.
 - When change to Sleep Mode.
 - When come back to work from Sleep Mode.
 - When changing time setting.
- Two short beep:
 - When starting each mode.
 - When turning on.

7 Setting Memorizing

LUXITE can recall a program mode last used. It is applied when turn off and on, reset, come back from Sleep Mode.


Caution

If you force the control button by hand, it can make problems of operation or malfunction.

7. Program Modes


1 Turbo Mode

You can use this mode when you need high power output immediately. Its output power is about more than 1600mW/cm². The select of time could be 5, 10 and 20 seconds.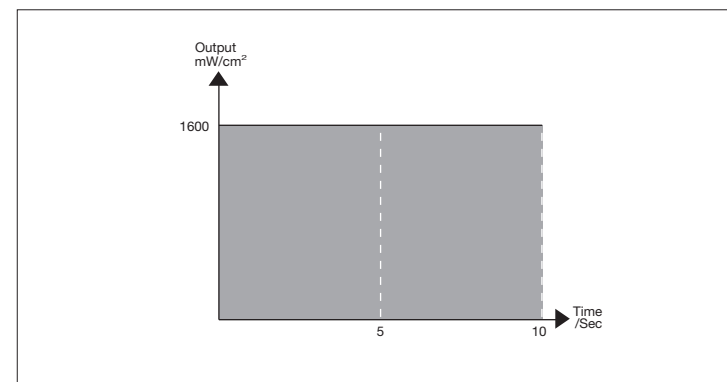


2 Normal Mode

Its output power is about more than 650mW/cm². The select of time could be 10, 20 and 30 seconds.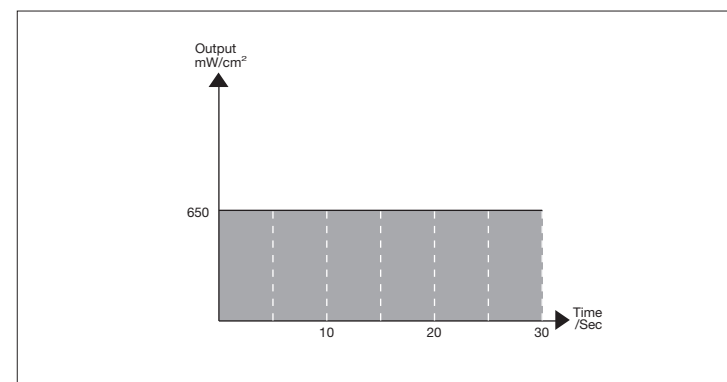


3 High Mode

Its output power is about more than 1200mW/cm².
The select of time could be 10, 20 and 30 seconds.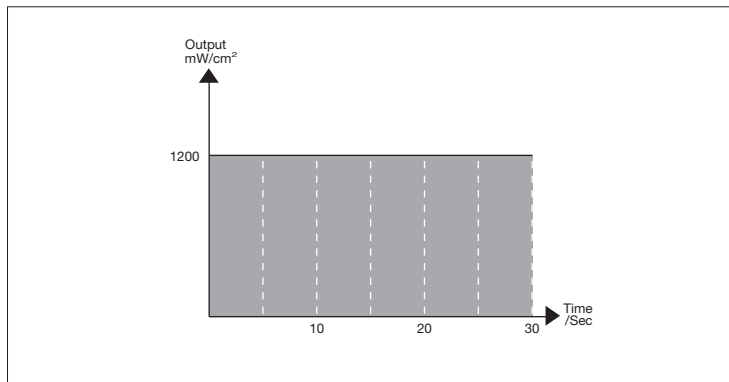


4 Soft Start Mode

During 5 seconds output power is raised from 0mW/cm² to around 650mW/cm² and it comes out more than 1600mW/cm² next during 15 seconds. Time setting is fixed as 20 seconds.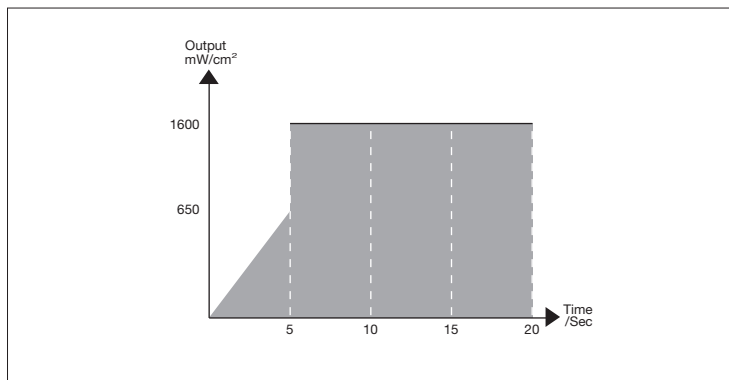


5 Pulse Mode

Output power is comes around 1600mW/cm² during 0.1 seconds, and takes a rest 0.05 seconds in repetition. The select of time could be possible 10 and 20 seconds.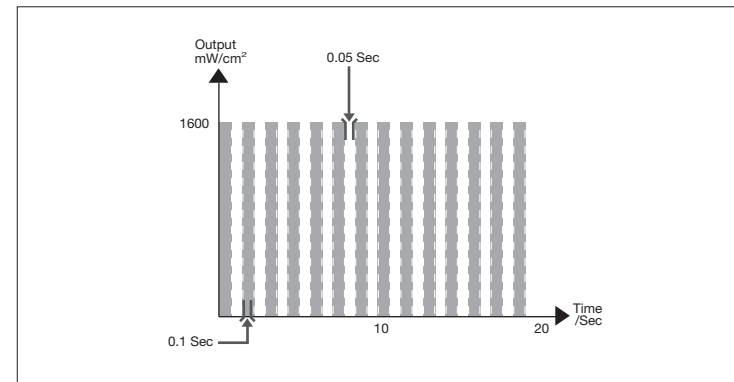

6 Pulse Soft Start Mode

During 5 seconds output power is raised from 150mW/cm² to around 1600mW/cm² like Pulse Mode and it comes out around 1600mW/cm² next during 15 seconds. Time setting is fixed as 20 seconds.

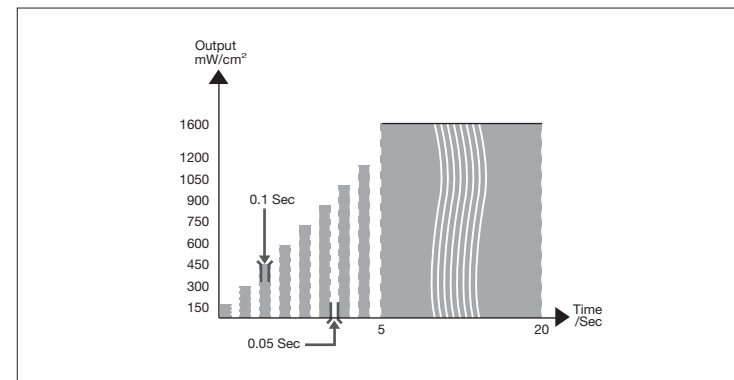8. Maintenance

GB

To prevent electrical shock during cleaning, please do not let water or other liquid which can make handpiece or charger soaked. Unplug power cord from power source when do cleaning.

1 Clean the body of unit by clean cloth softly.

- Wipe the handpiece and charger with a standard aldehyde-free disinfecting solution. Do not clean with highly aggressive solutions (e.g. solutions containing orange oil or solutions with and ethanol content of more than 50%), solvents (e.g. acetone), or pointed instruments, which damage or scratch the plastic.

2 Management of Light Probe

- Clean the Light Probe by soft and disinfected cloth. Do not use sharp or pointed instruments, since they may scratch the surface of the Light Probe and thus reduce light transmission. Light Probe can be autoclaved.
- If the Light Probe has been damaged, it should be exchanged with new one. Please contact to Itena or local dealer.
- Black spot in the Light Probe means breaking optical fiber. In this case Light Probe should be exchanged with new one.

3 Management of Battery Pack

- High temperature might get bad effect to the battery life. The ideal ambient temperature is between 10°C and 40°C. If the battery is too cold or too hot, the curing light might not work temporarily, even if the battery is fully charged.
- Battery Pack is Lithium-ion battery. For the environment, please discard Battery Pack at the appointed place as local environment guide line and law.

 Caution	Please do not sterilized handpiece and Battery Pack in autoclave. It makes lethal problem to product.
	Only Light Probe is allowed sterilization in autoclave.
	High voltage exists inside charger and handpiece. Do not open charger or handpiece arbitrarily.

9. Problem Solution

GB

If the product is not operated normally, please check the following methods.

Problem	Cause	Solution
Control Button is not operating normally	- It can be caused by strong press - It can be caused by a problem of programming	- Press the button softly - Recharge the curing light - Reset the curing light
LCD is not work	- It can be caused by Battery Pack - It can be caused by a problem of programming	- Check the Battery Packs charging condition - Reset the curing light
Turn off while operating	- It can be caused by Battery Pack - It can be caused by a problem of programming	- Try to press start button again - Check the Battery Packs charging condition - Reset the curing light
When the battery is not charged	It can be caused by wrong connection	- Check the connection between charger and handpiece - Check the connection between power cord and power source
When intensity of radiation is poor	- It can be caused by Light Probe - It can be caused by Battery Pack - It can be caused by the radio-meter	- Clean up the Light Probe again - Check the Battery Packs charging condition - Check the radio-meter condition

If LUXITE does not operate normally after check up all above list, please contact to Itena or local dealer.

10. Specification

GB

Dignified mien standard, parts composition, product shape etc. can be changed without advance notice for performance elevation of product.

· Operating Voltage (Charger)	5 V DC
· Battery	3.7 V Lithium Ion
· Light Source	High Power LED
· Wave Length Range	420 ~ 490 nm(peak 470nm)
· Light Output	MAX. 1600mW/cm ² (±10%)
· Dimension of Handpiece	L 97 W 216 H 37 (mm)
· Dimension of Charger	L 178 W 175 H 137 (mm)
· Weight of Handpiece	175 g
· Weight of Charger	225 g

※ This output data is measured by Itena.

11. Quality Assurance

GB

ITENA

Quality Assurance

We guarantee the quality of the product as follows

<Assurance details>

1. If there is any defect of manufacturer or product has any fault itself with in the assurance period, it will be replaced with the same product.
2. Below cases are not effective for the free remuneration.
 - Malfunction and damage according to user's carelessness,
 - Malfunction and damage according to natural disaster such as fire, earthquake and flood etc.
 - Malfunction and damage according to user's intention or accident.
 - Malfunction and damage according to using incompatible product.
3. The manufacturer does not accept any liability resulting misuse.
4. The compensation method which compensate it based on consumer law.

1. Name of product : 2. Name of model : 3. Customer information - Name of dental clinic : _____ - Name : _____ 4. Distributor - Seller : _____ - Address : _____ - Telephone : _____ 5. Assurance period - Date of purchase : from _____ , 20____ Product is for 2 years except accessories. • Battery Pack's term of guarantee is 1 year .	ITENA 153 Avenue Victor Hugo, 75116 Paris, FRANCE Tel. : +33 (0)1 45 91 30 06 Site internet : www.itena-clinical.com
--	--

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