Application

MaxiTemp® 1:1 in the cartridge:

Insert **MaxiTemp® 1:1** cartridge into a suitable dispensing system. There may be a slight difference in the levels in each chamber. Remove the cap and dispense the material until it is evenly distributed from each chamber. Attach the mixing tip, ensuring that the notches are lined up and turn clockwise until the tip locks. Extrude the material through the mixing tip.

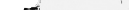
After use, do not remove the mixing tip, this will act as the cap. Change the tip when you are ready to use the product again. Check that the material can flow correctly from the cartridge.

min:s 0:00	min:s 0:30	min:s 1:00	min:s 1:30	min:s 4:00
application in the mouth	setting in the mouth	removal setting	complete	finishing

Indications/precautions:

- **MaxiTemp® 1:1** contains methacrylates, amines, terpenes, benzoylperoxide, BHT. **MaxiTemp® 1:1** should not be used in case of known hypersensitivities (allergies) to these ingredients.
- Before making the temporary, the prepared cores can be insulated for desensitisation (fluoride varnishes, core varnishes, etc.).
- In order to avoid fractures or injuries from debris the temporary should not be subjected to excessive loads (molar area).
- Linings and cores built up with resin-based materials (composites) should be sufficiently insulated (e.g. glycerine) to avoid removal with the temporary.
- Inadequate hygiene or intensive use of certain mouth rinses may lead to discolouration.
- Finishing can be completed once the material is fully set (approx. 4 mins after mixing).
- Restoration of defective temporaries:
In the rare occurrence that the temporary breaks the following procedures, the following procedures are recommended:
 - Air bubbles: fill with MaxiTemp® 1:1
 - In case of fracture immediately after the manufacturing of the temporary: re-connect the fracture with freshly mixed **MaxiTemp® 1:1**
 - Restoration of already used temporaries: Roughen the fracture with a bur and place undercuts. Apply bond on the prepared fractures and cure according to the respective instructions for use. Apply a flowable composite to the prepared areas. Reassemble the fractured pieces and cure according to the instructions for use.

Our preparations have been developed for use in dentistry. As far as the application of the products delivered by us is concerned, our verbal and/ or written information is given to the best of our knowledge.

Additional products:					
9000770	Universal Automix Dispenser 1:1				
9003022	Mixing canulas 1:1, light blue, 50 pcs				
9009069	A2 Starter kit	1		6	 blue 1:1
9009070	A3 Starter kit	1		6	 blue 1:1

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