- Ⓓ **Montageanleitung**
- ⒼⒷ **Mounting instructions**
- Ⓕ **Notice de montage**
- ⓃⓁ **Montage-instructie**
- Ⓘ **Istruzioni di montaggio**

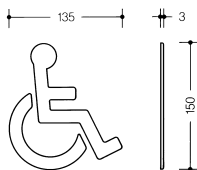
Artikel-Nr.  
Item number  
No de référence  
Type nr.  
Codice articolo

4 - 5


Warnhinweise  
Warning  
Mises en garde  
Let op  
Avvertenze importanti

6 - 8


Zweckbestimmung  
Proper use  
Affectation  
Toepassingen  
Conformità di norma

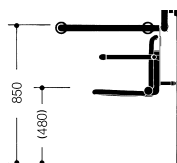9


Montagehinweis  
Mounting information  
Conseils de montage  
Montage aanwijzing  
Indicazioni per il montaggio

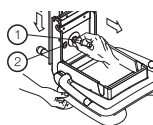10 - 11

Gebrauchsanleitung  
Instructions for use  
Mode d'emploi  
Gebruiksaanwijzing  
Istruzioni d'uso

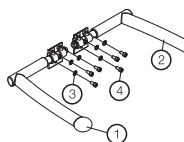12 - 15

Aufrüstanleitung  
Upgrading instructions  
Instructions de modification  
Upgrade-handleiding  
Istruzioni per l'equipaggiamento

16 - 17

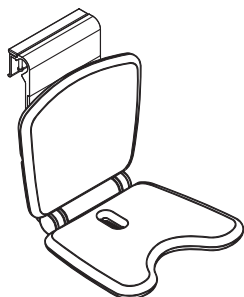
Wartung  
Maintenance  
Entretien  
Onderhoud  
Manutenzione

18 - 21

Pflegehinweise  
Care tips  
Conseils d'entretien  
Onderhoudsinstructies  
Consigli per la pulizia

22 - 23

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- 802.51.11092 anthrazitgrau
- 802.51.11097 lichtgrau
- 802.51.W110 White Edition

Removable hanging seat Comfort

- 802.51.11092 anthracite grey
- 802.51.11097 light grey
- 802.51.W110 White Edition

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- 802.51.11092 gris anthracite
- 802.51.11097 gris clair
- 802.51.W110 White Edition

Hangzitje Comfort

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### WARNING!

#### Use of HEWI removable hanging seat

The HEWI removable hanging seat is designed to withstand a maximum static working load of 150 kg (vertical). The armrests can each withstand a vertical load of 50 kg. Do not use this seat for weights in excess of this. The seat and / or armrests can suffer permanent damage if these loading limits are exceeded. Any non-compliance with loading limits can cause permanent damage to the seat. In such cases, the seat must be examined by a specialised company work prior to the seat being used again. If in doubt, contact HEWI's after-sales team (sanitary) on tel. +44 1634 377688 for further assistance.

Only use the HEWI removable hanging seat in the manner described in the instructions. Any improper use e.g. as a climbing aid can cause an accident with persons potentially suffering serious injury.

Assuming normal use, the hook-on seat is of a flexible nature thanks to the materials used in its manufacture. This flexibility reduces the risk of injury if anyone bumps into the seat. The fact that the seat bends to a certain extent under load is normal.

If you consider the seat to bend excessively under load, you can adjust its angle of inclination. The hook-on seat may only be hung from the HEWI support rails etc. listed on page 10. Test structure of wall. Lightweight partition walls must be fitted with timber noggins in places where the rail is fastened and the removable hanging seat's buffer area is located.

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The removable hanging seat is suspended from the pre-mounted support rail (Ø 33mm): **(GB)**

From a length of 1039 mm (W1/W2) on, an additional wall fixing is necessary because the distance between the supports may not exceed 850 mm (A, horizontal rail).

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
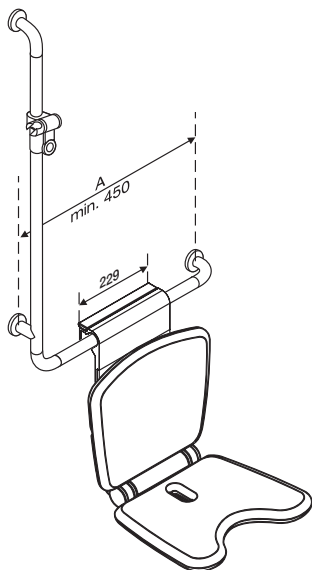
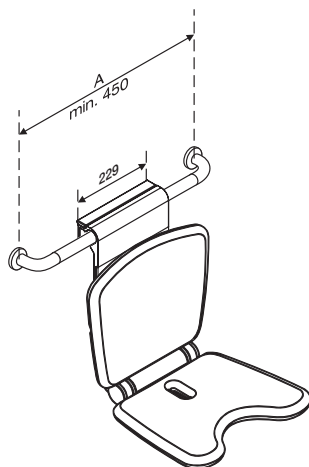
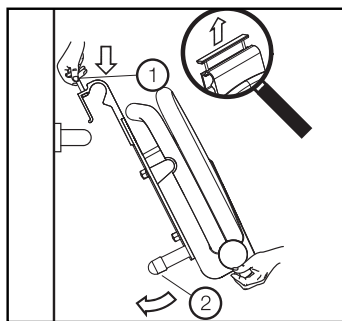
**(GB)** **1. Hooking seat on to HEWI support rail**  
 Hold seat at bottom with one hand and the hook-on element with the other. Hook seat in diagonal position (1) on to support rail. Tilt seat buffer (2) to wall.

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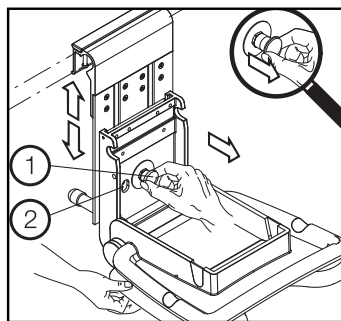**2. Changing seat height adjustment**  
 Fold seat surface and cover or backrest down. Support the seat from below with one hand and pull out spring-loaded securing bolt with the other. Push whole seat upwards or downwards until the required position is visible through seat bracket's viewing window.

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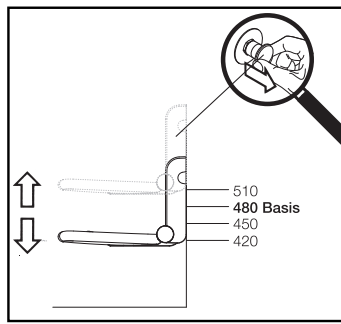
Release securing bolt – locks automatically in position due to spring loading.

**Warning!** Securing bolt must noticeably lock in place. Fold cover or backrest back up.

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- 4 = 510 mm
- 3 = 480 mm
- 2 = 450 mm
- 1 = 420 mm

### Seat height adjustment

Once the seat has been fitted as afore-described, there are 4 seat height options to choose from as indicated by the markings on the wall plate:

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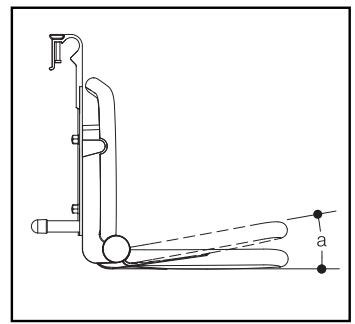
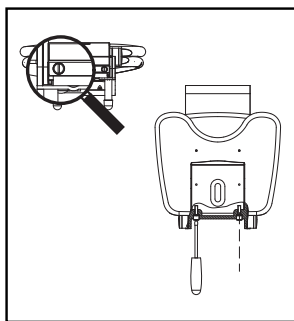
Fold seat surface up. Use thick-tipped screwdriver (approx. 1.8 mm thick) to increase or reduce buffer. 1 complete turn  $\approx 2^\circ$ .

**Warning!**

Both buffers must be set identically.

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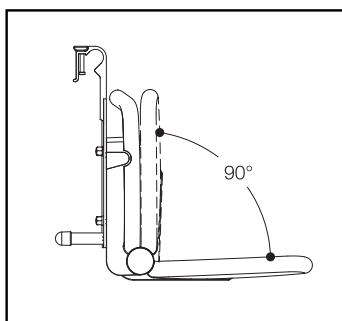
**4. Use of seat surface**

Seat can be folded up or down through  $90^\circ$ .

**Warning!** Folding movement is not controlled. The seat locks noticeably into position when fully upright.

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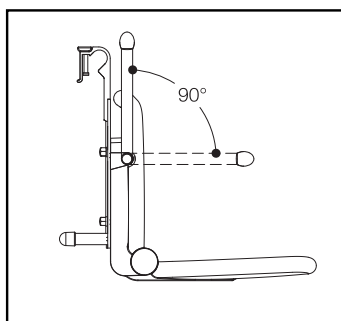
**5. Use of armrests  
Hook-on seat - Premium**

Armrests can be folded up or down through  $90^\circ$  and can be operated independently. Folding movement is controlled. The armrest locks in place when fully upright.

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Upgrading (converting)  
Removable hanging seat Comfort  
to removable  
hanging seat Premium

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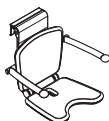
## **(GB)** Parts required: 802.51.010 armrest upgrade kit

- Item 1 1 armrest - left  
Item 2 1 armrest - right  
Item 3 6 B8,4 washers  
Item 4 6 M 8 x 16 socket screws.

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## 1<sup>st</sup> step

Fold backrest down. Use sharp knife to cut out recesses (1) for armrests at the correct points - on both sides of backrest.

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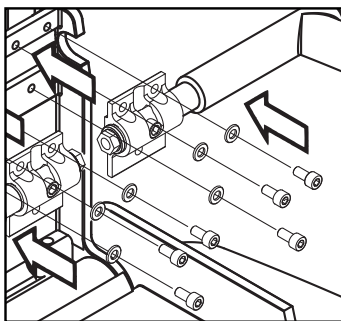
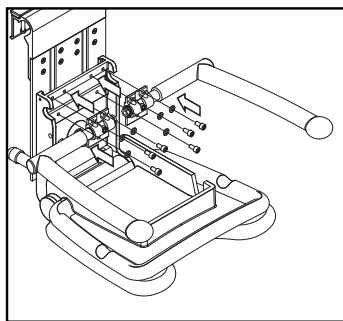
## 2<sup>nd</sup> step

Screw pre-mounted armrests in place (armrest right, armrest left). Ensure you use the screws and washers supplied.  
Tool: SW 6 socket spanner.  
Fold backrest up.

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**Maintenance of removable hanging seat with lift-up feature**

In purchasing the HEWI removable hanging seat, you have chosen a high-quality product. The seat is easy to transport and install. The seat surface folds up and down without any brake control. No special maintenance is required.

Seat height adjustment occurs by operating a spring-loaded securing bolt. Correct seat functionality should be checked once every year (firm mounting of spring-loaded securing bolt in seat bracket and effectiveness of spring) - see also "Seat height adjustment" section.

The folding seat should be taken out of service immediately and repaired by qualified specialists if the spring is no longer effective. In such cases, HEWI's after-sales service team on tel. +44 1634 377688 must be contacted.

Unprofessional repair work can result in accidents with persons suffering potentially serious injury. Furthermore, the entire system used to fasten the seat to the wall must be checked to ensure that it is in good working order.

**Damage in transit**

The seat leaves the factory in a perfect condition. When the customer takes possession of it, he/she should examine it for any damage that may have occurred in transit. Any defects have to be reported immediately to ensure that no potentially severe impairment to the seat functionality can subsequently occur, thus eliminating the risk of persons suffering serious injury. The surface of the seat is sensitive and can suffer scratching caused by dust or dirt on or near the installation site. Keep the seat in its packaging whenever possible.

**Exclusion of liability**

HEWI accepts no liability whatsoever for damage to persons or property that occurs or has occurred as a result of the improper installation, use or maintenance of the company's removable hanging seats, particularly if the safety instructions and other points to note listed above are not or have not been observed in full.

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**GB**

### **Care tips**

HEWI products are easy-care. Generally speaking, an occasional wipe with a damp cloth is enough. If, however, you feel you have to use detergents, you should take note of the following information.

Only use detergents with a pH-rating of between 6 and 8.

Detergents containing active substances such as phosphates, soap and tensides may be used.

Detergents containing acids, alkalis, bleach or scouring agents may not be used. Nor should utensils with a scouring effect such as brushes etc. be used.

Usage of any detergent requires full compliance with the manufacturer's instructions (concentration, soaking time etc.).

If this is not done, the surface finishes of the HEWI products may suffer as a result and sustain damage. HEWI accepts no liability whatsoever for damage occurring from incorrect handling

**For further information please contact HEWI, tel. +44 1634 377688.**

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HEWI Heinrich Wilke GmbH