FRANÇAIS

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PRODUCT DESCRIPTION AND RECOMMENDATION

This wrist blood pressure monitor is an electronic device intended to make human blood pressure and pulse rate at wrist of adult population.

- In order to get reliable measurements, keep taking and recording blood pressure measurement at the same time every day and avoid eating, bathing, smoking, exercising, ingest caffeine or alcohol at least 30 minutes prior to take measurement.
- Be seated and relax for at least 5 minutes before measuring.
- Make sure the cuff is at heart level for taking measurement.
- To make repeated measurements, the rest interval between measurements shall not be less than 30 seconds.
- Consult your physicians for interpretation of blood pressure measurement and follow their direction.


SECURITY

- This wrist blood pressure monitor can be only used for which it has been designed as explained in the instruction booklet.
- Consult your physician for health status and blood pressure range before taking measurement at home with this device.
- Do not press the power button if the cuff has not been properly wrapped.
- If the cuff causes you any discomfort during measurement, press the power button to turn off the device immediately.
- Pull off the Velcro strap to detach the cuff if the cuff pressure exceeds 300mmHg without an automatic exhaust.
- Do not modify or try to repair yourself this device.

MAINTENANCE

- Clean your device with a soft dry cloth, lens with a cotton swab moistened with water then wipe it dry.
- Do not store the device with direct sunlight and keep it away water.
- Do not twist the cuff or fully bend the tube.
- Remove batteries from your device if you don't use for a long time.


PRODUCT DESCRIPTION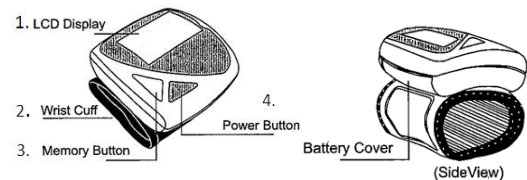


1-LCD display	2-Wrist cuff	3-Memory button	4-Power button
---------------	--------------	-----------------	----------------

Information you should know before using this product

- What is blood pressure?
Blood pressure is produced by the force of blood flowing against artery walls after the heart ejects blood into vessel system.
- What are systolic and diastolic pressures?
Systolic pressure represents the highest pressure during heart contraction and diastolic pressure is the lowest pressure while heart is resting.
- Is my blood pressure normal?
Your blood pressure can be compared with classification categories defined by 1999 World health Organization.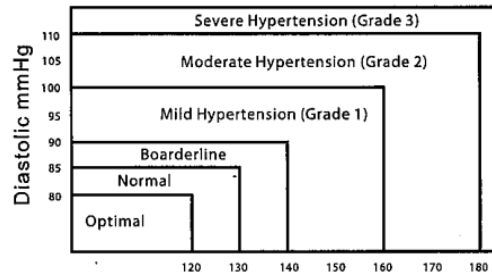

- Fluctuation and variation of blood pressure
Following diagram is showing human blood pressure fluctuation during 24 hours a day

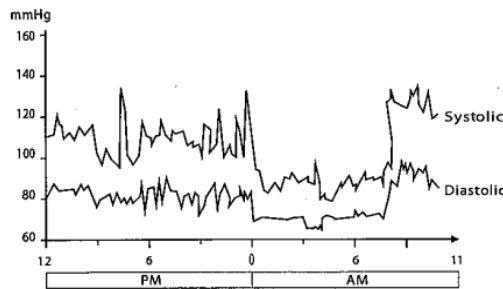Blood pressure measurement can be affected by the measurement position, posture, physiologic condition as well as factors such as eating, bathing, exercising,

smoking, drinking alcohol, stress, mental tension, breathing etc.

USING THE PRODUCT

Inserting and changing batteries

- Slide off the cover in order to open battery compartment, insert the batteries ensuring that the negative and positive ends are at the right place, and replace the cover.
- Using only same type of alkaline batteries is recommended to avoid incompatibility
- If the low indicator battery appears frequently, replace all batteries at the same time.

First use

- Wrap the cuff around bare skinned left wrist (right wrist can be used too). The display should be placed on palm side of the wrist.
- The cuff's upper edge should be 1cm (0,39inch) from the palm line.
- Before first use, you have to set the clock. When power is off, press "M" button until the display shows a blinking year. Press "P" button to adjust the year when it is blinking. Then press "M" button again and month will appear. Press the "P" button to adjust the month when it is blinking. Then date, hour and minute will follow as you press and release the "M" button again.
- For measuring blood pressure, press and release "P" button to start measuring automatically. When measurement is completed, display will show the measured blood pressure value and pulse rate. Press the "P" button to turn off the unit or it will turn itself after 2minutes30 seconds.
- When measurement is completed, if the IPD symbols appears, that indicates irregular pulse is detected during the measurement. If symbol appears regularly, please consult a qualified physician for professional advice.
- For recalling memories, press and release "M" button to recall the last reading. Display will show the memorized pressure value and date and time switching back and forth. Press "M" button to go to previous memories.
- For erasing one memory, press and release "M" button to display records and press "M" button until you find the record that you wish to delete. Press and hold "P" button until the display shows "dEl". After three beeps are heard, selected record is deleted. To delete all memories, follow steps to release one memory, then press and hold "M" button until "dEl all" appears on display. Press and hold "P" button. After three beeps are heard, all records are deleted.

Trouble shooting on display :

- (LLErr) Pressure measured was lower than 20mmHG, please measure again.
- (UUerr) Pressure measured was higher than 300mmHg. Please measure again.
- (PErr): Pumping error, check cuff and try again.
- rErr: Pressure cannot be measure due to signal noise. Please measure again.

- HI : Pumping pressure is higher than 300mmHg, please measure again.
- Low battery: Please check and replace the two batteries if necessary.

Technical specifications

- Measuring range : pressure: 20-300mmHg ; Pulse rate : 40-200
- Accuracy : pressure : +/- 3mmHg, pulse rate : +/- 5%
- Measuring method: oscillometric
- Inflation : electrical micro-pump
- Exhaust: solenoid valve
- Mémories : 85
- Operating range : 10 to 40°C, humidity <85%
- Storage : -20°C to 55°C, humidity <85%
- Power supply : 2 batteries AAA alkaline
- Auto power off : 2mn30s
- Battery life : approx. 250 operations
- Size and weight (with batteries): 80x76x26mm, approx. 130g
- Cuff size: 13,6cm-19,5cm

Compliance

- This blood pressure monitor is compliant with European medical certification (93/42.EEC) and following certification:
- EN1060-1, EN1060-3, ANSI/AAMISP10A-1996, AFFSAPS
 - CE0197
 - European representative: Polygreen Germany GmbH, Ruhlssdorfer strasse 95, D-14532, Stahnsdorf, Berlin – Germany
 - Manufacturer: Jitron Pte Ltd, #07-00F, Golden Wheel Building, 41 Kallang Pudding Road, Singapore 349316

Environmental protection: Deposit empty batteries in a container designed for this purpose so they may be collected and recycled. Do not mix empty batteries with new batteries. At the end of its lifespan, send this product to an electrical and electronic device equipment point, in line with European directive 2002/96/EC-WEE

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- Europejski przedstawiciel: Polygreen Niemcy GmbH, Ruhlssdorfer Strasse 95, D-14532, Stahnsdorf, Berlin - Niemcy
- Producent: Jitron Pte Ltd, # 07-00F, Golden Wheel Building, 41 Pudding Droga Kallang, Singapore 349316