


## REPLACING THE FOOTREST ROD


Tools Required: 4 mm (5/32") Allen Key (provided), 10-mm wrench

SERVICE LEVEL: 2

SAVE ALL REMOVED PARTS FOR USE DURING REASSEMBLY

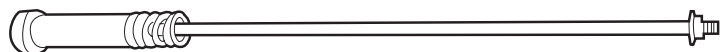

Footrest rod assembled

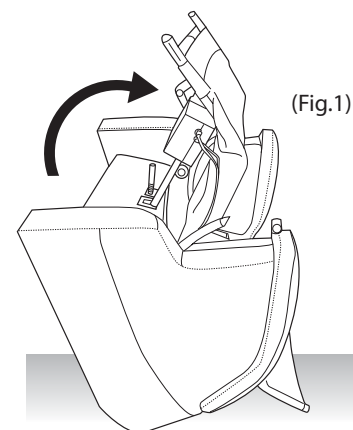


### GET READY

1. Gently tip the chair on its back (Fig.1).

### REMOVING THE FOOTREST ROD

1. Using a 10-mm wrench, turn the metal footrest rod nut counter-clockwise to detach the rod from the spacer barrel, then remove the rod from the chair frame.
2. Using the 4 mm (5/32") Allen Key (provided), remove the screw from the top of the black plastic spacer barrel to detach the spacer barrel from the bracket (Fig.2).

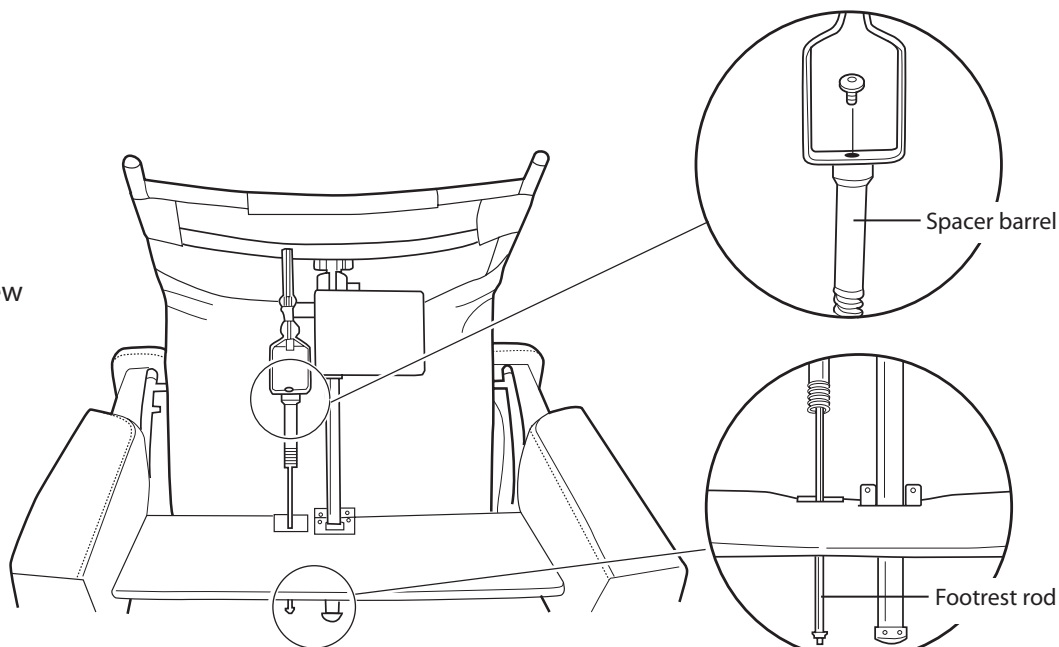

(Fig.2)

## INSTALLING THE NEW FOOTREST ROD

1. Remove the screw from the top of the black plastic spacer barrel (Fig.3).
2. Align the spacer barrel with the corresponding hole on the bracket, then secure it using the screw you removed in the previous step and the 4 mm (5/32") Allen Key (Fig.4).
3. From below the chair frame, insert the metal rod through the corresponding hole in the chair frame then into the black spacer barrel (Fig.4).
4. Turn the metal rod clockwise by hand to secure it, then tighten it using a 10-mm wrench.



(Fig.3)

