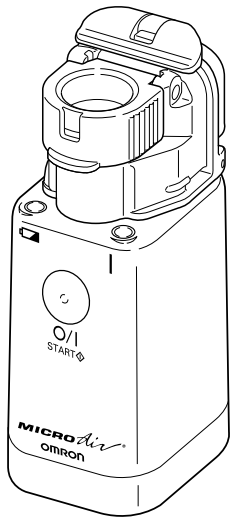
# INSTRUCTION MANUAL

## VIBRATING MESH NEBULIZER

**MICRO** *air*<sup>®</sup>

**Model NE-U22V**

Thank you very much for purchasing  
**OMRON** *MICRO air* Nebulizer.  
Read all instructions in the manual on the  
care and use of this unit before operating.  
Save these instructions for future reference.

### Before Using the Unit

Safety Information .....	2
Features of the Product .....	5
Components of the Product .....	6
Names and Functions of the Parts .....	7
How to Assemble the Main Unit .....	8

### Correct Use of the Unit

How to Use the Power Source .....	10
How to Fill the Medication Bottle .....	12
How to Select the Nebulization Mode .....	14
How to Inhale .....	15
How to Care for Unit After Use .....	16

### How to Disinfect the Unit

How to Disinfect .....	18
How to Replace the Mesh Cap .....	19

### Troubleshooting and Warranty

Troubleshooting .....	20
Specifications /	
Replacement Parts and Accessories .....	22
Warranty .....	23

English

Français

Español

Caution: Federal law restricts this device  
to sale by or on the order of a physician.

This unit is a medical device. Operate this unit only as instructed by your physician  
and/or Respiratory Therapist.


# Safety Information

- The warning signs and icons are shown below so that you use this product safely and correctly to prevent the risk of damage to yourself and others.

Warning signs	Contents
 <b>Warning</b>	Indicates matters in which the possibility of death or serious injury may arise as a result of incorrect handling.
 <b>Caution</b>	Indicates matters in which bodily harm or material damage* may arise as a result of incorrect handling.

\*Material damage refers to a wide range of damage involving your house, household goods, domestic animals, and pets.
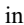
## Examples of signs

	The  icon indicates prohibitions (including warning and danger). Matters involving actual prohibitions are indicated by text or pictures in or near  . The icon to the left refers to “caution for electric shock”.
	The  icon indicates prohibitions (what you should not do). Matters involving actual prohibitions are indicated by text or pictures in or near  . The icon to the left refers to “do not to disassemble”.
	The  icon indicates something that is important (always follow). Matters involving actual important actions are indicated by text or pictures in or near  . The icon to the left refers to “important information”.

## Warning

For the type, dose, and regimen of the medication, be sure to follow the instructions of a doctor.

- Your symptom(s) may worsen.

When using the unit for the first time after purchasing it or after not using it for a long period of time, be sure to wash and disinfect Medication Bottle and Mesh Cap. (Refer to Page 14-18)

- Miscellaneous bacteria may be present and you could be infected.
- Wash and disinfect Mesh Cap, Inhalation Mask, Mouthpiece, and Mask Adapter after each use. (Refer to Page 18)

Be sure to dry the cleaned and disinfected parts promptly, and store them without getting re-contamination.

- Miscellaneous bacteria may be present and you could be infected.

If the unit is used by more than one person, be sure to use the disinfected Medication Bottle Unit, Child’s Inhalation Mask, Mouthpiece, and Mouth Adapter for each user.

- The user may be infected.

Keep the unit out of reach of infants or children.

- They may swallow small pieces such as Mesh Cap. The unit may be damaged or may cause injury to infants or children. In case a child swallows the Mesh Cap, immediately consult with a physician.

# Safety Information

## Warning

Do not plug in or pull out the AC Adapter with wet hands.

- You could suffer electric shock.

Do not wash the Main Unit and AC Adapter with water or hot water.

Avoid immersing them in water.

- Can damage unit.

## Caution

Rinse the disinfectant from the disinfected pieces with water thoroughly.

If you are not going to use the unit for a long period of time, be sure to unplug the power plug from the electric outlet.

Do not use disinfecting solution containing sodium hypochlorite.

- Mesh cap will rust.

Do not spill medication on the Main Unit or the AC Adapter. If you spill medication on the Main Unit or the AC Adapter, immediately wipe it off with a soft cloth.

Do not drop or apply strong shock to the Main Unit, AC Adapter, Medication Bottle, or the Mesh Cap.

Use only the Omron AC Adapter provided for this product.

Be sure to plug the AC Adapter into a 110 VAC outlet.

Do not disassemble, repair, or modify the unit. This may void the warranty.

General Advice

- Replace the worn batteries with new ones immediately.
- Do not use different types of batteries together.
- Do not use new and worn batteries together.
- Do not use dry batteries and rechargeable batteries together.
- Be careful not to insert batteries in the wrong way so that unit operates correctly.
- You can use commercially available rechargeable NiMH batteries, but do not use nickel cadmium batteries.
- Charge the rechargeable batteries with a commercially available charger. (The AC Adapter enclosed in the package cannot be used as a battery charger.)
- Wash Medication Bottle, Mesh Cap, and Vibrator with clean tap water or commercially available distilled water.
- Wash Medication Bottle, Mesh Cap, and Vibrator with clean tap water or the commercially available distilled water.
- When rinsing Medication bottle under the tap, use gentle pressure, not strong running tap water.
- Assemble the cleaned and disinfected parts after sufficiently drying them.
- Do not use benzene, thinner, or gasoline to clean the unit.
- Do not dry Main Unit, AC Adapter, Medication Bottle, and Mesh Cap in a microwave oven or boil them to clean..
- Do not store the unit under high temperature and high humidity, or under direct sunshine.
- Do not leave or carry the Medication Bottle Unit filled with medication or water.
- Remove batteries from the unit when you are not going to use the unit for a long period (approximately three months or more).

1 Pocket-sized unit can be easily carried anywhere you go

The unit is small enough to be placed in a carrying / storage case or purse, you can easily carry it around wherever you go. The new titanium alloy mesh creates a high efficiency aerosol.

2 Inhalation is possible at various angles

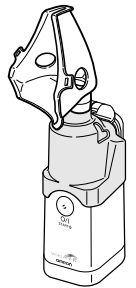
The new bottle structure enables inhalation at various angles with stable nebulization. Therefore, the unit can be used even at bedside.

3 Simple care and easy use

The unit does not need any complex care. Just like the traditional compressor type nebulizer, all you have to do is to remove the medication after inhalation and dry the bottle immediately after washing it.

To start inhalation, you must assemble the unit.

Before starting to use the unit, please read the section on “How to Assemble the Main Unit” and understand the points for assembling. (Refer to Pages 8 and 9)


(This is an image of the unit using an Inhalation Mask.)

# Components of the Product Please check the product.

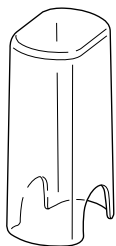
The following items are contained in the box.

If any item is missing, contact the store where you purchased the product or the nearest OMRON dealer.

**Main Unit Cover**

**Main Unit**

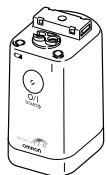**Medication Bottle**

**Mesh Cap (with Mesh)**

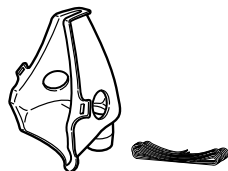
**Mask Adapter**


**Mouthpiece**


**Child's Mask (Optional) 9921**


**Instruction Manual**

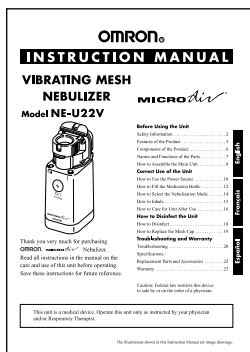
**Instructional Video**

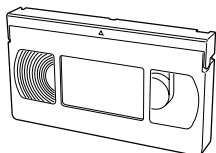**Storage Case**
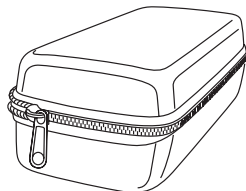
# Names and Functions of the Parts

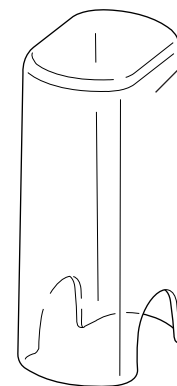**Main Unit Cover**  
Stores the main unit.

**Mesh Cap**  
An alloy mesh is installed for nebulization.  
– Handle mesh with care because it is very delicate.

**Medication Bottle Cap Locking Lever**  
Opens the Medication Bottle Cap for washing.  
– Do not open Medication Bottle Cap except for washing.

**Medication Bottle**

A container to store the medication

**Medication Port**

Fill the medication by opening the Mesh Cap.

**Electrode**

Supplies the power from the Main Unit to the Vibrator of Medication Bottle.

**Battery Low Indicator**

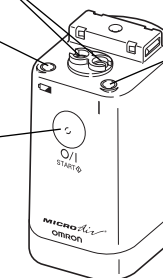
Orange light blinks when the batteries are worn.

**Power Indicator**

A green lamp turns on when the power is on.


**Power Button**

Turns Main Unit on and off.

**Bottom of Main Unit**  
Battery Cover  
Disengaging Lever  
AC adapter Jack on Main Unit

**AC Adapter (Optional)**
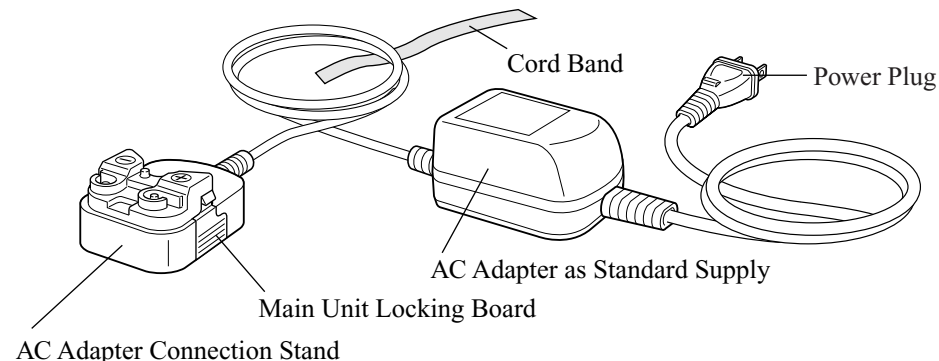
# How to Assemble the Main Unit

# How to Assemble the Main Unit

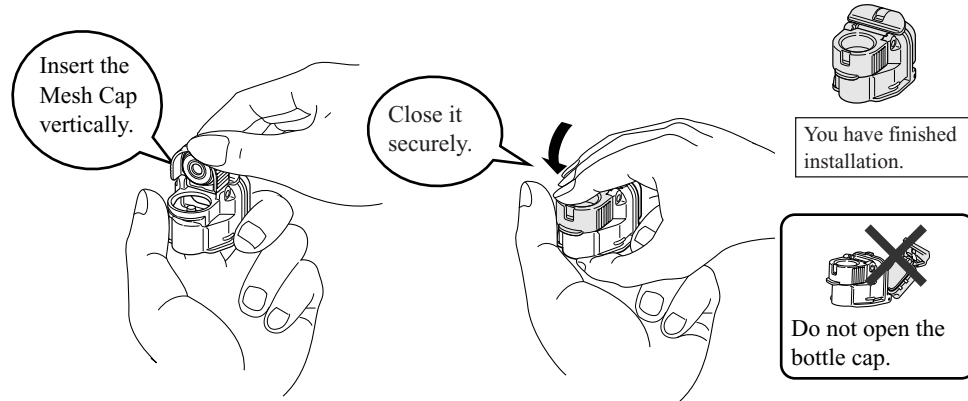
## This unit must be assembled before it is used.

- Parts may fit tightly since they are made to prevent the medication from leaking.
- You must wash and disinfect all removable parts before using the unit.  
Assemble the unit after washing, disinfecting, and drying.  
(For how to disinfect the unit, refer to Page 18)


### General Advice

- Hold the unit securely with both hands.
- Install the parts securely. (You may hear clicks depending on the parts at the time of installation.)


## 1 Attach the Mesh Cap on the Medication Bottle.

## 2 Attach the Medication Bottle to the Main Unit.

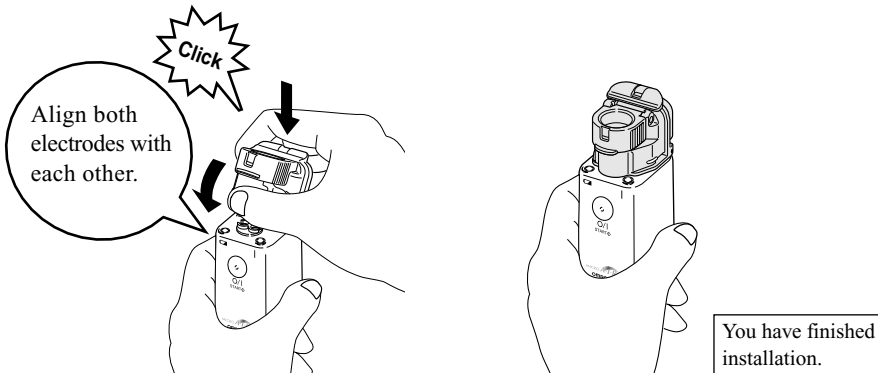
## 3 Attach the Mask Adapter to the Main Unit.

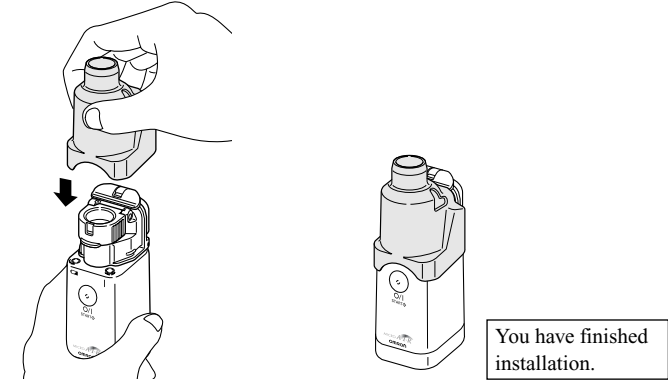
## 4 Attach the supplied Mouthpiece or Child's Inhalation Mask to the Mask Adapter.
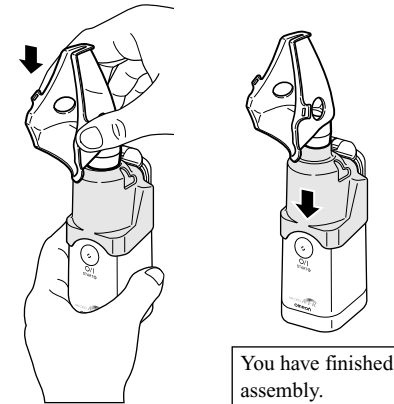
### • How to attach the Child's Inhalation Mask


### • How to attach the Mouthpiece
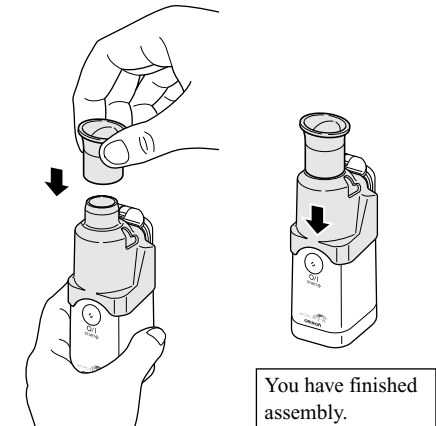
## Now you have completed assembling the Main Unit.

- If you are not going to use the unit soon after it is assembled, disassemble the unit (as shown in the Figure to the right) and store in the supplied Storage Case or store it in clean environment.

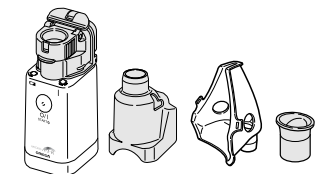(For the correct use of the unit, refer to Page 15)

## How to Insert Batteries

Batteries are not supplied. Use two commercially available alkaline or NiMH AA batteries.

### 1 Remove the Battery Cover.

(1) Slightly rotate the Battery Cover Disengaging Lever in the direction of an arrow.

(2) Remove the Battery Cover.

• The Battery Cover may not be removed easily when you try for the first time in order to prevent water drop from getting inside the Unit.

### 2 Insert the batteries correctly by aligning the polarities (+ and -) with the battery indication marks.

### 3 Put on the Battery Cover.

Put the Battery Cover on the Battery Case. Push in the Battery Cover until it catches the claws and clicks.

Push both ends firmly.

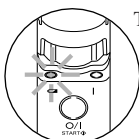
• You may find it difficult to push it through because the unit is made to prevent water from getting inside the Unit.

Bottom of the Main Unit

## Battery replacement and battery life

### When using alkaline batteries

The unit can be used for approximately 8 days (if it is used for 30 minutes a day.)

• The Battery Low Indicator (orange) flashes.

→ Batteries are low. Replace both batteries with new ones.

• The Battery Low Indicator (orange) turns on.

→ Batteries are worn out. As the unit will not nebulize, immediately replace both batteries with new ones.

### When using NiMH batteries

– You may use NiMH batteries (rechargeable batteries).

– If you use a commercially available battery charger suitable for the batteries you are using, you can save money by recharging the batteries again and again.

The unit can be used for approximately 8 days (if it is used for 30 minutes a day when the batteries are fully charged.)

• The Battery Low Indicator (orange) flashes.

→ The rechargeable batteries have no residual power. If the unit does not nebulize, immediately recharge the batteries with a commercially available charger.

• The supplied AC Adapter does not function as a battery charger.

Use a commercially available battery charger to recharge the batteries.

### Batteries

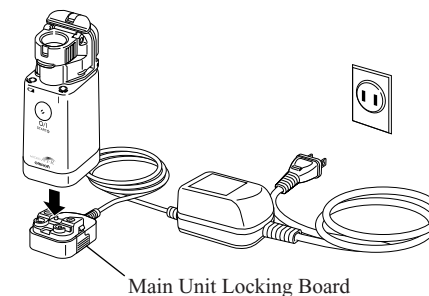
- If you store this unit with the batteries or rechargeable batteries in it, the batteries may leak and damage the unit.
- If you are not going to use the unit for a long period of time (more than three months), remove the batteries.
- Do not use worn and new batteries together or batteries of different types (such as regular batteries and rechargeable batteries) together.
- Follow the disposal method determined by the local community when it comes to disposal of used batteries.

## How to Use optional AC Adapter (optional)

Use the AC Adapter in a place where you have an electric outlet.


However, please note that this AC Adapter is not a battery charger.

### 1 Place the Main Unit on the AC Adapter Connection Stand as shown below.

Note: It will click and be locked to the Stand.

### 2 Plug in the power plug of the AC Adapter to the electric outlet.

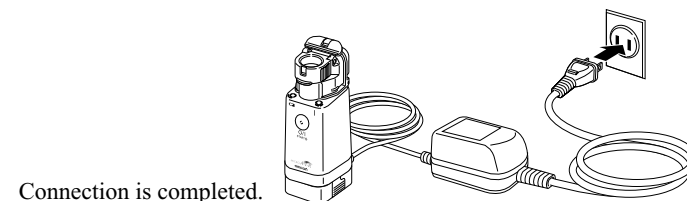
Connection is completed.

## How to remove the AC Adapter from the Main Unit

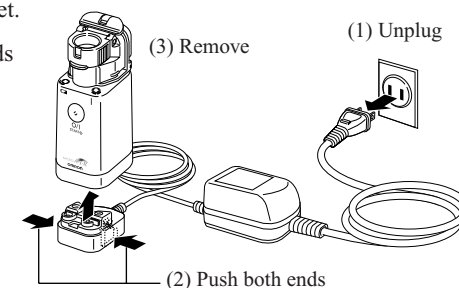(1) Unplug the power plug from the electric outlet.

(2) Unlock the Main Unit by pushing in both ends of the Main Unit Locking Board of the AC Adapter Connection Stand.

(3) Remove the Main Unit.

# How to Fill the Medication Bottle

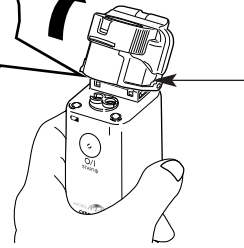
## Have you removed Mask Adapter and the supplied Mouthpiece?

(For how to assemble the Main Unit, refer to Page 8)

**1** Remove Medication Bottle from the Main Unit.

Press gray tabs on both side of Medication Bottle at the same time lift upwards.


Click

**2** Open the Mesh Cap.

Hold the medication bottle as if covering it with your hand.

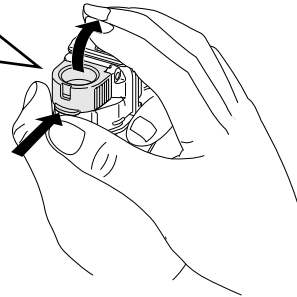Lift up the mesh cap while pressing your thumb hard on the brim.


Do not open the bottle cap.

**3** Fill Medication Bottle.

Be careful to prevent the Mesh Cap from closing as shown in the Figure.

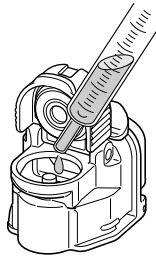
– The maximum capacity of Medication Bottle is 7 ml.

**4** Close the mesh cap.

– Be sure to check that the Mesh Cap is engaged. If it is not engaged, medication will leak.

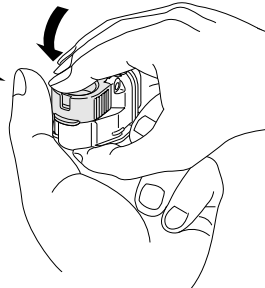Check that the Mesh Cap is closed.


# How to Fill the Medication Bottle



**5** Attach the Medication Bottle to the Main Unit.

- If the Medication Bottle is not installed correctly, the power will not turn on.
- If the Electrodes on the Main Unit are severely stained, the unit will not nebulize normally. In that case, wipe off the stain. (Refer to Page 17)

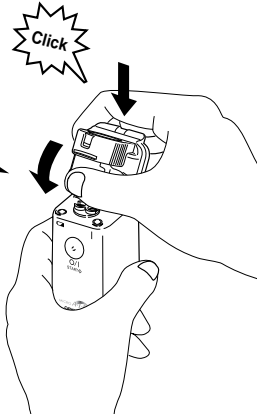Align both electrodes with each other.

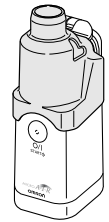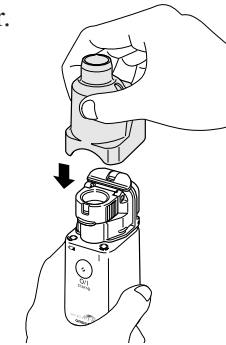**6** Attach the Mouthpiece, Mask Adapter.



You have finished assembly.



**7** Attach the Child's Inhalation Mask or the Mouthpiece.

• How to install the Child's Inhalation Mask

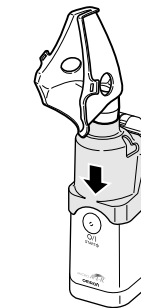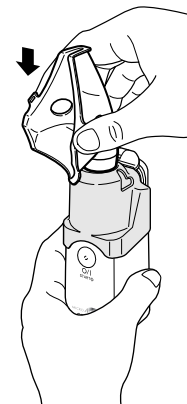• How to install the Mouthpiece

You have finished assembly.

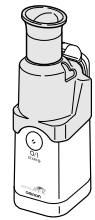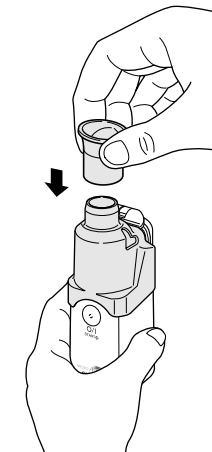You have finished assembly.


# How to Select the Nebulization Mode

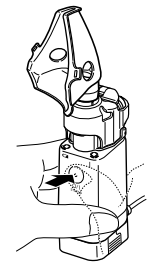You can use this unit either in the continuous nebulization mode or in the manual mode.

## Continuous Nebulization Mode

When you push the Button and immediately release finger from the Button (within approximately 1 second), the unit nebulizes in Continuous Nebulization Mode. When you push the Button again, the unit stops nebulization.


Push the Button and immediately release your finger (within 1 second).

## Manual Nebulization Mode

When you keep pressing the Button (approximately for 2 seconds or more), the unit nebulizes in Manual Nebulization Mode. The unit nebulizes only while you press the Button. You can inhale on demand.

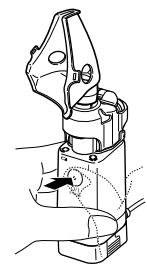Keep on pressing the Button (for approximately 2 seconds or more).

– The power indicator (green light) illuminates during nebulization.


# How to Inhale

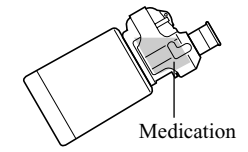**1 Hold the Main unit with your hand and start inhalation in relaxed posture.**


Slightly slant the Main Unit towards yourself.

- Immerse the Vibrator in the medication. If the Vibrator is not immersed, the unit will not nebulize.

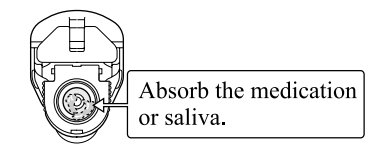- If saliva or medication collects on the electrode of Main Unit or on the Mesh, the unit stops nebulization. In that case, turn off the power and absorb the medication or saliva with tissue paper or a soft cloth.

### Caution

- Do not drop or apply strong pressure to Main Unit, Medication Bottle Unit, or Mesh Cap.
- The unit may be damaged and cannot be used.

**2 When you finish inhalation, push the Button to turn off the power.**

- The unit has a built-in timer to turn off the power in approximately 30 minutes after the power is turned on.
- When an AC Adapter is used, unplug the power plug from the electric outlet. (Refer to Page 11)

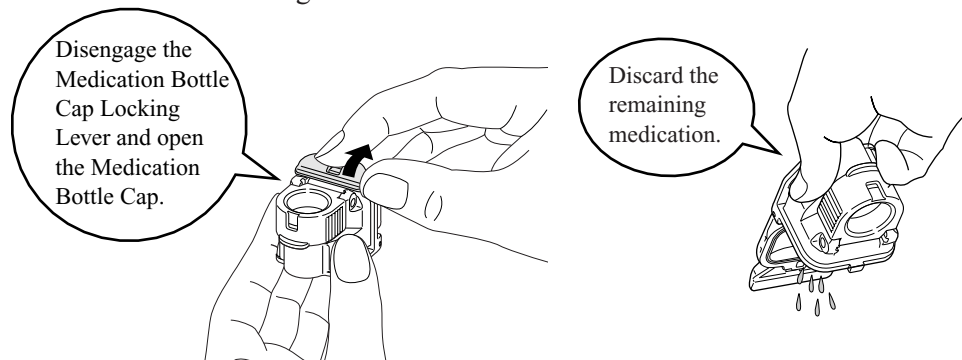
## Be sure to wash and store the unit after use.

If you do not wash the unit after inhalation, the medication will dry and adhere, and may not be nebulized. In that case, disinfect Mesh Cap and wash off dried medication. (Refer to Page 18)

### 1 Remove the parts from the Main Unit.

Remove the Child's Inhalation Mask or Mouthpiece, Mask Adapter, and Medication Bottle from the Main Unit. (For how to assemble the Main Unit, refer to Page 8 to 9)

### 2 Discard the remaining medication.


### 3 Wash the parts.

Wash the Inhalation Mask or Mouthpiece, Mask Adapter, and Medication Bottle in the water.

#### General Advice

Do not wash Medication Bottle and Mesh Cap under strong running tap water.

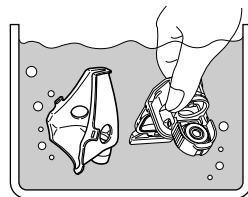- It may cause damage.


### 4 Dry parts thoroughly.

Immediately after washing the parts, wipe off excess water with a dry cloth, and allow to air dry.

Remove the Mesh Cap from the Medication Bottle and air dry. (Refer to Page 8)

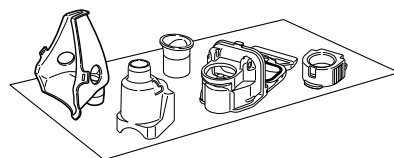– Do not touch the Mesh with your hand.

### 5 After assembling the unit, store it in a clean environment.

#### Warning

Wash and disinfect Medication Bottle, Inhalation Mask, Mouthpiece, and Mask Adapter after each use. (Refer to Page 16)  
Be sure to dry the cleaned and disinfected parts promptly, and store them without getting re-contamination.

#### When the main unit is dirty

Wipe the stain on the Main Unit with a soft cloth. If the Main Unit is heavily stained, wipe it with a soft cloth moistened with water or detergent and squeezed well. Then wipe it with a dry soft cloth.

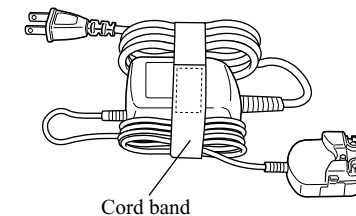
#### General Advice

- Do not clean the Main Unit with volatile chemical such as benzene or thinner.
- Do not immerse the Main Unit in Water.

## How to Carry the Unit

- You can put Mouthpiece and Inhalation Mask in the storage case to carry them with the unit.
- Put a Main Unit Cover over the Main Unit when carrying the unit.
- Do not put the AC Adapter in the storage case along with the Main Unit.  
The Main Unit may be scratched.

Bundle the power cord of the AC Adapter and fasten it to the Main Unit of the AC Adapter with the cord band.

#### General Advice

- Do not leave or carry the Medication Bottle filled with medication or water.
- The medication may drip or spill and damage the Main Unit.  
Do not store the unit under high temperature, high humidity, or in direct sunlight.

# How to Disinfect

The Micro-Air® nebulizer can be cleaned and disinfected easily because the Medication Bottle and the Mesh Cap can be removed from the Main Unit. The unit can be disinfected with benzalkonium chloride.

- 1 Make 0.1% dilute solution using benzalkonium chloride, and soak the Medication Bottle and Mesh Cap in the solution for 10 minutes.
- 2 Take the Medication Bottle and Mesh Cap out of solution to rinse it with water.
- 3 Wipe off the remaining water for faster drying.
- 4 If using commercial cleaners, follow manufacturers instructions.

## Caution

Do not use disinfection solution containing sodium hypochlorite.  
• The unit may not be able to be used after getting covered with rust.

- 5 The following parts can be disinfected:

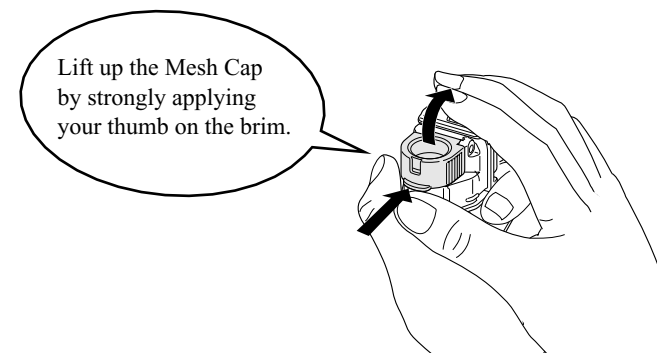
- Medication Bottle
- Mesh Cap
- Mask
- Mouthpiece
- Mouthpiece Adapter

# How to Replace the Mesh Cap


You can usually use the Mesh Cap for approximately 1 year (according to the test results of OMRON). However, this may vary depending on medication type and individual use and care. In some cases, the unit may not nebulize or may nebulize weakly even after disinfection is done.

**If the unit does not nebulize or nebulizes poorly even after cleaning the Mesh Cap,** purchase a new Mesh Cap from your nearest Omron dealer or where you purchased your unit from. The part number is U22-4.


- 1 Open the Mesh Cap.

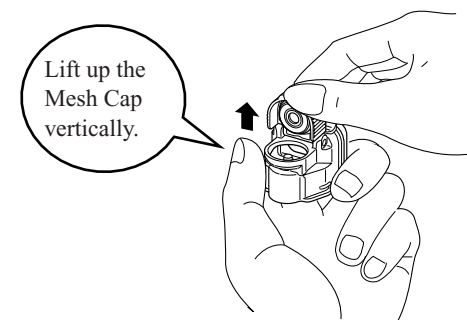- 2 Remove the Mesh Cap from the Medication Bottle.

- 3 Install a new Mesh Cap.

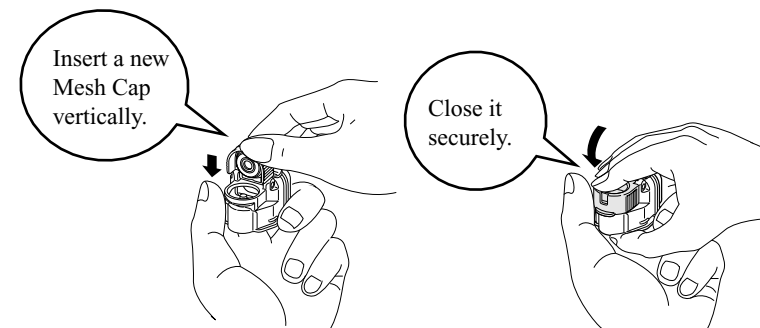If a problem occurs while you are using the unit, please check the following information before calling.

## The nebulization rate is extremely low, or the unit does not nebulize at all.

Where to Inspect	How to correct
Is the Battery Low Indicator turned on?	Replace the batteries with new ones. (Refer to Page 10)
Is the Mesh dirty?	Clean the Mesh Cap by disinfecting to remove any stains. (Refer to Page 18) • If the trouble is not corrected by the above mentioned procedure, replace the Mesh cap with a new one. (Refer to Page 19)

## The Power Indicator does not turn on and the unit does not nebulize.

Where to Inspect	How to correct
Are the polarities of batteries or rechargeable batteries placed in the wrong direction?	Insert batteries or rechargeable batteries in the correct alignment. (Refer to Page 10)
Are the batteries worn?	Replace the batteries with new ones. (Refer to Page 10)
Is the capacity of the rechargeable batteries (NiMH) low?	Recharge the batteries with the commercially available recharger or replace them with new ones.
Is the AC Adapter securely plugged into the electric outlet or the connection port of Main Unit?	Insert it correctly. (Refer to Page 11)
Is the Medication Bottle installed to the Main Unit correctly?	Install it correctly. (Refer to Page 13)

## The Power Indicator is illuminated, but the unit does not nebulize.

Where to Inspect	How to correct
Is the Medication Bottle filled with medication?	Fill with medication. (Refer to Page 12)
Is the Mesh broken, stained, or extremely soiled?	Clean the Mesh Cap by disinfecting. (Refer to Page 18) • If the unit does not nebulize even after carrying out the above-mentioned procedure, or the Mesh is broken, replace the Mesh Cap with a new one. (Refer to Page 19)
Is the medication being supplied to the Medication Bottle? (Is the medication in contact with the Vibrator?)	Use the unit slightly slanting the Main Unit (the side with the Button) towards yourself. (Refer to Page 15)
Has any medication or water collected on the electrode of the Main Unit?	Absorb the medication or water with a soft cloth. (Refer to Page 15)
Is the electrode of the Main Unit stained?	Wipe off the stain. (Refer to Page 17)

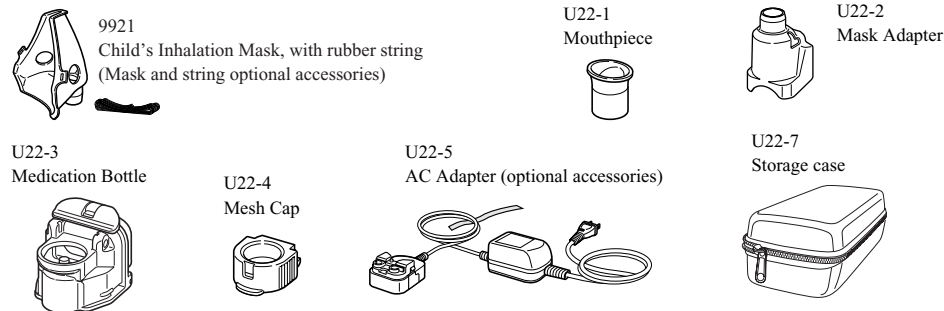
# Specifications

## Specifications

Product name:	OMRON Micro-Air® Vibrating Mesh Nebulizer
Model:	<b>NE-U22V</b>
Power supply:	110 VAC and 50/60 Hz (With the use of an AC Adapter) 3 VDC (two AA alkaline batteries) 2.4 VDC (two AA NiMH batteries)
Power consumption:	Approx. 1.5 W*
Vibration frequency:	Approx. 180 kHz
Nebulization rate:	0.4 to 0.5 mL/min average*
Particle size:	MMAD approx. 5µm* (1 to 5 µm for 80% of the entire volume particle distribution) (Measured by OMRON based on CEN/TC215)
Capacity of Medication Bottle:	Maximum of 7 mL
External dimensions:	38 (W) x 104 (H) x 51 (D) mm
Weight:	Approx. 97 g (not including batteries)
Battery life:	Approx. 4 hours with continuous operation* Approx. 8 days when used about 30 minutes a day (with the use of alkaline batteries)
Operating temperature and humidity:	10 to 40°C (No ice formation is allowed.) 30 to 85% RH
Accessories:	Child's Inhalation Mask, Mouthpiece, Mask Adapter, Storage case, AC Adapter, Instruction Manual

\* When physiological saline is used. These figures may vary depending on the type of medication.  
Specifications and appearance may be subject to change without prior notice for improvement.

## Replacement Parts and Accessories

For the inquiry on where to purchase the replacement parts and accessories, contact the store where you purchased the unit or the nearest OMRON dealer.

# Warranty

Your NE-U22V Omron Nebulizer is warranted to be free from defects in material and workmanship for a period of 2 years from the date of purchase. This warranty extends only to the original retail purchaser. This warranty does not cover failures resulting from misuse, abuse, alteration of the Omron Nebulizer System, use of improper components or failure to comply with the operating instructions. The 2 year warranty excludes mesh cap and accessories.

To make a warranty claim, return your Omron Nebulizer System, freight prepaid and insured (or assume the risk of loss or damage during shipment) to Omron Healthcare, Inc., 300 Lakeview Parkway, Vernon Hills, IL 60061, Attention: Service Department. All requests for warranty service should include the model number of your Omron Nebulizer System, proof of purchase (including date of purchase), your phone number and the nature of your service request. Omron will either repair or at its option replace free of charge any parts necessary to correct defects in material or workmanship covered by this limited warranty.

This limited warranty is in lieu of all other warranties, whether oral, written, express or implied. There are no warranties of merchantability or fitness for a particular purpose, except for those implied warranties arising by operation of law which are limited in duration to the 2 years term of this warranty. Omron's liability, whether based on contract, tort, warranty, strict liability, or any other theory, shall not exceed the price of the individual Omron nebulizer system which is the basis of the claim. In no event shall Omron be liable for any loss of profits, facilities, equipment or other incidental, consequential or special damages.

Some states do not allow limitations on how long an implied warranty lasts, or allow the exclusion or limitations on how long an implied warranty lasts, or allow the exclusion or limitation of incidental, consequential, or special damages, so the above limitations or exclusions may not apply to you. This warranty gives you specific legal rights. You may also have other rights which vary from state to state.

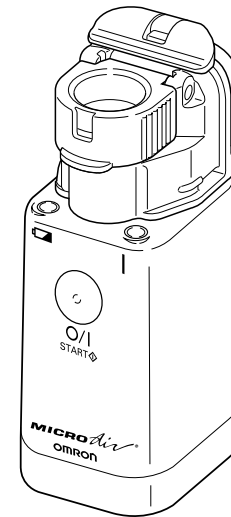
**For Customer Service**  
**Call Toll Free: 1-800-634-4350**

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Distributed by:  
**OMRON HEALTHCARE, INC.**  
300 Lakeview Parkway  
Vernon Hills, Illinois 60061  
Toll Free Customer Service: 1-800-634-4350  
www.omronhealthcare.com  
Copyright © 2002 Omron Healthcare, Inc.

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