

# GUARDIAN<sup>®</sup>

## *Guardian<sup>®</sup>* Tilt Automatic Overbed Table User Instructions

Model IC-6418

### **⚠ WARNING**

**POSSIBLE DAMAGE OR INJURY.** Be alert of hi/lo adjustment of overbed table when raising bed.

### **ASSEMBLY INSTRUCTIONS (FIGURE 1)**

1. Using the hex bolts and washers, attach the base to the post. Tighten the bolts securely.
2. Insert the tilt control bar into the post by loosening post screw. Tighten post screw, once tilt control bar has been inserted into post.
3. Attach the outer tube to the top board using the screws.
4. Slide the outer tube onto the tilt control bar. If it is too tight, release the rotating knob on the tilt control bar.
5. Tighten the rotating knob to set the top board.
6. Insert the casters into the base until they lock into place.

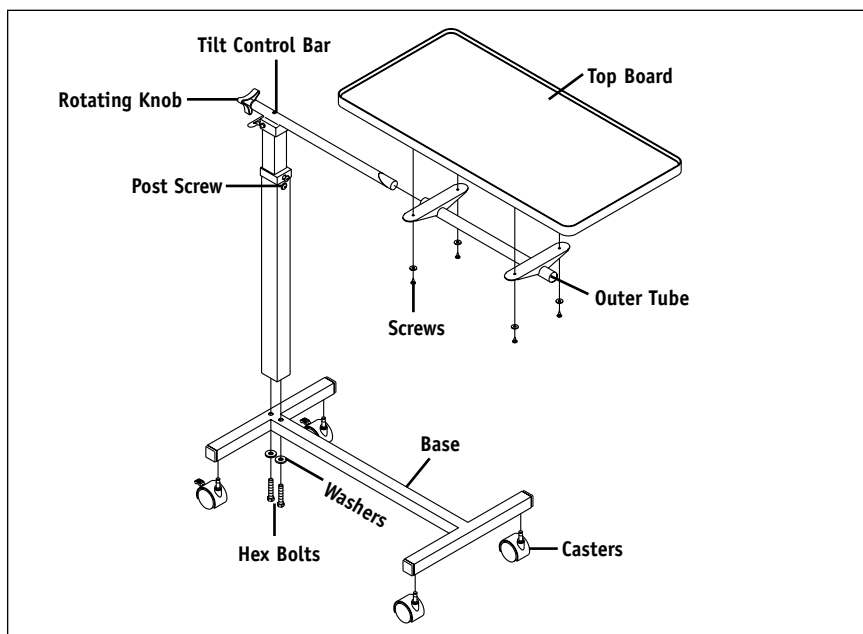


Figure 1

## OPERATING INSTRUCTIONS (FIGURE 2)

1. Remove fix screw located on the table post to release the inner tube and outer tube of the post.

**CAUTION**– Stabilize the top board as you remove the fix screw so that top board will not bounce back too quickly.

2. To raise the top board, place your hand underneath the top board and move it upward to the desired height. Remove pressure to set height.
3. To lower the top board, lift the adjustable plate and push top board downward to the desired height. To set the height, release the adjustable plate.
4. To rotate the top board, twist the knob of the tilt control bar to loosen. Rotate the top to the desired position. Tighten the knob to set the position.

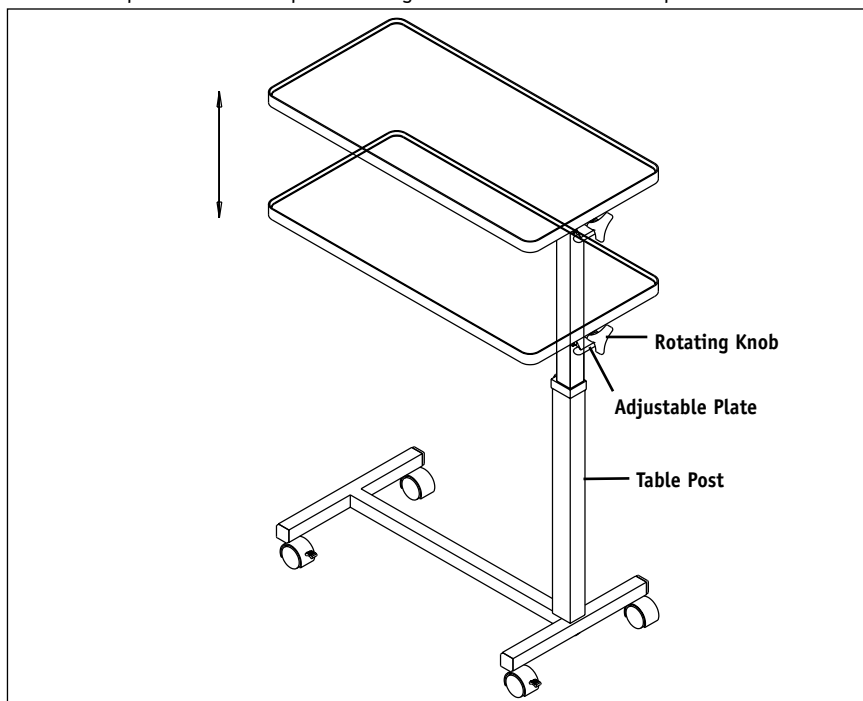


Figure 2



Manufactured for:  
**Sunrise Medical, Inc.**  
7477 East Dry Creek Parkway  
Longmont, Colorado 80503

[www.sunrisemedical.com](http://www.sunrisemedical.com)  
Phone: (303) 218-4500  
Toll Free Phone: (800) 333-4000  
In Canada: (800) 263-3390