

# Deko 190 Loading & Operating Instructions

**Before Loading** → Wear gloves → check chamber for soil & debris → Rotate jets to spin freely

## Commode Pots



1. Push horizontal rack up to dump commode contents. Bedpan rack must be flat against the door.



2. Lower horizontal rack and position pot over jet – note angle. Maximum 1 pot per load.

## Bedpans & Urinals



1. Flip horizontal rack up as shown above. Pull bedpan rack on door until spring locks rack in place.



2. Position bedpans with contents on hooks as illustrated. Slide urinals over jets so clip catches. Close door & contents will empty automatically.

## Cycle Operation



1. Use **Cycle 5** only – **DO NOT use any other cycle**



2. Wipe outside contact surfaces with disinfectant wipe e.g. Percept

**DO NOT interrupt the cycle for any reason**  
**DO NOT turn the Power Button to the OFF position**

### Cycle Lights:

- Red Light = Cycle failed to complete – items not safe for use – must re-run
- Orange Light = Cycle not yet complete, door cannot be opened
- Green Light = Cycle complete, all items are safe for use

## Trouble Shooting

- ❖ If soap alarm sounds: this indicates you must replace the soap bottle
- ❖ If washer leaks or displays an error: discontinue use and ask your unit clerk to submit a work request to FMO