

Cleaning and storage

1. If you do not plan to use the device for a long time, empty the reservoir before storing.
2. If MyPurMist has been in storage for an extended period of time, fill the water reservoir with MyPurFill and let the device run for 5 minutes to prevent any odor.
3. MyPurMist patented technology will not transmit germs in the mist that you breathe. However, periodically you may clean the outside of the device with soap and water. Make sure the device is unplugged while cleaning the outside.
4. Do not put the device in a dishwasher or immerse in water. Do not use abrasives to clean the device.
5. You may periodically remove the mask from the device and clean with soapy water.

Troubleshooting

There are no user serviceable parts inside the device. If MyPurMist is not functioning properly consult the troubleshooting guide available on www.mypurmist.com

One year limited warranty

MyPurMist comes with a limited warranty against defective material or workmanship for a period of one year from the date of purchase. Any defective part will be, at the discretion of the manufacturer, repaired or replaced at no charge if the device has not been tampered with and has been used according to this user manual. Retain your receipt to prove warranty. This warranty gives you specific legal rights, as well as other rights that may vary from state to state.

Vapore, Inc.

1130 Burnett Avenue, Ste. P
Concord, CA 94520
U.S.A.

© MyPurMist and MyPurFill are
trademarks owned by Vapore, Inc.



*MyPurMist shown with
optional child mask*

For your safety

When using electrical appliances, basic safety precautions should always be followed, including the following:

READ ALL INSTRUCTIONS

- Use MyPurMist (device) only for its intended use as described in this manual.
- To protect against a risk of electric shock, do not use device if it is wet or is immersed in water or other liquid. Do not use or store device where it may fall in water (such as a tub or sink).
- The device should always be turned "OFF" and be silent (see: How to Use" section) before unplugging from outlet.
- Do not operate device with a damaged cord or if the device has been dropped or damaged.
- Keep device and cord away from heated surfaces. Position cord to avoid pulling or tripping. Do not wrap cord tightly around device.
- Always unplug the device when filling or emptying the reservoir and when not in use.
- Use MyPurFill Demineralized Water in the device. Do not operate the device without liquid. Do not overfill. Using any additives in the water reservoir, any other liquid or water with minerals will damage the device and will void the warranty.
- Do not use device without the water reservoir cap in place.
- **Do not allow children under the age of 7 to operate the device unsupervised.** Hold the device for the child and let them put their face into the mask in a manner that is comfortable.
- Only use manufacturer supplied facemasks, and water reservoir cap.
- Do not attempt to fix device yourself. If it needs repair or replacement, contact the manufacturer.
- Do not use device near anything inflammable.
- During use do not block the air vents on the sides or bottom of the device such as wrapping the product in a blanket.
- Never insert any object or body part into any opening.

SPECIAL INSTRUCTIONS

- Polarized Plug: This device has a polarized plug (one blade is wider than the other). It will only fit into a polarized outlet one way. If it does not fit your outlet, contact a qualified electrician. Do not attempt to modify the plug in any way.
- Extension Cords: If an extension cord is absolutely necessary, use an approved extension cord. Care should be taken to arrange the cord so that it cannot be pulled or tripped over.

Contact us at customersupport@mypurmist.com or visit
www.mypurmist.com for further information.

mypurmist.com | 1-855-MYPURMIST

MyPurMist®
handheld steam inhaler

User manual



See back for safety information


How to Use MyPurMist

1. Pour MyPurFill Demineralized Water in reservoir and let sit for 30 seconds. For subsequent uses fill water reservoir before each use. Press the ON button.
2. Attach mask by tilting it forward and catching the lower lip of the mask under the tab in the warm mist opening [figure 1]. Do not push the mask straight down.
3. Plug the device into a standard wall outlet through the supplied power cord.
4. Press ON/OFF button for instant steam.
5. Use the temperature Up and Down button to vary the temperature between 105°F to 115°F (one to four blue lights).

6. Use MyPurMist by placing mask over nose and mouth lightly. Breathe comfortably through nose or mouth.
7. MyPurMist turns off automatically after 25 minutes of use. For longer use, refill reservoir with MyPurFill Demineralized Water and turn the device on again.
8. Each time after the device is turned OFF, the fan will run for 10 min to dry all the internal parts of the device. Do not unplug device when the fan is running.



Figure 1

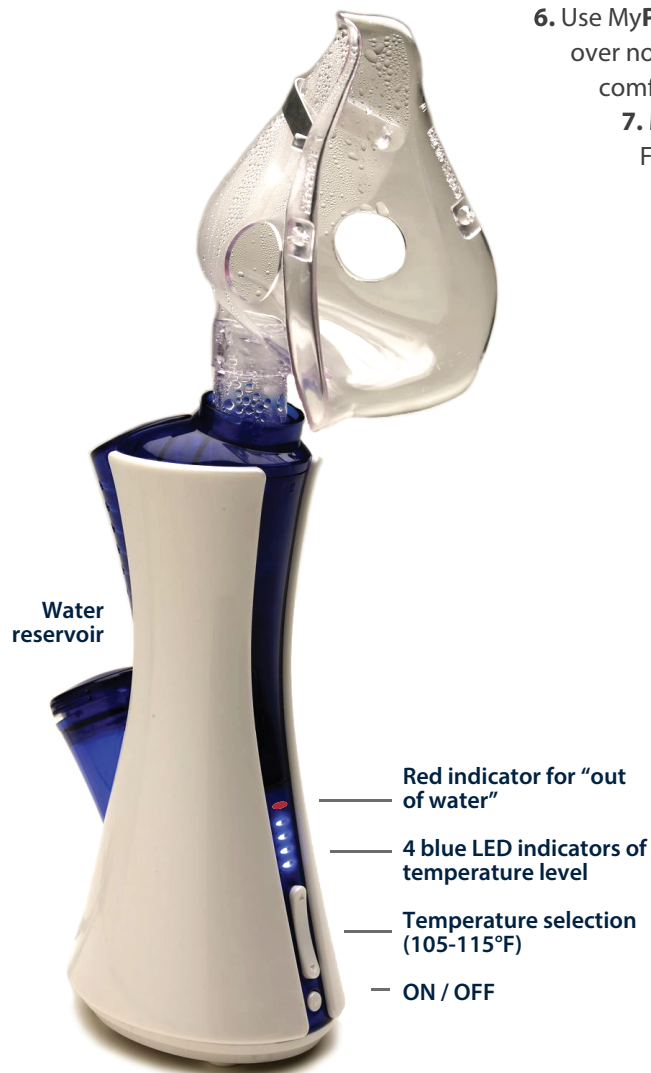

When to Use MyPurMist

1. MyPurMist can be used at anytime and for any length of time. It is the user's responsibility to determine when to use the device and the appropriate length of use.
2. A full water reservoir provides 25 minutes of use.
3. Natural steam is doctor recommended for sinus congestion, sinusitis, allergies, allergic asthma, common cold, flu, cough, croup, nosebleeds, dry mouth, snoring from sinus congestion and sinus headaches. Doctors and medical institutions recommend use of steam 2-4 times a day for symptomatic relief. **Consult your doctor.**
4. MyPurMist can be used at anytime of the day. Using MyPurMist in the evening or before sleeping will likely result in a more restful sleep.
5. MyPurMist can be used anywhere at home or office or while traveling - even overseas (110V to 240V). Remember to bring MyPurFill demineralized water.



MyPurFill Demineralized Water

Use MyPurFill Demineralized Water for optimal long-lasting operation of the device.

DO NOT use tap water or bottled drinking water as these have minerals, which will damage the device and void the warranty.

- We recommend using MyPurFill demineralized water, which is available in convenient smaller bottles. MyPurMist uses only 1.5 oz. of water to generate 25 minutes of very fine soothing steam – so the cost of using MyPurFill is less than a penny a minute. However, Distilled water is a type of purified water that could be appropriate to use in MyPurMist. Do not use other types of “purified water”.



Masks

Available masks include one for adults and one for children.



Contact us

Email: customersupport@mypurmist.com

Web (store): www.mypurmist.com

Toll-free call: 1-855-MYPURMIST

(1-855-697-8764)