YOU-Q QLASS

GEBRUIKERSHANDLEIDING
USER MANUAL
MANUEL D'UTILISATION
BEDIENUNGSANLEITUNG
BRUGSANVISNING
KÄYTTÖOHJE

6000033A

Nederlands	4
English	20
Français	36
Deutsch.....	52
Dansk	68
Suomi.....	84

Nederlands

© 2011 YOU-Q

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© 2011 YOU-Q

All rights reserved.

The information provided herein may not be reproduced and/or published in any form, by print, photoprint, microfilm or any other means whatsoever (electronically or mechanically) without the prior written authorization of YOU-Q.

The information provided is based on general data concerning the construction known at the time of the publication of this manual. YOU-Q executes a policy of continuous improvement and reserves the right to changes and modifications.

The information provided is valid for the product in its standard version. YOU-Q cannot be held liable for possible damage resulting from specifications of the product deviating from the standard configuration.

The available information has been prepared with all possible diligence, but YOU-Q cannot be held liable for possible errors in the information or the consequences thereof. YOU-Q accepts no liability for loss resulting from work executed by third parties.

Names, trade names, etc. used by YOU-Q may not, as per the legislation concerning the protection of trade names, be considered as being available.

2011-04

Table of contents

1	Introduction	22
	This manual	22
	Approval	22
	CE declaration of conformity	22
	Warranty	22
2	General description	22
	2.1 Product description QCLASS	22
3	Adjustment options (for qualified specialist)	23
	3.1 Seat depth	23
	3.2 Backrest angle	23
	3.3 Seat width	24
	3.4 Armrests	24
	3.5 Side panel height	25
	3.6 Central leg rest	25
	3.7 Central legrest gas spring adjustment	26
	3.8 Standard legrests	27
	3.9 Comfort legrests	28
	3.10 Headrest	28
	3.11 Fixed joystick holder	29
	3.12 Swing-away joystick holder	29
	3.13 Hip belt	30
4	Adjustment options	31
	4.1 Central leg rest gas spring adjustment	31
	4.2 Comfort leg rests	31
	4.3 Headrest	31
	4.4 Swing-away joystick holder	32
5	Getting in and out of the wheelchair (transfer)	32
6	Reducing the wheelchair in size	33
	6.1 Removing the seat cushion	33
	6.2 Folding down the backrest	33
	6.3 Removing the armrest	34
	6.4 Removing the leg rest	34
	6.5 Removing the headrest	34
7	Maintenance	35
	7.1 Maintenance table	35
	7.2 Cleaning the upholstery	35

1 Introduction

This manual

With this manual you can safely use and maintain (clean) the QLASS seating system. This manual is a supplement to YOU-Q's wheelchair manual. In case of doubt always contact your dealer.

Strictly follow the procedures with a caution symbol! These procedures can - if they are not executed properly - result in human injury or damage to the product or the environment.

Approval

The product meets the following requirements:

1. EN12183 (1999) Hand-operated wheelchairs or EN12184 (1999) Electrically driven wheelchairs.
2. ISO7176-8 Requirements for impact, static and fatigue strength.
3. ISO7176-9 Climate tests for wheelchairs and scooters.
4. ISO7176-16 Requirements of resistance to fire.
5. ISO 7176-19 Crash test.
6. EN-ISO 10993-5 (1999-11) Requirements of bio-compatibility.

CE declaration of conformity

The product is in conformity with the provisions of the Medical Devices Directive and thus has CE marking.

Warranty

The warranty stipulations applicable to the seating system are identical to and incorporated in the wheelchair manual.

2 General description

2.1 Product description QLASS

The wheelchair is equipped with the QLASS seating system. The QLASS seating system provides excellent support for the head, torso, arms and legs. Optimal body support ensures maximum independence for the wheelchair user.

The wheelchair must be adjusted by a qualified specialist. Incorrect settings may have negative consequences for the sitting posture.

Please study this manual carefully before you use the wheelchair.

3 Adjustment options (for qualified specialist)

3.1 Seat depth

Back tubes (fig 1)

1. Loosen the Allen bolts (A) on both sides of the seat frame a half turn (180°) with a 4 mm Allen key.
2. Slide the back tubes (B) in the desired position.
3. Tighten the Allen bolts (A) securely.

Make sure that there is at least 2 cm of free space between the back of the knee and the front of the seat to ensure good blood circulation.

Seat cushion (fig 2)

1. Pull the seat cushion upward to remove it from the seat.
2. Place the clamps (B) in the right position (A).
3. Click the seat cushion onto the seat frame.

Check if the seat is clicked in place securely.

3.2 Backrest angle

Backrest (fig 3)

1. Loosen the bolts (A and B) on the left side of the seat frame with a 13 mm spanner.
2. Select the desired backrest angle.
3. Tighten the bolts (A and B) securely.

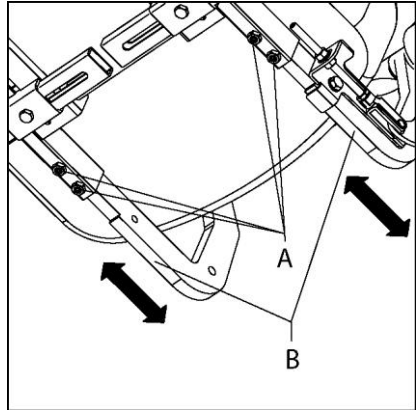
Figure 1

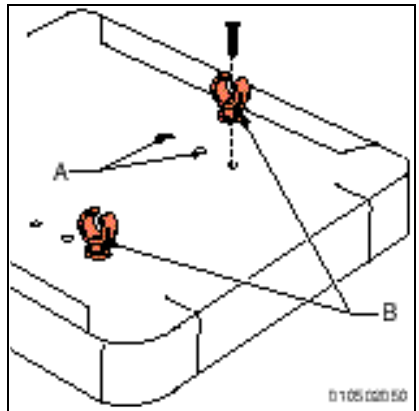
Figure 2

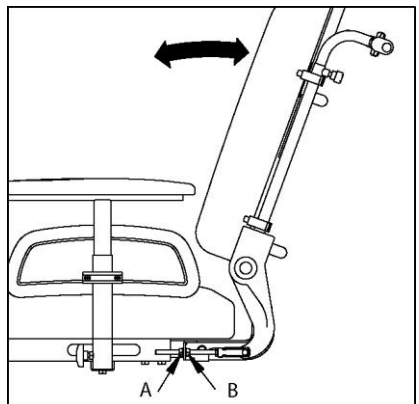
Figure 3

3.3 Seat width

XP seat width (fig 4)

1. Loosen the bolts (A) on both sides of the seat frame. Use a 13 mm spanner.
2. Slide the armrests (B) to the desired position.
3. Tighten the bolts (A) securely.

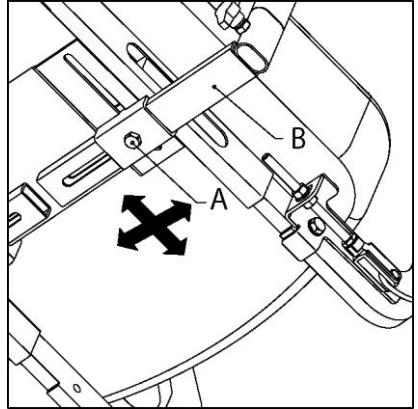
Figure 4

Beagle/Alex seat width (fig 5)

1. Loosen the two set screws, one on either side of the seat frame, using Allen key 4.
2. Slide the armrests into the desired position.
3. Tighten the bolts on each side securely.

3.4 Armrests

XP armrest depth (fig 4)

1. Loosen the bolts (A) on both sides of the seat frame. Use a 13 mm spanner.
2. Slide the armrests (B) in the desired position.
3. Tighten the bolts (A) securely.

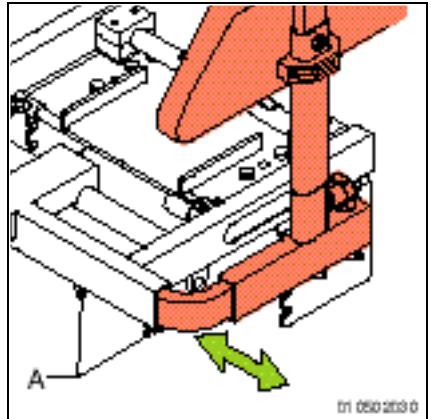
Figure 5

Beagle/Alex armrest depth (fig 6)

1. Loosen the bolts (A) on both sides of the seat frame using Allen key 4.
2. Slide the armrests (B) into the desired position.
3. Tighten the bolts securely.

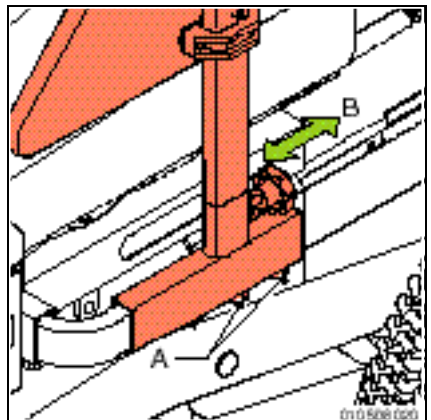
Figure 6

Armrest height (fig 7)

1. Loosen the Allen bolt (A) with a 4 mm Allen key.
2. Slide the armrests (B) in the desired position.
3. Tighten the Allen bolt (A) securely.

Make sure that the arms are supported well, without the shoulders being pushed upward.

3.5 Side panel height

Side panel height (fig 7)

1. Loosen the crosshead screws (C) with a few turns with a Phillips screwdriver.
2. Slide the side panel (D) in the desired position.
3. Tighten the crosshead screws (A) securely.

3.6 Central leg rest

Lower leg length (fig 8)

1. Loosen the Allen bolt (A) by a few turns using Allen key 5.
2. Adjust the length of the leg rest (B).
3. Tighten the Allen bolt securely.

Ensure that there is sufficient free space under the footplate for the wheelchair to drive over obstacles.

Leg rest depth (fig 9)

1. Loosen the Allen bolt (A) by a few turns using Allen key 6.
2. Slide the leg rest into the desired position.
3. Tighten the Allen bolt securely.

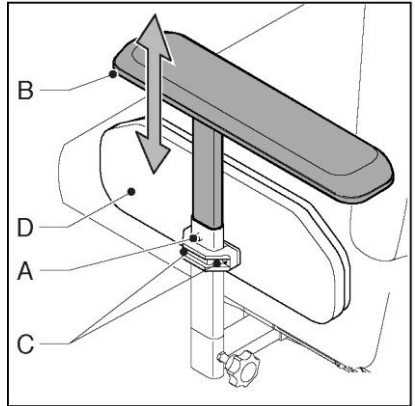
Figure 7

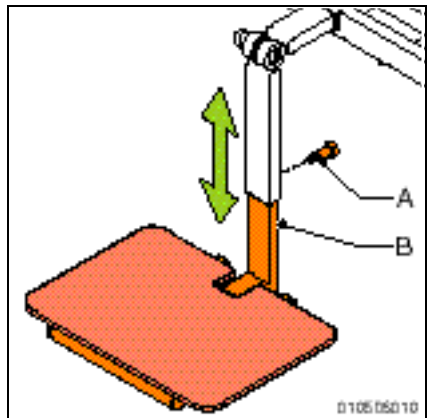
Figure 8

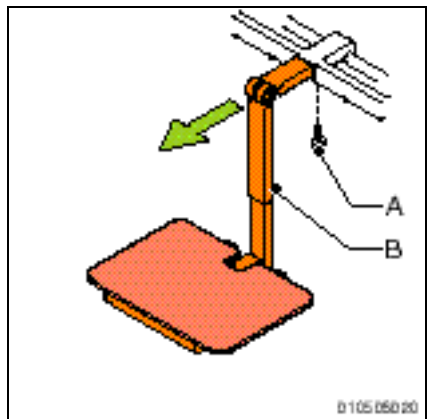
Figure 9

Adjustment options (for qualified specialist)

Knee angle (fig 10)

1. Using Allen key 5, loosen Allen bolt (A) to such an extent that the hinge teething is released, allowing the leg rest to be fully adjusted. Hold the cap nut (B) in place using ring spanner 13 while doing this.
2. Adjust the leg rest to the right angle (C) and push the hinge back into place by hand.
3. Tighten the Allen bolt securely.

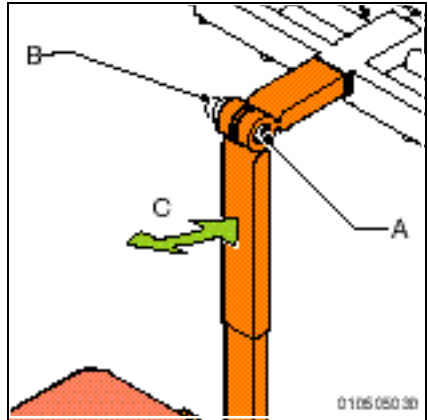
Figure 10

3.7 Central legrest gas spring adjustment

Lower leg length (fig 11)

1. Loosen the knob (B).
2. Slide the footplate (A) in the desired position.
3. Tighten the knob (B) securely.

Make sure that there is sufficient free space under the footplate to operate the wheelchair over obstacles.

Legrest depth (fig 12)

1. Loosen the Allen bolt (A) a few turns with a 5 mm Allen key.
2. Slide the legrest in the desired position.
3. Tighten the Allen bolt (A) securely.

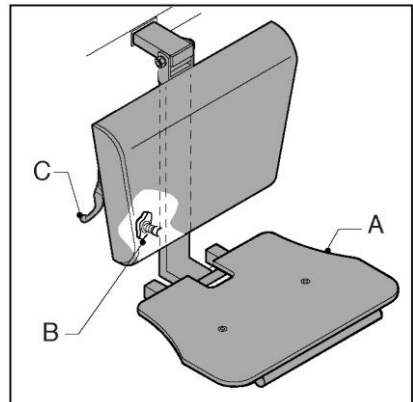
Figure 11

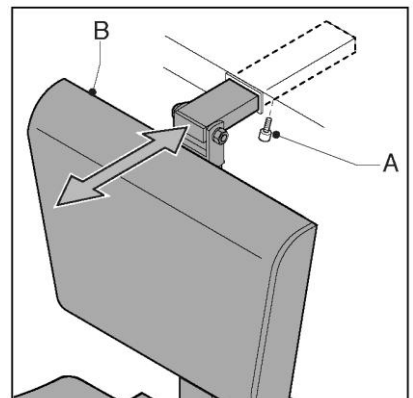
Figure 12

3.8 Standard legrests

Lower leg length (fig 13)

1. Loosen the setscrew (A) a few turns with a 4 mm Allen key.
2. Slide the footplate (B) in the desired position.
3. Tighten the setscrew (A) securely.

Make sure that there is sufficient free space under the footplates to operate the wheelchair over obstacles.

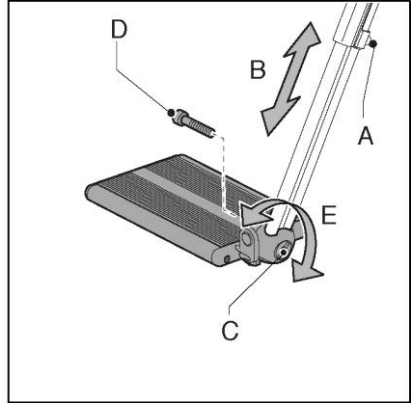
Figure 13

Legrest width (fig 14)

1. Remove the calfstrap.
2. Loosen the setscrews (C) a few turns with a 4 mm Allen key.
3. Place the legrests (D) in the desired position.
4. Tighten the setscrews (C) securely.

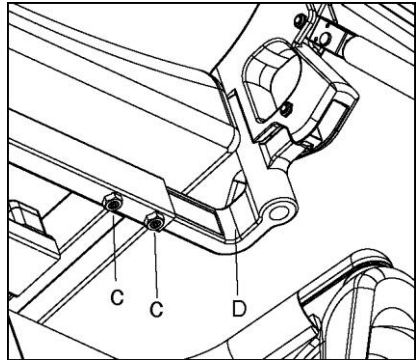
Figure 14

Legrest depth (option) (fig 15)

1. Loosen the Allen bolt (A) a few turns with a 5 mm Allen key.
2. Slide the legrest bracket (B) in the desired position.
3. Tighten the Allen bolt (A) securely.

- **Make sure that the castor wheels do not touch the footplates when manoeuvring. If this is the case, the legrests must be brought forward. There must be at least 1 cm between the castor wheels and the footplates.**
- **Make sure the suspension bracket does not push into the back of the knees causing discomfort. If this is the case, the seat must be adjusted forward.**

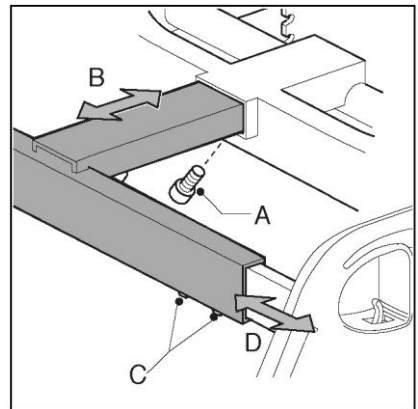
Figure 15

Adjustment options (for qualified specialist)

Footplate angle (fig 16)

1. Loosen the Allen bolt (C) a few turns with a 6 mm Allen key, so that the teeth of the hinge come completely free.
2. Adjust the desired footplate angle (D) and push the hinge manually together again.
3. Tighten the Allen bolt (C) securely.

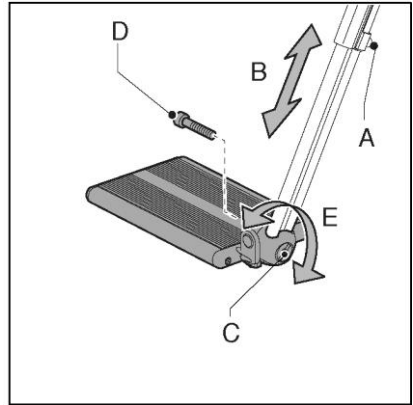
Figure 16

Footplate depth (fig 17)

The footplate is secured asymmetrically. The left and right footplate can be exchanged.

1. Loosen the setscrew (A) with a 4 mm Allen key.
2. Remove the footplate together with the profile.
3. Slide the footplate in the top tube of the other legrest.
4. Tighten the setscrew (A) securely.

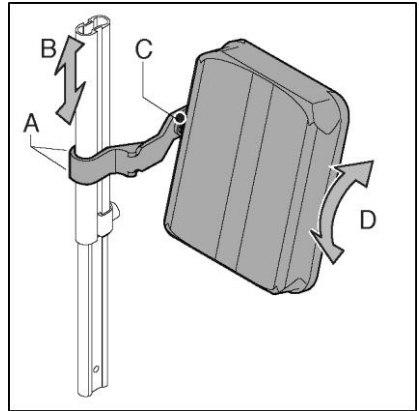
Figure 17

3.9 Comfort legrests

Calf pad height (fig 17)

1. Loosen the Allen bolt (A) a few turns with a 4 mm Allen key.
2. Slide the calf pad (B) in the desired position.
3. Tighten the Allen bolt (A) securely.

3.10 Headrest

Height (fig 18)

1. Loosen the wing nut (A) a few turns.
2. Slide the headrest (B) in the desired position.
3. Tighten the wing nut (A) securely.

Position (fig 18)

1. Loosen the three bolts (C) a few turns with a 10 mm spanner.
2. Adjust the desired position (D).
3. Tighten the three bolts (C) securely.

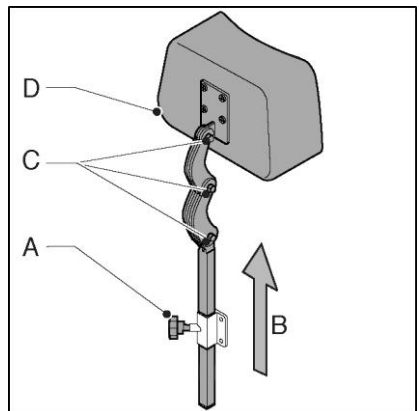
Figure 18

3.11 Fixed joystick holder

Depth (figure 19)

1. Loosen set screw (A) by a few turns using Allen key 4.
2. Slide the control box (B) up into the desired position (forwards or backwards).
3. Tighten the set screw securely.

Height (figure 20)

1. Loosen set screw (A) by a few turns using Allen key 3.
2. Slide the control box (B) up into the desired position.
3. Tighten the set screw securely.

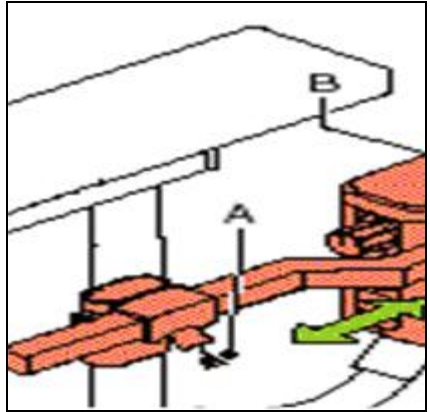
Figure 19

3.12 Swing-away joystick holder

Depth (figure 19)

1. Loosen set screw (A) by a few turns using Allen key 4.
2. Slide the control box (B) up into the desired position (forwards or backwards).
3. Tighten the set screw securely.

Height (figure 21)

1. Loosen set screw (A) by a few turns using Allen key 3.
2. Slide the control box (B) with its swivel mechanism up to the desired height.
3. Tighten the set screw securely.

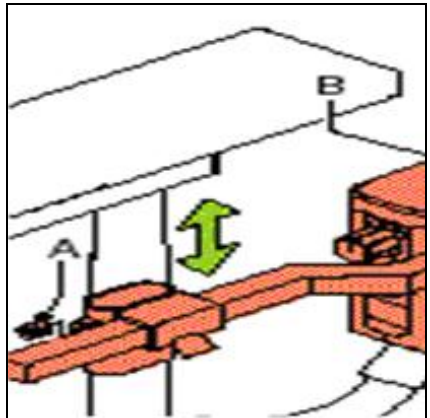
Figure 20

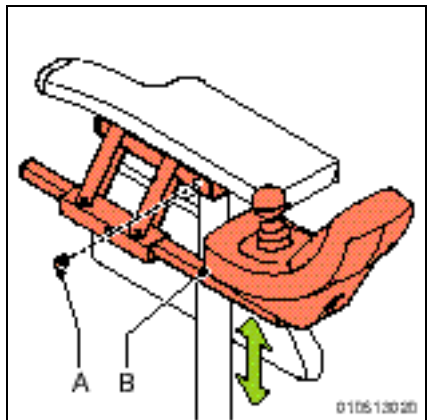
Figure 21

3.13 Hip belt

It is possible for the wheelchair to be fitted with a positioning belt. The hip belt offers extra seating stability under normal use.

The hip belt is attached as follows (figure 22):

1. Attach the belt buckle (A) using a nut and bolt.
2. Pass the belt through the buckle.

- **Check that the hip belt is correctly adapted to the wheelchair user. The belt should lie firmly over the hip at an angle of approx. 45°.**
- **Placing the belt over the hip, against the soft skin of the stomach, can result in the wrong sitting position and the user sliding forward in the wheelchair.**
- **Make sure the belt is clean and that the locking mechanism works at all times. The belt and locking mechanism can be cleaned using a damp cloth.**
- **Never use the hip belt as a safety belt when transporting the user in the wheelchair.**

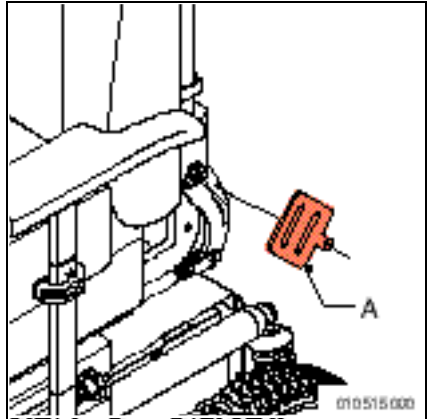
Figure 22

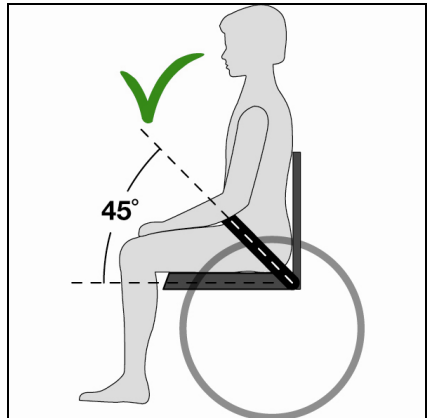
Figure 23 Correct placing of belt

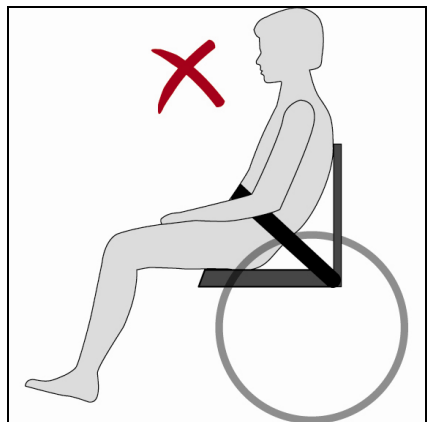
Figure 24 Incorrect placing of belt

4 Adjustment options

4.1 Central leg rest gas spring adjustment

Knee angle (figure 25)

1. Press gas spring handle (C).
2. Adjust the leg rest to the desired position.
3. Release the gas spring (C).

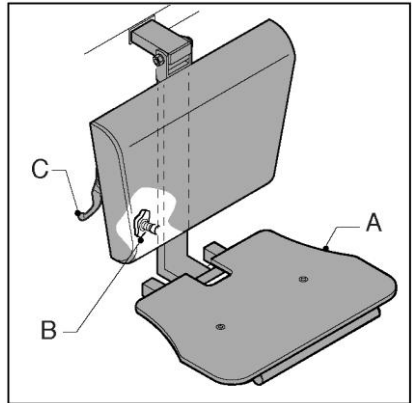
Figure 25

4.2 Comfort leg rests

Knee angle (figure 26)

The angle of the comfort leg rests can be adjusted independently of each other.

1. Push handle (B) forwards.
2. Lift your leg.
3. Release the handle when in the desired position.

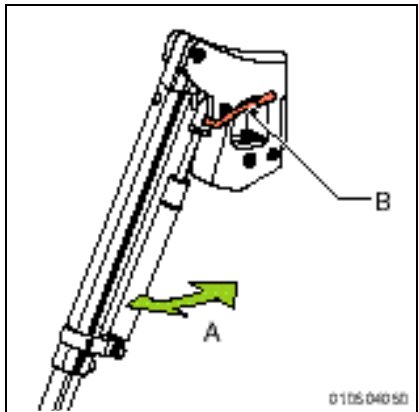
Figure 26

4.3 Headrest

Position (figure 27)

1. Loosen the three clamps by a few turns.
2. Adjust to the desired position.
3. Tighten the three clamps securely.

Figure 27

Getting in and out of the wheelchair (transfer)

4.4 Swing-away joystick holder

Swing away (figure 28)

1. Apply light sideways pressure to the control box (B).
2. The control box swings out parallel to the armrest.

This allows the user to drive close to a table, while still being able to operate all of the functions of the control box.

 **Ensure that nothing gets caught between the swivel mechanism when the control box swings out.**

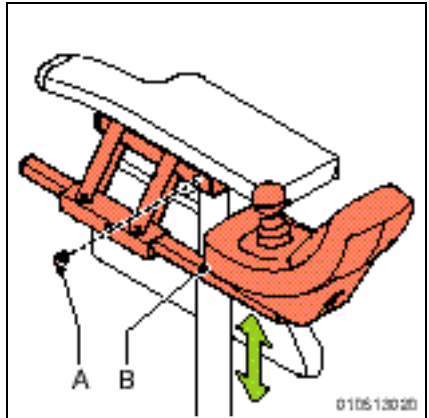
Figure 28

5 Getting in and out of the wheelchair (transfer)

 **Do not stand on the foot-plates. These have not been designed to accommodate the full weight of a person. In addition, this could cause the wheelchair to topple over.**

Forwards transfer (fig 29)

1. Fold up the foot supports
2. Swing the calf plates to the side (if applicable)
3. If possible, swing the legrests to the side
4. Seat yourself in the chair

Sideways transfer (fig 30)

1. Unscrew the cap and remove the armrest
2. Seat yourself in the chair

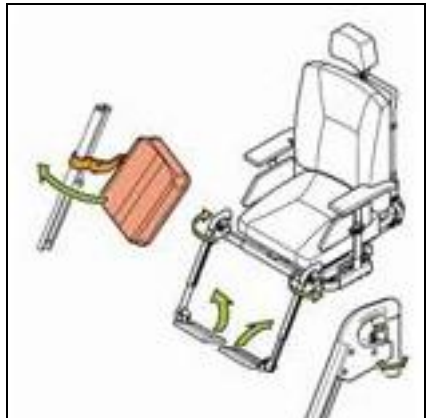
Figure 29

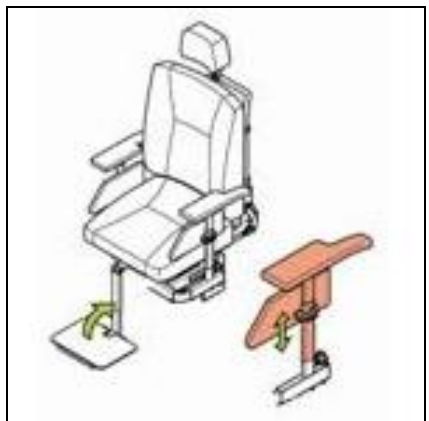
Figure 30

6 Reducing the wheelchair in size

The wheelchair can be reduced in size for transportation without a passenger (figure 31). Any components of the wheelchair that can easily be detached should be removed. Store such components in a safe place. The backrest can also be folded down if the car's boot space is small.

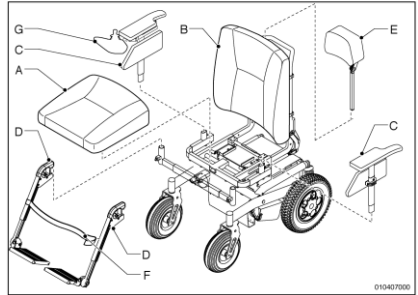
Figure 31

6.1 Removing the seat cushion

1. Lift the seat (A) (figure 32)

When replacing the seat, the clips (B) must click onto the tubing (C) in the seat frame.

Check to see whether the seat has clicked into place.

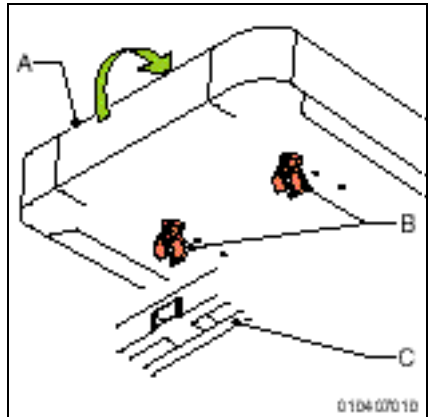
Figure 32

6.2 Folding down the backrest

1. Release the circlip (A) from the adjustment bar (B) (figure 33).
2. Pull the lock pin out of the hinge joints.
3. Fold down the backrest.

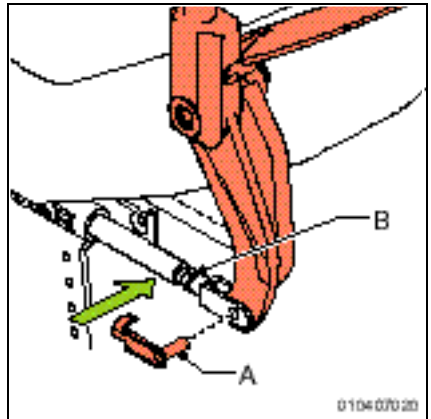
Figure 33

Reducing the wheelchair in size

6.3 Removing the armrest

1. Loosen the star knob (A) (figure 34).
2. Remove the armrest (B) together with the side panel.
3. Unplug the lighting (if the wheelchair is fitted with lights).

The control box remains connected to the wheelchair by its cable.

6.4 Removing the leg rest

1. Loosen the calf band.
2. Push the catch (A) to the rear in order to unlock the leg rest (figure 35).
3. Swing the leg rest outwards.
4. Remove the leg rest (B) from its bracket by lifting.

You should hear a click when replacing the leg rest. This indicates that the leg rest has locked into place.

6.5 Removing the headrest

1. Loosen the star knob (A) by a few turns (figure 36).
2. Slide the headrest (B) upwards to remove.

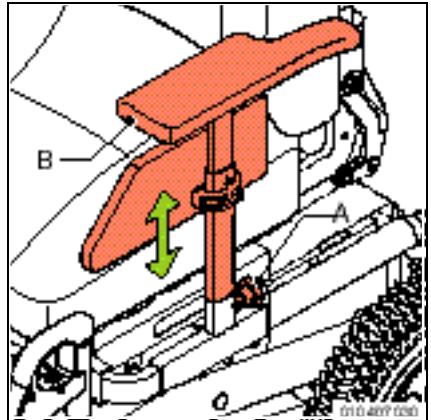
Figure 34

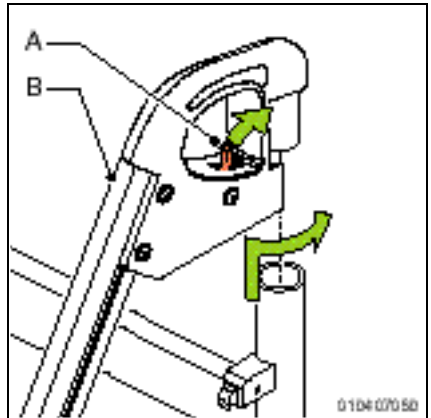
Figure 35

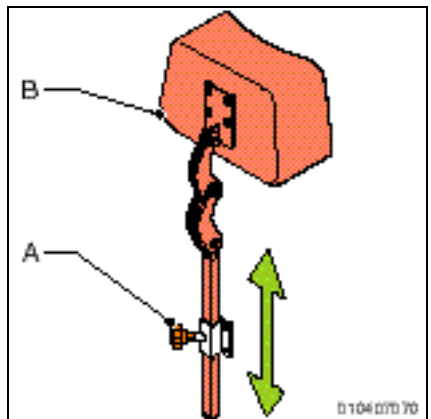
Figure 36

7 Maintenance

7.1 Maintenance table

It is recommended to have your wheelchair serviced by your dealer once a year or, in case of intensive use, every six months. The table below shows the maintenance activities you can perform yourself.

Frequency	Description
Monthly	Cleaning the upholstery, if necessary. See 3.2.

7.2 Cleaning the upholstery

Clean the upholstery with a damp cloth and household soap. Then wipe the upholstery dry with a soft dry cloth.

- **Do not use organic solvents such as thinner, benzene or white spirit.**
- **Do not dry clean, iron or spin-dry the upholstery.**

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YOU-Q B.V.