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© 2008 Handicare

All rights reserved.

The information provided herein may not be reproduced and/or published in any form, by print, photoprint, microfilm or any other means whatsoever (electronically or mechanically) without the prior written authorization of Handicare.

The information provided is based on general data concerning the construction known at the time of the publication of this manual. Handicare executes a policy of continuous improvement and reserves the right to changes and modifications.

The information provided is valid for the product in its standard version. Handicare cannot be held liable for possible damage resulting from specifications of the product deviating from the standard configuration.

The available information has been prepared with all possible diligence, but Handicare cannot be held liable for possible errors in the information or the consequences thereof. Handicare accepts no liability for loss resulting from work executed by third parties.

Names, trade names, etc. used by Handicare may not, as per the legislation concerning the protection of trade names, be considered as being available.

2008-01

Table of contents

Introduction.....	15
This manual	15
1 General description	15
1.1 Product description Sedeo	15
2 Adjustment options	16
2.1 Seat depth.....	16
2.2 Backrest angle	16
2.3 Seat width	17
2.4 Armrests.....	17
2.5 Side panel height	17
2.6 Central legrest.....	18
2.7 Standard legrests.....	18
2.8 Comfort legrests.....	20
2.9 Headrest	21
3 Maintenance	21
3.1 Maintenance table.....	21
3.2 Cleaning the upholstery	21

Introduction

This manual

With this manual you can safely use and maintain (clean) the Sedeo seating system. This manual is a supplement to Handicare's wheelchair manual. In case of doubt always contact your dealer.

Strictly follow the procedures with a caution symbol! These procedures can - if they are not executed properly - result in human injury or damage to the product or the environment.

Approval

The product meets the following requirements:

1. EN12183 Hand-operated wheelchairs or EN12184 Electrically driven wheelchairs.
2. ISO7176-8 Requirements for impact, static and fatigue strength.
3. ISO7176-9 Climate tests for wheelchairs and scooters.
4. ISO7176-16 Requirements of resistance to fire.
5. ISO 7176-19 Crash test.
6. EN-ISO 10993-5 (1999-11) Requirements of bio-compatibility.

CE declaration of conformity

The product is in conformity with the provisions of the Medical Devices Directive and thus has CE marking.

Warranty

The warranty stipulations applicable to the seating system are identical to and incorporated in the wheelchair manual.

1 General description

1.1 Product description Sedeo

The wheelchair is equipped with the Sedeo seating system. The Sedeo seating system provides excellent support for the head, torso, arms and legs. Optimal body support ensures maximum independence for the wheelchair user.

The wheelchair must be adjusted by a qualified specialist. Incorrect settings may have negative consequences for the sitting posture.

Please study this manual carefully before you use the wheelchair.

2 Adjustment options

2.1 Seat depth

Back tubes

1. Loosen the Allen bolts (A) on both sides of the seat frame a half turn (180°) with a 4 mm Allen key.
2. Slide the back tubes (B) in the desired position.
3. Tighten the Allen bolts (A) securely.

Make sure that there is at least 2 cm of free space between the back of the knee and the front of the seat to ensure good blood circulation.

Seat cushion




1. Pull the seat cushion upward to remove it from the seat.
2. Place the clamps (B) in the right position (A).
3. Click the seat cushion onto the seat frame.

Check if the seat is clicked in place securely.

2.2 Backrest angle

1. Loosen the bolts (A and B) on the left side of the seat frame with a 13 mm spanner.
2. Select the desired backrest angle.
3. Tighten the bolts (A and B) securely.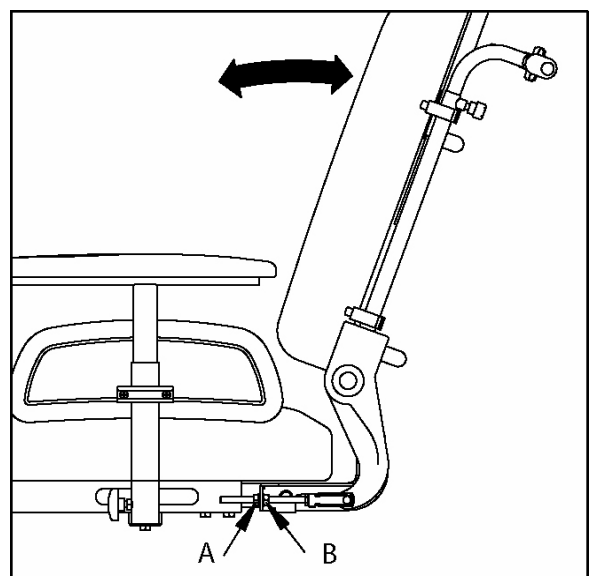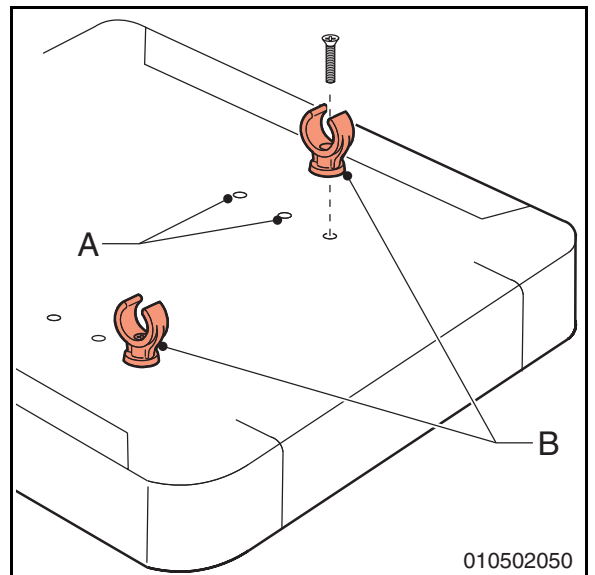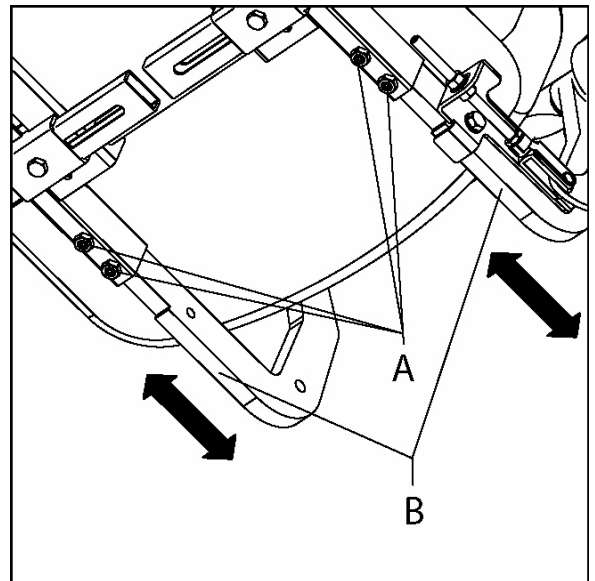

2.3 Seat width

1. Loosen the bolts (A) on both sides of the seat frame. Use a 13 mm spanner.
2. Slide the armrests (B) to the desired position.
3. Tighten the bolts (A) securely.

2.4 Armrests

Armrest depth



1. Loosen the bolts (A) on both sides of the seat frame. Use a 13 mm spanner.
2. Slide the armrests (B) in the desired position.
3. Tighten the bolts (A) securely.

Armrest height

1. Loosen the Allen bolt (A) with a 4 mm Allen key.
2. Slide the armrests (B) in the desired position.
3. Tighten the Allen bolt (A) securely.

Make sure that the arms are supported well, without the shoulders being pushed upward.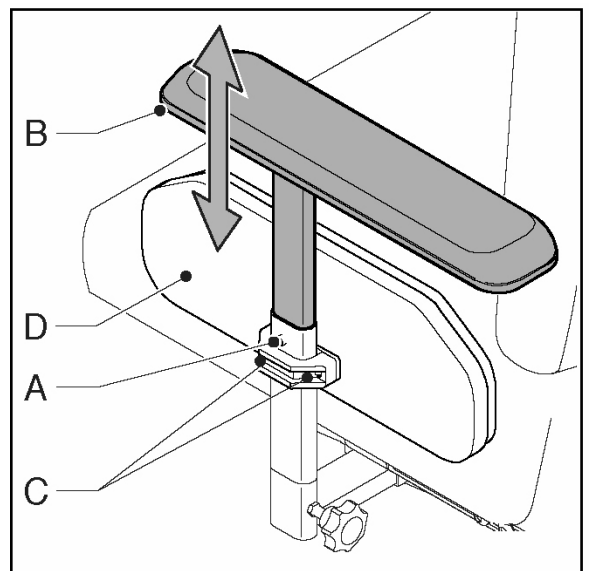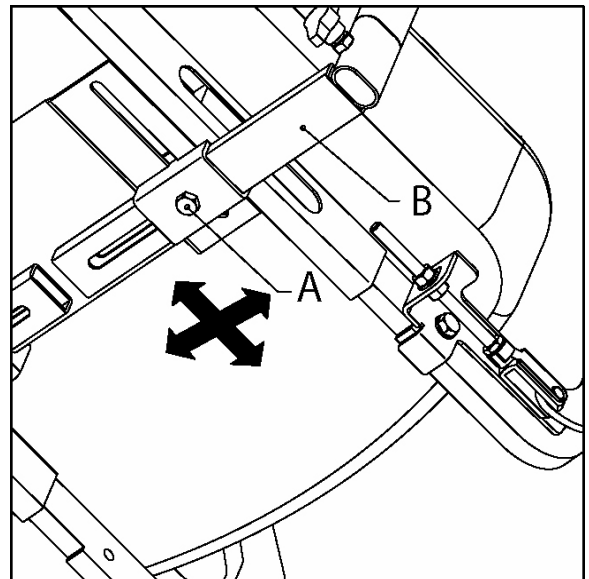

2.5 Side panel height

1. Loosen the crosshead screws (C) with a few turns with a Phillips screwdriver.
2. Slide the side panel (D) in the desired position.
3. Tighten the crosshead screws (A) securely.


2.6 Central legrest

Lower leg length

1. Loosen the knob (B).
2. Slide the footplate (A) in the desired position.
3. Tighten the knob (B) securely.

Make sure that there is sufficient free space under the footplate to operate the wheelchair over obstacles.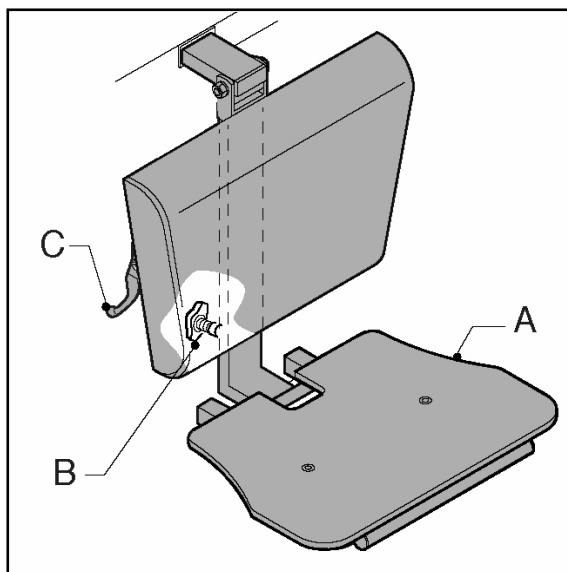


Legrest depth

1. Loosen the Allen bolt (A) a few turns with a 5 mm Allen key.
2. Slide the legrest in the desired position.
3. Tighten the Allen bolt (A) securely.

Knee angle

1. Push the gas spring adjustment handle forward (C).
2. Place the legrest in the desired position.
3. Release the gas spring (C).

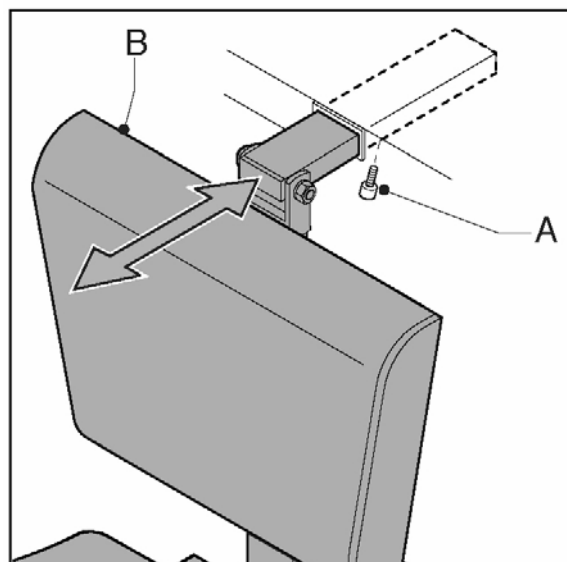
2.7 Standard legrests

Lower leg length

1. Loosen the setscrew (A) a few turns with a 4 mm Allen key.
2. Slide the footplate (B) in the desired position.
3. Tighten the setscrew (A) securely.


Make sure that there is sufficient free space under the footplates to operate the wheelchair over obstacles.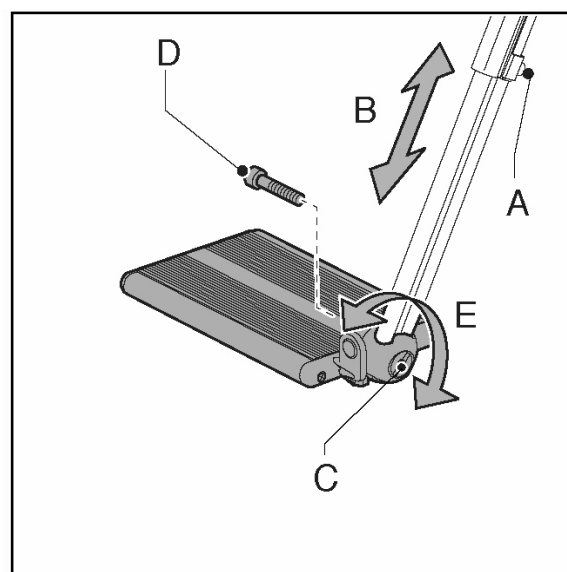

Legrest width

1. Remove the calfstrap.
2. Loosen the setscrews (C) a few turns with a 4 mm Allen key.
3. Place the legrests (D) in the desired position.
4. Tighten the setscrews (C) securely.

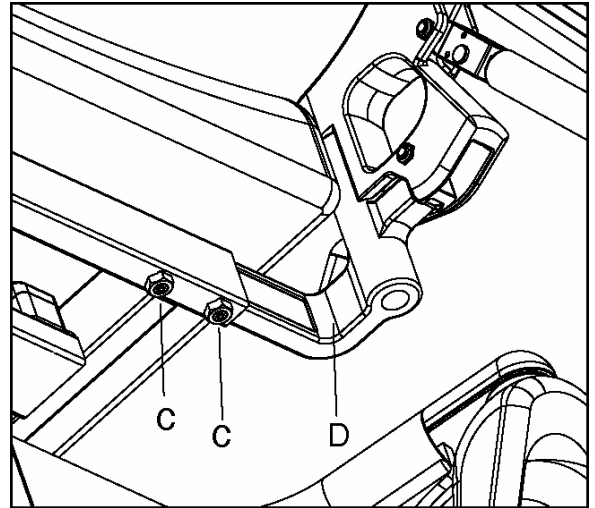
Legrest depth (option)

1. Loosen the Allen bolt (A) a few turns with a 5 mm Allen key.
2. Slide the legrest bracket (B) in the desired position.
3. Tighten the Allen bolt (A) securely.


- **Make sure that the castor wheels do not touch the footplates when manoeuvring. If this is the case, the legrests must be brought forward. There must be at least 1 cm between the castor wheels and the footplates.**
- **Make sure the suspension bracket does not push into the back of the knees causing discomfort. If this is the case, the seat must be adjusted forward.**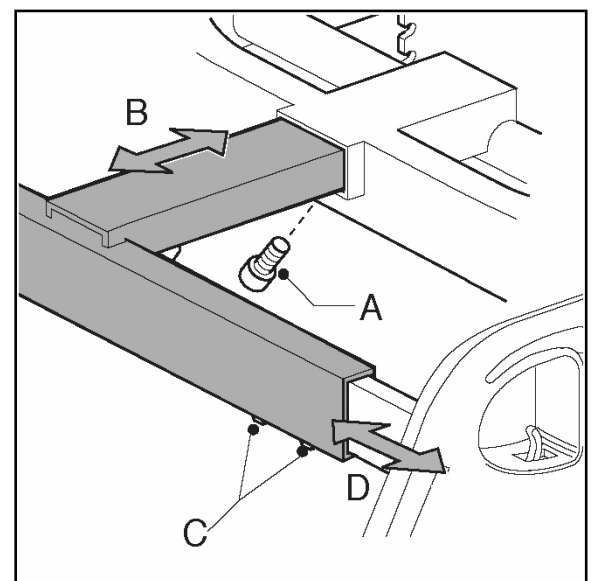

Footplate angle

1. Loosen the Allen bolt (C) a few turns with a 6 mm Allen key, so that the teeth of the hinge come completely free.
2. Adjust the desired footplate angle (D) and push the hinge manually together again.
3. Tighten the Allen bolt (C) securely.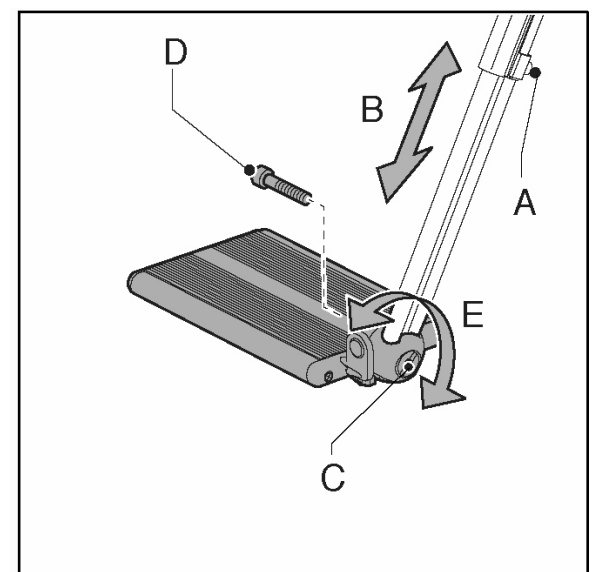


Adjustment options

Footplate depth

The footplate is secured asymmetrically. The left and right footplate can be exchanged.

1. Loosen the setscrew (A) with a 4 mm Allen key.
2. Remove the footplate together with the profile.
3. Slide the footplate in the top tube of the other legrest.
4. Tighten the setscrew (A) securely.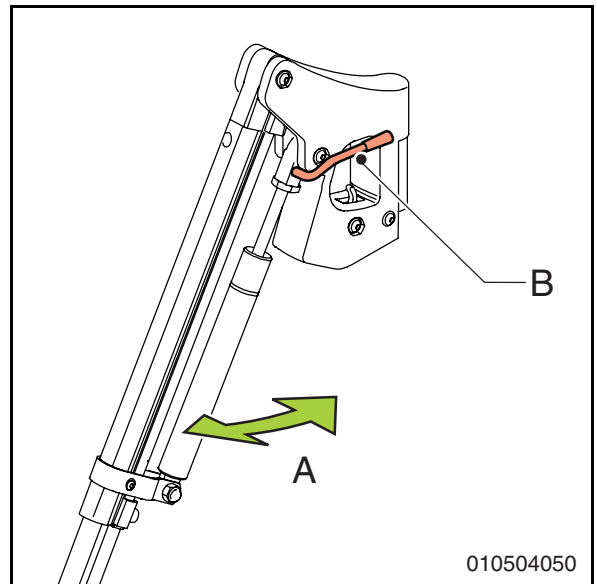


2.8 Comfort legrests

Angle adjustment

The comfort legrests are angle adjustable independently of each other. The gas spring delivers enough force to allow the legrest to go up easily.

1. Push the handle (B) forward.
2. Move your leg upward.
3. Release the handle in the desired position.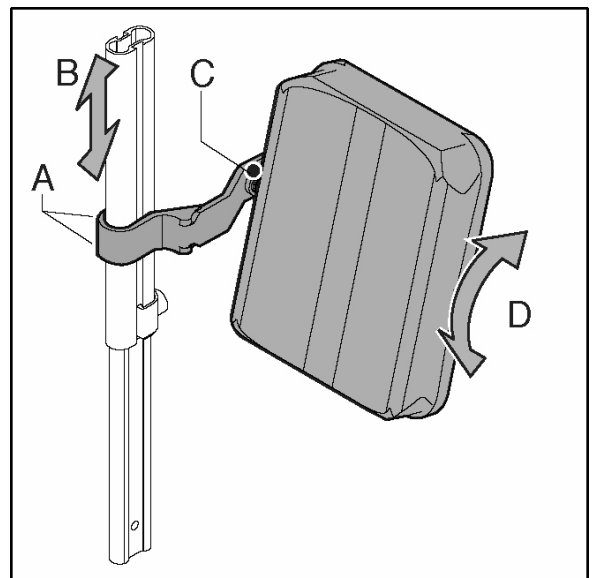

Calf pad height

1. Loosen the Allen bolt (A) a few turns with a 4 mm Allen key.
2. Slide the calf pad (B) in the desired position.
3. Tighten the Allen bolt (A) securely.

Calf pad angle

1. Loosen the nut (C) a few turns with a 13 mm spanner.
2. Place the calf pad (D) in the desired position.
3. Tighten the nut (C) securely.


2.9 Headrest

Height

1. Loosen the wing nut (A) a few turns.
2. Slide the headrest (B) in the desired position.
3. Tighten the wing nut (A) securely.

Position

1. Loosen the three bolts (C) a few turns with a 10 mm spanner.
2. Adjust the desired position (D).
3. Tighten the three bolts (C) securely.

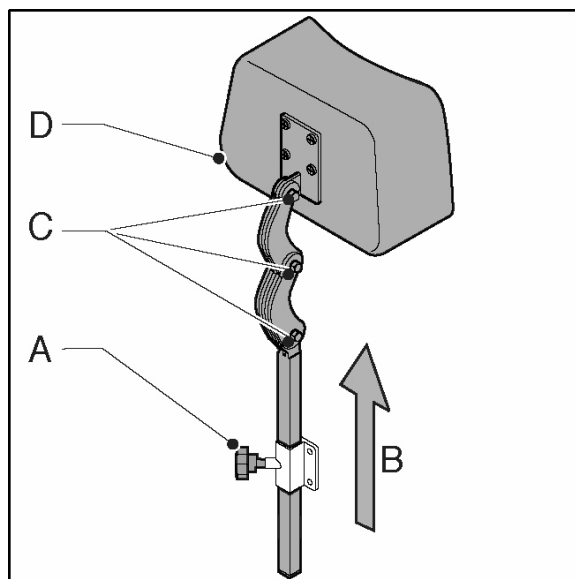3 Maintenance

3.1 Maintenance table

It is recommended to have your wheelchair serviced by your dealer once a year or, in case of intensive use, every six months. The table below shows the maintenance activities you can perform yourself.

Frequency	Description
Monthly	Cleaning the upholstery, if necessary. See 3.2.

3.2 Cleaning the upholstery

Clean the upholstery with a damp cloth and household soap. Then wipe the upholstery dry with a soft dry cloth.

- Do not use organic solvents such a thinner, benzene or white spirit.
- Do not dry clean, iron or spin-dry the upholstery.

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Handicare B.V.