

Size:89x50mm

Warranty

Your vibrating alarm watch comes with one-year limited warranty. We warrant during this warranty period, from the date of purchase, to the original consumer, the watch to be free from defects in materials and workmanship under intended home use. In the event that the watch fails to function properly within one year of original purchase due to defects in materials or workmanship (Batteries Excluded), return the watch (freight prepaid) with proof of purchase (sales receipt or packing slip, no exceptions) to Serene Innovations. Serene Innovations will either repair or replace the unit (with a refurbished unit or unit of equal condition) and return it to you (using UPS/USPS ground shipping) at no cost to you if the unit is returned within 30 days of purchase. If the unit is returned after 30 days of purchase but within the warranty period, there will be a warranty handling charge of \$9.95 for each return.

This warranty does not apply to any watch that has been accidentally damaged, abused, misused or negligent. Defects or errors caused by unauthorized alterations, repairs, and/or tampering are also not covered by this warranty. This Warranty gives you specific legal rights, and you may also have other legal rights that vary from state to state.



SERENE
innovations

Vibrating Alarm Watch
Model:VQ200

www.sereneinnovations.com

Serene Innovations, Inc. 13421 Pumice Street Norwalk, CA.90650 Tel:(562) 407-5400

Vibrating Alarm

When the alarm sounds off, the watch will vibrate for about 15 seconds. If the alarm is not stopped, the vibration will repeat at least two more times at 3 minutes apart before it stops and resets (it will sound off again in twelve hours). To stop the vibration (and to set the alarm OFF), just push the Alarm Knob in (if the knob is pushed while it is vibrating, the vibration may continue momentarily).

Batteries

This watch uses two batteries, one to keep time and one to power the vibration. When the watch looses time or when the vibration turns weak, please have the battery(ies) replaced at any watch repair shop.



Alarm Knob

Off On

1 2 3
Time Knob

To set time

Pull the Time Knob to position 3, then turn knob to set time. When finished, push knob all the way in.

To set alarm time

Pull the Time Knob to position 2, then turn knob to set alarm time. When finished, push knob all the way in.

To set alarm ON/OFF

Pull Alarm Knob out to set alarm ON; push it back to set alarm OFF and to stop the vibrating alarm.