

**Congratulations to your new VoiSec.** We have strived to deliver an easy-to-use quality product in three parts:

- the **VoiSec-button** for recording, storing and playing short speech messages.
- the soft rubber **holder** is used to prevent involuntary recording and to hold the VoiSec-button. The button should always be placed in the holder except when recording or changing batteries.
- the **attachment means** are used to attach the holder to other objects, e.g. the lid of a tin. VoiSec is delivered with double-sided adhesive patches but other kinds of attachment means are available.

### 1. Playing a message



VoiSec should be placed in the holder when to play. Press the lid to play a recorded message. The **VoiSec Multimessage** will play all messages in a sequence from the start. Press again to stop playing.


### 2. Recording a new message



- Remove VoiSec from the holder, turn it over and keep it pressed together. Do not block the little protruding spike or the little opening next to it.

- Wait for two beeps before talking. Talk straight towards the bottom surface of the button at a distance of approximately 10 - 20 cm.
- When the button is released, a two-beep signal and the recorded message should be played.
- The **VoiSec Singlemessage** will erase the previous message when recording a new one.
- The **VoiSec Multimessage** will add one message after another until the memory is full. After that, a new recording will erase all previous ones and start from the beginning again.


### 3. Changing batteries

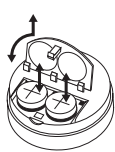

N.B.! **Changing batteries will not erase** any recorded messages.

- Open the battery cover at the bottom of the button by pressing down at the notch on the outside. Remove the cover.
- Insert two type SR44 batteries with the flat end facing outwards and the protruding end inwards. We recommend using batteries from Libego AB (available from your dealer).
- Re-mount the cover and close well.
- Put the VoiSec-button back into the holder.

### 4. Attaching VoiSec to other objects

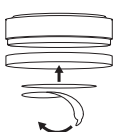

- Use the included adhesive patches or other attachment means from the accessories assortment.
- Attach the patch at the center underneath the holder.
- Make sure the surface to be stuck to is dry and clean.
- Remove any protecting material.
- Press the holder and the attachment means against the object to be stuck to.

## 5. Error handling

**VoiSec is pressed but nothing is heard.**

- No message is recorded.
- The button is not fully depressed.
- The batteries are not properly inserted.
- The battery cover is not fully closed.
- The batteries are finished or the wrong type. When the batteries are beginning to run out of energy, VoiSec may have difficulties in instant re-playing. The button will manage about twenty more play-outs with some pausing inbetween.
- The little protruding spike underneath is not fully depressed by the holder.
- VoiSec has been shortcut by humidity. Remove the holder, the battery cover and the batteries. Let it dry and try again with fresh batteries.

**The sound volume is too low.**

- The message was recorded with too low a voice, the button facing the wrong way, beside or far away from the mouth.
- The hearing distance is too far.

*The national warranties apply, contact your dealer if you suspect a manufacturing error.*

## 6. Technical facts

Measures: diameter 42 mm, height w. tray 18 mm.

Capacity: maximum 70 seconds.

Runtime: typically 400 playouts of a 30 second message (volume dependant).

Power: two 1.5 V type SR44 buttoncell batteries.

VoiSec may be cleaned with a dampened cloth but not immersed in water.

This product has been CE-certified by an authorised laboratory.

## 7. Disposal

Used batteries should be disposed of in accordance to local regulations.

VoiSec should be handled as electronic waste. The plastics may be recycled.

## 8. Accessories

Batteries, holders and different products for fastening or wearing VoiSec may be ordered. Contact your dealer or see [www.voisec.se](http://www.voisec.se) for up-to-date information.

## 9. Consumer contact

Please contact your point-of-purchase or

**Libego AB**

**Teknikringen 7,  
SE-583 30 Linköping,  
Sweden**

**E-mail: [info@libego.se](mailto:info@libego.se)**

**Web: [www.voisec.se](http://www.voisec.se)**

**Phone: +46(0)13 342 01 60**

**Fax: +46(0)13 342 01 61**

VoiSec is patented, a registered trade mark and the design is protected.