INVACARE INTERNATIONAL

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France: Les Roches, F-37230 Fondettes.

United Kingdom: South Road, Industrial Estate, Bridgend - UK - CF31 3PY.

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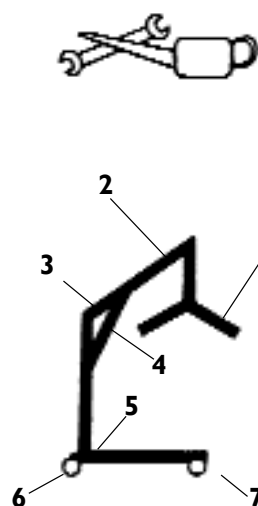
Fig 4

WARRANTY

Your Reliant Lifters are warranted for a period of 36 months from date of purchase, which the exception of the battery, which is warranted for 12 months from date of purchase, subject to the following conditions;

1. Only INVACARE products at full price are warranted against defective workmanship and materials.
2. If a defect or fault is discovered, the dealer from whom the appliance was purchased should be notified immediately.
3. The Manufacturer will not accept responsibility for damage caused by misuse or the non-observance of instructions set out in the user's guide.
4. During the period of warranty, any parts that have become defective due to faulty workmanship or materials, will be renewed or repaired without charge by the supplying dealer.
5. This warranty will be forfeited should any unauthorised alteration be made to the product.
6. Your statutory rights as a customer are not affected.

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RELIANT 100/150/200/250

USER GUIDE INTRODUCTION

Our INVACARE range of mobile lifters is one of the most comprehensive in Europe. It covers a wide spectrum of user needs from a basic, compact hydraulic model to a high-performance heavy-duty electrically operated model.

All our lifters are built to the most stringent high-quality standards where SAFETY and RELIABILITY are the overriding criteria. They are easy to operate and user-friendly, both for the patient and the attendant. Please read this manual carefully before you start to use the lifter. It contains useful tips and information on safety, operation and maintenance.

In the event of any problems or queries, do not hesitate to contact the local INVACARE dealer who will be at your service for advice and assistance.

This manual is subject to change without prior notice in the interest of product improvement.

CONTENTS

	Page
GENERAL SPECIFICATION	1
ASSEMBLY	1
OPERATING INSTRUCTIONS	2
Adjustment of Chassis Width	2
Brakes	2
Moving the Lifter	2
Attaching Slings	2
Raising/Lowering	2
Reliant 100 & 200	2
Reliant 150 & 250	3
BATTERY CHARGING	3
MAINTENANCE	3
General	3
Lubrication	3
Chassis Assembly	3
Chassis Adjustment Mechanism	4
Wear or Damage	4
Cleaning	4
TROUBLESHOOTING	4
WARRANTY	BACK COVER

ASSEMBLY

The Lifter in the Invacare range are designed for easy assembly. The only tools required are an adjustable spanner and a 5mm Allan key.

CAUTION

**TAKE THE UPMOST CARE WHEN LIFTING COMPONENTS DURING ASSEMBLY
ALWAYS REMEMBER TO ADOPT THE CORRECT LIFTING POSITION. ASSEMBLED
LIFTER WEIGHTS ARE AS FOLLOWS:**

REL100-32KG, REL150-34KG, REL200-34KG, REL250-36KG.

To Assemble

1. Remove all parts from the carton.
2. Place the chassis assembly on the floor and apply the brakes, by pressing on the round pedal located on both rear castors.
3. For increased stability, widen the chassis legs by pressing down the r/h pedal at the back of the Chassis.
4. Insert the Mast and Jib/actuator assemblies into the Socket located in the chassis.
5. In the Plastic Bag supplied, you will find an 85mm long Screw, Handknob and a Hexagon Wrench.
6. From the rear of the Lifter insert the 85mm long Screw through the location holes in Chassis and Mast ensuring the Threaded end protrudes through to the inside of the Lifter. Using the Hexagon Wrench to secure Tighten Handknob.
7. Electric Models connect the Remote Control Cable to the Control Module.

- Using a Spanner, mount the spreader bar to the moving part of the mast. Do not over-tighten the fixing nut.

Dismantling is the reversal of the assembly procedure.

It should never be necessary to dismantle the actuator from the jib, but in the unlikely event of having to do so always ensure the actuator is refitted to its original position, never in the hole blanked off by grommets.

OPERATING INSTRUCTIONS

IMPORTANT SAFETY NOTICE:

INVACARE DO NOT RECOMMEND THE OPERATION OF ANY MOBILE LIFTING EQUIPMENT WITHOUT PRIOR PROFESSIONAL TRAINING IN MOVING AND HANDLING TECHNIQUES. NEVER EXCEED THE MAXIMUM RECOMMENDED LIFT CAPACITY: REL100-130kg, REL150-130kg, REL200-175kg, REL250kg-175kg

Adjustment of Chassis Width

- Standing behind the lifter take a firm hold of both handles.
- To open (widen) chassis legs press the right foot pedal.
- To close (narrow) chassis legs press the left foot pedal.

Brakes

- Standing behind the lifter take a firm hold of both handles.
- To apply the brakes press down the knob-shaped foot pedal in both rear castors. A red indicator button will show that brakes are applied.
- To release the brakes press the locking knob again.

Moving the Lifter

- Standing behind the lifter take a firm hold of both handles.
- Narrow chassis legs.
- Release brakes in both rear castors.
- Push or pull the lifter and steer it as required. As the lifter is fitted with castors at both ends, it needs careful guiding and steering, especially when a patient is suspended in the sling. However, with some practice the operator will be able to take full advantage of the increased manoeuvrability provided by this design, in particular when moving in tight areas as well as moving sideways.

TO NEGOTIATE RAISED OBSTACLES IT IS RECOMMENDED TO PULL THE LIFTER REARWARDS.

It is recommended not to move the patient with the jib at maximum height. Take great care when moving the hoist on wet or slippery surfaces. Always take extra care when using on uneven or rough surfaces and never use on slopes.

Attaching Slings

- Select the sling best suited for the patient in question. Do not attempt to lift any patient without prior professional training. If in doubt ask your Invacare Dealer for advise.
- Fit the sling to the patient as instructed in the Invacare Sling User's Guide. Take great care to ensure that the sling is fitted correctly, so that the patient is comfortable and safe.
- Manoeuvre the lifter, so that the spreader bar can be lowered to the sling loops as close as possible.
- Widen the chassis legs for increased stability.
- Lower the spreader bar and attach the 4 loops of the sling, sliding the loops under the safety clips in the hooks.
- If the patient is in a sitting position, lower the spreader bar to his/her chest, then move the lifter nearer to the patient.

Ensure brakes are engaged at all times during patient lifting or lowering.

CAUTION

NEVER ATTEMPT TO LIFT A PATIENT OUTSIDE THE BASE AREA OF THE LIFTER. WHEN LIFTING A PATIENT FROM THE FLOOR, THE PATIENT MUST ALWAYS BE LOCATED BETWEEN THE TWO LIFTER LEGS, AND NEVER OUTSIDE THE AREA. TO AVOID INJURY, UTMOST CARE MUST BE TAKEN WHEN RAISING AND LOWERING A PATIENT. NEVER EXCEED THE RECOMENDED MAXIMUM LIFT WEIGHT.

Raising & Lowering - Reliant 100 & 200 Hydraulic Models only (refer to Fig 2)

1. Check that the handwheel at the base of the pump lever is turned fully clockwise.
2. Operate the pump lever in firm strokes until the spreader bar is sufficiently raised.
3. To lower.
 - a) Without a Patient: Turn the handwheel at the base of the pump lever anticlockwise through two full turns.
 - b) With a Patient: Turn the handwheel at the base of the pump lever **THROUGH A QUARTER TURN ONLY**, in order to ensure a gentle descent and correct positioning of the patient. Turn the knob clockwise to reset the control.

Raising & Lowering - Reliant 150 & 250 Electric Models

1. To raise the mast, press the UP Arrow on the hand control.
2. To lower the mast, press the Down Arrow on the hand control.

EMERGENCY LOWERING DEVICES

The Reliant 150 & 250 lifters feature the latest Emergency Lowering Devices in line with the 1997 CEN Standard for Patient Transfer Systems;

Emergency Powered Lowering (refer to Fig 3)

Should the hand control on the Reliant 150 or 250 fail, the mast can be lowered by pushing a pointed object, such as a pen or paper clip, into the 'Emergency Lower' circular membranous switch located on the face of the charger unit, just below the red 'Emergency Stop' button. The mast will descend for as long as the 'Emergency Lower' switch is depressed.

Should it be necessary to use this switch, please consult your Invacare Dealer before using the lifter again.

Emergency Mechanical Lowering

In case of either total power failure or the battery becoming flat whilst the lifter is in use, the Reliant 150 & 250 models are fitted with a fail safe mechanical lowering device. To lower the mast in this instance, pull the red 'Emergency Toggle' located on the top surface of the actuator. The mast will continue to descend as long as the toggle is pulled out.

Should it be necessary to use this system, please consult your Invacare Dealer before using the lifter again.

To activate the safety system, insert a screw driver into the hole of the red plastic mechanism and unscrew for five turns until an adjustment screw is visible. This procedure must always take place before the equipment is installed. To adjust the emergency descent speed in relation to the patient's weight, screw in or out the adjustment screw, unscrew to increase speed, screw in to decrease speed. This adjustment must be made upon delivery at the place of use.

Note: The Emergency Mechanical Lowering System will only operate when there is an individual in the lifter.

BATTERY CHARGING

General

The Reliant 150 & 250 models are supplied with an integral battery charger for optimum convenience. The batteries must be charged at regular intervals to ensure maximum lifter performance and long battery life. The battery system is fitted with an audible warning system to alert the user when the charge is becoming low. It is recommended that the lifter is put on charge as soon as is feasible after this audible warning is heard.

Charging

Charge regularly overnight if the lifter has been in use during the day and immediately when the audible warning sounds during operation.

To charge the lifter, insert the charging lead into the socket at the base of the battery box. Connect the charger input plug into the mains and switch on the mains supply. A full charging cycle lasts approximately 12 hours, but the charger will switch off automatically as soon as the battery is fully charged. Charging must be carried out in a well ventilated room.

While charging takes place the lifter cannot be used.

MAINTENANCE

a) General

The new INVACARE range of lifters are designed for minimum maintenance, however a 6 regular monthly check and quick lubrication should ensure continued safety and reliability. Ensure batteries are regularly charged, preferably EVERY NIGHT. This will prolong their life and maintain peak performance. Keep lifter and slings clean and in perfect working order - any defect should be noted and reported to your dealer service agent as soon as possible.

b) Lubrication (refer to Fig 4)

Lubricate all pivot points. Ensure all surplus of lubricant is removed from surface.

c) Chassis Assembly

There is no maintenance of either the castors or brakes, other than cleaning, lubrication and checking axle and swivel bolts for tightness.

Remove all debris, fluff etc ... from the wheel and swivel bearings.

Regrease as shown on lubrication chart.

If the tyres, bearings or general condition require any further attention, it is recommended new parts are fitted - refer to spare parts document.

d) Chassis Adjustment Mechanism

The chassis adjustment should not require any attention other than to check the squareness of the legs when set to narrow and to adjust the Ackerman linkage rods if necessary.

Check the free movement of the linkage rods and tighten or lubricate if necessary.

Check the response of the leg adjustment foot pedal when you press it with the foot. There should be some resistance to prevent the legs from opening or closing accidentally, but there should be sufficient free movement of the mechanism to operate satisfactorily in either direction when the pedals are fully pressed down. Any irregularity can be corrected as follows:

1. From the pedal swivel bracket, remove the nut cap.
2. Remove the locking pin from the castellated nut.
3. To correct sluggishness in the foot pedals, tighten the nut.
4. To correct excessive stiffness in the foot pedal response, slacken the nut slightly.
5. Refit locking pin to castellated nut.
6. Replace nut cap.

e) Wear or Damage

IT IS IMPERATIVE TO INSPECT AT REGULAR INTERVALS ALL STRESSED PARTS, SUCH AS SLINGS, SPREADER BAR AND ALL PIVOT POINTS FOR ANY SIGNS OF FRAYING, CRACKING, DEFORMATION OR DETERIORATION. ANY DEFECTIVE PARTS MUST BE REPLACED IMMEDIATELY AND THE LIFTER MUST NOT BE USED UNTIL IT HAS BEEN REPAIRED.

f) Cleaning

Slings should be washed regularly in accordance with the washing instruction label. The lifter structure can be cleaned with a non-abrasive cleansing cream. It is also recommended to remove the mast assembly from the chassis and to clean and coat the mating surfaces of mast and mounting socket with vaseline before re-assembly.

TROUBLESHOOTING

Symptoms	Faults	Remedy
Lifter feels loose	Foot pressure loose	Refer to section d of MAINTENANCE INSTRUCTIONS
Castors/Brakes noisy or stiff	Fluff or debris in bearings	Refer to section e of MAINTENANCE INSTRUCTIONS
Noisy or dry sound pivots	Needs lubrication	Refer to section b of MAINTENANCE INSTRUCTIONS
Oil leaking from Hydraulics	Hydraulic pump in need of service	Contact your supplier
Hydraulic pump fails to lift when pumped	a) Handwheel not fully closed b) Hydraulic pump in need of service	Close handwheel Contact your supplier
Electric actuator fails to lift when switch depressed	a) Hand-control or actuator connector loose b) Batteries low c) Red isolating knob down	Push connectors fully home Charge batteries Rotate clockwise to release

Electric actuator fails
to lower when the
switch is depressed

- a) Hand control or connector
is loose.
- b) Battery low
- c) Red isolating knob pushed in.
- d) Hand control failure
- e) Electric fault

Push connectors fully home

Charge battery
Rotate clockwise to release
Press 'Emergency Lower Switch'
Pull 'Emergency Toggle'

*If problems are not remedied by the suggested means, please contact your dealer service agent.
The frame and lifting actuator are guaranteed for three years, the battery is guaranteed for one year.*

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