Owner's Manual  
for model: **BSP-4007**

**SAMSUNG AMERICA, INC.**

Made in China

105 Challenger Road, 6th Floor  
Ridgefield Park, NJ 07660

[www. SAMUNGHealthyLiving.com](http://www.SAMUNGHealthyLiving.com)

PREMIUM DELUXE

## Premium Deluxe Thinline Blood Pressure Monitor

- Personal Alert Level
- Irregular Heartbeat Detector
- Thin, Lightweight Design
- 3 Zones of 80 Memory - 240 Memory Total
- Memory Average of Last 3 Measurements
- Multi Measurement Options
- Time and Date Indications
- AC Capability and Adapter

## **Important Warnings/Medical Disclaimer**

Before using your blood pressure monitor, you should read and understand all instructions and follow all warnings.

Information in this manual is provided for informational purposes only. This product and manual are not meant to be a substitute for the advice provided by your own physician or other medical professional. You should not use this product, or the information contained herein, for diagnosing or treating a health problem, disease, or prescribing any medication. If you have or suspect that you have a medical problem, promptly contact your medical provider.

## Table of Contents

Important Warnings/Medical Disclaimer _____	1
Healthy Living® _____	3-4
Commonly Asked Questions About Blood Pressure _____	5-8
Intended Use _____	8
Measurement Method _____	9
Important information before using the unit _____	10-11
Main Unit and Accessories _____	12
Special Features _____	13
Accuracy _____	14
Special Conditions _____	14
Battery Installation _____	14
Digital Display Symbols _____	15
Setting Date, Time and Levels _____	16-17
Applying the Cuff _____	18
Taking a Blood Pressure Measurement _____	19
Storing, recalling, and erasing measurement data _____	20-21
3 Zones of 80 Memory _____	21
Multi-Measurement Options _____	22
Personal Computer Link _____	23
Taking Care of Your Blood Pressure Monitor _____	24-25
Troubleshooting _____	26-27
Lifetime Limited Warranty _____	28-29
Specifications _____	30
Blood Pressure Diary _____	31-32

## Healthy Living®

Purchasing this blood pressure monitor is your first step in living a healthier life, but it is only the beginning. It is also important to:

**STAY ACTIVE** - Start out by taking just a short walk each day. Over time you'll begin to feel more energetic. Walking is a fun, easy way to help lose weight and also aids in building stronger bones and a healthier heart!

**EAT HEALTHY** - Adding more fruits and vegetables to your diet is a great way to start. Also, look at incorporating more whole grains, fish and low fat dairy products and keep a check on your sodium intake. If you're using a blood pressure monitor, you should already be aware of how sodium can affect your blood pressure. You may want to explore the potential benefits of certain botanicals such as onion and garlic. As always, consult your physician before beginning any dietary changes.

## Healthy Living®

**STAY POSITIVE** - A healthy mind goes hand in hand with a healthy body. Daily relaxation techniques such as meditation and yoga may be just what you need to alleviate stress. Keep your mind focused on the positive things in life. Start out by giving yourself a pat on the back for those walks you are now taking or that apple you just ate!

**WATCH YOUR WEIGHT** - Sometimes just a small amount of weight loss may have a positive effect on your blood pressure. If you have already started exercising and eating healthier, the weight loss may just happen on its own!

**REMEMBER:** Consult your physician before you start or change your exercise routine, diet, vitamin/supplement intake and/or blood pressure monitoring.

## Commonly Asked Questions About Blood Pressure

### ***Why Should I Monitor My Blood Pressure At Home?***

By monitoring at home you can manage your health care between visits to the doctor. Also, you now have the means to keep a record of your blood pressure measurements, a valuable tool for both you and your healthcare provider.

Home monitoring may help alleviate White Coat Hypertension. This term refers to what may cause a person's blood pressure to be higher when taken in a medical setting. At times the stress of just being in a doctor's office may cause blood pressure to rise. When monitoring at home you may get a truer indication of your blood pressure because you are in a relaxed, comfortable space.

### ***What is Blood Pressure?***

The term "blood pressure" refers to the force or pressure that is exerted on the body's arteries as blood flows through them. Each time the heart beats or contracts, it produces pressure in the arteries. This blood pressure is what moves blood through the body, supplying oxygen and nutrients to every organ.

## Commonly Asked Questions About Blood Pressure

When the heart contracts it sends blood out into the body. This pressure is referred to as **systolic**.

The pressure that is created as the heart pulls blood back through is called **diastolic**.

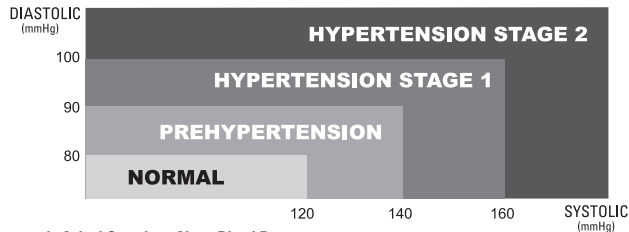
Blood Pressure is recorded as **systolic over diastolic**.

### **What is considered Normal Blood Pressure?**

The Seventh Report of the Joint National Committee on Prevention, Detection, Evaluation, and Treatment of High Blood Pressure from the National Heart, Lung and Blood Institute has set forth the following guidelines regarding high blood pressure. Guidelines provided are for information purposes only. Please consult with your physician for proper diagnosis.

## Commonly Asked Questions About Blood Pressure

### **Why Does My Blood Pressure Fluctuate?**

Blood pressure changes with every beat of the heart and is in constant fluctuation throughout the day. In addition to natural changes, other factors that may affect blood pressure are:

- Temperature (too hot or cold)
- Humidity
- Restlessness
- Lack of sleep
- Physical exercise
- Constipation
- Fear
- Anger
- Anxiety
- Food / beverage consumption
- Your physical condition
- Stress

Be sure to speak with your healthcare professional to determine what is affecting your blood pressure.

### **How Do I Know If I Have High Blood Pressure?**

People have the misconception that they can "feel" their blood pressure rise. In reality it is nearly impossible to tell if your pressure is elevated without having it checked. This is why hypertension has been named "the silent killer". Start monitoring your blood pressure today and discuss the results with your healthcare professional.

## Commonly Asked Questions About Blood Pressure

### *Why Is It Important To Control My Blood Pressure?*

Left uncontrolled hypertension can place an incredible amount of stress on your heart and arteries. They are both forced to work harder to keep blood flowing through your body. Over time this stress may result in health related problems including enlargement of the heart, loss of artery elasticity, heart disease, stroke or death. Making simple lifestyle changes now may help you stay healthy in the future.

## Intended Use

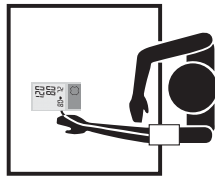
Healthy Living® blood pressure monitors are designed for home use. They are noninvasive; no part enters your body, and provide systolic, diastolic and pulse rate measurements. These monitors are recommended for use by people over 18 years of age and are not suitable for clinical use.

## Measurement Method

Healthy Living® monitors measure blood pressure and pulse rate by using an oscillometric method, meaning the fluctuations in pressure are measured. Once the cuff is wrapped around the upper arm, the monitor is turned on and the cuff is automatically inflated. The inflation of the cuff creates pressure around the arteries inside the arm. Within the cuff is a gauge which senses the fluctuations (oscillations) in pressure. The fluctuations measured are the arteries contracting with each heart beat as a result of the pressure the cuff has placed on the arm. The monitor measures these contractions and converts the information to a digital value. That is the result displayed on the monitor screen. Once the measurement is complete the cuff will automatically deflate.


## Important information before using the unit

- Blood pressure measurements should be interpreted by a physician or trained health professional who is familiar with one's medical history. By using the unit regularly and recording the results for the physician to interpret, one can keep the physician informed of the continuing trends in one's blood pressure.
- Sit quietly for about 10 minutes. This will allow your body to return to its normal resting state.
- Wrap the cuff snug around your upper arm and lay your arm on a flat surface at the same level as your heart. Your feet should be flat on the floor.
- Do not move the device during the taking of a measurement; this may cause an inaccurate measurement.
- Perform the measurement in a quiet environment, in a relaxed position at room temperature. For consistency, we recommend using the same arm (left arm if possible) and testing around the same time each day.

## Important information before using the unit

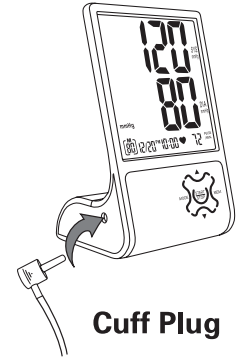- If taking multiple measurements, please allow 10 minutes between measurements. This will allow your blood vessels to return to their normal state.
- If you have consumed beverages containing caffeine or have smoked, please allow 30 - 45 minutes before taking your measurement.
- Do not place the cuff over a jacket or sweater sleeve; it must be wrapped around your exposed upper arm.
- Keep in mind that blood pressure naturally varies from time to time throughout the day and is affected by many different factors, such as smoking, alcohol consumption, medication, and physical activity.
- Blood pressure measurements taken with this unit are equivalent to those obtained by a trained observer using the cuff/stethoscope auscultation method within the limits prescribed by the American National Standard for Electronic or Automated Sphygmomanometers.
- Make sure the cuff plug is attached to the monitor as shown above.

**Cuff Plug**

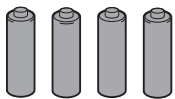
## Main Unit and Accessories

### Main Unit

(See page 15 for descriptions of screen icons.)



### Accessories

- 4 AA Alkaline Batteries
- 1 Storage/Travel Pouch
- 2 Form Fitting Cuffs (Regular and Large)

### Using the AC Adapter

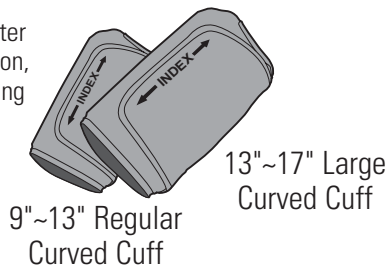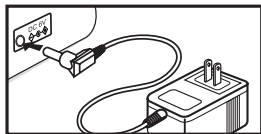- \*Use Output: 6VDC, 1A, AC/DC adapter
- \*In order not to lose stored information, plug in the AC adapter before removing batteries.

## Special Features

### Heart Sense®

The Heart Sense® Inflation System senses your blood pressure before the measurement is complete, signaling the monitor to adjust the air pressure amount. The result is a quick, comfortable measurement each time you check your blood pressure.

### Quiet Inflation

A unique, low noise, rolling motor system allows our monitors to inflate quietly.

### Large 3 Row Display Screen

To make reading your result easier, the monitor comes with a large 3 row LCD display screen.

### Personal Alert Level

If the Systolic or Diastolic rate of your reading result is greater than your set personalized level, the result on screen will flash.

### Irregular Heartbeat Detector

This symbol will appear on left of screen indicating an irregular heartbeat was detected.

### Multi Measurement Options

The monitor can be set to automatically take multiple (3 or 5) measurements and select the middle reading as your result.

### PC Link Function

With this Blood Pressure Monitor you can transfer the measurements to a notebook or PC. See separate instructions included with your monitor.

### Memory Feature

Your Healthy Living® Deluxe Inflate Monitor features memory capability of up to 240 measurements and provides the average of the 3 most recent readings!

### Time and Date

The monitor will automatically keep track of the time and date.

## Accuracy

Healthy Living® monitors have been clinically tested against a scientific device called a *sphygmomanometer*, considered the gold standard in blood pressure measurement. All Healthy Living® monitors have performed equivalent to measurements taken with this scientific device and are within the accuracy limits prescribed by the American National Standard for Electronic or Automated Sphygmomanometers.

## Special Conditions

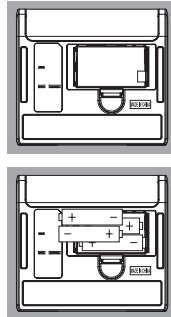
All Healthy Living® blood pressure monitors are safe and accurate to use with arrhythmias such as atrial or ventricular premature beats and atrial fibrillation.

## Battery Installation

Using the 4 AA Alkaline batteries included, locate the battery compartment on the bottom of the monitor. Remove the battery cover and insert the batteries, making sure the polarities (+/-) are correct. Snap the battery cover back into place.


When the display screen shows "⊗" it is time to replace with 4 new AA batteries.

\* To make the batteries last longer, you may want to remove them if you are not using your monitor for an extended period of time.


## Digital Display Symbols
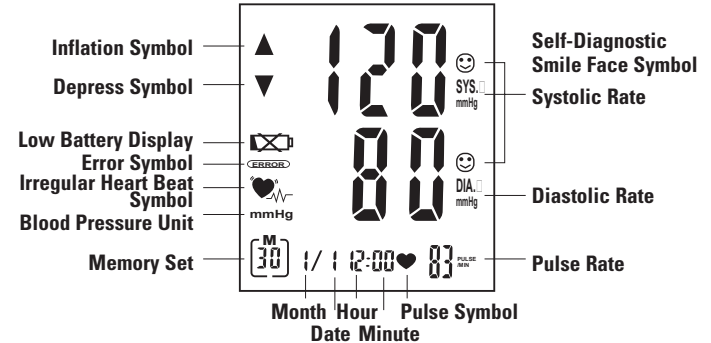
### Mesurement Reading Screen

### At Rest Screen
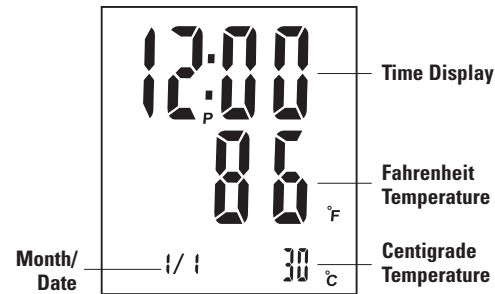
### Note:

\* Ambient temperature will respond to the change of environment slowly.

\* The temperature value might be a little bit higher if multiple measurements are being taken continuously.

## Setting Date, Time and Levels

### Setting date and time

- Press "MODE" button ("month" starts flashing)  
Press "[▲]" and "[▼]" button to set and adjust the correct month (1, 2, 3 ...12)
- Press "MODE" button again ("date" starts flashing)  
Press "[▲]" and "[▼]" button to set and adjust the correct date
- Press "MODE" button again ("hour" starts flashing)  
Press "[▲]" and "[▼]" button to set and adjust the correct hour in 12-hour format. A "PM" will appear for PM and disappear for AM.
- Press "MODE" button again ("minute" starts flashing)  
Press "[▲]" and "[▼]" button to set and adjust the correct minute (00, 01, 02 ...59)

**If the batteries have run down or are removed or the adapter has been unplugged, the time and date will have to be reset.**

*After setting the minute you will automatically be prompted to set the personalized level for Memory Zone 1. The storage of 240 memory readings is divided into 3 memory zones, each maintaining up to 80 memory readings. The following explains how to set your personalized level in each zone.*

*Your personalized level should be decided between you and your healthcare provider.*

**Before setting your levels it is important to know:**

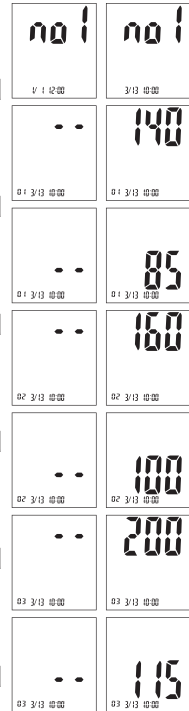
\*Your systolic rate is set between 70 and 250 mmHg.

\*Your diastolic rate is set is between 40 and 115 mmHg.

## Setting Date, Time and Levels

### Setting personalized levels


- Press "MODE" button  
(" - - " will flash for 1st systolic self-management)  
Press "▼" or "▲" button to set/adjust the personalized systolic rate
- Press "MODE" button  
(" - - " will flash for 1st diastolic self-management)  
Press "▼" or "▲" button to set/adjust the personalized diastolic rate
- Press "MODE" button  
(" - - " will flash for 2nd systolic self-management)  
Press "▼" or "▲" button to set/adjust the personalized systolic rate
- Press "MODE" button  
(" - - " will flash for 2nd diastolic self-management)  
Press "▼" or "▲" button to set/adjust the personalized diastolic rate
- Press "MODE" button  
(" - - " will flash for 3rd systolic self-management)  
Press "▼" or "▲" button to set/adjust the personalized systolic rate
- Press "MODE" button  
(" - - " will flash for 3rd diastolic self-management)  
Press "▼" or "▲" button to set/adjust the personalized diastolic rate

The examples above illustrate setting personalized levels in all 3 zones. After setting your Personalized Alert Level, you will automatically be given the option to set a Multi-Measurement for that zone. If you choose not to set a Multi-Measurement, just enter 1. After entering these settings you will be returned to the Rest mode and can take a measurement by pressing START/STOP.



## Applying the Cuff

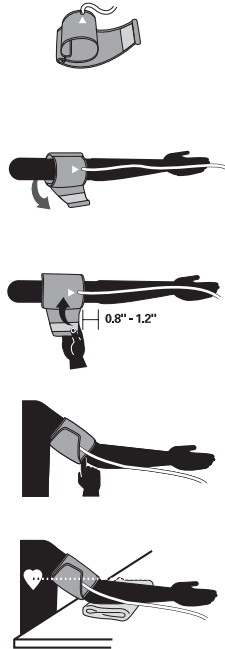- Insert the Cuff Plug into the Cuff Plug Jack on the Monitor.
- Pull the Velcro tab so the cuff is laying flat with the Velcro facing up at you.
- Pull the cuff to the left of the metal ring to form an opening for your arm.
- Put your left arm through the cuff loop. The bottom of the cuff should be approx. 0.5 inch above elbow. The cuff tube should lie over the brachial artery on the inside of the arm.
- If the cuff is properly positioned the tube should be running down your arm as in the illustration to the right.
- Pull the cuff so that the top and bottom edges are snug around your arm. Do not tighten to the point of being uncomfortable.
- When the cuff is positioned properly, press the Velcro firmly against the pile side of the cuff.
- Sit in a chair and place your arm on the table so that the cuff is at the same level as your heart.
- Relax your arm and sit with your palm facing upward.
- Assure there are no kinks in the air tube.

## Taking a Blood Pressure Measurement

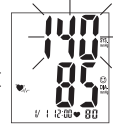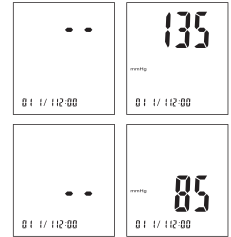- Wrap the cuff around the arm (refer to previous page)
- Sit upright in a chair with feet flat on the floor.
- Press "▼" or "▲" button to select a particular memory zone then press the "START/STOP" button to start the measurement. The monitor will automatically inflate to the level that is right for you.
- When taking repeat measurements, make sure to select the same memory zone before pressing "START/STOP", so the measurement is registered to the appropriate memory.
- If the result of your Systolic or Diastolic rate of your reading result is greater than (not equal to) your set personalized level the DIGIT will flash. In addition the measurement that is out of range, either Systolic or Diastolic, will flash. If both are out of range both will flash.  
If the result of your Systolic or Diastolic rate of your reading result is equal or less than your set personalized level a SMILING symbol will show out and flash on screen.

The appearance of this symbol, , signifies that a certain pulse irregularity was detected during the measurement. Talking, moving, shaking or an irregular pulse during the measurement can result in the appearance of this icon. Usually this is not a cause for concern, however if the symbol appears often, we recommend you seek medical advice. The device does not replace a cardiac examination, but serves to detect pulse irregularities at an early stage.

- To stop a reading in process, press the "START/STOP" button.
- Press and hold "Mode" button to check the pre-set personalized level.

## Storing, recalling, and erasing measurement data

### • Storing data

\*After each measurement, the systolic and diastolic rates, pulse rate and the time & date will be automatically stored. The memory holds data for the latest 80 measurements in each of 3 memory zones. If more than 80 measurements are entered, the memory will automatically clear out the oldest data.

### • Recalling data

\*Press the "▲" or "▼" button to select the memory zone.

\*Press the "MEM" button to get your stored memory data. If there is no data in the memory nothing will appear.

\*If there is data in the memory, the first reading will be an average of the last three recorded set of measurements (systolic and diastolic pressure and pulse rate).

\*Press the "MEM" button again and the latest recorded set of measurements will appear along with the recording time and date.

\*Press the "▼" button to read the next set of stored data. The data displayed starts with the most recent reading and continues backwards to the oldest.

\*If your memory readings are out of range of your preset personal Alert levels. They will flash or show SMILING FACE symbol as explained on the previous page.

\*No Irregular Heartbeat Detector Record will be stored in memory.

## Storing, recalling, and erasing measurement data

### • Erasing data

\*Press "MEM" into Memory mode.

\*Press and hold on "MODE" button and then press "▲" button, the pre-set Personal Alert Level and all measurement data will be erased.

\*To confirm the data has been erased, press the "MEM" button and no data should appear.

## 3 Zones of 80 Memory

When you take a measurement, the monitor automatically stores it in memory space number one. As you continue to take measurements, the monitor will push the previous measurement back one space. In other words, the value in space one will always be the most recent blood pressure measurement taken.

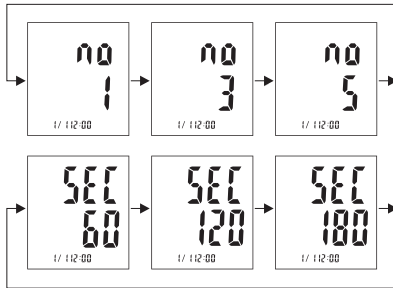
Your monitor features a 3 party memory capability or 3 "zones" - each zone stores 80 measurements each for a total capacity of 240 measurements. This feature is useful for a user to track their blood pressure throughout the various periods of the day: morning, afternoon, and night. Press the "▲" button to toggle between memory zone No.1, 2, and 3. Once you select a zone that you want to view the measurements in, press the "MEM" button once to view the most recent measurement along with the time and date that it was taken. Press the "MEM" button again in succession and you will see all of the measurements registered to that zone. You can then press the "▲" button to switch to the next zone to view measurements. The monitor will automatically power off after about one minute of inactivity.

**\*When taking measurements, be sure and select the correct zone number using the SET button so that the measurement is registered to the appropriate zone.**

## Multi-Measurement Options

### Setting Multi-Measurement

- a. Press the MODE button repeatedly until the figure below shows on the screen.

- b. Press "▲" to select 1, 3 or 5 measurements. When set to 1 the reading is a typical single measurement.
- c. Press "MODE" button ("60" starts flashing).
- d. Press "▲" to set/adjust the time between readings; 60, 120 or 180 seconds.
- e. Press the START/STOP button to start measurements.
- f. The monitor will show 2 bars (--) on the Systolic and Diastolic display when each measurement ends.
- g. After all measurements have been taken, the middle reading of each Systolic, Diastolic and Pulse reading will be displayed on the screen.

## Personal Computer Link

Use this software to transfer your blood pressure monitor readings to your personal computer. Separate instructions are included with your monitor.

**\*PC program compatible with Microsoft® Windows® ME, 2000, XP Home Edition and XP Professional edition in English version only.**

## Taking Care of Your Blood Pressure Monitor

### Maintenance

#### \* Lose of Power

If you use the AC adapter and have lost power, your stored data will not be affected. You will have to reset the time, date and personalized levels once power is restored or batteries are used, but the data in the memory zones will remain.

Use a damp cloth to wipe the monitor and dry immediately with a dry cloth.

Use only a dry cloth to wipe the cuff.

Do not use strong cleaning agents for cleaning.

When the unit is not to be used for a long time or you use only the AC Adapter, it is best to remove the batteries.

### Safe Keeping

Always keep the unit in the carrying case after its use.

Do not store the monitor in direct sunlight, high temperatures, or in humid or dusty places.

Do not store in extremely low (less than -4 °F / -20 °C) or high (more than 122 °F / 50 °C) temperatures.

## Taking Care of Your Blood Pressure Monitor

### To prevent malfunctions

Use the cuff only as instructed and be careful not to twist it. Do not inflate the cuff when it is not wrapped around the arm.

Do not attempt to disassemble or change any monitor parts, including the cuff.

Treat the monitor with care, do not drop it and prevent all strong impacts.

### Warnings

The regular cuff size is suitable for adult arms 9 ~ 13 inches only. If your arm measures 13 ~ 17 inches around, use the large cuff. Both size cuffs are included with your monitor.

The device is not to be used if your arm has a wound or injury.

In the case that the cuff does not stop inflating, remove the cuff at once.

Do not disassemble or modify the device in any way. This could result in inaccurate readings and voids the device warranty.

Do not use the cuff on any body part except your upper arm. Keep out of reach of children.

## Troubleshooting

Error/Symbol	What Does It Mean?	How To Fix It
<b>ERROR</b>	Monitor may have been inflated without the cuff around the arm <i>OR</i> Cuff may not be positioned correctly on the arm.	Read page 18 (Applying the Cuff).
	The cuff tube may be folded <i>OR</i> You may have moved while the cuff was inflating.	Read pages 10 & 11 (Important information before using the unit).
	You may have moved or spoken during the measurement process.	Remember that it is important not to move and speak as it may elevate measurements.
	You may have been startled by a noise or vibration while measuring.	Make sure you are in a quiet and comfortable place when taking a measurement.

## Troubleshooting

Error/Symbol	What Does It Mean?	How To Fix It
<b>ERROR</b>	The batteries have run down.	Replace with four new "AA" batteries. Make sure the polarities are positioned correctly in the battery compartment before continuing to measure your blood pressure.
	The batteries have run down.	Replace with 4 AA batteries.
Blood pressure value seems too high or too low	Cuff may not be aligned correctly	Read page 18 (Applying the Cuff).
	Circuitry locked.	Remove and reinsert the batteries and then proceed to take measurement again.
Unable to take a measurement after receiving an error symbol.	The cuff may not be positioned correctly.	Read page 18 (Applying the Cuff).

## Lifetime Limited Warranty

Samsung America, Inc. will, at its option, repair or replace your monitor, including the cuff, with the same or comparable model free of charge (except you must pay for shipping charges set forth below) for the lifetime of the original owner from the date of original purchase in the event of a defect in materials or workmanship occurring during normal use. This product is for home use only and is not intended for commercial or clinical use. This warranty only extends to the original retail purchaser. Purchase receipt or other proof of date of original purchase is required before warranty performance.

This warranty extends solely to failures due to defects in materials or workmanship occurring during normal use. It does not cover normal wear of the product. This warranty is void if: the product housing has been removed, if the product's label, logo or serial number have been removed or if the product fails to function properly as a result of accident, misuse, abuse, neglect, mishandling, misapplication, defective batteries, faulty installation, setup, adjustments, improper maintenance, alteration, maladjustment of controls, modification, power surges, commercial or clinical use of the product, service by anyone other than an authorized service center or acts beyond the manufacturer's or distributor's control.

The warranty and remedy provided herein are exclusive and in lieu of all other express and implied warranties and unless stated herein, any statements or representations made by any other person or entity are void. NEITHER THE MANUFACTURER, DISTRIBUTOR NOR THEIR AFFILIATES SHALL BE LIABLE FOR ANY LOSS, INCONVENIENCE, INJURY, OR DAMAGE INCLUDING DIRECT, SPECIAL, INCIDENTAL, OR CONSEQUENTIAL DAMAGES RESULTING FROM THE USE, IMPROPER USE, FAILURE TO HEED ANY WARNING, DISCLAIMER OR INSTRUCTION OR INABILITY TO USE THIS PRODUCT, WHETHER ARISING OUT OF ANY BREACH OF THIS WARRANTY OR ANY OTHER LEGAL THEORY. ALL EXPRESS AND IMPLIED WARRANTIES,

## Lifetime Limited Warranty

INCLUDING THE WARRANTIES OF MERCHANTABILITY, ARE LIMITED TO THE APPLICABLE WARRANTY PERIOD SET FORTH ABOVE. Manufacturer and Distributor assume no responsibility for injuries, damages or penalties incurred or suffered resulting from the use of this product in an improper manner or manner or location other than for that for which it is intended or from failure to heed any warning, disclaimer or instruction.

Some states do not allow the exclusion or limitation of incidental or consequential damages, or limitations on how long an implied warranty lasts, so the above exclusions or limitations may not apply to you. This warranty gives your specific legal rights and you may also have other rights which vary from state to state.

Should your product prove defective during this warranty period, please return the product in the original carton prepaid to:

**SAMSUNG AMERICA, INC.**  
**c/o Healthy Living Division**  
**105 Challenger Road, 6th Floor**  
**Ridgefield Park, NJ 07660**  
**Attn: Customer Service**

Include \$11.95 for return shipping & insurance. (CHECK OR MONEY ORDER ONLY PLEASE, NON-REFUNDABLE). Sorry, we cannot deliver to P.O. Boxes or outside the U.S. Please allow 6-8 weeks for delivery.

Include with your monitor a letter stating: your name, street address, daytime phone number, detailed description of the problem, a copy of your dated cash register receipt or other proof of purchase.

This warranty applies only to products purchased in the U.S. For products purchased in, but used outside, the U.S., this warranty covers only warranty services within the U.S. (and does not include shipping outside the U.S.).

**Specifications**

<b>Model</b>	<b>BSP-4007</b>
Monitor	Premium Deluxe Thinline
Measurement Method	Oscillometric
Display	Liquid crystal digital display
Measurement Range	Pressure: 0 ~ 300 mmHg Pulse: 40 ~ 199 beats/Minute
Accuracy	Pressure: within ± 3 mmHg Pulse: within ± 5%
Memory	Stores up to 80 measurements for each zone
Automatic Power Off	Approximately 1 minute after measurement is complete
Battery Type	4 AA batteries (Alkaline)
Battery Life	Approximately 2.5 months when used once a day, less when using a large cuff
Storage Temperature	Temperature: -4 °F ~ 122 °F/-20 °C ~ 50 °C Relative Humidity: 10 ~ 85% R.H.
Operating Temperature	Temperature: 50 °F ~ 104 °F/10 °C ~ 40 °C Relative Humidity: 30 ~ 85% R.H.
Dimensions (approx.)	Monitor: 7 1/4 " (L) x 4 3/8 " (W) x 3 15/16 " (H) Regular Cuff: fits arms 9 " ~ 13 " around Large Cuff: fits arms 13 " ~ 17 " around
Weight (approx.)	17.5 oz (including batteries)

*For purposes of improvement, specifications are subject to change without notice.*

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