

JVC


AC POWER ADAPTER

Instructions

AA-V112E

Dear Customer,

Thank you for purchasing the JVC AC Power Adapter AA-V112E. Before use, please read the safety precautions below to ensure safe use of your new AC Power Adapter.

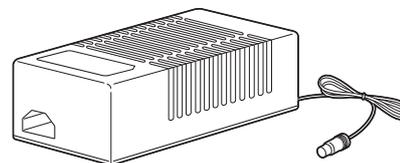

AC Power Adapter

For Customer Use:

Enter below the Model No. and Serial No. which are located on the bottom of the cabinet. Retain this information for future reference.

Model No. _____

Serial No. _____

Safety precautions

WARNING:

To prevent fire or shock hazard, DO NOT expose this equipment to rain or moisture.

WARNING:

This equipment is not sterilized.

INTRODUCTION

The model you have purchased is designed for use together with the Isolation Transformer AA-V31E and MICRO HD CAMERA DZ-VCA1SE for medical applications.

INSPECTION

Remove the AC Power Adapter from the shipping container and inspect it for shipping damage. Do not use the product if it appears to be damaged in any way. If damaged, notify the carrier and seller immediately.

ENVIRONMENTAL CONSIDERATIONS

This AC Power Adapter has environmental requirements similar to other medical devices. It has been designed for indoor use only, in areas where it will not be exposed to excessive dust or moisture. Make certain that there is adequate air flow around the unit. Do not place objects on top of or near the unit which could obstruct the air vents. For usage, please refer to the MICRO HD CAMERA DZ-VCA1SE Instruction Book.

SIZING INFORMATION AND INSTALLATION PROCEDURE

The name plate on the top panel lists the AC Power Adapter voltage and current ratings. Observe that the input current rating is not exceeded when connecting equipment. Ask that installation be performed by a JVC authorized dealer.

AA-V112E INFORMATION

This information contains instructions on using AA-V112E. Please read it to use properly.

The way of use and instructions

- AA-V112E is inlet type AC power adapter. For the input cord, be sure to use the provided AC inlet cable.
- Rated input voltage is AC120V continuously.
- Inside this AC Power Adapter, there is a high voltage part. Never disassemble this.

Symbols indicated on this equipment have the following meanings.

	Attention, consult ACCOMPANYING DOCUMENTS
	Alternating current
	Direct current

SPECIFICATIONS

AC Power Adapter

- **Manufacturer** : Victor Company of Japan Limited
- **Model No.** : AA-V112E
- **Input** : AC 120 V \sim 50 Hz
MAX. 0.8 A
- **Output** : 12 V DC --- 3.0 A

TECHNICAL AND SERVICE ASSISTANCE

JVC offers technical and customer service assistance. Please contact us at the following address.

JVC PROFESSIONAL PRODUCTS (UK) LIMITED
ULLswater House, Kendel Avenue, London W3 OXA,
United Kingdom
TEL: (0181) 896-6000
FAX: (0181) 896-6060

JVC PROFESSIONAL PRODUCTS GmbH
Grüner Weg 10, 61169 Fiedberg/Hessen, Germany
TEL: (06031) 6050
FAX: (06031) 605180