

Zoom-Ex®

Installation Guide

ABISee, Inc.

77 Powder Mill Rd
Acton, MA 01720
www.abisee.com
1-800-681-5909

Table of Contents

Table of Contents 2

What is Eye-Pal 3

What's in the Box? 3

Minimum System Requirements 3

Setting Up Eye-Pal® 4

1. Install Eye-Pal Software. 4
2. Connect Eye-Pal® To Your Computer. 4

You Are Ready to Start! 5

What is Zoom-Ex

Zoom-Ex is an electronic scanner that aids low vision computer users to read printed material. Zoom-Ex connects to your PC or laptop through the USB 2.0 port, no extra batteries or power packs required.

Zoom-Ex is a portable camera that scans and the software program that runs on your computer. Since it folds easily and weighs about 1 pound, it fits in your laptop bag and is easy to carry.

Eye-Pal can be used to listen to the text-to-speech voice of your choice, to read books, magazines, newspapers, documents, medicine bottle labels, and anything else that has printed text or to read on a Braille display of your choice.

Zoom-Ex is:

1. **Fast** - it captures the whole page up to 11'x8.5' in a second and reads it aloud or shows on a Braille display or wrapped magnified text on a computer monitor.
2. **Smart** – it distinguishes text from pictures, allows you to read through columns, describes the page as sighted person sees it.
3. **Easy to use** – you need click only one key to start reading and click another key when you turn the page.
4. **Portable** – it weights 1lb and folds like an umbrella
5. **Easy to install** – connects to PC or laptop using USB 2.0 port.

What's in the Box?

Your box contains:

- Zoom-Ex® - the portable scanner-reader.
- Zoom-Office® Software CD.
- Bag.

Minimum System Requirements

- Windows XP (recommended) or Vista
- Intel® Pentium® 1.3 MHz
- 512 MB of system memory (RAM)
- 300 MB hard disk space
- USB 2.0 port

Setting Up Zoom-Ex®

Note: Install Zoom-Office software prior to connecting the Zoom-Ex® device to your computer

1. Install Zoom-Ex Software.

Insert Zoom-Office CD into the CD-ROM drive, watch the Installation Wizard dialog appear, and follow the Installation Wizard instructions on your screen.

If the Installation Wizard dialog does not start automatically:

- a. Click Start on taskbar and choose Run.
- b. Type D:\Setup. Substitute correct letter for your CD-ROM drive
- c. Click OK
- d. Follow instructions on the screen.

2. Connect Zoom-Ex® To Your Computer.

- a. Take the Zoom-Ex® out of its packaging.
- b. Bring the Zoom-Ex® into the upright position.

As you gently pull out the Zoom-Ex® “legs” and “head”, they make a light clicking sound when they reach their final secure position.

1. Lift the Zoom-Ex® “head” up to be at the right angle to the “spine”.
2. Move its “legs” out until they are at the right angle with the magnifier’s vertical “spine”.
3. Turn the levers to lock the legs.

Place the Zoom-Ex® in its upright position on a hard surface next to your computer or laptop.

Plug the Zoom-Ex® into the USB 2.0 computer port, usually located at the front of a desktop computer, or on the side of a laptop.

“Found New Hardware” dialog will appear on the screen.

Select “*No, not this time*” and click *Next*.

- c. On the next screen select “*Install the software automatically*”, and click *Next*.

Zoom-Ex software is now installed. The next dialog will inform you that the installation is complete.

You Are Ready to Start!

Plug your Zoom-Ex into your computer. Zoom-Ex® is ready to use.

Find and double-click Zoom-Office icon on your computer screen.



| Put your favorite book under the camera and click E. Zoom-Ex will start reading the text in a few seconds.

Now, please, open The Training Guide and User Manual that you can find on your Zoom-Office CD. You are ready to start reading anything you want!

Best luck.