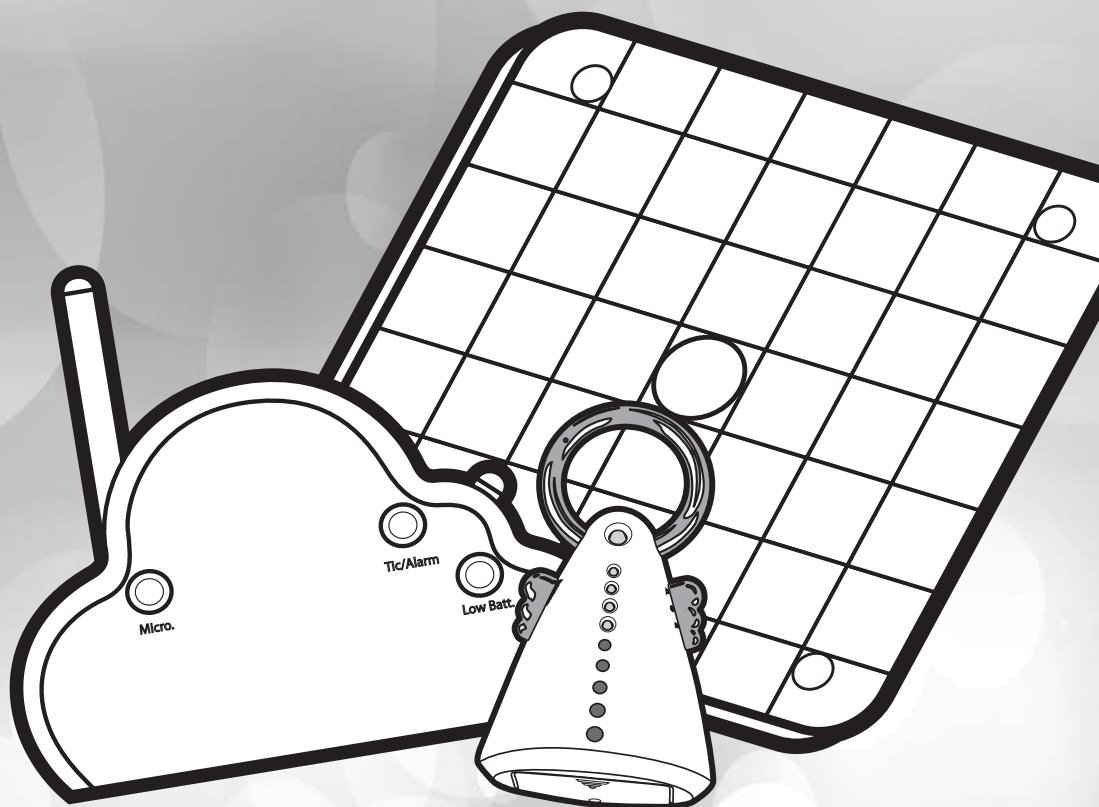
Angelcare®

FOR YOUR PEACE OF MIND

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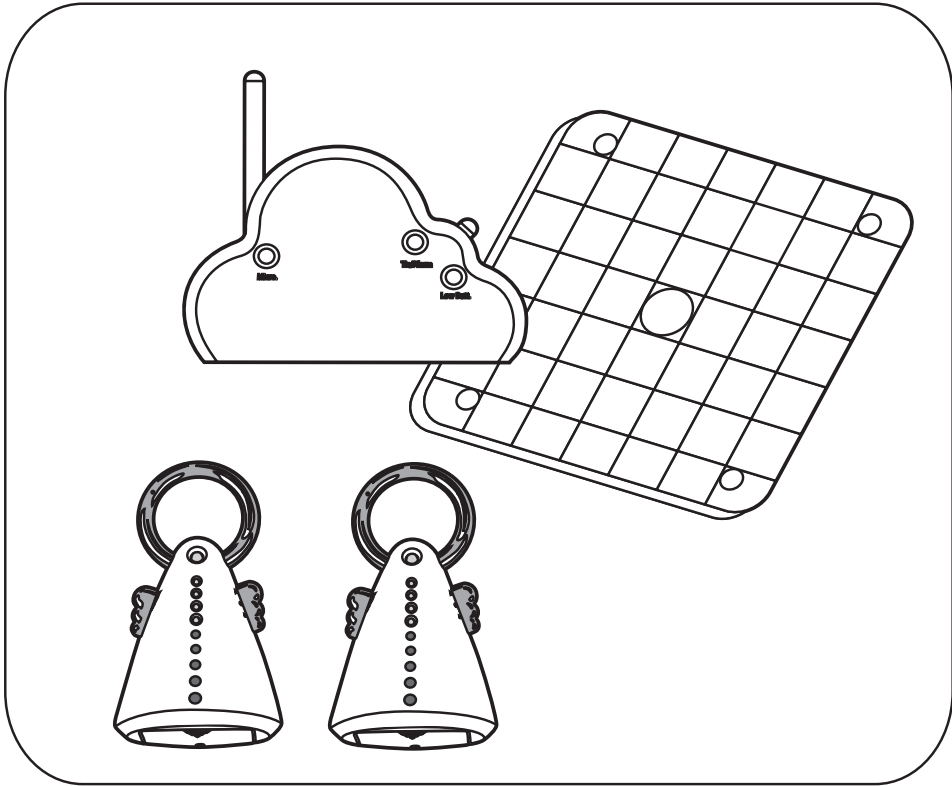
AC201 AC201-2P

OWNER'S MANUAL
KEEP INSTRUCTIONS FOR FUTURE
REFERENCE.

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Angelcare®

Model AC201-2P

Your monitor includes 2 Parent units. Make sure all units are on the same channel.

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Owner's manuel

KEEP THIS INSTRUCTION
MANUAL FOR FUTURE REFERENCE

PLEASE READ THE WARNINGS AND
INSTRUCTIONS CAREFULLY BEFORE
ASSEMBLING & USING THE PRODUCT.

FAILURE TO FOLLOW THE WARNINGS
AND THE INSTRUCTIONS COULD RESULT
IN SERIOUS INJURY OR DEATH
OR DAMAGES TO THE PRODUCT

⚠️ WARNING

IMPORTANT SAFETY INSTRUCTIONS

THIS PRODUCT CANNOT REPLACE RESPONSIBLE ADULT SUPERVISION. This product is designed to aid in the monitoring of your child. When using the monitor, you must still provide the supervision necessary for the continued safety of your child. Monitoring of premature babies, or those considered to be at risk, should always be carried out under the supervision of a health professional. This monitor is not intended to be used as a medical monitor.

STRANGULATION HAZARD. Children have STRANGLED in cords. Keep the adapter cords and sensor pad cord out of the reach of children. Do not put the Nursery Unit inside a crib or play yard. Because of the serious strangulation risk to infants and toddlers, parents and caregivers should never place the nursery unit within 3 feet (1 meter) of a crib. NEVER use extension cords with AC adapters. ONLY use the AC adapters provided.

Route sensor pad cord only below crib mattress and out of reach of children.

When an AC adapter is plugged into an electrical outlet, do NOT touch the exposed end.

ALWAYS be sure that both the transmitter and receiver are working properly and are within range of each other. Test monitor before first use, periodically and when changing location of the camera.

ALWAYS be sure to immediately check on baby whenever an alarm sounds.

Do NOT use monitor near water (such as bath tub, sink, ect.).

Keep monitor AWAY from heat sources (such as stoves, radiators, ect.).

ALWAYS place both the Nursery Unit and Parent Unit on a flat surface in an upright position and allow for proper ventilation. Do NOT place on sofas, cushions, beds, etc. which may block ventilation.

This monitor uses public airwaves to transmit signals. The parent Unit may pick up signals or interference from other nursery monitors in your area. Your monitor may even be picked up by other households. To protect your privacy, make sure both units are turned off when not in use. If you experience interference, change the channel on both units.

This product is NOT a toy. Do NOT allow children to play with it.

This product contains small parts. Adult assembly required. Exercise care when unpacking and assembling the product and keep small parts away from children.

Use and store the monitor at a temperature between 50°F (10°C) and 104°F (40°C).

⚠️ BATTERY WARNING:

Do NOT mix old and new batteries.

Do NOT mix alkaline, standard or rechargeable batteries.

Insert batteries with correct polarity.

Do NOT short-circuit supply terminals.

ONLY use the recommended batteries or equivalent (volts and size) and the correct adapter.

Remove batteries when product is stored for long periods of time or when batteries are exhausted.

Dispose exhausted batteries properly.

Do NOT recharge non-rechargeable batteries.

Keep all batteries away from children.

⚠️ IMPORTANT SAFETY INSTRUCTIONS

1) Read these instructions- All the safety instructions should be read before the product is operated.

2) Keep these instructions. The safety and operating instructions should be retained for future reference.

3) Heed all warnings – All warnings on the product and in the operating instructions should be adhered to.

4) Follow all instructions – All operating and use instructions should be followed.

5) Water and moisture- Do NOT use this apparatus near water. For example, near a bath tub, wash bowl, kitchen sink, laundry tub, in a wet environment or near a swimming pool and the like. Care should be taken so that objects do not fall onto units and that liquids are not spilled into either unit or sensor pad through its openings.

6) Cleaning - Disconnect all units before cleaning. Do NOT immerse in water any part of the Nursery Unit, Parent Units, or docking station. Clean only with dry cloth. Do NOT spray with cleaning products or solvents. Wipe sensor pad with a cloth moistened very slightly with an antiseptic or mild detergent.

7) Ventilation - Do NOT block any ventilation openings. Allow for proper ventilation when units are in use. Do NOT cover the Nursery Unit or Parent Units with any object such as a blanket. Do NOT place it in a drawer or in any location which would muffle the sound or interfere with the normal flow of air.

8) Heat sources- Do NOT install near any heat sources such as radiators, heat registers, stoves, TVs or other apparatus (including amplifiers) that produce heat. Heat can damage the case or electrical parts.

9) Power cord protection- Protect the sensor pad cord and adapter's power cord from being walked on or pinched by furniture or other items – Power supply cords should be routed so that they are not likely to be walked on or pinched by items upon or against them, paying particular attention to cords and plugs, convenience receptacle, and the point where they exit from the product.

10) Accessories- Only use attachments/accessories specified by the manufacturer. Do NOT place this product on an unstable cart, stand, tripod, bracket or table. The product may fall, causing serious injury to a child or adult and serious damage to the product. Use only with a cart, stand, tripod, bracket or table recommended by the manufacturer or sold with the product. Any mounting of the product should follow the manufacturer's instructions.

11) Unplug this appliance during lightning storms or when not used for long periods of time.

12) Servicing- Do NOT attempt to service this product yourself as opening or removing covers (except battery covers) may expose you to dangerous voltage or other hazards.

13) Damage required servicing- Unplug this product from the wall outlet and refer all servicing to qualified service personnel. Servicing is required when the apparatus has been damaged in anyway, such as power supply cord or plug is damaged, liquid has been spilled or objects have fallen into the apparatus, the apparatus has been exposed to rain or moisture, does not operate normally, or has been dropped.

14) Replacement parts - When replacement parts are required, be sure the service technician has used replacement parts specified by the manufacturer or have the same characteristic as the original part. Unauthorized substitution may result in fire, electric shock or other hazards.

15) Safety check- Upon completion of any service or repair to the product, ask the service technician to perform safety checks to determine that the product is in proper operating condition.

16) Usage-This baby monitor is intended as an aid. It is NOT a substitute for responsible and proper adult supervision and should not be used as such. Use this appliance ONLY for its intended use as described in this manual.

17) Power Sources- Use ONLY with 110-120 volt AC electrical outlets.

18) Overloading- Do NOT overload wall outlets, extension cords or integral convenience receptacles as this can result in a risk of fire or electric shock.

19) Object and Liquid Entry- NEVER push objects of any kind into this product through casing slots as they may touch dangerous voltage points or short out parts that could result in a risk of fire or electric shock. NEVER spill liquid of any kind on the product.

20) Antenna -The antenna used for this transmitter must be installed to provide a separation distance of at least 20 cm (7.9") from all persons and must not be collocated or operating in conjunction with any other antenna or transmitter.

21) Power Lines- An outside antenna system should NOT be located in the vicinity of overhead power lines or other electric light or power circuit, or where it can fall into such power lines or circuits as contact with them may be fatal.

22) STRANGULATION HAZARD. Children have STRANGLED in cords. Keep the adapter cords and sensor pad cord out of the reach of children. Because of the serious strangulation risk to infants and toddlers, parents and caregivers should NEVER place the nursery unit within 3 feet (1 meter) of a crib. NEVER use extension cords with AC adapters. ONLY use the AC adapters provided.

23) Interference- This monitor uses public airwaves to transmit signals. The Parent Unit may pick up signals or interference from other nursery monitors in your area. Your monitor may even be picked up by other households. To protect your privacy, make sure both units are turned off when not in use. If you experience interference, change the channel on both units.

24) Vibration Source- When the monitor is in use, do NOT use a crib mobile or other crib accessories that may produce vibrations. Other sources of vibrations may include fans, washing machines, loud music, etc. These sources of vibration may interfere with the performance of your Angelcare® monitor and the alarm will not sound if it continues to detect any movement other than your baby's. Be sure all vibration sources are eliminated before using the monitor.

You may experience false alarms for several reasons. The most common reason is the Nursery Unit has not been turned off after you have removed your baby from the crib. Another reason is the Sensor Pad has not been placed under the mattress properly. Be sure the Sensor Pad is on a flat, firm surface with

the printed side facing up. The sensitivity level may also need to be adjusted. Refer to the Movement Sensitivity Adjustment section. Always be sure to immediately check on baby whenever an alarm sounds.

25) Sensor Pad- Your sensor pad can be used in any crib that is stationary and designed to safely hold a baby. If the crib has a spring or slat base, place a hard board between the crib base and the Sensor Pad measuring at least 13" x 13" (33 x 33 cm) and 1/4 inch (6 mm) thick. Only use your Angelcare® monitor in a stationary sleeping environment such as a crib where the Sensor Pad can rest on a completely flat, firm surface.

ALWAYS route the sensor pad cord below crib mattress ONLY and out of reach of children.

IMPORTANT SAFETY INSTRUCTIONS RELATED TO USE OF BATTERIES

1) Use ONLY the type and size of batteries specified in the user manual. There is a risk of explosion if batteries are replaced with an incorrect type.

2) ONLY use the AC adapters included with your monitor.

3) Do NOT use these adapters with any other equipments.

4) When unplugging the AC adapter from the nursery unit, be sure it is switched to the off position. If it isn't, the unit will revert to battery mode if batteries are installed.

5) Do NOT dispose batteries in fire. They may explode. Check with local regulations for possible special disposal instructions.

6) Exercise care in handling batteries in order not to short out the batteries with conducting materials such as rings, bracelets and keys. Overcharging, short circuiting, reverse charging, mutilation, or incineration of batteries must be avoided to prevent one or more of the following occurrences: release of toxic materials, release of hydrogen and/or oxygen gas, rise in surface temperature.

7) Do NOT attempt to recharge batteries provided with or identified for use with this product that are not intended to be charged. The batteries may leak corrosive electrolyte or explode.

8) Do NOT attempt to rejuvenate the batteries with or identified for use with this product by heating them. Sudden release of battery electrolyte may occur, causing burns or irritation to eyes or skin.

9) If and when necessary, the batteries must be replaced with identical new ones or equivalent (volts and size). If a battery to be replaced is connected with other batteries in series, replace the other batteries at the same time as well. Do NOT mix old and new batteries.

10) Reverse polarity installation of the batteries in the end product must be avoided. Reverse insertion of batteries can cause charging, and that may result in leakage or explosion.

11) Remove batteries from this product if you store the product over 30 days because the batteries could leak and damage the product.

12) Discard 'dead' batteries as soon as possible since 'dead' batteries are more likely to leak in a product. If a battery has leaked or vented, it should be replaced immediately using protective gloves.

13) Do NOT store this product, or the batteries provided with or identified for use with this product, in high-temperature areas. Batteries that are stored in a freezer or refrigerator for the purpose of extending shelf life should be protected from condensation during storage and defrosting. Batteries should be stabilized at room temperature prior to use after cold storage.

COMPLIANCE INFORMATION

WARNING

Modifications not authorized by the manufacturer may void users authority to operate this device.

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications.

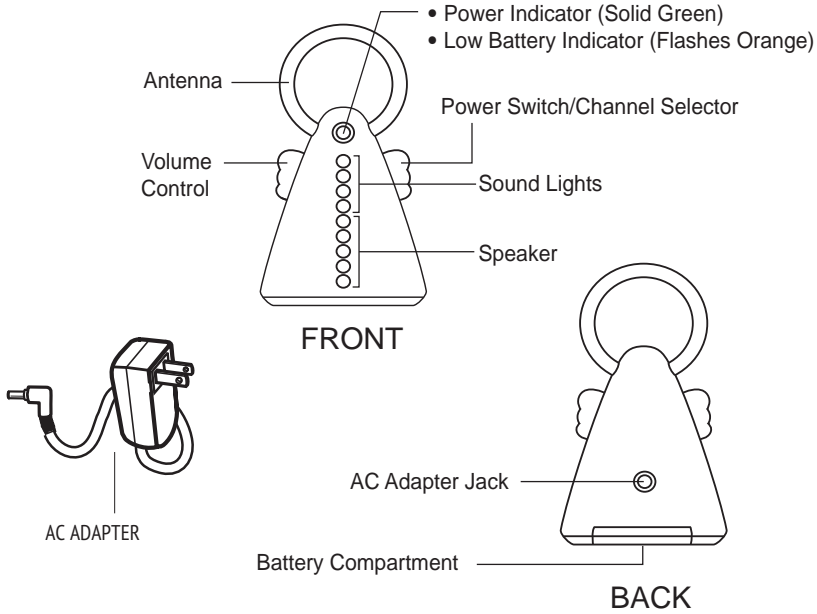
However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.


This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions:

- (1) This device may not cause harmful interference, and
- (2) This device must accept any interference received, including interference that may cause undesired operation.

Features Parent Unit


Nursery Unit

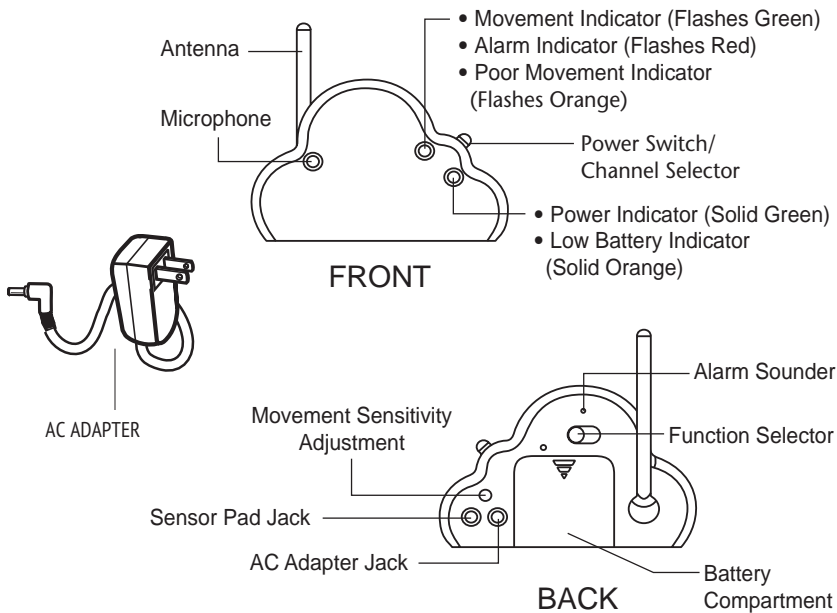Installing Batteries in Parent Unit

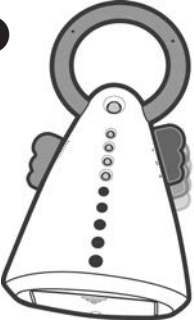
1


To insert batteries, remove the battery cover on the bottom of the unit.

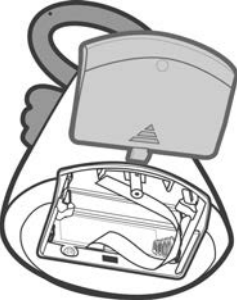2

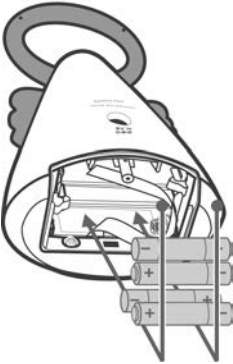
The Parent Unit requires four AAA (LR03) batteries (not included). To insert batteries, remove the battery cover on the back of the unit and insert the batteries matching positive (+) to positive (+) and negative (-) to negative (-) and replace the battery cover.

Replace the batteries when the Low Battery Indicator flashes and turns orange instead of green.

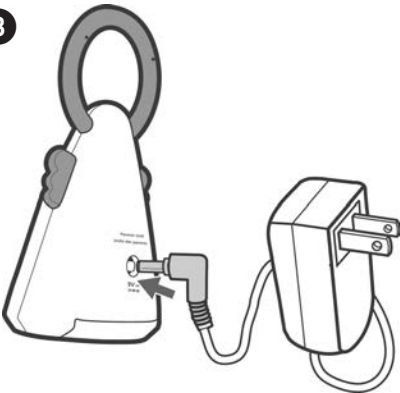3

IMPORTANT: Use only the AC adapters included with your monitor. Do not use these adapters with any other equipment.

Insert the AC adapter into the jack marked “9V (6LR61)” and plug the adapter into an electrical outlet. Unit will run off of the AC adapter unless there is a power outage, in which case the unit will begin using the batteries.

Installing Batteries in Nursery Unit

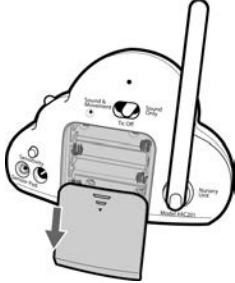
1


The Nursery Unit requires four AAA (LR03) alkaline or rechargeable batteries (not included).

2


To insert batteries, remove the battery cover on the back of the unit.

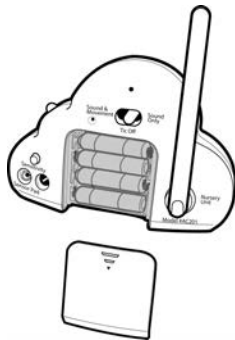3


Insert the batteries matching positive (+) to positive (+) and negative (-) to negative (-). Replace the battery cover.

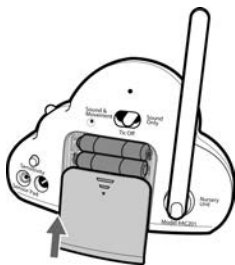4

Replace the battery cover. Replace the batteries when the Low Battery Indicator turns orange and a rapid ticking is heard. If the batteries are not replaced, the light will eventually turn red and the monitor will stop working.

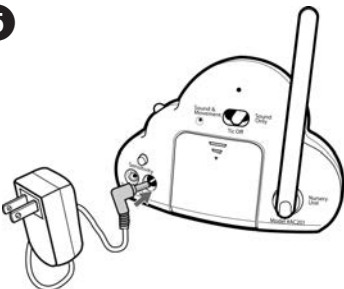5

IMPORTANT: Use only the AC adapters included with your monitor. Do not use these adapters with any other equipment.

Insert the AC adapter into the jack marked "9V" and plug the adapter into an electrical outlet. Unit will run off of the AC adapter unless there is a power outage, in which case the unit will begin using the batteries.

Setting Up Your Monitor

SENSOR PAD (See figs. 1 and 2)

- Connect the Sensor Pad cord to the Nursery Unit.
- Place the Sensor Pad in the middle of the crib under the mattress, as illustrated below. Make sure the printed side faces up. Do NOT place any bedding between the mattress and the sensor pad.
- Secure the sensor pad cord to crib using a tie wrap. Run the cord on the floor to ensure it is out of the baby's reach.
- The Sensor Pad must rest on a completely flat and firm surface. If the crib has a spring or slat base, place a hard board between the crib base and the Sensor Pad measuring at least 13" x 13" (33x33 cm) and 1/4 inch (6 mm) thick. Make sure the sensor pad's cord passes underneath the mattress as shown below.
- The sensor pad works with all baby bed sizes and all mattresses, except memory foam mattresses. You will have to test your sensor pad on p.16.
- Once you are done using the Sensor Pad, please remove it from the baby's crib and store it away.
- The Angelcare Sensor Pad feature is not designed for use with adults, children or infants older than 24 months.

WARNING

STRANGULATION HAZARD - Children have STRANGLED in cords. Keep the adapter cords and sensor pad cord out of the reach of children. Do not put the Nursery Unit inside a crib or play yard. Because of the serious strangulation risk to infants and toddlers, parents and caregivers should never place the nursery unit within 3 feet (1 meter) of a crib. NEVER use extension cords with AC adapters. ONLY use the AC adapters provided.

Figure 1


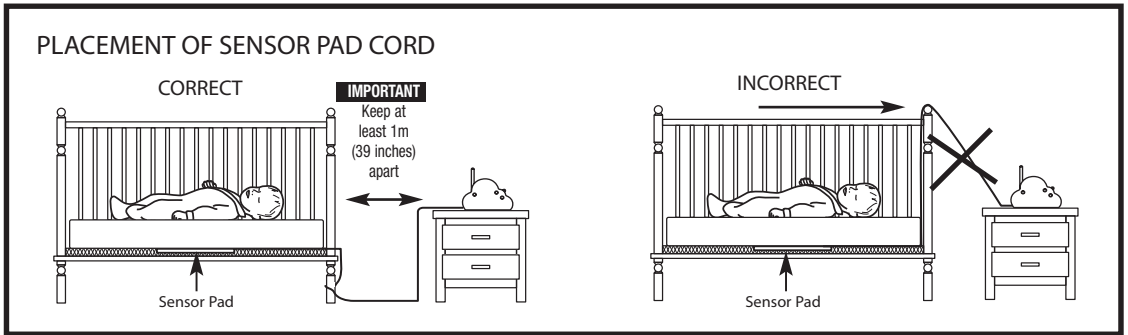
Figure 2

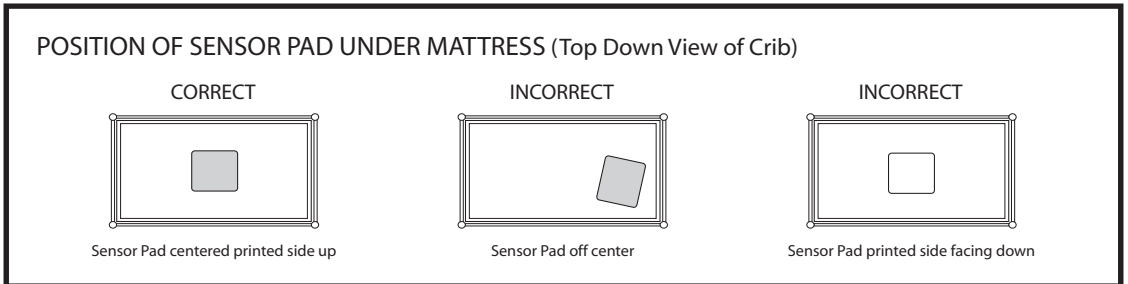Movement Sensitivity Control

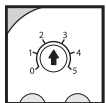
Figure 3

IMPORTANT: Only use the AC adapters included with your monitor. Do not use these adapters with any other equipment. Movement Sensitivity Control. The movement sensitivity control is on the back of the Nursery Unit. The level has been preset to suit most conditions of movement detection (see fig.3). Do not adjust this unless you receive false alarms. Refer to the Movement Sensitivity Adjustment section on for more information. It is extremely important to follow our instructions for setting the sensitivity level. Now plug the Sensor Pad cord into the jack on the back of the Nursery Unit marked "Sensor Pad." Turn the Nursery Unit on using the switch on the left. The green Power Indicator will light up.

Activating Units

Always test your monitor before using it the first time and then on a periodic basis.
Before testing your monitor, complete the setup process as described above.

On the back of the Nursery Unit select one of the following three functions:

- Sound & Movement: You will be able to hear your baby's sounds, see sound lights, and hear a "Tic" sound with every movement your baby makes. If no movement is detected, the "Tic" will stop and after 20 seconds the alarm will sound.
- Tic Off: In this mode you will not hear a "Tic" with every movement. You will still hear your baby's sounds and see sound lights, and your baby's movement is still being detected. The default setting is "Tic Off."
- Sound Only: The Movement Sensor is disabled but you will still be able to hear your baby's sounds and see sound lights. Your Angelcare® works like a regular sound monitor when set to "Sound Only."

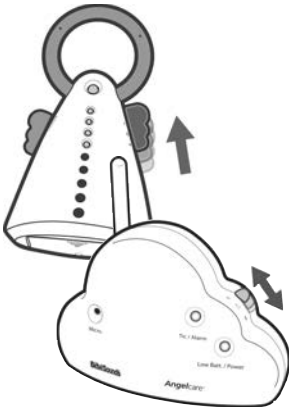
Sound Only

1


Select "Sound Only" as shown.

2

Turn the Parent Unit ON by selecting either channel "A" or "B". Make sure it is set to the same channel as the Nursery Unit. The green power indicator light will light up. Choose the channel that gives you the best reception.

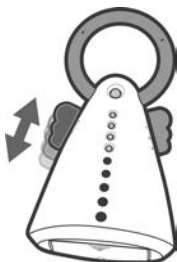
3

Adjust the volume to the desired level as shown.

Sound & Movement • • • • • • • • • •

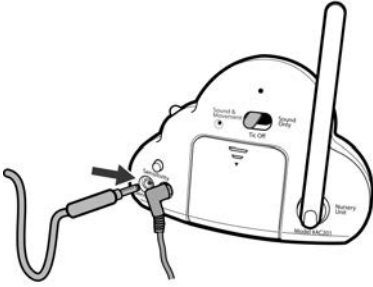Note: Make Sure your Sensor Pad is set up before proceeding with this section. If you haven't set up your Sensor Pad please go to instructions on how to properly set up the Sensor Pad.

- Sound & Movement: You will be able to hear your baby's sounds, see sound lights, and hear a "Tic" sound with every movement your baby makes. If no movement is detected, the "Tic" will stop and after 20 seconds the alarm will sound.

Switch the Nursery Unit to the "Sound & Movement" function. Turn the Parent Unit ON by selecting either channel "A" or "B". Make sure it is set to the same channel as the Nursery Unit. The Movement Indicator Light on the Nursery Unit will flash red, green, and then emit a single "Tic" sound. Gently pat your hand on the mattress. The green Movement Indicator Light will continue to flash with every movement. The Nursery Unit will also tick with every movement.

Remove your hand from the mattress. Due to no movement, the ticking will stop, the Movement Indicator Light will stop flashing, and a pre-alarm beep will sound after 15 seconds. Five seconds later the alarm will sound and the red Alarm Indicator Light will flash continuously.

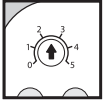
Tic Off

You can also test for movement with the Nursery Unit set to the "Tic Off" position. In this case, you will not hear a constant ticking when you touch the mattress. When you remove your hand from the mattress the Movement Indicator Light will stop flashing, a pre-alarm beep will sound after 15 seconds, and five seconds later the alarm will sound and the red Alarm Indicator Light will flash continuously.

To turn off the alarm, turn the Nursery Unit off or pat your hand lightly on the mattress so the Sensor Pad will redetect movement. The red Alarm Indicator Light on the Nursery Unit will continue to flash for one minute and it will then revert to green.

If the alarm does not sound when you are not touching the mattress, the Sensor Pad may be detecting a continuous vibration from another source from inside and/or outside your baby's room. Common sources of this interference are crib mobiles, fans, washing machines, or even loud music. Therefore, it is very important to make sure all vibration sources are eliminated before using the monitor. Place the crib near a solid supporting wall and away from strong air currents. Do not use crib mobiles or accessories that may cause vibrations. Avoid contact with the crib when the monitor is in use. If you find the alarm will still not sound when you are not touching the mattress or crib, the monitor is on, and you have removed all external sources of vibration, the sensitivity level may need to be decreased. Please refer to the Movement Sensitivity Adjustment section.

Special Features

Movement Sensitivity Adjustment

The movement sensitivity adjustment dial has been preset to a level which will suit most situations. The highest sensitivity level is 5 (most sensitive) and the lowest sensitivity level is 1 (least sensitive).

You may get false alarms if your baby is in a very deep sleep or has moved to the edge of the crib. If you get false alarms due to these circumstances, you should increase the sensitivity level by using the sensitivity dial on the back of the Nursery Unit.

NOTE: Adjust the sensitivity level if you receive false alarms and you have followed the advice in the Troubleshooting section for false alarms. Make a note of the position of the sensitivity dial before you change it should you wish to return to that setting.

To increase the sensitivity level, start by increasing it by half a level. For example, if the level is currently at 2.5, increase it to 3. Use the monitor as you normally would. If you still experience false alarms, increase it by another half level. If you are still experiencing false alarms when you have reached level 5, please consult our customer service link on our website at www.angelcare-monitor.com

Because the Sensor Pad may detect movement from many sources from both inside and outside your baby's room, be sure all vibration sources are eliminated before changing the movement sensitivity level. Check to see if the Sensor Pad is resting on a flat, firm surface. Place the crib near a solid supporting wall and away from strong air currents. Do not use crib mobiles or accessories that may cause vibrations. Avoid contact with the crib when monitor is in use.

To make sure you have avoided all external sources of vibration, take your baby out of the crib while the monitor is on. If the alarm does not sound after you take your baby out of the crib, the sensitivity level may need to be decreased. To decrease the sensitivity level, adjust by half a level by going from 2.5 to 2. If you are still experiencing this problem when the level is set to 1, please consult our customer service link on our website at www.angelcare-monitor.com

Sound Lights

The Parent Unit features Sound Lights so you can “see” your baby's sounds if background noise prevents you from hearing your baby. As the sounds get louder, more lights illuminate.

Optional “Tic” Feature

When the “Tic” feature is selected, the Parent Unit will continuously tick as your baby's movement is detected by the Sensor Pad. If no movement is detected, the ticking will stop. If you do not want to hear the ticking, slide the switch to the “Tic Off” position.

Movement Indicators

The Movement Indicator Light is located only on the Nursery Unit. When the Nursery Unit is set to “Sound & Movement” or “Tic Off,” the Movement Indicator Light will flash green with your baby's every movement. The flashing green light will let you know that movement is being detected. This will allow you to check on your sleeping baby in the dark without disturbing.

Tips For Using Your Monitor

- When the monitor is in use, do not use a crib mobile or other crib accessories that may produce vibrations. Other sources of vibrations may include fans, washing machines, loud music, etc. These sources of vibration may interfere with the performance of your Angelcare® monitor and the alarm will not sound if it continues to detect any movement. Be sure all vibration sources are eliminated before using the monitor.
- There are several reasons you may experience false alarms. The most common reason is the Nursery Unit has not been turned off after you have removed your baby from the crib. Another reason is the Sensor Pad has not been placed under the mattress properly. Be sure the Sensor Pad is on a flat, hard surface with the printed side facing up. The sensitivity level may also need to be adjusted. Refer to the Movement Sensitivity Adjustment section.
- Your monitor can be used in any crib that is stationary and designed to safely hold a baby. If the crib has a spring or slat base, place a hard board between the crib base and the Sensor Pad measuring at least 13" x 13" (33x33 cm) and 1/4 inch (6 mm) thick. Only use your Angelcare® monitor in a stationary sleeping environment such as a crib where the Sensor Pad can rest on a completely flat and firm surface.
- When unplugging the AC adapter from each unit, be sure the unit is switched to the off position. If it isn't, the unit will revert to battery mode if batteries are installed.
- You can use rechargeable batteries, however, a rechargeable battery cannot be recharged by plugging in the AC adapter. Remove batteries from the unit to recharge.
- The range of this monitor is approximately 300 feet. However, the range will vary depending on the surroundings. Test the range before use. NOTE: This monitor is not equipped with an out-of-range indicator. No sound will be received if the Nursery Unit and Parent Unit are too far apart.

Testing Your Monitor

Always test your monitor before using it the first time and then on a periodic basis. Before testing your monitor, complete the set-up process as described above.

1. Testing for Sound. Turn the Nursery Unit and Parent Unit on, making sure they are both set to the same channel. On the Nursery Unit, switch to the "Sound Only" function. Have someone else speak into the Nursery Unit or place a radio near the Nursery Unit while you take the Parent Unit into another room. You should hear the sounds being transmitted from the Nursery Unit. If there is no sound, adjust your volume control and/or refer to the Troubleshooting section. If you experience interference or static, switch both units to the other channel.

2. Testing for Movement. Switch the Nursery Unit to the "Sound & Movement" function. The Movement Indicator Light on the Nursery Unit will flash red, green, and then emit a single "Tic" sound. Gently pat your hand on the mattress. The green Movement Indicator Light will continue to flash with every movement. The Nursery Unit will also tick with every movement. Remove your hand from the mattress. Due to no movement, the ticking will stop, the Movement Indicator Light will stop flashing, and a pre-alarm beep will sound after 15 seconds. Five seconds later the alarm will sound and the red Alarm Indicator Light will flash continuously.

You can also test for movement with the Nursery Unit set to the "Tic Off" position. In this case, you will not hear a constant ticking when you touch the mattress. When you remove your hand from the mattress the Movement Indicator Light will stop flashing, a pre-alarm beep will sound after 15 seconds, and five seconds later the alarm will sound and the red Alarm Indicator Light will flash continuously.

To turn off the alarm, turn the Nursery Unit off or pat your hand lightly on the mattress so the Sensor Pad will redetect movement. The red Alarm Indicator Light on the Nursery Unit will continue to flash for one minute and it will then revert to green.

If the alarm does not sound when you are not touching the mattress, the Sensor Pad may be detecting a continuous vibration from another source from inside and/or outside your baby's room. Common sources of this interference are crib mobiles, fans, washing machines, or even loud music. Therefore, it is very important to make sure all vibration sources are eliminated before using the monitor. Place the crib near a solid supporting wall and away from strong air currents. Do not use crib mobiles or accessories that may cause vibrations. Avoid contact with the crib when the monitor is in use.

If you find the alarm will still not sound when you are not touching the mattress or crib, the monitor is on, and you have removed all external sources of vibration, the sensitivity level may need to be decreased. Please refer to the Movement Sensitivity Adjustment section.

3. Testing for Battery Power. Verify that the units are operating on battery power by installing four AAA batteries in each unit and unplugging the AC adapters. The green Power Indicator should light up on all units.

Care and Maintenance

Disconnect all units and remove all batteries before cleaning. Clean the exterior surface of the units with a soft dry or slightly damp cloth. Do not spray with cleaning products or solvents. Do not submerge in water or any other liquid.

During long periods of non-use, remove all batteries from their compartments and unplug the AC adapters.

Troubleshooting

Problem	Possible Cause	Solution
False alarms	Baby was removed from crib and the Nursery Unit was not switched off.	Turn the Nursery Unit off.
	The printed side of the Sensor Pad is not facing up.	Make sure the printed side is facing up.
	The Sensor Pad is not detecting your baby's movements because the sensitivity level is too low.	The sensitivity level may need to be increased. Refer to the Movement Sensitivity Adjustment section.
	Sensor Pad is not in full contact with the crib mattress or is not resting on a flat, firm surface.	Make sure there is no bedding between mattress and the Sensor Pad. The Sensor Pad must rest on a completely flat and firm surface. If the crib has a spring or slat base, place a hard board between the crib base and the Sensor Pad measuring at least 13" x 13" (33x33 cm) and 1/4 inch (6 mm) thick.
	Baby has moved far away from the Sensor Pad.	Make sure that the sensor pad is installed as shown. The sensitivity level may also need to be increased. Refer to the Movement Sensitivity Adjustment section.

Problem	Possible Cause	Solution
False alarms	Sensor Pad cord is not connected properly to the Nursery Unit.	Unplug and reconnect the cord.
Alarm will not sound	Nursery Unit is detecting movement when crib is being touched.	Avoid contact with the crib when monitor is on.
	Nursery Unit is detecting movement from outside the crib such as motorized devices, strong drafts, etc.	Place crib near a solid supporting wall to avoid Sensor Pad from detecting movement from outside the crib. Turn off crib mobile, fans, loud music or anything else that may cause vibrations.
	Sensor Pad sensitivity level is set too high.	You may need to decrease the sensitivity. Refer to the Movement Sensitivity Adjustment section.
Movement Indicator Light on Nursery Unit is flashing orange (monitor is barely detecting movement).	Sensor Pad is not in full contact with crib mattress, or the mattress is not resting on a hard surface.	Make sure there is no bedding between mattress and the Sensor Pad. The Sensor Pad must rest on a completely flat and firm surface. If the crib has a spring or slat base, place a hard board between the crib base and the Sensor Pad measuring at least 13" x 13" (33x33 cm) and 1/4 inch (6 mm) thick.
	Baby is in a very deep sleep or has moved away from Sensor Pad to the edge of the crib.	Make sure Sensor Pad is positioned correctly in crib. The sensitivity level may need to be adjusted. Refer to the Movement Sensitivity Adjustment section.
No sound is being transmitted.	Parent and Nursery Units are too far apart.	Bring Parent Unit closer to Nursery Unit.
	Batteries are low or improperly installed.	Check or replace batteries.
	AC adapters are not connected properly or electrical outlet is out of order.	Check connections or change to another electrical outlet.
	One of the units is not on.	Make sure both units are on.
	Units are set to different channels.	Make sure both units are on the same channel, either both on A or B.
	Volume level is too low.	Increase volume level.

Problem	Possible Cause	Solution
Power Indicator (green light) does not light up.	Batteries are low or improperly installed.	Check or replace batteries.
	AC adapters are not connected properly or electrical outlet is out of order.	Check connections or change to another electrical outlet.
	One of the units is not on.	Make sure both units are on the same channel, either both on A or B.
There is very weak reception.	Parent and Nursery Units are too far apart.	Bring Parent Unit closer to Nursery Unit.
	One or both units are placed near a large metal mass.	Change the position of one or both units.
	One or both units are not in the upright position.	Place units on flat surface away from baby's reach.
	Batteries are low.	Check or replace batteries.
There is static, distortion, interference, etc.	Parent and Nursery Units are too far apart.	Bring Parent Unit closer to Nursery Unit.
	Units are not set to the same channel.	Make sure both units are on the same channel, either both on A or B.
	One of the units is not on.	Make sure both units are on.
	The Parent Unit is located near motorized devices, fluorescent lights, television, etc.	Rotate Parent Unit or move it away from source of interference.
There is feedback (squealing noises).	Units are too close to each other.	Move units farther apart (at least 10 feet apart).
	The volume is too high.	Decrease volume on Parent Unit.

IMPORTANT! This Angelcare® monitor is a personal care product. Please do not return this product to the store. If you have questions or need help, please call Angelcare before returning this product.

Replacement Parts
Warranty Information (USA & Canada only) www.angelcare-monitor.com
angelcare@cdecinc.ca - 1-877-281-8827

REGISTER ANGELCARE PRODUCTS

Register your Angelcare monitor and other Angelcare products on our website: www.angelcare-monitor.com
3980, rue Saint Ambroise, Montréal, Québec, H4C 2C7, Canada

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The ANGELCARE® trademark is a registered trade mark of the Angelcare Development Inc.
La marque ANGELCARE® est une marque enregistrée propriété de Les Développements Angelcare Inc.
La marca ANGELCARE® es una marca registrada de Angelcare Development Inc.