# EXL30 AMPLIFIED TELEPHONE

Line Powered

User Guide

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Manuale d'istruzioni

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## HELP LINE TELEPHONE NUMBERS

United Kingdom  
0800 410014

Germany  
0800 9323400

France  
0825 0825 99  
(0,15 € TTC/mn)

Switzerland  
0800 932340

Spain  
902 415191

Netherlands  
0800 7526876

Luxembourg and Belgium  
00800 75268766

Austria  
0800 242500

Sweden  
0200 214681

Portugal  
+34 91 640 47 44

Denmark  
80884610

Finland  
0800 117095

Ireland  
1 800 551896

Norway  
800 11336

Italy  
800 950934

# DIAGRAMS

FIG. A

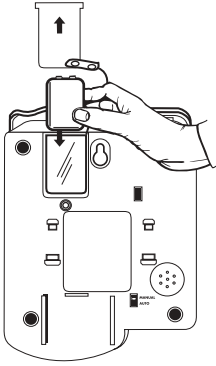
FIG. B

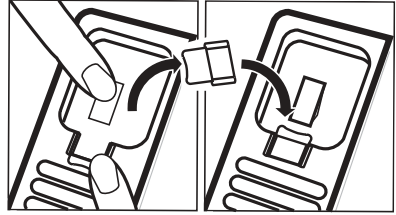
FIG. C



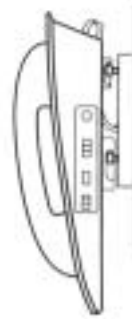
FIG. D
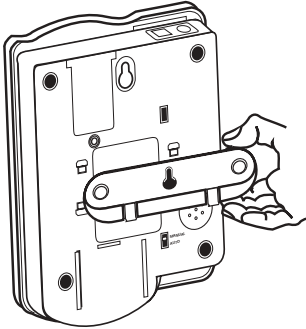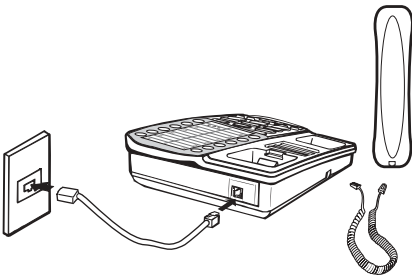
# DIAGRAMS

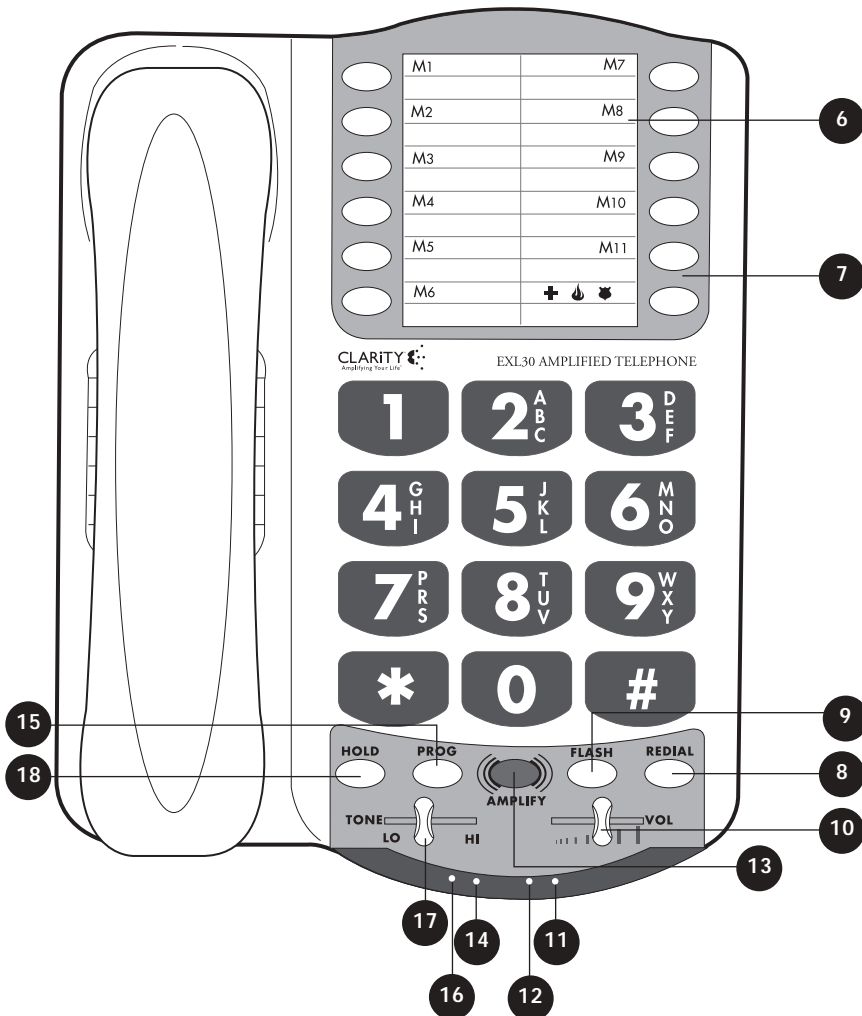
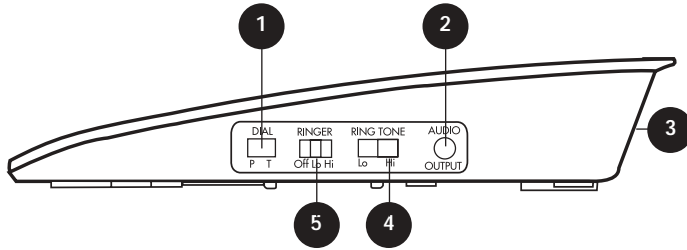
FIG. E

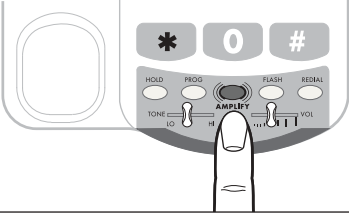
FIG. F

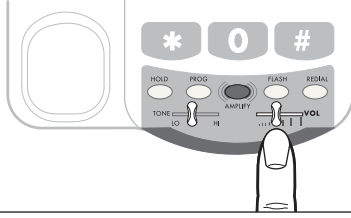
FIG. G

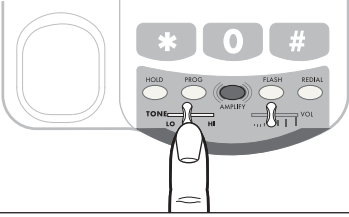
FIG. H

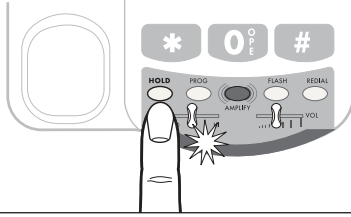
FIG. I

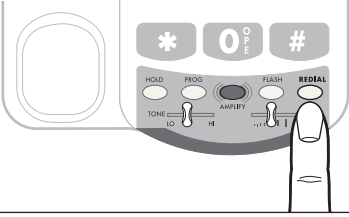
FIG. J

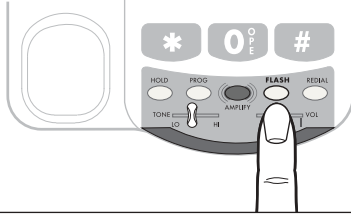
FIG. K

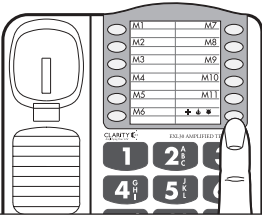
FIG. L

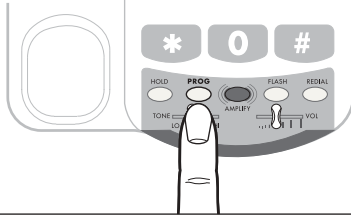
FIG. M


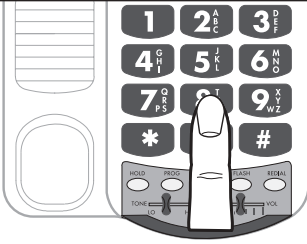
FIG. N
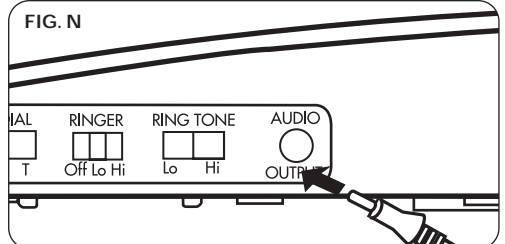
|                      |                     |
|----------------------|---------------------|
| <b>UK ENGLISH</b>    | <b>Page 6-13</b>    |
| <b>D DEUTSCH</b>     | <b>Seite 14-21</b>  |
| <b>E ESPAÑOL</b>     | <b>Página 22-29</b> |
| <b>F FRANÇAIS</b>    | <b>Page 30-37</b>   |
| <b>I ITALIANO</b>    | <b>Pagina 38-45</b> |
| <b>NL NEDERLANDS</b> | <b>Pagina 46-53</b> |

# WELCOME

---

## **EXL30 Amplified Telephone from Clarity**

Thank you for selecting the EXL30 Amplified Telephone from Clarity.

This User Guide provides you with the information you need to use your EXL30 effectively and safely.

Read this user guide thoroughly before using your telephone. Keep the manual near the telephone for easy reference.

The EXL30 Amplified Telephone is Hearing Aid compatible.

## DIAGRAM KEY

---

- |   |                               |    |                        |
|---|-------------------------------|----|------------------------|
| 1 | Dial Mode Switch (Pulse/Tone) | 10 | Volume Control         |
| 2 | Audio Output Socket           | 11 | Low Battery Indicator  |
| 3 | Telephone Line Connection     | 12 | Visual Ring Indicator  |
| 4 | Ring Tone Switch              | 13 | Amplify Button         |
| 5 | Ringer Volume Switch          | 14 | Amplify Indicator      |
| 6 | Telephone Number Directory    | 15 | Programme Button       |
| 7 | Memory Buttons                | 16 | Call On Hold Indicator |
| 8 | Redial Button                 | 17 | Tone Control           |
| 9 | Flash/Recall Button           | 18 | Call Hold Button       |

## IMPORTANT SAFETY INFORMATION

---

### PLEASE READ THE FOLLOWING INSTRUCTIONS BEFORE USE:

When using your telephone equipment these basic safety precautions should always be followed to reduce the risk of fire, electric shock, and injury to persons including the following:

Read and understand all instructions.

Follow all warnings and instructions marked on the telephone.

Please ensure this product is disconnected from the telephone line cable before installing or replacing batteries.

Do not dispose of batteries in a fire. They may explode. Take empty batteries to a collection point for chemical waste.

Do not install or use this telephone near water.

Avoid using a telephone (other than a cordless type) during a storm. There may be a remote risk of electrical shock from lightning.

Unplug this telephone from the wall outlets before cleaning. Do not use liquid cleaners or aerosol cleaners on the telephone. Use a damp cloth for cleaning.

Do not cover the slots and openings on this telephone. This telephone should never be placed near or over a radiator or heat register. This telephone should not be placed in a built-in installation unless proper ventilation is provided.

Do not place anything on the telephone line cable. Install the telephone where no one will step or trip on the cable.

Never push any objects through the slots in the telephone. They can touch dangerous voltage points or short out parts that could result in a risk of fire or electrical shock. Never spill liquid of any kind on the telephone.

To reduce the risk of electrical shock, do not take this phone apart. Opening or removing covers may expose you to dangerous voltages or other risks. Incorrect reassembly can cause electric shock when the appliance is subsequently used.

Never install telephone wiring during a lightning storm.

**Hereby, Plantronics BV declares that the Clarity™ EXL30 Amplified Telephone is in compliance with the essential requirements and all other relevant provisions of Directives 1999/5/EC and 93/42/EEC.**

To view the Declaration of Conformity for this product please see [www.plantronics.com/documentation](http://www.plantronics.com/documentation)

**Class I medical device**

**When the Amplified Indicator (14) is illuminated, the volume may be very loud and in excess of 118dB. This mode is intended specifically for hearing impaired users.**

## **GETTING STARTED**

---

### **INSTALL BATTERIES**

**Please ensure this product is disconnected from the telephone line cable before installing or replacing batteries.**

To add extra brightness to the EXL30's Visual Ring Indicator (12), install one 9 Volt alkaline battery (not included).

### **TO INSTALL THE BATTERY**

**Fig. A** Use a pen to open the battery door (see instructions underneath the telephone unit)  
Slide open the battery compartment cover on the bottom of the telephone, install the battery, and replace the cover.

### **SETTING THE DIALLING MODE**

Set the Dial Mode Switch (1) to T (tone) if you have touch tone DTMF service. Otherwise set the switch to P (pulse).

## **CONNECTING FOR WALL MOUNT OR DESKTOP USE**

---

### **WALL MOUNTING**

Drill two holes in the wall 10.2cm (4") apart adjacent to the mounting holes on the telephone. Use the correct screws to hang the telephone on the wall.

**Fig. B** Slide the handset hook out as shown in Fig. B and reverse its position so that the hook points up and will hold the handset when you hang up.

**Fig. C** Insert the tabs of the phone base into the slots in the lower position of the base as shown in Fig. C. Push the mounting bracket in and down firmly until it snaps into place. Position on the wall close to your telephone wall socket.

**Fig. D** Connect the Telephone Line Cable and the Handset Cable to the telephone. If required, use the supplied adapter.

Lift the handset and listen for a dial tone.

The telephone is ready to use.

## DESKTOP

**Fig. D** Connect the Telephone Line Cable and the Handset Cable to the telephone. If required, use the supplied adapter.

Lift the handset and listen for a dial tone. The telephone is ready to use.

## USING YOUR EXL30

---

### INCOMING VOICE AMPLIFICATION

**Fig. E** Lift the handset and press the Amplify Button (13) to turn on the amplifier.

**Fig. F** You can adjust the volume of incoming calls by sliding the Volume Control (10). The amplifier will turn off when you hang up.

**CAUTION:** Operation of the Amplification feature will make the receiver very loud. It is designed specifically for hearing impaired users.

Amplification can be activated by pressing the Amplify Button (13) the Amplify Indicator (14) indicates that Amplification is in operation.

By activating the Amplification feature you are choosing to increase the level of amplification that allows any abnormal event to exceed 118dB. This is potentially harmful. The volume control should be increased gradually to find the minimum level of amplification that is comfortable for you. Avoid prolonged or unnecessary exposure to high volume levels

### INCOMING VOICE TONE

The EXL30 provides extra amplification at the sound frequency you need to boost. To identify the sound frequency range that best suits your needs, follow these steps:

Lift handset and listen for a dial tone.

Press the Amplify Button (13) to turn on the amplifier (refer to Fig. E on page 4).

**Fig. G** Slide the Tone Control (17) either right or left for maximum clarity.

The next time and every time you press the the Amplify Button (13), your EXL30 will give you the best listening clarity.

### RINGER VOLUME AND TONE

You can adjust the Ringer Volume Switch (5) as high as 85+ decibels. The settings available are OFF, LO and HI, as shown in the main diagram on page 3. The Ring Tone Switch (4) has two pitch settings: LO and HI, as shown in the main diagram on page 3.

### HOLD

**Fig. H** To put the call on hold, press the Call Hold Button (18) and return the handset to its cradle. You will notice the Call On Hold Indicator (16) illuminates.

**NOTE:** If you pick up an extension phone on the same line when the EXL30 is on HOLD, the EXL30 will release the HOLD and you can continue your conversation on the extension telephone.

## LAST NUMBER REDIAL

**Fig. I** To redial the last number called, lift the handset and press the Redial Button (8). The telephone will redial up to 31 digits.

## FLASH/RECALL

**Fig. J** Press the Flash/Recall Button (9) to access special services available from your local telephone company e.g. Call Waiting and 3-Way Calling.

(Subject to your local telephone service).

## PROGRAMMING THE MEMORY BUTTONS

**Fig. K** You can automatically dial twelve programmed telephone numbers using the Memory Buttons (7). Each button can be programmed (or reprogrammed using the same procedure) for a telephone number up to 15 digits long.

**Fig. L** To program a button, follow these steps:

Lift the handset and press the Programme Button (15).

**Fig. M** Enter the phone number on the keypad just as you would dial it normally.

Press Redial Button (8) if you want to insert a brief pause between two numbers.

Press a Memory Button (7) to store this number on that button.

Immediately hang up the handset.

**NOTE:** Any number previously stored at that button will be overwritten.

To change a stored number, repeat the programming process starting with Fig. K.

**NOTE: Do not programme emergency service numbers into any memory button.**

## DIRECT AUDIO OUTPUT

**Fig. N** The Audio Output Socket (2) allows you to connect the EXL30 to a hearing aid, neck loop, cochlear implant or other assistive listening devices.

To use this port, plug in a compatible auxiliary cable and connect it to your assistive listening device.


Speak into the handset when you use the Audio Output connection.

**NOTE:** This port is a TNV connection with respect to EN41003. Only connect equipment that complies with EN60950 to this port.

## CONNECTING A SPEECH PROCESSOR COCHLEAR IMPLANT ADAPTER CORD

**Caution:** Before plugging any cord into your speech processor, you must consult your implant manufacturer.

- Plug the mono end of the plug into the speech processor.
- Plug the stereo end of the plug into the Audio Output socket on the telephone.
- Speak into the handset when using the Cochlear Implant Adapter Cord.

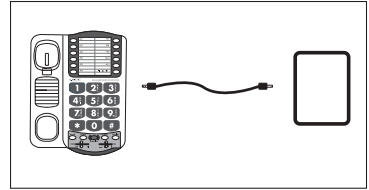## OUTGOING VOICE AMPLIFICATION

The EXL30 can amplify your outgoing voice by up to 15dB. To use this feature, set the Outgoing Voice Amplifier switch to AMPLIFY located underneath the telephone unit.

**NOTE:** The factory pre-setting is NORMAL.

## TROUBLE SHOOTING

---

### PROBLEM

---

*No dial tone.*

Check all the telephone cable connections.

### PROBLEM

---

*Call cannot be dialled, or dials very slowly.*

Reset the Dial Mode Switch (see Setting the Dial Mode).

### PROBLEM

---

*Visual Ring Indicator is not very bright.*

Check and replace the 9 Volt battery.

### PROBLEM

---

*No number is dialled when the memory button is pressed.*

No number stored in that Memory Button (see Programming the Memory Buttons).

### PROBLEM

---

*Call comes in but the telephone does not ring.*

Check that the Telephone Line Cable is correctly connected.

## **WARRANTY**

- The following 2 year warranty and service information applies only to product purchased and used in the countries listed on page 1. You may also contact your local retail dealer for warranty information in your country.
- Clarity guarantees the satisfactory condition of the equipment with regard to manufacturing and material defects for 2 years from the date of purchase. (Please keep your original receipt safely).
- This product must be installed and used in accordance with the instructions contained in this User Guide, and serviced at an approved Plantronics Service Centre. Failure to comply with these conditions may render the warranty invalid.

## **2 YEAR WARRANTY AND REPAIR SERVICE**

- Products under warranty are replaced free of charge.
- All units serviced to full production standards using original factory parts.

## **CUSTOMER SERVICE**

If you need support in installing or using your EXL30 telephone, then please contact the Clarity Customer Service (see page 1).

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Plantronics B.V.