 **SENNHEISER**

SENNHEISER ELECTRONIC KG., D-3002 WEDEMARK

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User's Guide

Mode d'Emploi

Infrarot-Empfänger

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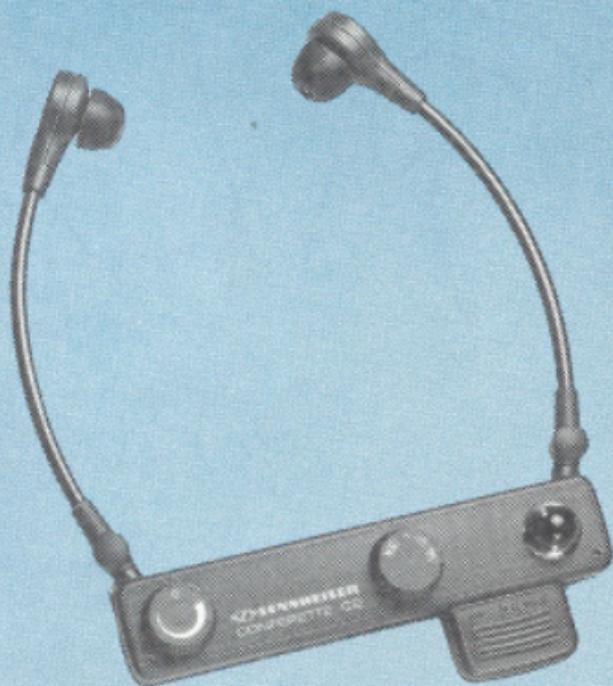
Infrared Receiver

+ Hearing Amplifier

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Conferette C 2-1

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INFRARED RECEIVER + HEARING AMPLIFIER »CONFERETTE C 2-1«

Introduction

The "Conferette C 2-1" is a hearing amplifier for people who only require such a device in certain circumstances, in order to make use of it when demanded by the situation. The size and shape of the unit are already an indication that the "Conferette C 2-1" differs from all conventional hearing aids. However, these are not the significant characteristics of the "Conferette C 2-1". The real difference between this device and other hearing aids is that it amplifies sound stereophonically as a complete unit, and not monaurally in the usual manner. Technically, this innovation was made possible by fitting two very small electret microphones which are physically arranged in such a way that the natural, stereophonic hearing is retained. After individually adjustable amplification, the spoken word is applied to two high-quality dynamic earphones. The voice of the person wearing the "Conferette C 2-1" is also picked up by the microphones; in order to prevent this voice from reaching the ears at an unnaturally high level after passing through the amplification stages, the user's own voice is suppressed by means of a compensation circuit.

However, the function of the "Conferette C 2-1" is not merely restricted to receiving audible signals, but also allows reception of the signals of an infrared transmitter (e.g. SI 405 S or SI 406 S); when this transmitter is connected to a television set, the system serves for wireless TV sound transmission. The modulated, infrared signal emitted by the transmitter is received by the "Conferette C 2-1" and converted to audible signals. Since the microphones are also operating during the wireless sound transmission, other people can still talk to the user at any time. In order to prevent ambient noise from impairing the wireless sound transmission, the microphones only switch on automatically in this mode when a defined sound level is reached, i.e. when the person wearing the device is spoken to with normal speech volume from a distance of 1 m.

The user can select one of two modes:

1. MIC

In this mode the "Conferette C 2-1" operates purely as an audio amplifier. The infrared receiver section is switched off.

2. IR + MIC

In this mode, audio signals can be received as well as the infrared signals.

This description has covered the basic functioning of the unit. The following relates to operating the unit:

The User's Guide is subdivided into two sections. The first section describes **adjustments which are only to be made by hearing-aid specialists**. The second section contains the adjustments required to operate the unit.

A. Basic adjustment

Disassembly

Pull the rear part of the case to the rear, away from the unit. No tools are required for this operation, because the front and rear parts of the case are only interconnected by a snap-fit.

Balance adjustment

With this adjustment, the volume relationship between the two earphones is adapted to the reduced hearing capability of the user, which is not usually the same for both ears. The control is designated PB in Fig. 2. The setting range is ± 30 dB.

Microphone activation threshold for infrared mode

Ex works the basic adjustment is such that the automatic activation circuit responds to the sound pressure produced by a person speaking at a normal speech level, and at a distance of approximately 1 m from the listener. The respond threshold of the automatic circuit, with respect to distance, can be varied within limits by means of PE.

Adjustment of compensation circuit

The "Conferette C 2-1" contains a compensation circuit which prevents the voice of the person wearing the C 2-1 from being amplified at a unnaturally high volume. The degree of compensation may be adjusted with potentiometer PU (fig. 2) according to the main application.

Adjustment: Potentiometer fully anticlockwise: for application as hearing amplifier.
Potentiometer fully clockwise: if C 2-1 is exclusively used for hearing and speech practice.

B. Operating the unit

1. Insert the rechargeable battery plug ③ into the unit. Observe the instructions for use on the red label.
2. Select the mode with switch ② "0" position: unit is switched off. "MIC" position: microphones switched on, infrared receiver section switched off. "IR + MIC" position: infrared receiver section is switched on and microphones are switched on via threshold setting.
3. Place the earpieces in the ears as shown in Fig. 1.
4. Adjust control ① for the desired value. If only hissing noise is heard when the infrared mode has been selected ("IR + MIC"), the infrared transmitter is not yet in operation.
5. For infrared reception ("IR + MIC") lens ④ must point in the direction of the infrared transmitter. Wear the unit in such a way that the lens is always directed towards the transmitter, i.e. the television set, and so that it is not covered by your hand or clothing.

7

8

C. Charging the rechargeable battery plug

The rechargeable battery plug can power the «Conferette C 2-1» for about four hours a day. Then simply pull it out of the unit and plug it into an AC power socket so that it will be recharged. After twenty hours, it is again ready for use. If the unit is used for more than four hours per day, please use the spare plug supplied.

If you observe the following points carefully, the rechargeable battery plug will prove to be reliable for a long time.

1. After each period of use, insert the rechargeable battery plug into an AC power socket for recharging. If the battery is not used for a long time, remove it from the AC power socket.
2. Remember to switch off the unit. A full discharge of the rechargeable battery can shorten its service life (total discharge). If a total discharge has taken place, the recharging period is approximately sixty hours.

Technical data

Hearing amplifier section

Acoustical mode of operation	Electret pressure microphone
Electrical AF-response	300-12000 Hz
Amplification	approx. 60 dB max. (depending on the volume adjusted)

Infrared receiver section

Electrical AF-response	50-12000 Hz
Receiving frequency	95 kHz

General data

Max. sound pressure level	approx. 112 dB (120 dB with silicon ear pads)
Power source	Rechargeable battery plug GZS 406-1, 2.2 - 2.8 V
Operating time	approx. 4 hours (depending on the volume adjusted)
Charging time	approx. 20 hours
Weight	approx. 80 g

We reserve the right to alter specifications, in particular with regard to technical improvements.

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