**SAVE THESE INSTRUCTIONS**

NOTE: Check ALL parts for shipping damage. If shipping damage is noted, DO NOT use. Contact carrier/dealer for further instruction.

**⚠ WARNING**

- **DO NOT** use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as owner's manuals, service manuals or instruction sheets supplied with this product or optional equipment. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment - otherwise, injury or damage may occur.
- Each individual should **ALWAYS** consult with their physician or therapist to determine proper adjustment and usage.
- **DO NOT** use the Walklite or Rollite as a wheelchair or transport device. Walklites and Rollites are not intended to be propelled while seated.
- A physical/occupational therapist should assist in the height adjustment of the product for maximum support and stability.
- Make sure that all hardware is secure, attachments are securely tighten and locked in the open position, and that casters and moving objects are in good working order before using this or any mobility aid.
- The wheels and glide brakes must be in contact with the floor at all times during use.
- The glide brakes are intended to provide additional stability while in the seated position. The glide brakes are not intended to be used for stopping the Walklite during ambulation.
- **DO NOT** rock the Walklite or Rollite while sitting on the seat.
- **ALWAYS** observe the weight limit on the labeling of your product. Check that all labels are present and legible. Replace if necessary.
- If the Walklite or Rollite is exposed to extreme temperature (above 100°F or below 32°F), high humidity and/or becomes wet, prior to use, ensure handgrips **DO NOT** twist on the Walklite or Rollite handles - otherwise damage or injury may occur.
- **DO NOT** attempt to reach objects if you have to move forward in the seat. Reaching for these objects will cause a change to the weight distribution of the Walklite or Rollite and may cause the unit to tip, resulting in injury or damage.
- Before attempting to reach objects or pick them up from the floor by reaching down between your knees, place feet securely on the floor. Use extreme caution when reaching for any object.
- **DO NOT** hang anything from the frame of the Walklite or Rollite. Items should be placed in the basket.
- The basket has a weight limitation of 11 lbs (5 kg). Items placed in the basket should not protrude from the basket as this may cause the product to tip.

**⚠ ACCESSORIES WARNING**

- Invacare products are specifically designed and manufactured for use in conjunction with Invacare accessories. Accessories designed by other manufacturers have not been tested by Invacare and are not recommended for use with Invacare products.

**Patient Height Specifications**

NOTE: For models not listed, please check labeling on product.

| WALKLITE MODELS      | PATIENT HEIGHT MIN. / MAX. |
|----------------------|----------------------------|
| 6300-JRA             | 4'11" - 6'0"               |
| 6300-ATA             | 5'5" - 6'5"                |
| ROLLITE MODELS       | PATIENT HEIGHT MIN. / MAX. |
| 65100, 65100R, 68100 | 5'4" - 6'0"                |
| 65100-JR, 65100R-JR  | 4'11" - 5'7"               |
| 68100-TA             | 5'11" - 6'6"               |

**Opening/Closing the Walklite or Rollite**

**Opening**

1. Swing the side frames outward until the release paddles lock.

NOTE: An audible "click" will be heard.


2. Lower the seat and snap the seat supports onto both side braces.

NOTE: An audible "click" will be heard.

3. Grab the side frames and attempt to fold them in towards the front crossbrace. The unit should remain open.

**Closing**

1. Grab the handle and lift the seat up.
2. Depress either release paddle and pivot the side frame in towards the front crossbrace.
3. Repeat STEP 2 to close the other side frame.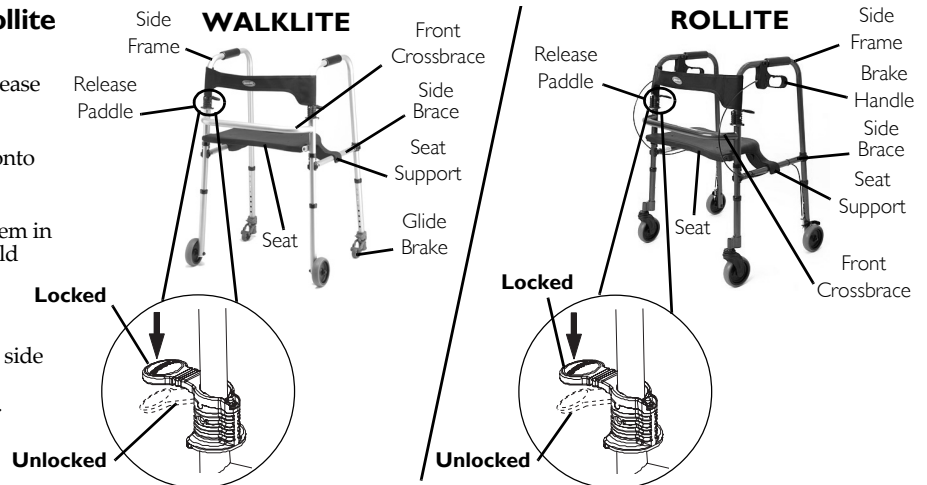

**Using the Rollite Basket**

NOTE: The Rollite Basket is an optional accessory.

Place the hooks of the basket onto the front crossbrace of the Rollite.

**Using the Glide Brakes (Walklites Only)**

**To Become Stationary**

Put the seat down and carefully sit down. Adjust your weight as necessary until evenly distributed on the seat.

**To Continue Mobility**

Grasp both side handles and carefully lift yourself up into a standing position.

**Using the Hand Brakes (Rollites Only)**

**To Become Stationary**

Pull up on the brake handles. To lock the hand brakes, push down on the brake handles until they lock into place.

**To Continue Mobility**

Release the brake handles. If the brakes are locked, slightly pull up on the brake handles until the brakes release.

For more information regarding Invacare products, parts, and services, please visit [www.invacare.com](http://www.invacare.com)

## Installing/Adjusting Leg Extensions /Brake Cables

**NOTE:** Adjust the product's height so that when the user's arm is down to their side, the hand grip is at wrist height. This will ensure the arms are at an approximate 20° - 30° bend when using the product.



1. Press the snap buttons on each of the front legs and slide the extension up or down to the desired height.

**NOTE:** Ensure that the snap button protrudes through the same height adjustment hole on the each leg extension.

**NOTE:** An audible "click" will be heard.

2. Push/pull the brake cables along the side braces so that excess brake cable is near the rear leg extensions (Rollite models only).
3. Repeat STEP 1 for the rear leg extensions (Rollite models only).
4. Coil the excess brake cable by pulling the cable through the cable tie wraps until the excess cable loops out in front of the side frame (Rollite models only).

**NOTE:** For the brakes to function properly, the brake cable **MUST** be flush along the leg extension and side brace and the excess brake cable should loop in front of the side frame.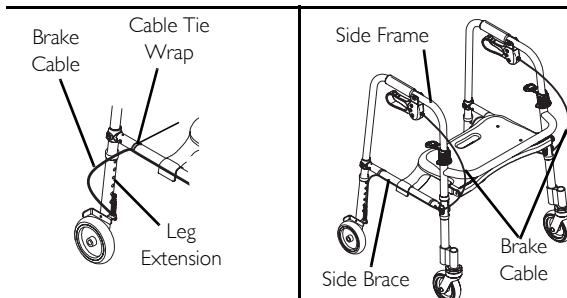
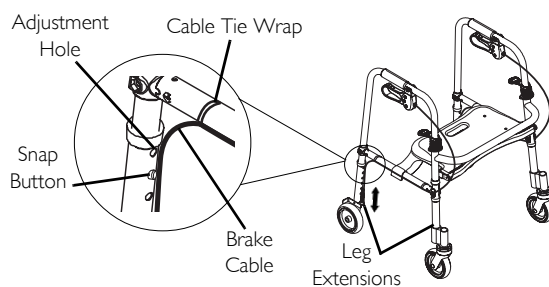


## Adjusting the Hand Brakes (Rollites Only)

### Brake Handle

1. Turn the brake handle adjustment nut clockwise to loosen.
2. Turn the brake handle nut counterclockwise to loosen, or clockwise to tighten.
3. Turn the brake handle adjustment nut counterclockwise to secure in place.
4. Repeat STEPS 1 - 3 for opposite side, if necessary.

**NOTE:** If further tension is needed, refer to [Cable Adjuster Unit](#).

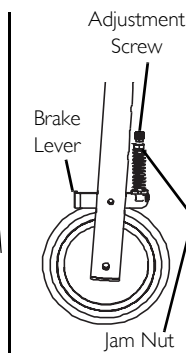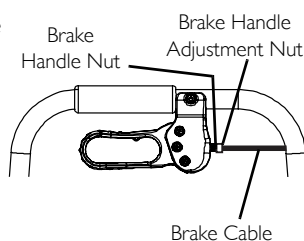### Cable Adjuster Unit

1. Loosen the adjustment screw on the brake clamp.
2. Adjust the brake wire through the brake clamp to make it tighter or looser, as needed.
3. Tighten the adjustment screw.

**NOTE:** There may be excess brake wire. It may be necessary to coil the excess wire to keep it from interfering with brake action. Refer to [Installing/Adjusting Leg Extensions /Brake Cables](#).

4. Repeat STEPS 1-3 on the opposite brake, if necessary.

## Care and Maintenance

1. Verify operation of the brakes and adjust them as necessary.
2. If handgrips are loose, DO NOT use the product. Contact your Invacare dealer.
3. Inspect wheels periodically for wear and damage.
4. Replace any broken, damaged or worn items IMMEDIATELY.
5. Periodically inspect the casters and caster stems for tightness and verify that the wheels are free of hair, lint and other debris that could interfere with free wheel operation.

## Limited Warranty

**PLEASE NOTE: THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.**

This warranty is extended only to the original purchaser/user of our products.

This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state.

Invacare warrants the brakes to be free from defects in materials and workmanship for five years, and all other components for one year for the original purchaser (excluding tires and wheels). If within such warranty period any such product shall be proven to be defective, such product shall be repaired or replaced, at Invacare's option. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of any such product. Invacare's sole obligation and your exclusive remedy under this warranty shall be limited to such repair and/or replacement.

For warranty service, please contact the dealer from whom you purchased your Invacare product. In the event you do not receive satisfactory warranty service, please write directly to Invacare at the address on the back page. Provide dealer's name, address, model number, date of purchase, indicate nature of the defect and, if the product is serialized, indicate the serial number.

Invacare Corporation will issue a return authorization. The defective unit or parts must be returned for warranty inspection using the serial number, when applicable, as identification within thirty (30) days of return authorization date. DO NOT return products to our factory without our prior consent. C.O.D. shipments will be refused; please prepay shipping charges.

**LIMITATIONS AND EXCLUSIONS: THE WARRANTY SHALL NOT APPLY TO PROBLEMS ARISING FROM NORMAL WEAR OR FAILURE TO ADHERE TO THE ENCLOSED INSTRUCTIONS. IN ADDITION, THE FOREGOING WARRANTY SHALL NOT APPLY TO SERIAL NUMBERED PRODUCTS IF THE SERIAL NUMBER HAS BEEN REMOVED OR DEFACED; PRODUCTS SUBJECT TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE; OR PRODUCTS MODIFIED WITHOUT INVACARE'S EXPRESS WRITTEN CONSENT INCLUDING, BUT NOT LIMITED TO: MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS; PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE; PRODUCTS DAMAGED BY CIRCUMSTANCES BEYOND INVACARE'S CONTROL; PRODUCTS REPAIRED BY ANYONE OTHER THAN AN INVACARE DEALER, SUCH EVALUATION SHALL BE SOLELY DETERMINED BY INVACARE.**

**THE FOREGOING EXPRESS WARRANTY IS EXCLUSIVE AND IN LIEU OF ANY OTHER EXPRESS WARRANTIES WHATSOEVER, WHETHER EXPRESS OR IMPLIED, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE AND THE SOLE REMEDY FOR VIOLATIONS OF ANY WARRANTY WHATSOEVER, SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. THE APPLICATION OF ANY IMPLIED WARRANTY WHATSOEVER SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESS WARRANTY PROVIDED HEREIN. INVACARE SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.**

**THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE/PROVINCIAL LAWS AND REQUIREMENTS.**

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All rights reserved. Trademarks are identified by the symbols ™ and ®. All trademarks are owned by or licensed to Invacare Corporation unless otherwise noted.
© 2008 Invacare Corporation