

Personal PA™ R35 Receiver

Instructions For Use & Care

SAFETY INFORMATION

Hearing Safety:



CAUTION!

This product is designed to amplify sounds to a high volume level which could potentially cause hearing damage if used improperly. To protect your hearing and the hearing of others:

1. Make sure the volume is turned down before putting on the earphone or headphone before adjusting the volume to a comfortable level.
2. Set the volume level at the minimum setting that you need to hear.
3. Do not allow children or other unauthorized persons to have access to this product.

BATTERY SAFETY & DISPOSAL



CAUTION!

Do not attempt to recharge disposable batteries, which may explode, release dangerous chemicals, cause burns, or other serious harm to the user or product.

PACEMAKER SAFETY



CAUTION!

1. Before using this product with a pacemaker or other medical device, consult your physician or the manufacturer of your pacemaker or other medical device.
2. If you have a pacemaker or other medical device, make sure that you are using this product in accordance with safety guidelines established by your physician or the pacemaker manufacturer.



RECYCLING INSTRUCTIONS



Help Williams Sound protect the environment! Please take the time to dispose of your equipment properly.

Product Recycling:

Please do NOT dispose of your Williams Sound equipment in the household trash. Please take the equipment to a electronics recycling center; OR, return the product to the factory for proper disposal.

Battery Recycling:

Please do NOT dispose of used batteries in the household trash. Please take the batteries to a retail or community collection point for recycling.

4/11/07

APPROVALS

This product meets approval standards established for:

FCC RoHS WEEE



OPERATING INSTRUCTIONS

1. Install two (2) AA alkaline or NiMH rechargeable batteries. If you're using rechargeable batteries, they must be charged before using.
2. Plug the earphone into the "EAR" jack on the top of the unit.
3. Turn the receiver on by rotating the volume control knob clockwise. The receiver's ON indicator should illuminate green.
4. Adjust the volume for your comfort.
5. To turn the receiver off, rotate the volume control knob counter-clockwise until it clicks off. The receivers' ON indicator should go dark.

Note: To maximize battery life, turn the receiver off when it's not in use.

DISCLAIMER: FCC RULES LIMIT USE OF THIS EQUIPMENT TO AUDITORY ASSISTANCE.

EARPAD CLEANING

Do not immerse the earphone in water or other cleaning agent. Foam pads may be removed and washed with a mild laundry soap solution, rinsed thoroughly, and air-dried. Foam pads may also be replaced with new pads.

BATTERY INFORMATION

Alkaline batteries (BAT 001):	100 hours
Rechargeable NiMH batteries (BAT 026):	50 hours per charge

Batteries from other suppliers may provide different operating life.

NOTE: The ON indicator will illuminate RED to indicate a low battery

LIMITED WARRANTY


Williams Sound products are engineered, designed and manufactured under carefully controlled conditions to provide you with many years of reliable service. Williams Sound warrants the Personal PA™ R35 Receiver against defects in materials and workmanship for FIVE (5) years. During the first five years from the purchase date, we will promptly repair or replace the Personal PA™ R35 Receiver.

Earphones, headphones, batteries, cables, carry cases, and all other accessory products carry a 90-day warranty. Chargers carry a one year warranty.

WILLIAMS SOUND HAS NO CONTROL OVER THE CONDITIONS UNDER WHICH THIS PRODUCT IS USED. WILLIAMS SOUND, THEREFORE, DISCLAIMS ALL WARRANTIES NOT SET FORTH ABOVE, BOTH EXPRESS AND IMPLIED, WITH RESPECT TO THE PERSONAL PA™ R35 RECEIVER, INCLUDING BUT NOT LIMITED TO, ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. WILLIAMS SOUND SHALL NOT BE LIABLE TO ANY PERSON OR ENTITY FOR ANY MEDICAL EXPENSES OR ANY DIRECT, INCIDENTAL OR

CONSEQUENTIAL DAMAGES CAUSED BY ANY USE, DEFECT, FAILURE OR MALFUNCTION OF THE PRODUCT, WHETHER A CLAIM FOR SUCH DAMAGES IS BASED UPON WARRANTY, CONTRACT, TORT OR OTHERWISE. THE SOLE REMEDY FOR ANY DEFECT, FAILURE OR MALFUNCTION OF THE PRODUCT IS REPLACEMENT OF THE PRODUCT. NO PERSON HAS ANY AUTHORITY TO BIND WILLIAMS SOUND TO ANY REPRESENTATION OR WARRANTY WITH RESPECT TO THE PERSONAL PA™ R35 RECEIVER. UNAUTHORIZED REPAIRS OR MODIFICATIONS WILL VOID THE WARRANTY.

The exclusions and limitations set out above are not intended to, and should not be construed so as to contravene mandatory provisions of applicable law. If any part or term of this Disclaimer of Warranty is held to be illegal, unenforceable or in conflict with applicable law by a court of competent jurisdiction, the validity of the remaining portions of this Disclaimer of Warranty shall not be affected, and all rights and obligations shall be construed and enforced as if this Limited Warranty did not contain the particular part or term held to be invalid.

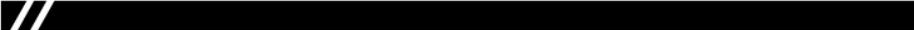

If you experience difficulty with your system, call Toll-Free for Customer Assistance:

1-800-843-3544 (U.S.A.) or +1 952 943 2252 (Outside the U.S.A.)

If it is necessary to return the system for service, your Customer Service Representative will give you a Return Authorization Number (RA) and shipping instructions.

Pack the system carefully and send it to:

Williams Sound Corp.
Attn: Repair Dept.
10321 West 70th Street
Eden Prairie, MN 55344



INSTALLING AND REPLACING BATTERIES

1. Open the battery compartment by lifting the tab on the back of the receiver with a finger.
2. To remove depleted batteries, pull up on the fabric strip.
3. If Alkaline (non-rechargeable) batteries are being installed, slide the battery selection switch above the battery compartment to the “Alkaline” position. If installing NiMH (or rechargeable) batteries, slide the battery selection switch to the “NiMH” position.
4. Press the batteries into place over the fabric strip. **Be sure to observe proper polarity (+/-).** Damage due to improper battery installation may void the warranty on the product. Close the lid.
5. When the sound becomes weak or distorted, replace or recharge batteries.

BATTERY CHARGING

Rechargeable batteries must be charged overnight before first time use. Drop-in charger points are located on the bottom of the R35 unit. For charger availability, please contact Williams Sound at (800) 843-3544.

CHANNEL TUNING

First, set the transmitter to the desired channel. Remove the antenna and connect an audio source. Move the receiver 25 feet away from the transmitter for tuning. Open the battery compartment on the R35 by lifting the tab on the back of the receiver with a finger. Locate the metal tuning slug above the batteries next to the “Channel Tuning” label. Using a small screw driver or tuning wand, turn the tuning slug in a counter-clockwise direction about two turns. Then, slowly turn the tuning slug in the clockwise direction until the signal is heard. There may be two signal points heard. The one which is received first is a false response. Be sure to continue tuning slightly further to the correct point, which will be much louder. Tune back and forth to find the center of the point of best response to the program being heard.



WILLIAMS SOUND®

Helping People Hear

10321 West 70th St., Eden Prairie, MN 55344 U.S.A.
800.843.3544 | 952.943.2252 | FAX: 952.943.2174
www.williamsound.com

FRM 452E ©2007 Williams Sound Corp.