LIFE SOURCE®

ADVANCED/DELUXE ONE STEP AUTO-INFLATION

Blood Pressure Monitor

Trilingual Instruction Guide – MODEL UA-767 Plus
English • Français • Español

THIS MONITOR IS CLINICALLY VALIDATED

This LifeSource® blood pressure monitor has undergone and passed a Clinical Validation study to determine the performance and accuracy. This monitor is clinically validated according to the British Hypertension Society (BHS) standard. The BHS is the most prestigious international blood pressure organization. The published study was performed by a reviewing committee consisting of physicians and/or nurses. Among blood pressure manufacturers, A&D Medical is proud to have the highest number of Clinically Validated monitors with a positive recommendation from professionals that are available to consumers.

The monitor is recommended by the Canadian Hypertension Society.

Fast. Easy. Accurate.

Congratulations on purchasing a state-of-the-art LifeSource® blood pressure monitor—one of the most technologically advanced monitors available today. Designed for ease of use and accuracy, this monitor will facilitate your daily blood pressure regimen.

Physicians agree that daily self-monitoring of blood pressure is an important step individuals can take to maintain cardiovascular health and prevent the serious consequences associated with undetected and untreated hypertension.

A&D Medical has been manufacturing high quality healthcare products for over 20 years. We are committed to providing you and your family with monitoring devices that provide the utmost in accuracy and convenience.

LifeSource - Designed for Life.

WHAT DISPLAY SYMBOLS MEAN

Display Symbol	Condition/Cause	Recommended Action
	The symbol shown while measurement is in progress. It blinks while detecting the pulse.	Measurement is in progress, remain as still as possible.
	An irregular heartbeat or body movement may have occurred. Refer to page E-8 for more information on irregular heartbeats.	Take measurement again and consult with your physician.
	Previous measurement stored in MEMORY.	
 Battery Full	The battery power indicator during measurement.	Replace all batteries with new ones when the symbol blinks. Alkaline batteries recommended.
 Battery Low	The batteries are low when it blinks.	
	Unstable blood pressure due to excessive body movement.	Try the measurement again. Remain very still during the measurement.
	The systolic and diastolic values are within 10 mmHg of each other.	Fasten the cuff correctly, and try the measurement again.
	The pressure value did not increase during inflation.	Check for air leaks along the tube and around the air socket.
 	The cuff is not fastened correctly.	Refasten the cuff and retake measurement.
	There is an air leak in the cuff or monitor.	Make sure tube is properly connected to cuff and monitor.
 PULSE DISPLAY ERROR	The pulse is not detected correctly.	Try the measurement again. Remain very still during the measurement.
	Cuff Inflation Meter	Measurement is in progress, remain as still as possible.
	Pressure Rating Indicator™	Refer to the section "About Pressure Rating Indicator" for further explanation.

HOW UA-767 PLUS WORKS

LifeSource® blood pressure monitors are easy to use, accurate and digitally display full measurement readouts. Our technology is based on the "oscillometric method" – a noninvasive blood pressure determination. The term "oscillation" refers to any measure of vibrations caused by the arterial pulse. Our monitor examines the pulsatile pressure generated by the arterial wall as it expands and contracts against the cuff with each heartbeat.

The cuff is inflated until the artery is fully blocked. The inflation speed is maximized and the pressure level is optimized by the device. The monitor takes measurements while the cuff is both inflating and deflating. This results in a faster measurement providing greater comfort to the user.


BEFORE YOU START

You must install 4 type AA (1.5 volt) batteries (alkaline batteries recommended), or use the AC Adapter (see page E-11 for using the Monitor with AC Adapter) and attach the cuff to the monitor before using it. To install batteries (or replace them if the "Low Battery" symbol appears on display), proceed as follows:

1. Remove battery compartment cover by gently pushing down on arrow and sliding cover forward.
2. Put in bottom row of batteries first. Place the batteries in compartment with positive (+) and negative (-) terminals matching those indicated in the compartment. Be sure batteries make contact with compartment terminals.
3. Replace cover by sliding it into the compartment and gently pressing into place.

NOTE: Rechargeable batteries will not work with this monitor.

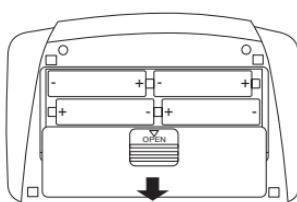
TAKING YOUR BLOOD PRESSURE

Tips for Blood Pressure Monitoring:

- ☞ Relax for about 5 minutes before measurement.
- ☞ Do not smoke or ingest caffeine at least 30 minutes prior to measurement.
- ☞ Remove constricting clothing and place cuff on bare arm.
- ☞ Unless your physician recommends otherwise, use left arm to measure pressure.
- ☞ Do not talk during measurement.
- ☞ Do not cross legs and keep your feet flat on floor during measurement.

Now you are ready. Follow these simple steps:

1. Sit comfortably with your left arm resting on a flat surface so that the center of your upper arm is at the same height as your heart.

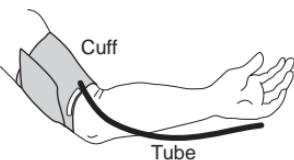2. Lay left arm on the table, palm up and thread cuff end through metal loop, smooth side against arm. Then position the tube off-center toward the inner side of arm in line with the little finger.

3. Pull the end of the cuff to tighten it, fold back the extra material, and fasten securely. The cuff should be snug but not too tight. You should be able to insert two fingers between the cuff and your arm.

IMPORTANT: Measure pressure at the same time each day.

4. Confirm that the index ▲ points within the proper fit range.

WHAT IS AN IRREGULAR HEARTBEAT

The UA-767 Plus Blood Pressure Monitor provides a blood pressure and pulse rate measurement even when an irregular heartbeat occurs. The Irregular Heartbeat symbol «» will appear in the display window in the event an Irregular Heartbeat has occurred during measurement. An irregular heartbeat is defined as a heartbeat that varies by 25% from the average of all heartbeat intervals during the blood pressure measurement. It is important that you relax, remain still and refrain from talking during measurements.

NOTE: We recommend contacting your physician if you see this «» symbol frequently.

ABOUT CUFF INFLATION METER

The Cuff Inflation Meter is located on the left side of the display screen to tell you when the blood pressure monitor is inflating and deflating the cuff. The Cuff Inflation Meter moves up during inflation and moves down during deflation.

Inflation in Progress	Inflation Complete	Deflation/Measurement in Progress

ABOUT MEMORY

This monitor automatically stores up to thirty (30) blood pressure and pulse measurements in memory. Measurements stored in memory are assigned an index number in the order of the newest to the oldest. The oldest reading displays as "n01". The M symbol in the upper left corner of the display screen indicates that you are viewing a previous measurement stored in memory.

To retrieve readings, follow these simple steps:

1. When the display screen is blank, press and quickly release the Memory ("M") button. You will see the average blood pressure reading displayed. Press and quickly release the Memory button again. You will see the index number of the most recent reading followed by the measurement.
2. If you want to retrieve other measurements in memory, press and release the Memory button repeatedly until the desired index number is reached (e.g. n04). The measurement will then be displayed.
3. The display will shut off automatically a few seconds after the readings are displayed.

To clear the measurement history from memory, press and hold the Memory button for at least five seconds while the display screen is blank. Release the button when you see the M symbol flash and disappear in the upper left hand corner of the display. This indicates that the memory has been cleared.

If there are no measurements stored in memory, you will see two 0s displayed vertically followed by a blinking "A00".

NOTE: Stored measurements will be lost if batteries become low or are removed.

ABOUT BLOOD PRESSURE

■ What Is Blood Pressure?

Blood pressure is the force exerted by blood against the walls of the arteries. Systolic pressure occurs when the heart contracts; diastolic pressure occurs when the heart expands. Blood pressure is measured in millimeters of mercury (mmHg).

■ What Affects Blood Pressure?


Blood pressure is affected by many factors: age, weight, time of day, activity level, climate, altitude and season. Certain activities can significantly alter one's blood pressure. Walking can raise systolic pressure by 12 mmHg and diastolic pressure by 5.5 mmHg. Sleeping can decrease systolic blood pressure by as much as 10 mmHg. Taking your blood pressure repeatedly without waiting an interval of 5 minutes between readings, or without raising your arm to allow blood to flow back to the heart, can also affect it.

In addition to these factors, beverages containing caffeine or alcohol, certain medications, emotional stress and even tight-fitting clothes can make a difference in the readings.

■ What Causes Variations In Blood Pressure?

An individual's blood pressure varies greatly from day to day and season to season. For hypersensitive individuals, these variations are even more pronounced. Normally, blood pressure rises during work or play and falls to its lowest levels during sleep.

Fluctuation within a day (case: 35 year old male)

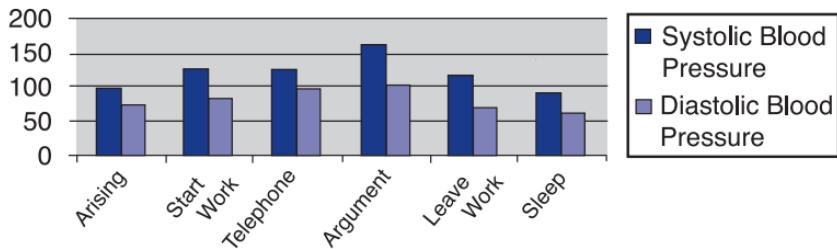ABOUT BLOOD PRESSURE

■ What Is Hypertension?

Hypertension (high blood pressure) is the diagnosis given when readings consistently rise above normal. It is well known that hypertension can lead to stroke, heart attack or other illness if left untreated. Referred to as a "silent killer" because it does not always produce symptoms that alert you to the problem, hypertension is treatable when diagnosed early.

■ Can Hypertension Be Controlled?

In many individuals, hypertension can be controlled by altering lifestyle and minimizing stress, and by appropriate medication prescribed and monitored by your doctor. The American Heart Association recommends the following lifestyle suggestions to prevent or control hypertension:

- **Don't smoke.**
- **Reduce salt and fat intake.**
- **Maintain proper weight.**
- **Exercise routinely.**
- **Have regular physical checkups.**
- **Monitor your blood pressure at periodic intervals.**

■ Why Measure Blood Pressure at Home?

It is now well known that, for many individuals, blood pressure readings taken in a doctor's office or hospital setting might be elevated as a result of apprehension and anxiety. This response is commonly called "white coat hypertension." In any case, self-measurement at home supplements your doctor's readings and provides a more accurate, complete blood pressure history. In addition, clinical studies have shown that the detection and treatment of hypertension is improved when patients both consult their physicians and monitor their own blood pressure at home.

ABOUT BLOOD PRESSURE

■ Establishing Baseline Measurements

The most important method to get an accurate blood pressure measurement is consistency. To get the most benefit out of your monitor, it is important to establish a “baseline measurement.” This helps build a foundation of measurements that you can use to compare against future readings. To build this baseline measurement, devote two weeks for consistent blood pressure monitoring. This involves doing everything the same way when you measure (e.g. measuring during the same time of day, in the same location, sitting in the same chair, using the same cuff, etc.). Once you establish your baseline measurement, you can start evaluating if your measurement has been affected based on things lifestyle changes or medication treatment.

■ How Do I Record My Blood Pressure?

Blood pressure readings are typically recorded with the systolic pressure written first, followed by a slash mark and the diastolic pressure. For example, 120 mmHg systolic and 80 mmHg diastolic measurements are written as 120/80. Pulse is simply written with the letter “P” followed by the pulse rate—P 72, for example. Please see the back of the manual for the blood pressure tracking record.

■ Customer Support Tools Online

Additional tools are available on www.LifeSourceOnline.com (in Canada - www.autocontrol.com) to help you get the most out of your blood pressure monitoring. These include:

- Large Print Instruction Manuals
- Animated Operating Instructions for select models
- Additional Logbook Sheets

SPECIFICATIONS

Model	UA-767P (UA-767 Plus)
Type	Oscillometric
Display	Digital character height Pressure (16.0 mm)/pulse (10.0 mm) displayed simultaneously
Memory	30 readings
Measurement range	Pressure: 20 mmHg to 280 mmHg Pulse: 40 pulses to 200 pulses per minute
Accuracy	Pressure: ± 3 mmHg or $\pm 2\%$, whichever is greater Pulse: $\pm 5\%$
Pressurization	Automatic, using micropump
Depressurization	Automatic constant speed
Power source	4 type "AA" (1.5 volt) alkaline batteries (not included) or 120V AC Adapter
Battery life	Approximately 6 months with one daily measurement
Operating environment	50°F to 104°F (10°C to 40°C) Less than 85% relative humidity
Storage environment	-4°F to 140°F (-20°C to 60°C) Less than 95% relative humidity
Dimensions	Length: 4.3" (110 mm) Width: 5.6" (142 mm) Height: 2.5" (64 mm)
Weight	10.6 oz. (300 g) without batteries

Blood pressure measurements determined by the UA-767 Plus are equivalent to those obtained by a trained observer using the cuff/stethoscope auscultation method within the limits prescribed by the American National Standards Institute for electronic or automated sphygmomanometers.

IMPORTANT!

If You Need Assistance with the Set-Up or Operation

We Can Help!

Please call us FIRST before contacting your retailer at

LifeSource Health Line

1-888-726-9966 - Toll Free

Auto Control Medical

1-800-461-0991 - In Canada

A specially trained representative will assist you

**A division of A&D Engineering, Inc.
1756 Automation Parkway
San Jose, CA 95131
www.LifeSourceOnline.com**

Pressure Rating Indicator is a trademark and LifeSource is a registered trademark of A&D Medical.
Pressure Rating Indicator is patent pending.

©2007 A&D Medical. All rights reserved.

Specifications subject to change without notice.

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