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Features:

- Provides assistance to those who have difficulty stepping over the bathtub walls
- Anodized aluminum frame is lightweight, durable, and rust resistant
- Drainage holes in seat minimizes water build up
- Reversible backrest to accommodate any bathroom.
- Support Arm support aids in lateral transfer
- Seat height is adjustable in ½" increments
- 350lbs maximum weight capacity – 7927A
- 400lbs maximum weight capacity – 7929
- Designed to meet the requirements of HCPCS code: E0247/E0248 (for 7929A)
- Lifetime Limited Warranty

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© 2009, GF Health Products, Inc., 2935 Northeast Parkway, Atlanta, Georgia 30360.
Made in China. www.grahamfield.com

Specifications subject to change without notice
Especificaciones son sujetas a cambio sin aviso

GF0500029RevB09

Transfer Bench de Banco de Transferir

Items 7927A and 7929

User Instructions and Warranty Information

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Instructions for Use – Please Read All Instructions Before Use.

Attaching the Leg Extensions *(if not attached already)*

1. Place frame on its side on a level surface.
2. Attach leg extensions with suction feet to the frame on the same side as where the support arm will be. Attach leg extensions with flange tips to frame on opposite side of frame.
3. The transfer bench legs with suction feet should be the two legs that are to be placed inside the tub.

Attaching the Backrest

1. Determine which direction the user will face when sitting on the bench in the tub.
2. Position the bench on a level surface.
3. Locate the channel guides (4 each) on underside of seat. Align backrest support tubes with channel guides.
4. Slide backrest seat support tubes through channel guides until seat support tube push buttons protrude fully through channel guide holes.

NOTE: Ensure seat support tubes buttons protrude fully through channel guide holes.
CAUTION: Backrest is not designed to support the full weight of the user.

CAUTION: Ensure backrest is facing down correctly before inserting to avoid improper installation.

Attaching the Support Arm

1. To attach the support arm, position the seat arm against the leg frame and slide the armrest tubing into the tube openings. Make sure that the push buttons fully protrude through the holes in the bracket.
NOTE: An audible “click” will be heard.
2. To remove the seat arm, depress the push buttons and pull up.

NOTE: The end with the Support Arm and Suction Feet is the side that should be inside the tub.

Adjusting Leg Height

1. Depress the height adjustment button and slide the lower leg upward or downward until the height adjustment button is fully engaged in the desired hole.
NOTE: An audible “click” will be heard.
2. Repeat for other 3 legs.

CAUTION: Ensure that all four legs are adjusted to the same height, and that all height adjustment buttons protrude fully through adjustment holes before use.

Using Your Transfer Bench

NOTE: Be sure to move slowly throughout use to prevent injuries.
NOTE: Children or those who need assistance should NOT be left unattended during use of this product.

From a seated position (such as a wheelchair):

1. Place your wheelchair next to the bench, facing the front of the bathtub.
2. Engage wheelchair locks.
3. Transfer your torso to the transfer bench first, and then lift your legs into the tub. Move slowly to prevent injuries.
4. After bathing, lift your legs out of the tub first, then your torso.

From a standing position:

1. Sit on the side of the transfer bench that is outside of the tub.
2. Extend your arms to grab the support arm of the bench and slide your body over onto the bench. Lift each leg over the tub slowly.
3. Position yourself to a comfortable position in center of bench.
4. After bathing, lift your legs out of the tub first, then your torso slowly.

Maintenance

- Use a mild soap and water solution, rinse with clear water, then dry with a soft cloth. Ensure that product is well rinsed before using.
- Never use organic solvents, abrasive cleaners, or scouring pads on any part of the product.
- Check fit and tightness of all nuts, bolts and knobs weekly to ensure that product is safe to use.
- To guarantee the best service from your product, always specify Lumex replacement parts, available from your GF Health Products, Inc authorized distributor.

For more information regarding GF Health Products, Inc. products, parts and services, please contact your distributor or visit us online at www.grahamfield.com.

Safety Guidelines – Please Read Before Use
NOTE: Check ALL parts for shipping damage. If shipping damage is noted, DO NOT use. Contact carrier/dealer for further instruction.

WARNING

WARNING: DO NOT use this product without first completely reading and understanding these instructions and any additional instructional material such as owner's manuals, service manuals or instruction sheets supplied with this product. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment - otherwise, injury or damage may occur.

WARNING: Users with limited physical capabilities should be supervised or assisted when using the transfer bench.

WARNING: Item 7927A has a maximum weight capacity of 350 lbs
Item 7929 has a maximum weight capacity of 400lbs

WARNING: GF Health Products Inc. assumes no responsibility for any damage or injury caused by improper installation, assembly, or use of this product.

WARNING: Leg extensions with rubber tips MUST be in contact with the floor at all times.

WARNING: Ensure that the push buttons fully protrude through the same height adjustment hole of each leg extension. This ensures that the leg extensions are securely locked in position and that an even height adjustment is achieved.

WARNING: Make sure that all attaching hardware, screws, nuts and/or bolts are tight at all times.

WARNING: Inspect rubber tips on the leg extensions for rips, tears, cracks or wear or if they are missing. Replace them IMMEDIATELY if any of these conditions exist.

WARNING: The transfer bench seat MUST be installed before sitting on the transfer bench.

WARNING: If components are damaged or missing, contact your GF Health Products Inc. distributor immediately.

Lifetime Limited Warranty

Quality is the hallmark of all Lumex® products. Trusted by healthcare professionals and end-users for over 50 years, Lumex features a full line of patient aids and specialty healthcare seating.

Lumex products are designed and built to precise standards and are carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products. We are dedicated to providing products that will give our customers years of dependable service.

GF Health Products, Inc. hereby warrants specific parts of the Lumex products listed on the cover of this manual for the lifetime of the product against defective workmanship and/or material defects when used for the purpose intended, under normal conditions, and provided the product receives proper care

Parts covered under the lifetime warranty include: (frame, welding, screws, frame locking components). Specific items "excluded" from this warranty of any kind are: "ALL" other parts not specifically listed above. The warranty does not extend to non-durable components, which are subject to normal wear and require periodic replacement.

Should a product defect that is covered by this warranty arise, we will, at our option, repair or replace the product.

For additional information on this Lumex product or this warranty, please contact an authorized GF Health Products, Inc. Distributor.

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