Deluxe Automatic Inflate Blood Pressure Monitor with Heart Sense®

Made in Taiwan

Walgreens c/o SAI

105 CHALLENGER ROAD, 6FL
RIDGEFIELD PARK, NJ 07660

Toll Free Customer Service

1.800.261.0130

- Large Three Row Digital Display
- 3 Zones of 30 Memory - 90 Memory Total
- Time and Date
- Pulse Display
- Memory Average of last 3 Measurements
- AC Capability and Adapter

Distributor: Walgreen Co.
Deerfield, IL 60015

In This Manual

Important Warnings/Medical Disclaimer	1
Healthy Living®	2-3
Commonly Asked Questions	4-7
Intended Use	7
Accuracy	8
Measurement Method	8
Special Conditions	9
Special Features	9
Main Unit and Accessories	10
Digital Display Symbols	11
Battery Installation	11
AC Capability and Adapter	12
Selecting Time and Date	13
Important Reminders	14-15
Applying the Cuff	16
Taking a Blood Pressure Measurement	17-18
Storing Measurements	19
Recalling Stored Measurements	19-20
Reset Function	20
Taking Care of Your Monitor	21
Troubleshooting	22-23
Limited Lifetime Warranty	24-25
Specifications	26
Blood Pressure Diary	27-29

Important Warnings/Medical Disclaimer

Before using your blood pressure monitor, you should read and understand all instructions and follow all warnings.

Information in this manual is provided for informational purposes only. This manual and product are not meant to be a substitute for the advice provided by your own physician or other medical professional. You should not use the information contained herein or this product for diagnosing or treating a health problem or disease, or prescribing any medication. If you have or suspect that you have a medical problem, promptly contact your healthcare provider.

This blood pressure monitor is the first step to living a healthier life. But monitoring your blood pressure at home is just the beginning, it's also important to:

EXERCISE REGULARLY - Start out by taking a short walk each day. Before you know it, you'll feel energized. That's because along with being an easy way to shed a few pounds, walking also helps bones and hearts to become stronger.

EAT RIGHT - Remember, you are what you eat! So when you reach for a snack, try grabbing a piece of fruit or a vegetable. When cooking meals at home try to use fats and oils sparingly. Sometimes a healthy diet is the best medicine of all! It's also a good idea to monitor your sodium intake because about one-third of individuals who have high blood pressure are also sodium sensitive.

BE HAPPY! - A positive attitude is a big part of a healthy body. Try not to get too stressed out over things. Daily relaxation techniques such as gentle yoga or even quiet time by yourself can do a lot of good for the mind and body. Stay focused on the good things in life. Begin by congratulating yourself for that short walk you just took or the smart food choice you made. It's your first step to a healthier life!

KEEP AN EYE ON YOUR WEIGHT! - Surprisingly, losing even a small amount of weight can often times help control high blood pressure. Talk with your doctor to find a "healthy" weight for you and strive to come within a range of it.

Remember to consult your physician prior to beginning exercise, vitamin/mineral/dietary changes or programs and blood pressure monitoring.

Commonly Asked Questions

Why Should I Monitor My Blood Pressure At Home?


By monitoring at home, you can manage your health between doctor visits. Home monitoring also provides you with a record of blood pressure measurements over time to give to your doctor - a valuable tool in monitoring hypertension.

Home monitoring may also help alleviate *White Coat Hypertension*. This term refers to the medical phenomena which causes many individuals' blood pressure to be higher when taken in a medical office setting. That's because many times the stress of being in the doctor's office may cause blood pressure to rise as much as 20-50 points higher than normal. When monitoring at home, you can get a truer indication of your blood pressure in a normal, relaxed state.

What Is Blood Pressure?

The commonly used term "blood pressure" refers to the force or *pressure* that is exerted on the body's arteries as blood flows through them. Each time the heart beats or *contracts* it produces pressure in the arteries.

This blood pressure is what moves blood through the body, supplying oxygen and nutrients to every organ.

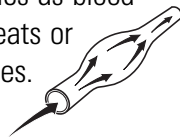Commonly Asked Questions

When the heart contracts it sends blood out into the body. This force of blood within the arteries is called **systolic** pressure.

The pressure that remains as the heart relaxes between beats is called **diastolic** pressure.


systolic

diastolic

pulse

What Is Considered Normal Blood Pressure?

The well-respected World Health Organization (WHO) has set forth the following guidelines regarding high (commonly referred to as hypertension), low and borderline blood pressure.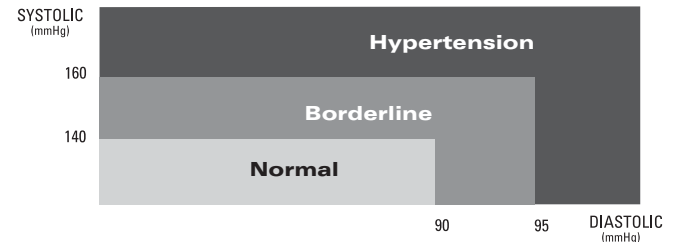

Commonly Asked Questions

Why Does My Blood Pressure Fluctuate?

Because blood pressure changes with every beat of the heart it is in constant fluctuation throughout any given day. In addition to these natural changes there are other factors that may affect blood pressure such as:

- Temperature (too hot or cold)
- Humidity
- Restlessness
- Lack of sleep
- Physical exercise
- Constipation
- Stress
- Fear
- Anger
- Anxiety
- Food and beverage consumption
- Individual physical conditions

In addition to these factors you should talk with your physician to see what may be affecting you.

How Do I Know I Have High Blood Pressure?

It's nearly impossible to know if your blood pressure is elevated without having it checked. People have the common misconception that one can "feel" their blood pressure rise. It is because of this mystery that hypertension has been named "the silent killer." Don't let it sneak up on you! Start monitoring your blood pressure every day with your new Walgreens monitor.

Commonly Asked Questions

Why Is It Important To Control My Blood Pressure?

Left uncontrolled, hypertension can place an incredible amount of stress on your heart and arteries, both of which become forced to work harder to keep blood flowing in a healthy manner throughout the body. Over time, this stress may result in health problems including over-enlargement of the heart, loss of elasticity of the arteries, heart disease, stroke, and even death. Making some simple lifestyle changes now may help you to stay healthy in the future.

Intended Use

Walgreens blood pressure monitors are intended for home use. They are noninvasive, meaning no part of the monitor enters your body, and provide systolic, diastolic and pulse measurements. Walgreens monitors are recommended for use by people over the age of 18 and are not suitable for clinical use.

Accuracy

Walgreens monitors have been clinically tested against a scientific device called a *sphygmomanometer*, considered the gold standard in blood pressure measurement. All Walgreens monitors have performed equivalent to measurements taken with this scientific device and are within the accuracy limits prescribed by the American National Standard for Electronic or Automated Sphygmomanometers.

Measurement Method

Walgreens monitors measure blood pressure and pulse by using what's called an *oscillometric* method, meaning they measure the fluctuations in pressure. This is done by first wrapping a fitted cuff around the upper arm. The monitor then automatically fills the cuff with air creating pressure around the arteries inside of the arm. Mounted inside of the cuff is a special gauge that is capable of sensing the small *oscillations* (fluctuations) in pressure. These fluctuations are produced as a result of the pressure the cuff has placed around the arm and are, in a basic sense, the arteries pulsing (getting larger and smaller) with each heart beat. The monitor then measures how high and how quickly these pulses occur and converts that information into a digital value. A special valve, called an *electrode*, deflates the monitor automatically when the measurement is complete.

Special Conditions

All Walgreens blood pressure monitors are safe and accurate to use with arrhythmias such as atrial or ventricular premature beats and atrial fibrillation.

Special Features

This Walgreens monitor with Heart Sense® technology senses your blood pressure before the measurement is complete - the monitor then adjusts the air pressure amount according to your individual level. The results are quick, comfortable measurements each time you check your blood pressure.

Large Three Row Display

A large LCD displays the time, date and your blood pressure - systolic over diastolic - to make measuring your blood pressure easy to read.

Memory Feature

Your Walgreens Deluxe Automatic Inflate monitor features a 3 party memory capability or 3 "zones" - each zone stores 30 measurements for a total storage capacity of 90 measurements. This unique feature allows 3 users to use and store their blood pressure readings separately all on one monitor!

Time and Date


This monitor automatically keeps track of the time and date and remains on when the monitor is not in use for easy reference.

Main Unit and Accessories


Main Unit
Date


Accessories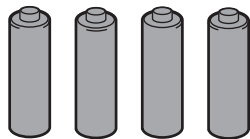


4 "AA" Alkaline Batteries

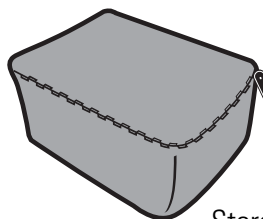Storage Pouch

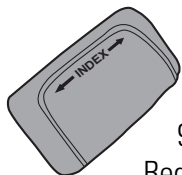
9"~13"
Regular Cuff

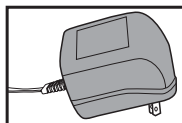AC Adapter

Digital Display Symbols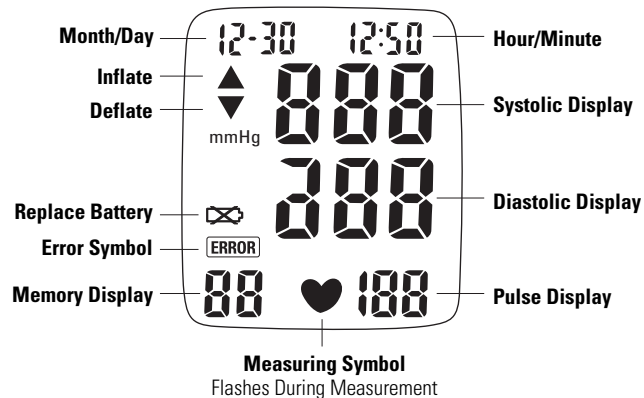



Battery Installation

When the display shows , it's time to replace the batteries. This monitor uses 4 "AA" alkaline batteries.

Insert the batteries into the monitor making certain to align the polarities correctly.

Close the battery cover securely. It will snap into place when it is fully closed.

* It's a good idea to remove the batteries if you know you will not be using your batteries for an extended amount of time.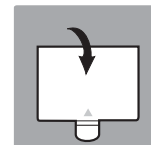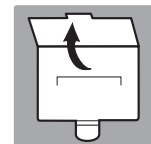


AC Capability and Adapter

This Walgreens blood pressure monitor is equipped with AC capability should you choose to use electric power instead of batteries to operate your unit.

Use only the authorized AC adapter included with your monitor. Use of any other adapter may cause damage to the monitor and will void the user warranty. For warranty or replacement information refer to pages 24-25.

AC Adapter Output: 6V DC 1A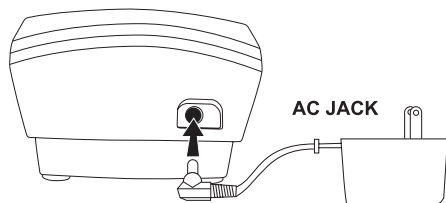

*It is suggested that you remove the batteries from the compartment if you are going to use an AC adapter for an extended amount of time.

Selecting Time and Date

Your Walgreens monitor will keep track of the time and date unless the batteries have run down or are removed from the monitor.


- Press the MODE button on the unit. The month will begin to flash. Press the ▲ (Up) or ▼ (Down) button until the correct month appears on the display.
- Press the MODE button once again to store and now the day will begin to flash. Press the ▲ (Up) or ▼ (Down) button until the correct day appears on the display.
- Press the MODE button to store and the hour will begin to flash. Follow the same process as setting the month and day to set the hour and minute.
- Once you have selected the minute, press the MODE button. The unit will now display the time and date.

Important Reminders

Here are some important reminders:

Before Taking a Measurement

- ✓ Sit quietly for about 10 minutes. This will allow your body to return to its normal, resting state.
- ✓ Take your blood pressure in a comfortable environment - not too cold, not too hot - as temperature may affect your measurement.
- ✓ If you've just consumed beverages containing caffeine such as coffee, non-herbal tea, or cola wait about 30-45 minutes. Also, don't smoke just before taking a measurement.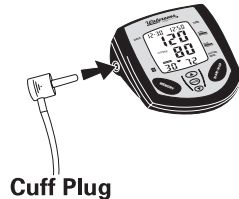


- ✓ Insert the cuff plug into the monitor before taking a measurement.
- ✓ Make sure the cuff is properly assembled before use. To assemble, insert the end of the cuff marked INDEX into the metal ring with the velcro side facing out. The smooth material should be on the inside of the cuff loop.

Important Reminders

While Taking a Measurement


- ✓ Sit with your legs uncrossed and your feet flat on the floor.
- ✓ For consistency, it's a good idea to measure your blood pressure using the same arm and at about the same time each day. If possible, we suggest using the left arm.
- ✓ Relax. Sit still and quietly while measuring. Talking or moving may elevate measurements. Do not touch the cuff or monitor during the measurement.
- ✓ When taking multiple measurement right after each other, make to wait at least 8-10 minutes in between. Waiting allows your blood vessels to return to their normal state.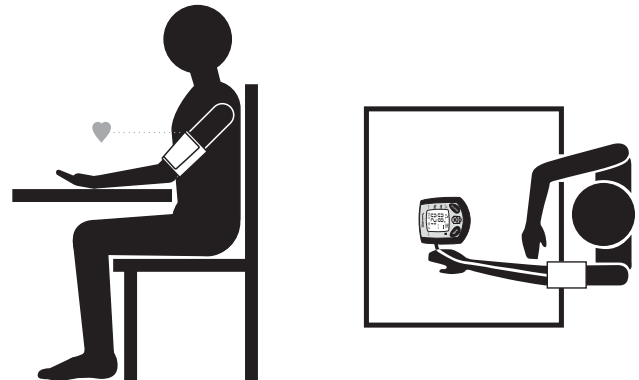


Applying the Cuff

1. If your arm measures 9"-13" around, use the regular cuff included with your monitor. If your arm measures 13"-17" around, please use a Walgreens Large Cuff (Model #BC-200W).

2. Before taking a measurement, read the Important Reminders on pages 14-15. Make certain the cuff plug is inserted securely in the monitor and the cuff is assembled properly. Remove any constricting or thick clothing from your upper arm.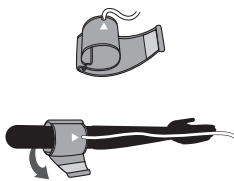


3. Slide your arm through the cuff loop. The cuff should be positioned on your bicep about 1/2" above the elbow. With your palm facing up, the red arterial marker should be in the center of your arm over the brachial artery. The cuff tube should run down the center of your arm. Instructions are also printed on the cuff for reference.

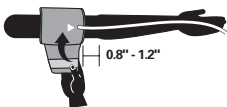4. Secure the cuff around your arm using the velcro so it fits securely but is not too tight. You should be able to place 2 fingers in between the cuff and your arm.

5. The velcro tab should fall within the "OK" range printed on the cuff for proper use. If your arm is larger than 13" around and the cuff does not fall within the "OK" range, please use a Walgreens Large Cuff (Model #BC-200W).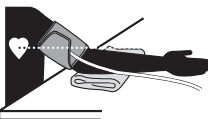


6. Relax your arm and place your elbow on the table.

Taking a Blood Pressure Measurement

1. Once the cuff is properly applied and you are seated with your feet flat on the floor you are ready to take a measurement.


2. Press the ▲ (Up) or ▼ (Down) button and you will see a number appear: 1, 2, or 3. These are each of the 3 party memory zones explained on page 9 (Special Features).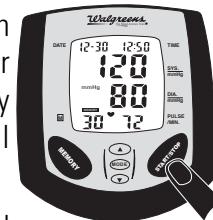

*If you select zone No. 1 your blood pressure will be recorded as one of the 30 readings available for that zone. The same is true for zones 2 and 3. When taking measurements, be sure and select the correct zone number so that the measurement is stored in the appropriate zone.

3. Press the START/STOP button. The monitor will begin to inflate to the level that's right for you. An upwards facing arrow on the digital display indicates the monitor is inflating. Remain still and do not talk during the measurement process.


4. You will feel the cuff get tighter around your arm - this is normal. It should not inflate to the point of discomfort. If this occurs, the cuff may be secured too tightly. To stop the inflation, press START/STOP. The cuff will deflate automatically. Rewrap the cuff allowing more room between your arm and the cuff.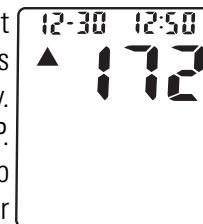

Inflation


Taking a Blood Pressure Measurement

5. Inflation will stop automatically to begin the measurement process. You will see a downward facing arrow and the numbers decreasing as the cuff is deflating. You will also see a flashing heart to indicate the measurement is taking place.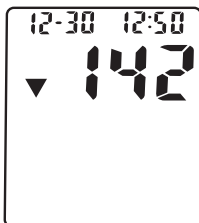

Deflation

Note: Occasionally, to get an appropriate measurement, the monitor may reinflate the cuff to 30mmHg higher than the previous inflation and begin the measurement process again.

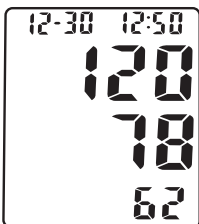6. When your measurement is complete the monitor will deflate automatically, releasing the air pressure in the cuff.

7. The entire measurement process should take less than a minute. Your blood pressure and pulse readings are displayed and automatically stored in the memory.

* The monitor will automatically power off after about one minute.

NOTE: Blood pressure constantly changes during the course of a day. Occasionally, high or low readings do not necessarily mean your monitor is malfunctioning or that you are ill. It is important to speak with your doctor to learn what your normal blood pressure is if you experience abnormal measurements.

Storing Measurements

When you take a measurement the monitor automatically stores it in memory space number one. As you continue to take measurements the monitor will push the previous measurement back one space. In other words, the value in space one will always be the most recent blood pressure measurement taken.

Your monitor features a 3 party memory capability or 3 "zones" - each zone stores 30 measurements a for a total memory capacity of 90 readings. This unique feature allows 3 users to use and store their blood pressure measurements separately all on one monitor. This feature is also useful for one user to track their blood pressure throughout the day: morning, afternoon, and evening.

Recalling Stored Measurements

1. Press the ▲ (Up) or ▼ (Down) button to select a user zone. You will see a number appear: 1, 2, or 3. (Fig. 1)
2. Press the MEMORY button and the average of the last 3 readings will be displayed for that zone. For example, if you select zone No. 1 and you press MEMORY, the average of the last 3 measurements stored in zone No. 1 will appear on the display.
3. Press ▼ (Down) to view the next stored measurement; press ▲ (Up) to go back to the previous measurement.
4. To go back to the main screen for "Recalling Stored Measurements" and to choose another zone, press ▲ (Up) to scroll through the measurements until you reach the most recent measurement and press once more. You can also press START/STOP to return to the main screen.

Recalling Stored Measurements

5. Follow steps 1-3 once again to review measurements in another zone.

*Fig.1

*Fig.2

Reset Function

Your Walgreens monitor will not lose the data stored in the memory even when you change the batteries. To erase all of the data in memory zone, refer to the instructions below.

Note: Single measurements cannot be erased.

Warning: Resetting the memory will permanently erase all of the stored data in the memory zone that you select.

- Press ▲ (Up) ▼ (Down) to choose a zone: 1, 2, or 3.
- Press the MEMORY button. (Fig. 3)
- Press the MODE and ▲ (Up) button at the same time to clear all of the stored data. (Fig. 4)

*Fig.3

*Fig.4

Taking Care of Your Monitor

It's important to treat your monitor with care. Here are some basic tips:

- ✓ Keep the monitor out of direct sunlight, high humidity and extreme temperature changes. 
- ✓ Try not to drop it on the floor.
- ✓ Do not take the monitor or the cuff apart. None of the monitor's parts are fixable at home. If your monitor needs repair, see details in the warranty section of this manual.
- ✓ Keep the monitor free of dust and other materials. Avoid getting any part of the monitor or cuff extremely wet. If the monitor needs cleaning it's best to wipe it with a cloth slightly dampened with water. We recommend that you clean your monitor about once a month.
- ✓ Do not harshly fold or bend the cuff as  this may damage it on the inside.
- ✓ Do not inflate the monitor's cuff if it is not wrapped around your arm.
- ✓ Always remove the cuff tube from the monitor with care.
- ✓ For your safety, do not wrap the cuff around any other body part other than your arm.

Troubleshooting

Error/Symbol What Does It Mean? How To Fix It

Error/Symbol	What Does It Mean?	How To Fix It
ERROR	Monitor may have been inflated without the cuff around the arm <i>OR</i> Cuff may not be positioned correctly on the arm.	Read page 16 (Applying the Cuff).
	The cuff tube may be folded <i>OR</i> You may have moved while the cuff was inflating.	Read pages 14 & 15 (Important Reminders).
	You may have moved or spoken during the measurement process.	Remember that it is important not to move and speak as it may elevate measurements.
	You may have been startled by a noise or vibration while measuring.	Make sure you are in a quiet and comfortable place when taking a measurement.

Troubleshooting

Error/Symbol What Does It Mean? How To Fix It

Error/Symbol	What Does It Mean?	How To Fix It
ERROR	You have been ignoring the flashing replace battery symbol ().	Replace with four new "AA" alkaline batteries. Make sure the polarities are positioned correctly in the battery compartment before continuing to measure your blood pressure.
	The batteries have run down.	Replace with 4 "AA" batteries so you may continue to measure your blood pressure without any interruptions.
Blood pressure value seems too high or too low	Cuff may not be aligned correctly.	Read page 16 (Applying the Cuff).
Blood pressure value seems too high or too low	Circuitry locked.	Remove then reinsert the batteries and then proceed to take measurement again.
Unable to take a measurement after receiving an error symbol.	The cuff may not be positioned correctly.	Read page 16 (Applying the Cuff).

Limited Lifetime Warranty

Manufacturer warrants to the original purchaser that this Deluxe Blood Pressure Monitor (the "Product") will be free from defects in material and workmanship for the life of the original owner under normal use. This Product is for home use only and is not intended for commercial or clinical use. If the Product does not work properly because of defects in material or workmanship, Manufacturer will, at its option, repair or replace the Product, including the cuff, with the same or a comparable model free of charge (excluding shipping & insurance charges). This warranty only extends to the original retail purchaser. Purchase receipt or other proof of date of original purchase is required before warranty performance.

This warranty extends solely to failures due to defects in materials or workmanship occurring during normal use. It does not cover normal wear of the Product. This warranty is void if: the Product housing has been removed, the Product's label, logo or serial number have been removed or if the Product fails to function properly as a result of accident, misuse, abuse, neglect, mishandling, misapplication, defective batteries, faulty installation, setup, adjustments, improper maintenance, alteration, maladjustment of controls, modification, power surges, commercial or clinical use of the product, service by anyone other than an authorized service center or acts beyond the manufacturer's or distributor's control.

The warranty and remedy provided herein are exclusive and in lieu of all other express and implied warranties and unless stated herein, any statements or representations made by any other person or entity are void. NEITHER THE MANUFACTURER, DISTRIBUTOR NOR THEIR AFFILIATES SHALL BE LIABLE FOR ANY LOSS, INCONVENIENCE, INJURY, OR DAMAGE INCLUDING DIRECT, SPECIAL, INCIDENTAL, OR

Limited Lifetime Warranty

CONSEQUENTIAL DAMAGES RESULTING FROM THE USE, IMPROPER USE, FAILURE TO HEED ANY WARNING, DISCLAIMER OR INSTRUCTION OR INABILITY TO USE THIS PRODUCT, WHETHER ARISING OUT OF ANY BREACH OF THIS WARRANTY OR ANY OTHER LEGAL THEORY. ALL EXPRESS AND IMPLIED WARRANTIES, INCLUDING THE WARRANTIES OF MERCHANTABILITY, ARE LIMITED TO THE APPLICABLE WARRANTY PERIOD SET FORTH ABOVE. Manufacturer and Distributor assume no responsibility for injuries, damages or penalties incurred or suffered resulting from the use of this product in an improper manner or manner or location other than for that for which it is intended or from failure to heed any warning, disclaimer or instruction.

Some states do not allow the exclusion or limitation of incidental or consequential damages, or limitations on how long an implied warranty lasts, so the above exclusions or limitations may not apply to you. This warranty gives your specific legal rights and you may also have other rights which vary from state to state.

Should your Product prove defective under this warranty, please return the Product in the original carton prepaid to:

Walgreens c/o SAI
105 Challenger Road, 6th Floor
Ridgefield Park, NJ 07660

Your dated register receipt or other proof of purchase is required along with your returned unit. Also include your name, address, daytime phone and detailed description of the problem. Sorry we cannot deliver to PO Boxes.

Include US\$8.95 check or money order (NON-REFUNDABLE) for return shipping & insurance within the continental U.S. (\$19.95 outside the continental U.S.)

Specifications

Model	BD-7181W
Monitor Type	Deluxe Automatic Inflate
Measurement Method	Oscillometric
Display	Liquid crystal digital display
Measurement Range	Pressure: 0-260 mm Hg Pulse: 40-199 beats/Minute
Accuracy	Pressure: within \pm 3mm Hg Pulse: within \pm 5%
Memory	Stores up to 30 measurements for each zone (90 memory total)
Automatic Power Off	About 1 minute after measurement is complete
Battery Type	4 AA batteries (Alkaline)
Battery Life	About 6 months when used once a day, less when using a large cuff
Storage Temperature	Temperature: -4°F to 140°F Relative Humidity: less than 95%
Operating Temperature	Temperature: 50°F to 104°F Relative Humidity: less than 95%
Dimensions (approx.)	Monitor: 4 $\frac{3}{4}$ " (w) x 5 $\frac{1}{8}$ " (d) x 2 $\frac{7}{8}$ " (h) Regular Cuff: fits arms 9"-13" around
Weight (approx.)	14 oz (including batteries)

For purposes of improvement, specifications are subject to change without notice.

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